

DCM Converter



Version: 1.0.0

Revision 1

Contents

1	Introduction and Overview	3
1.1	Medit DCM Converter Overview	3
1.2	System Requirements	3
1.3	Installation Guide	4
2	User Interface	7
3	Running Medit DCM Converter	8
3.1	Running Medit DCM Converter from Medit Link.....	8

1 Introduction and Overview

1.1 Medit DCM Converter Overview


Medit DCM Converter is a software application that converts 3Shape DCM files to a format usable in the Medit ecosystem. Medit DCM Converter can be run from both clinic and lab accounts in Medit Link.

1.2 System Requirements

	Laptop	Desktop
CPU	Intel Core i5 2.6 GHz or higher	
RAM	16 GB or higher	
Graphics	NVIDIA GeForce GT 760 2 GB (or equivalent AMD video card) or higher	
OS	Windows 8 64-bit or higher (Unavailable in 32-bit)	


1.3 Installation Guide

- ① Log in to your Medit Link account and go to App Box.

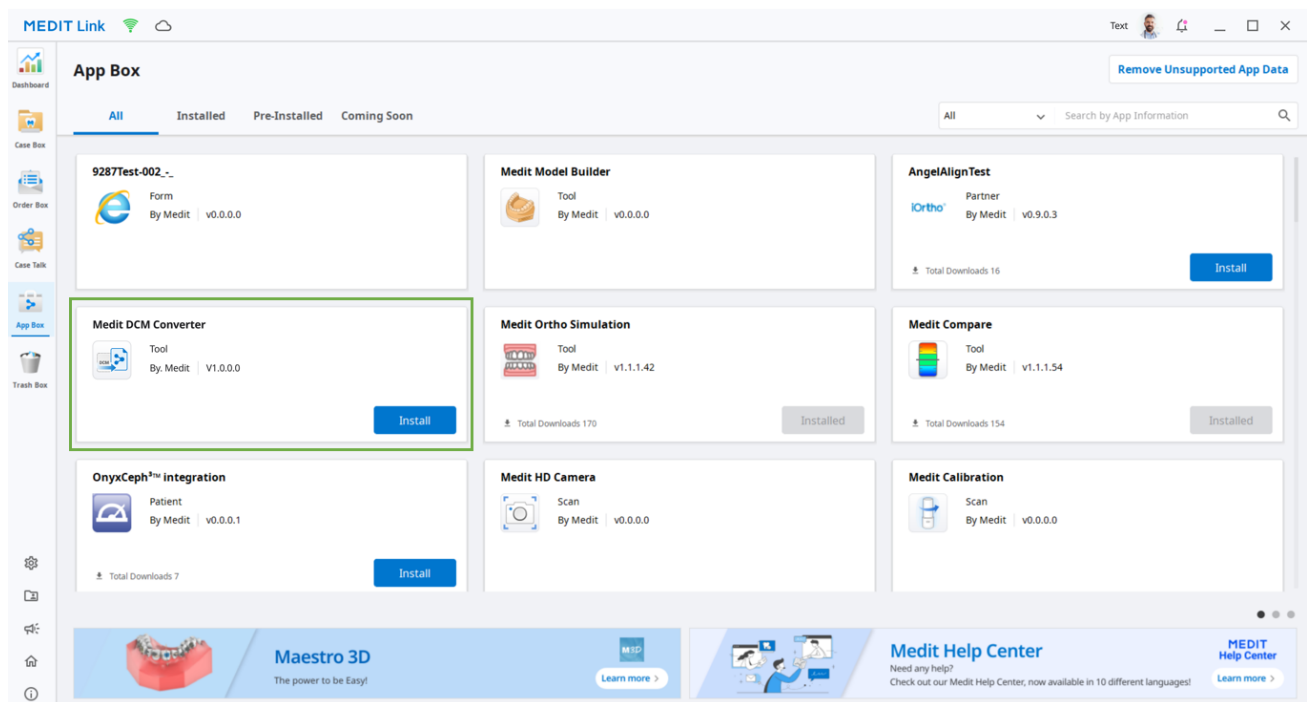

- ② Find the “Medit DCM Converter” app and click on the “Install” button.
- ③ Once the download is complete, the Medit DCM Converter installer will be run automatically.


- ④ Read and agree to the License Terms and Conditions.

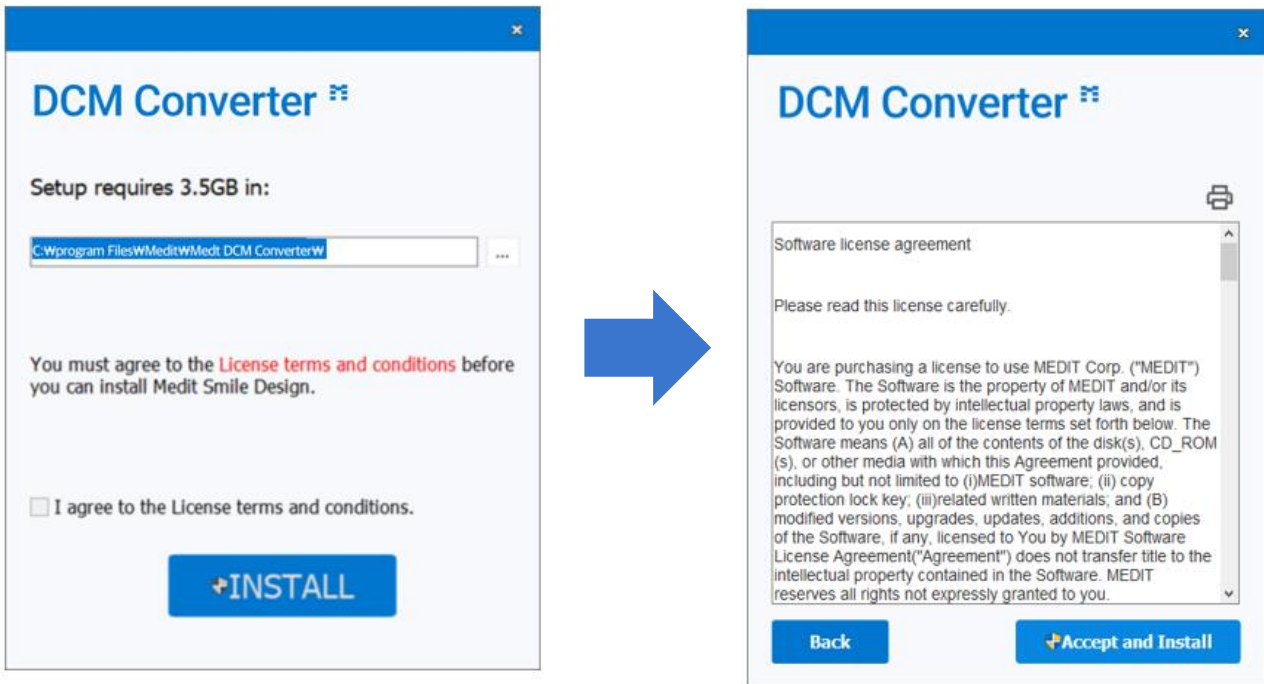

- ⑤ It may take several minutes to finish the installation process. Please do not turn off the PC until the installation is complete.




⑥ Press “Finish” to complete the installation.

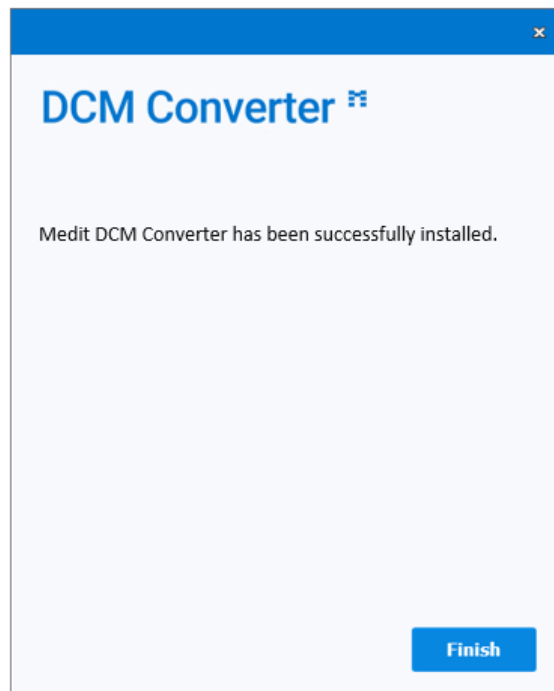

⑦ Restart Medit Link.

2 User Interface

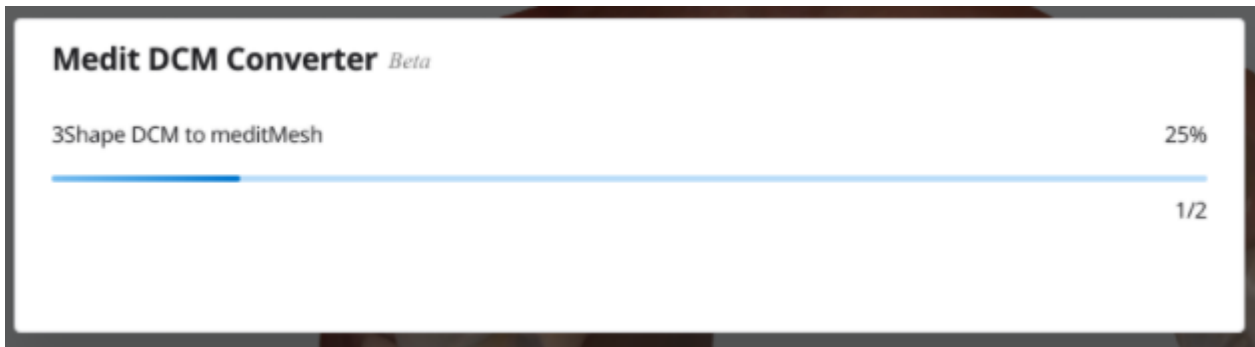



This progress bar shows the file conversion process.


3 Running Medit DCM Converter


3.1 Running Medit DCM Converter from Medit Link

- 1 Go to Case Box (for clinic account) or Work Box (for lab account) and choose a case with 3Shape DCM files attached to it.

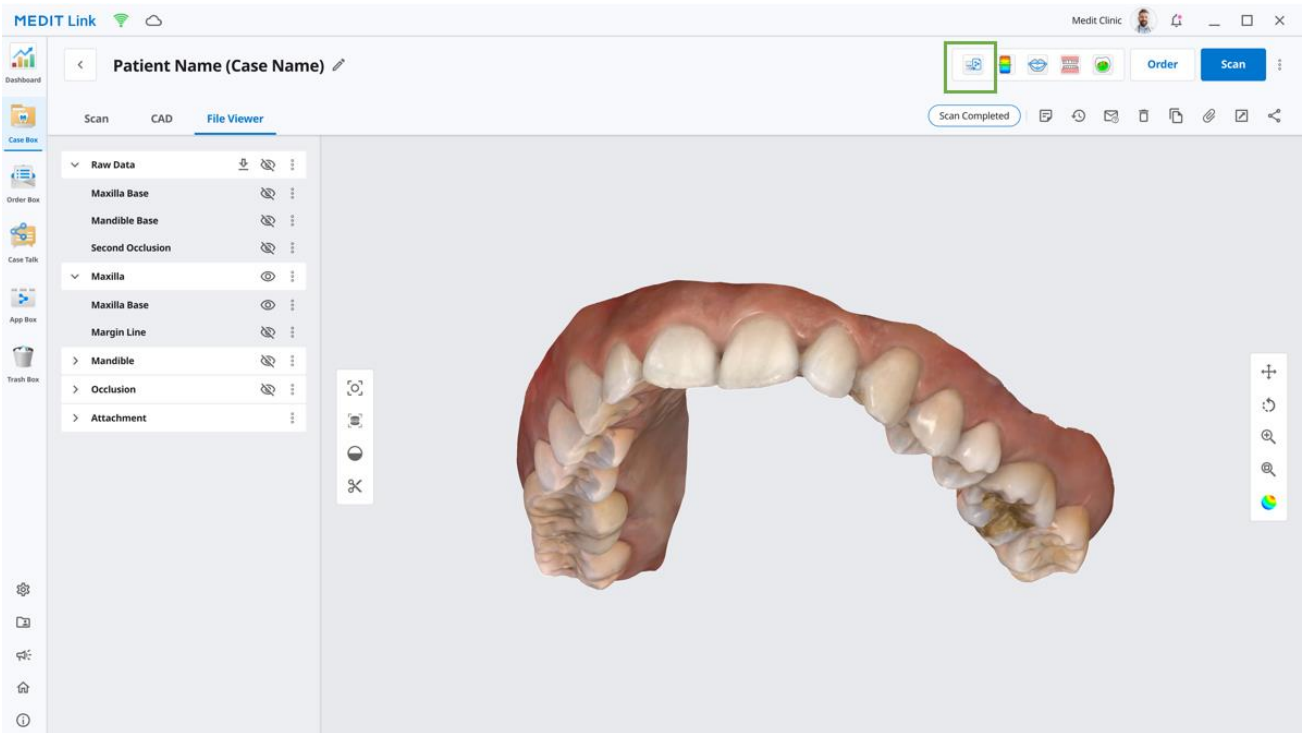

- 2 Press the “Medit DCM Converter”  icon in the upper right corner of the Case Detail window in Medit Link, which will automatically appear once you install the app and restart Medit Link.
- 3 In case there are 3Shape DCM files attached to the case, the conversion will begin straight away.

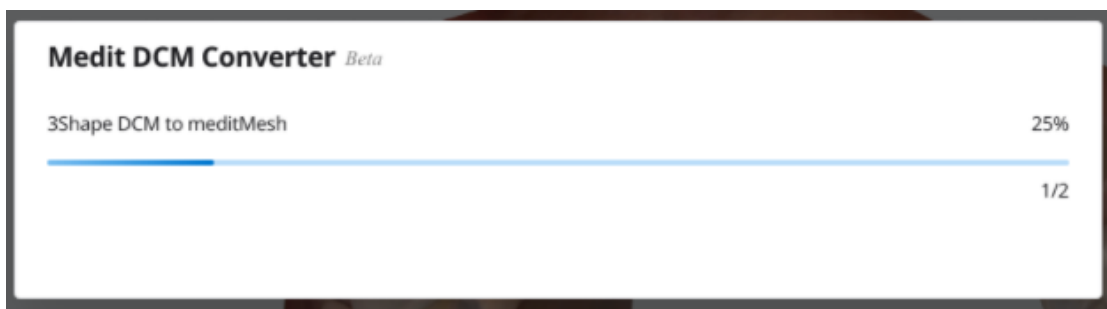

 Note that the DICOM files cannot be converted.

- 4 Once the conversion is completed, the files will be saved as attached, and you will be able to check the files through File Viewer.