Quick Guide

Revision 1 (February 2021)

CONTENTS

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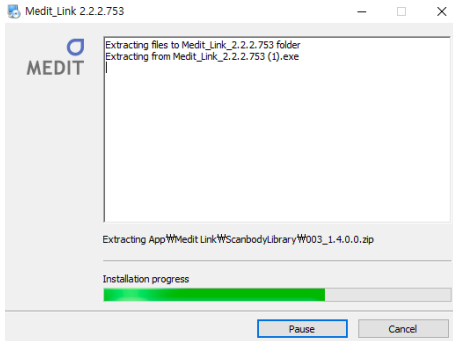
English

1 Software Installation	6
2 Hardware Installation	7
3 Components	8
Accessory Components	9
4 Preparation of accessories	11
5 Product Specification	12
6 Precautions for the User	12
General Precautions	12
During Installation	12
Before Using the Equipment.....	13
Equipment Usage	13
Maintenance.....	13
Disposal	14
Storage	14
Environmental Conditions	14
User Guide Download	15

Software Installation

- ① Run Medit_Link_X.X.X.exe.


- ② Select the folder for the installation of this program. Please agree to the “License terms and conditions” after reading them carefully and then click “Install.”

- ③ The installation might not be successful if the scanner is connected to the PC. Please make sure you disconnect the USB 3.0 cable from your PC before installation.

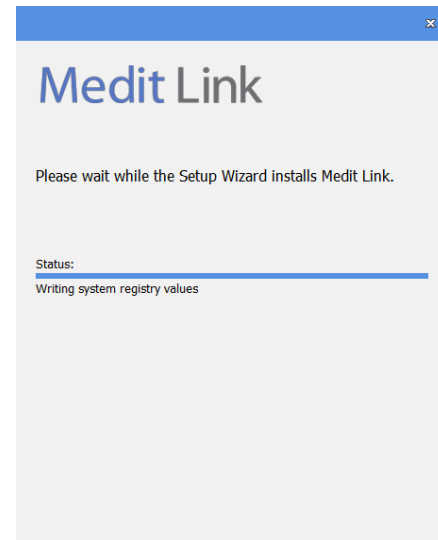- ④ The program will be installed automatically. It may take a few minutes. Do not turn off or unplug your PC until the installation is complete.


- ⑤ Please restart your PC once the installation is complete. If you do not have a scanner you can proceed without restarting the PC.

- ⚠ Please restart your PC once the installation is complete. If you do not have a scanner you can proceed without restarting the PC.

Hardware Installation

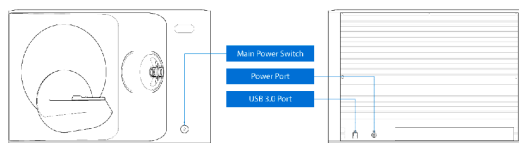How to connect the product :

You can begin installing the hardware after completing software installation and rebooting your PC.

⚠ Caution

The scanner cables should be properly connected to the PC.
(The package includes a power cable and a USB cable.)

* Please only use a USB 3.0 port while connecting the scanner to your PC.

1. Connect the power cable.

2. Connect the USB cable via the USB 3.0 (indicated with Blue color) port. (*Important)

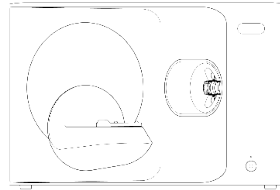
3. Turn on the switch located on the front of the 3D scanner device.


Components


1. 3D Table Top Scanner

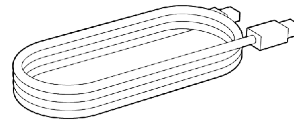2. USB Cable

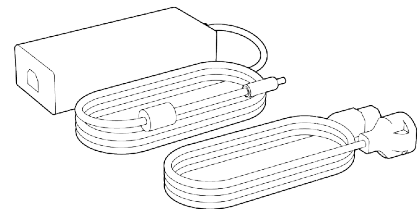3. Power Cable & External Adapter


4. LED Protector


Accessory Components

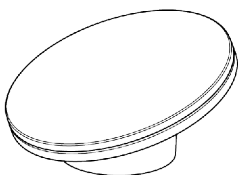5. Calibration Panel

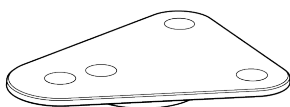6. Articulator Plate


7. USB for Installation


* Installation Guide is included in the manual.

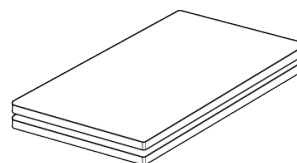8. Blu Tack

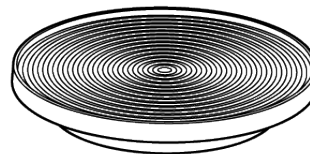9. Flexible Multi-die


10. Single die (16ea)

11. Spray Supporter (2ea)

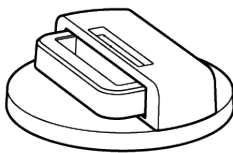
12. Impression Jig


* Not available for T510, T310, and T4.

**Optional Components
(purchased separately)**

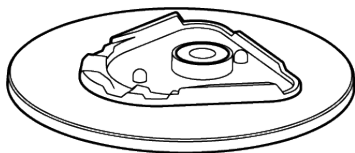
13. KAS Jig

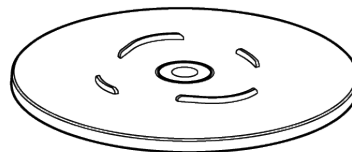14. AM Jig

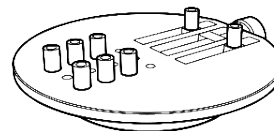
15. Screw Jig


Preparation of accessories

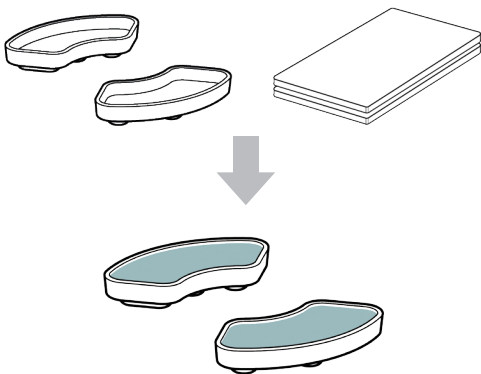
Fill each of the Spray Supporters(2ea) with Blu Tack.

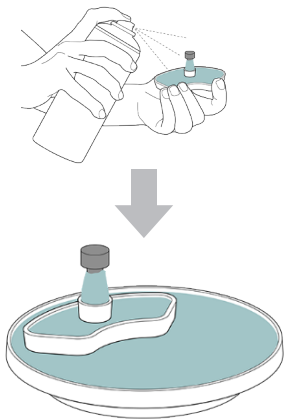
Spray the scan spray onto an object on the spray supporter and fix it to the flexible multi-die for scanning.

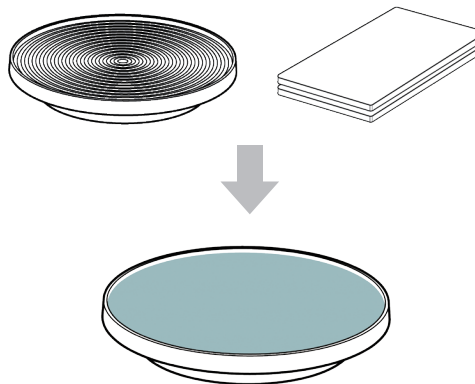Prepare Blu Tack to use on the surface of the Flexible Multi-die.

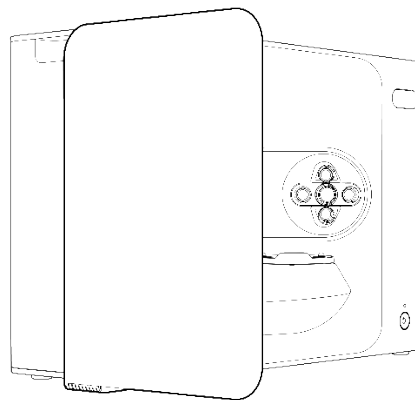Use the LED protector to cover the light of the scanner projector.

Product Specification

Category	3D Scanner	Category	Scanner
Camera resolution	Mono 5.0 (MP)	Light Color	Blue LED
Point spacing	0.040 mm	Scanning of textures	Color textures
Scan Region	103mm x 99mm x 89mm	Connection method	USB 3.0 B Type
Scan Principle	Phase-shifting optical triangulation	Power consumption	Stand BY : 20W (IDLE : 35W, SCAN : 48W)
Size	505mm x 271mm x 340mm	Power requirements	POWER SUPPLY : 100~240VAC, 50/60Hz
Weight	15 kg	Rating	24V --- 5A
Light Source	LED, 150 ANSI-lumens		

Precautions for the User

General Precautions

- This device should only be used by trained professionals.
- Prior to operation of the scanner, the user should be trained on how to use the system or have read and fully understand this User Guide.

During Installation

- This device should be used only by trained professionals.
- Install the device in a dust-free environment with proper ventilation and minimum changes in air pressure, temperature, humidity, sodium and ions.
- Please be aware of the safety conditions such as tilting the device, vibration, or shock.
- Do not install the device in a place with chemical storage or near gas generating points.
- Do not install the device in a place with poor ventilation.
- Please consider the power requirements and consumption.
- To prevent access to the Moving Part, a distance of 1.5 M should be maintained between the scanner and the PC with the operating system.
- Ensure that all packaged components are free from physical damage. Safety cannot be guaranteed if there is any physical damage to any component of the units.
- Install and use only approved programs to ensure proper functionality of the 3D scanner system.
- Hand Hazard
 - ⚠️ ⚠️ Keep hands off while running.
- To prevent access to the Moving Part, a distance of 1.5 M should be maintained between the scanner and the PC with the operating system.
- Access should only be made while the Moving part is not working.

- Do not access (Scan during moving part operation) and maintain a certain distance (1.5M) from the product.

Before Using the Equipment

- Please make sure that all the components and cables are connected properly.
- Please make sure that the device is seen in the Device Manager.
- Avoid using force when manually moving the axis of the scanner.

Equipment Usage

- Make sure that the device does not get shocked during use.
- Do not turn off the scanner during use.
- Do not block any open spots on the device during use.
- If you notice any smoke or smell coming out of the device, turn off the device immediately, disconnect the cables and contact the manufacturer.
- Unplug the power cable if the device is unattended or not being used for a long period of time.
- If there is a problem, do not disassemble or assemble the product on your own. After confirming a problem, please contact your local authorized resellers for repair.
- Please do not modify the product.
- If the equipment fails to operate adequately (e.g., accuracy issues), stop using the device and contact the manufacturer or authorized resellers.
- Manipulating the scanner should only be done while the Moving Part is not in operation.
- Do not manipulate the scanner during scanning or Moving Part operation and maintain a distance of 1.5M from the device.

- Do not station the device so that it is difficult to manage and detach accessories.
- If the device or equipment is used in a manner not specified by the manufacturer, the protection provided by the equipment may be impaired.
- Do not place objects over 1.5 kg on the moving part.
- In case there is an error during the scanning process, it will automatically stop and the LED indicator light on the scanner will turn red.
 - » If the LED indicator light on the scanner turns red, press the 'Initialize Axis' button in the software to fix the problem.
- LED Caution

Risk Group 2

- » Do not stare at the lamp for long periods during operation, it can be harmful to the eyes.

Maintenance

- If the device was not in use for a long time, please make sure it is properly installed and calibrated before reusing it. (The recommended calibration period is one month.)
- Please check if the device is accepting scan data properly.
- Clean the equipment regularly with a dry cloth to avoid getting water inside the device.
- Make sure to unplug the power cable before examining the device.
- Do not apply the scanning spray to the scanner or to any objects located on the Moving Part.
- Do not touch the mirror inside the scanner.
- Replacement of any parts should only be done by service personnel with the use of the appropriate tools.

Disposal

- All components are designed to conform with following Directives :
 - » RoHS, Restriction of the Use of Certain Hazardous Substances in Electrical and Electronic Equipment. (2011/65/EU)
 - » WEEE, Waste Electrical and Electronic Equipment Directive. (2012/19/EU)

Storage

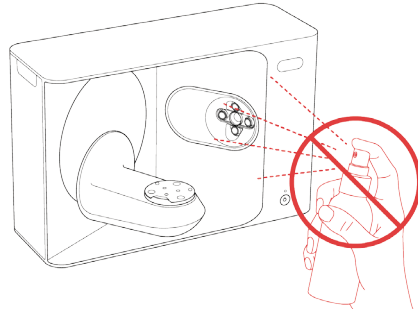
- Wipe the device surface gently with a dry cloth. Make sure that no alien objects or liquids meet it. In case of such contact, immediately wipe the surface without applying force.
- Store the device in a safe place to avoid any damage.

Environmental Conditions


Warning : Observe the following environmental conditions.

- Operating Conditions
 - Temperature : 18 ~ 28°C
 - Humidity : 20 ~ 75% (Non-condensing)
 - Atmospheric pressure : 800hPa ~ 1,100hPa
 - Altitude : Up to 2000m
 - Indoor use
- Storage & Transportation Conditions
 - Temperature : -5 ~ 45°C
 - Humidity : 20 ~ 80% (Non-condensing)
 - Atmospheric pressure : 800hPa ~ 1,100hPa (No condensation)


Do not sprinkle spray inside the scanner.

Do not touch the lenses.

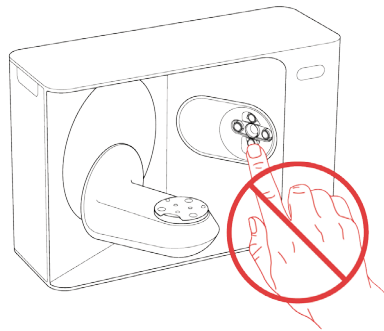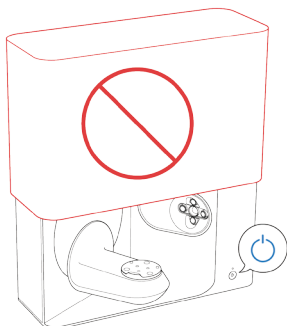Do not operate the scanner with cover on.

Do not store items on top of the scanner.

User Guide Download

- You can download the User Guide from the Menu at the top left of the window > Help.

Manufacturer

Medit Corp

23, Goryeodae-ro 22-gil, Seonbuk-gu, Seoul, 02855 Rep. of Korea

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