
MEDIT Link

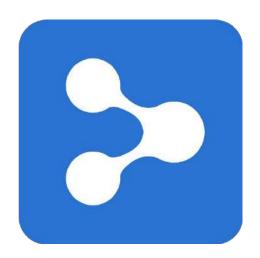

ME-UG-702H Revision 10 (18.03.2025) SW version 3.4.1



Medit Link

Revision 10 (18.03.2025)

Table of Contents

Medit Link	
Introduction	1
Medit Link	1
Service Overview	1
Cloud Service	2
Workflow Management Platform with Ordering System	2
Medit Link > Installation	3
System Requirements	
System Requirements for Windows	3
System Requirements for macOS	3
System Requirements for Web Service	3
Recommended for Browsers	3
Minimum for Mobile Devices	3
Installation on Windows	4
Installation on macOS	6
Medit Link > Getting Started	
Medit Link > Getting Started Medit Link Website	
_	10
Medit Link Website	10 10
Medit Link Website Sign Up Login and Welcome Page Medit Link > Medit Link Web	
Medit Link Website	
Medit Link Website Sign Up Login and Welcome Page Medit Link > Medit Link Web	
Medit Link Website Sign Up Login and Welcome Page Medit Link > Medit Link Web Dashboard	
Medit Link Website Sign Up Login and Welcome Page Medit Link > Medit Link Web Dashboard Dashboard Overview	
Medit Link Website Sign Up Login and Welcome Page Medit Link > Medit Link Web Dashboard Dashboard Overview Medit Insights	
Medit Link Website Sign Up Login and Welcome Page Medit Link > Medit Link Web Dashboard Dashboard Overview Medit Insights Subscribe Medit Insights	
Medit Link Website Sign Up Login and Welcome Page Medit Link > Medit Link Web Dashboard Dashboard Overview Medit Insights Subscribe Medit Insights UnSubscribe Medit Insights	
Medit Link Website Sign Up Login and Welcome Page Medit Link > Medit Link Web Dashboard Dashboard Overview Medit Insights Subscribe Medit Insights UnSubscribe Medit Insights Patient	
Medit Link Website Sign Up Login and Welcome Page Medit Link > Medit Link Web Dashboard Dashboard Overview Medit Insights Subscribe Medit Insights UnSubscribe Medit Insights Patient Search Patient	

	Filter by status	20
	Filter by Search	21
	Case Details	21
Or	der Box/In Box	22
	Order Details	22
	Tabs in Order Box/In Box	22
Ca	se Talk	23
	Creating a Case Talk	23
	Setting the sharing permission type and sharing options	24
	Case Talk details	26
1	Web Viewer	27
	Annotation	28
	Comments	29
	Real-time screen sharing	29
	Data Tree	29
	Data Tree Controls	29
	3D Model Controls	30
Pa	rtners	31
	Search for Partners	31
	My Partner List	32
	Partner Invitation	32
Me	embership	33
	Medit Cloud Storage Plan	33
	2 GB Plan	
	10 TB Plan	34
	Cloud Storage	35
	Subscribe	
	Cancel Subscription	38
	My Subscription	39
	Details	
	Resubscribe and Cancel Subscription	40
	How to add a payment method	
	How to change a payment method	
	Redeem	
	Details	43
	Register Redeem Code	44
	Activate Redeem Code	45
	Payment Details	45
	View Bill Details	
M۱	/ Info	
_	My Account	
	My Clinic/My Lab (Admin Account Only)	
	My Scanner	

Other Menu	50
Notice & Update	50
Related Websites	51
Download Medit Link App	51
Medit Link > Medit Link App	
Overview	
Title Bar	
My Information	
Notifications	
Notice & Update	
Settings	
Network Status	
Settings	
General	
Storage Management	
Set Period	
Local Storage	
Cloud Storage	
Set Connected Program	
Export Options	
Case Converting Tool	
Export Case	
Import Case	64
Proxy Options	66
About	66
User Guide	66
Menu	
Dashboard	67
Patient	67
Add New Patient	68
Delete Patient	68
Merge Patients	69
Case Box/Work Box	70
Tabs in Case Box	70
Functions in Case Box/Work Box	71
Order Box/In Box	72
Tabs in Order Box/In Box	73
Case Talk	73
App Box	74
All	74
Medit Apps	74
Downloaded	75
Trash Box	75
Case Synchronization Manager	76
Upload	76

Related Websites	77
Nelated Websites	77
File Viewer	78
Data Tree	78
Model Control Tools	79
3D Model Controls	79
2D Image Control	80
Function Tools	81
Image Capture and Memo	81
Ortho Capture	81
Fill Holes	
Trimming	
Optimize Occlusion Alignment	
Mandibular Movement	
Manage Multi Occlusion Data in File Viewer	
How to manage multiple results	
View Options	
Status Filters	
View Type	
Grouping	
Sorting	
Filtering and Search	
Filter by Patient	
Filter by Date	
Advanced Search Capabilities	
	95
·	
Medit Link > Workflow Create/Edit/Clone/Delete Case	96
Medit Link > Workflow Create/Edit/Clone/Delete Case	96 96
Medit Link > Workflow	96 96
Medit Link > Workflow Create/Edit/Clone/Delete Case Creating Cases Create Case in Case Box (Clinic)	96 96 96
Medit Link > Workflow	96 96 96 97
Medit Link > Workflow Create/Edit/Clone/Delete Case Creating Cases Create Case in Case Box (Clinic)	96 96 96 97 98
Medit Link > Workflow Create/Edit/Clone/Delete Case Creating Cases Create Case in Case Box (Clinic) Create Case in Work Box (Lab) Editing Cases	96 96 96 96 97 98
Medit Link > Workflow Create/Edit/Clone/Delete Case Creating Cases Create Case in Case Box (Clinic) Create Case in Work Box (Lab) Editing Cases Edit Case in Case Box (Clinic)	96 96 96 97 98 98
Medit Link > Workflow Create/Edit/Clone/Delete Case Creating Cases Create Case in Case Box (Clinic) Create Case in Work Box (Lab) Editing Cases Edit Case in Case Box (Clinic) Edit Case in Work Box (Lab)	96 96 96 97 98 98 99
Medit Link > Workflow Create/Edit/Clone/Delete Case Creating Cases Create Case in Case Box (Clinic) Create Case in Work Box (Lab) Editing Cases Edit Case in Case Box (Clinic) Edit Case in Work Box (Lab) Cloning Cases	96 96 96 97 98 98 99 100
Medit Link > Workflow Create/Edit/Clone/Delete Case Creating Cases Create Case in Case Box (Clinic) Create Case in Work Box (Lab) Editing Cases Edit Case in Case Box (Clinic) Edit Case in Work Box (Lab) Cloning Cases Clone Case in Case Box (Clinic)	96 96 96 97 98 98 99 . 100 . 100
Medit Link > Workflow Create/Edit/Clone/Delete Case Creating Cases Create Case in Case Box (Clinic) Create Case in Work Box (Lab) Editing Cases Edit Case in Case Box (Clinic) Edit Case in Work Box (Lab) Cloning Cases Clone Case in Case Box (Clinic) Clone Case in Case Box (Clinic) Deleting Case Delete Case (Clinic and Lab)	96 96 96 97 98 98 99 . 100 . 100 . 101 . 101
Medit Link > Workflow Create/Edit/Clone/Delete Case Creating Cases Create Case in Case Box (Clinic) Create Case in Work Box (Lab) Editing Cases Edit Case in Case Box (Clinic) Edit Case in Work Box (Lab) Cloning Cases Clone Case in Case Box (Clinic) Clone Case in Work Box (Lab) Deleting Case	96 96 96 97 98 98 99 . 100 . 100 . 101 . 101
Medit Link > Workflow Create/Edit/Clone/Delete Case Creating Cases Create Case in Case Box (Clinic) Create Case in Work Box (Lab) Editing Cases Edit Case in Case Box (Clinic) Edit Case in Work Box (Lab) Cloning Cases Clone Case in Case Box (Clinic) Clone Case in Case Box (Clinic) Deleting Case Delete Case (Clinic and Lab)	96 96 96 97 98 99 . 100 . 100 . 101 . 101
Medit Link > Workflow Create/Edit/Clone/Delete Case Creating Cases Create Case in Case Box (Clinic) Create Case in Work Box (Lab) Editing Cases Edit Case in Case Box (Clinic) Edit Case in Work Box (Lab) Cloning Cases Clone Case in Case Box (Clinic) Clone Case in Work Box (Lab) Deleting Case Delete Case (Clinic and Lab) Form Information	96 96 96 97 98 99 . 100 . 100 . 101 . 101 . 101
Medit Link > Workflow Create/Edit/Clone/Delete Case Creating Cases Create Case in Case Box (Clinic) Create Case in Work Box (Lab) Editing Cases Edit Case in Case Box (Clinic) Edit Case in Work Box (Lab) Cloning Cases Clone Case in Case Box (Clinic) Clone Case in Work Box (Lab) Deleting Case Delete Case (Clinic and Lab) Form Information Registering Form Information in Case Box	96 96 96 97 98 98 99 . 100 . 100 . 101 . 101 . 101 . 101
Medit Link > Workflow Create/Edit/Clone/Delete Case Creating Cases Create Case in Case Box (Clinic) Create Case in Work Box (Lab) Editing Cases Edit Case in Case Box (Clinic) Edit Case in Work Box (Lab) Cloning Cases Clone Case in Case Box (Clinic) Clone Case in Case Box (Clinic) Clone Case in Work Box (Lab) Deleting Case Delete Case (Clinic and Lab) Form Information Registering Form Information in Case Box Enter on the Form Tab	96 96 96 97 98 99 . 100 . 100 . 101 . 101 . 101 . 101 . 101
Medit Link > Workflow Create/Edit/Clone/Delete Case Creating Cases Create Case in Case Box (Clinic) Create Case in Work Box (Lab) Editing Cases Edit Case in Case Box (Clinic) Edit Case in Work Box (Lab) Cloning Cases Clone Case in Case Box (Clinic) Clone Case in Work Box (Lab) Deleting Case Delete Case (Clinic and Lab) Form Information Registering Form Information in Case Box Enter on the Form Tab Registering Form Information in Work Box	96 96 96 97 98 99 . 100 . 100 . 101 . 101 . 101 . 101 . 101 . 102 . 102

Medit Scan for Labs	. 103
General options	. 103
Options by teeth	. 103
Adding to Favorite	. 104
Add to Favorites	. 104
Manage Favorite Templates	. 104
Work on Scanning/CAD	. 105
Scan Process	. 105
CAD Process	. 105
Utilize Add-in Apps	. 106
Add-in Quick Launcher	. 106
App Launcher Management	. 106
Quick Launcher	. 106
All Apps	. 106
Get Result Files	. 107
Export Scan Result Files	. 107
Export Result Files from Case List	. 109
Export Result Files from File Viewer	. 111
Export Result Files from Data Tree	. 113
Export Case List as CSV	. 115
Export Margin Line	. 116
Export Margin Line as a Color	. 116
Export Margin Line as a File	. 117
Download Files from Medit Link Web	. 117
Case Box/Work Box	. 118
Web Viewer	. 118
Order/Manage Orders	. 121
Ordering Cases	. 121
Confirm Order (In Box)	. 122
Cancel Order (Order Box)	. 122
Reject Order (In Box)	
Checking Ordered Cases	. 123
Entering Shipment Information (In Box)	. 124
Managing Delivery Date	. 125
Setting Requested Delivery Date (Clinic account)	. 125
Setting Expected Delivery Date (Lab account)	. 126
Completing Case	. 127
Complete (Work Box/In Box)	. 127
Complete a case from the case detail	. 127
Complete the multiple cases from the case list	. 128
Complete (Case Box/Order Box)	
Complete a non-ordered case in the Case Box	
Communicate Using Tags	
Add Tags When Creating a New Case	
Add Tags When Ordering	
Check Tags on Case Details	
-	

Medit Link

Introduction

Medit Link

Medit Link is a collaborative tool designed to facilitate communication between clinics and labs to provide an optimized workflow for each party.

It offers differentiated features based on the type and permissions of each account. The program provides a user-friendly user interface for managing the workflow, from scanning with oral and model scanners, ordering cases, and designing and producing prostheses.

Medit Link consists of a web service and a computer application.

Service Overview

		Web	Арр
		 Register and manage the account Search and manage partners Share cases via Web Viewer with a sharable link Access cases made from the App Manage orders 	 Enter form information Scan Manage orders Link with CAD Manage patients
Clinic	Admin	Manage the clinic account, search and manage partners, and share cases. • Dashboard • Patient • Case Box • Order Box • Case Talk • Partners • Membership • My Info	Enter form information, scan, place orders, and manage patients. • Dashboard • Patient • Case Box • Order Box • Case Talk • App Box • Trash Box
	Member	PatientCase BoxOrder BoxCase TalkMy Info	PatientCase BoxCase TalkApp BoxTrash Box

Lab	Admin	Search and manage partners and your lab, and share cases. Dashboard In Box Work Box Case Talk Partners Membership My Info	Enter form information, scan, link with CAD, and manage orders. • Dashboard • In Box • Work Box • Case Talk • App Box • Trash Box
	Member	In BoxWork BoxCase TalkMy Info	In BoxWork BoxCase TalkApp BoxTrash Box

Cloud Service

Medit Link supports the automatic upload/download of databases and files through AWS (Amazon Web Services).

The data stored in the cloud is automatically synchronized to the computers of multiple users in the organization so that they can access the same data from any computer at anytime. Cloudbased backup and recovery keep all your data safe. It keeps your data safe even in case of data loss due to sudden computer failure.

The cloud service allows partnered dentists and labs to quickly share data without the hassle of emailing or manually transferring the scans. All data, including patient information, is securely encrypted, stored, and transmitted. Medit complies with medical laws and regulations like HIPAA and GDPR.

Workflow Management Platform with Ordering System

Medit Link offers its ordering system.

- Medit Link allows users to establish partnerships between dentists and dental laboratories and share data quickly and easily.
- You can check and manage the ordering status.

Medit Link > Installation

System Requirements

₽ Note

Please refer to the system requirements for Medit Scan for Clinics and Medit Scan for Labs if you want to use the scanning software with your scanner.

System Requirements for Windows

	Laptop & Desktop	
CPU	Intel Core i5 2.6 GHz or higher	
RAM	16 GB or higher	
Graphics	NVIDIA GeForce GTX 760 (2GB) or higher/or equivalent AMD video card	
os	Windows 10 64 Bit (unavailable in 32 Bit) or higher	

System Requirements for macOS

	Мас	
Processor	M1 with 8-core, 7-core GPU, 16-core Neural Engine	
RAM	16 GB or higher	
os	Monterey 12 or higher	

System Requirements for Web Service

Recommended for Browsers

	Browsers	
PC	Chrome, Firefox, Safari, Edge (Chromium)	
Android	Chrome, Firefox	
iOS	Safari, Chrome, Firefox	


Minimum for Mobile Devices

	Android	iOS
Device	Galaxy S7 equivalent or higher	iPhone 7 equivalent or higher
os	Android 7.1 Nougat	iOS 11.0
RAM	3 GB or higher	2 GB or higher
AP	Above Snapdragon 710	A10 Fusion

Installation on Windows

Medit Link is installed as a package with scan acquisition programs - Medit Scan for Clinics and Medit Scan for Labs.

- 1 Run Medit LinkSetup.exe.
- 2 Select a language for installation.
- 3 Select the folder to install Medit Link. Please read and agree to the license terms and conditions and click "INSTALL."

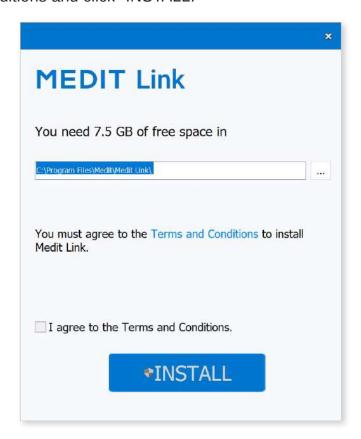


Make sure that the scanner cable is disconnected from the computer before proceeding to the next step. The installation may not proceed if the scanner is connected to your PC.

(5) The program will be installed automatically and may take a few minutes. Do not turn off or unplug your PC until the installation is complete.



6 Please restart your PC after the installation is complete. You can proceed without restarting the PC if you do not have the scanner.

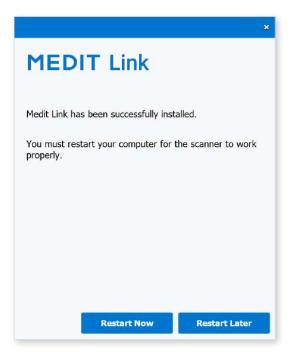


⊘ Note

If you do not restart your PC after the installation is complete, the scanner may not function properly.


Installation on macOS

Medit Link is installed as a package with Medit Scan for Clinics.


① Run Medit_Link_x.x.x.pkg and click "Continue."

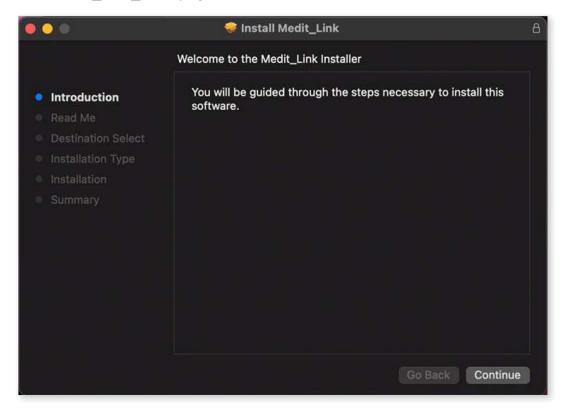

② Read the End User License Agreement and click "Continue."

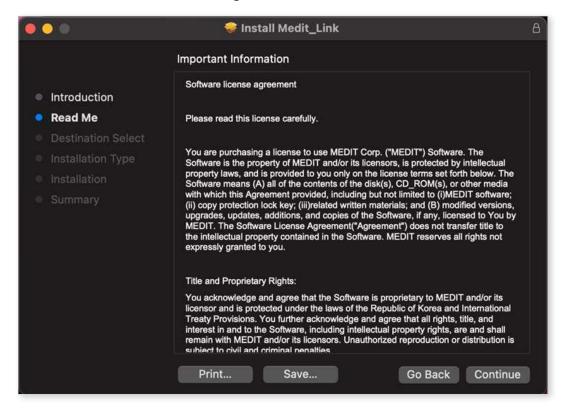

3 Select the disk to install and click "Continue."

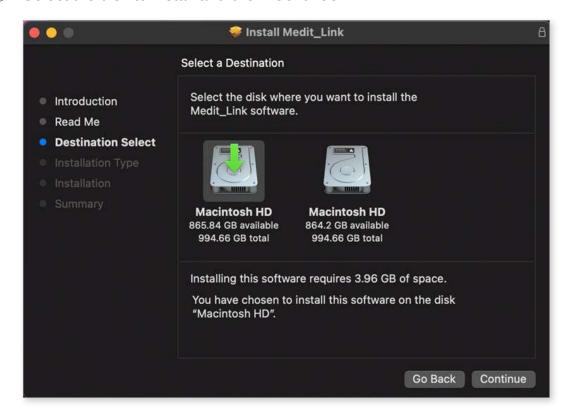


4 Click the "Change Install Location..." to change the installation destination.




⑤ Click "Install" and type your Mac password to allow installation if requested.



6 Wait until the installation is completed.

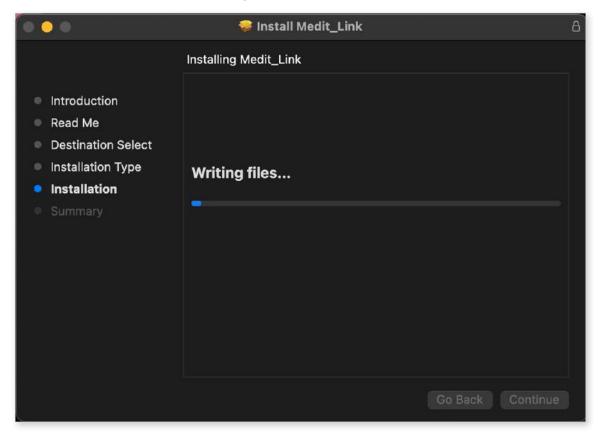


7 Click "Close" to exit the installation window.

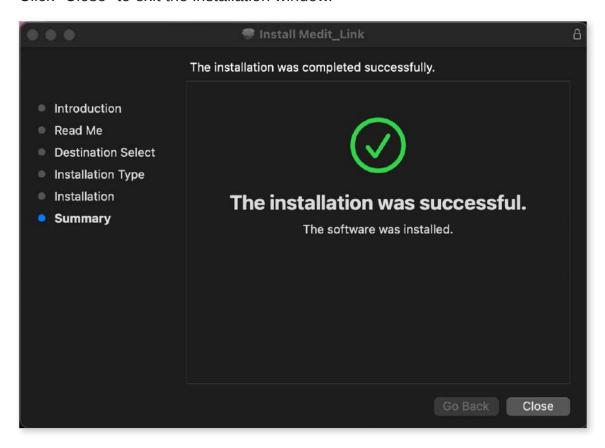

Medit Link > Getting Started

Medit Link Website

Sign up for Medit Link service at www.meditlink.com.

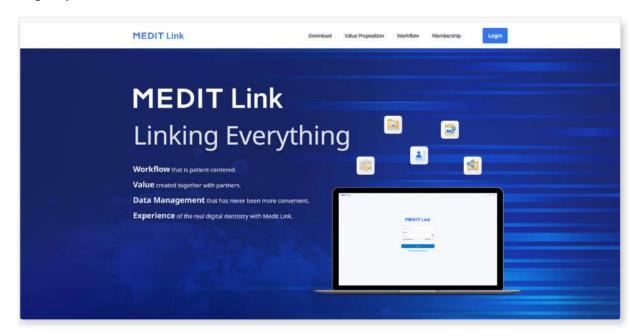

Sign Up

To use the Medit Link service, you must sign up for an account. You can utilize features provided in Medit Link after creating an account as a clinic or lab.


Users can select one of the following four account types to register for membership and use the service with full or limited features and privileges, depending on the account type.

- Clinic Admin
- Clinic Member
- Lab Admin
- Lab Member


The administrator is usually a representative of your organization and will have full access to all functions and services of Medit Link. In contrast, the member account has limited access.

To sign up for membership, follow these steps:


① Click "Don't have an account yet? Sign up." on the login page.

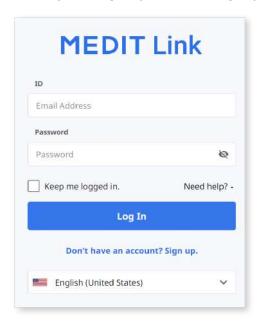

② After reading the provided descriptions, click the "Register" button for the desired account type.

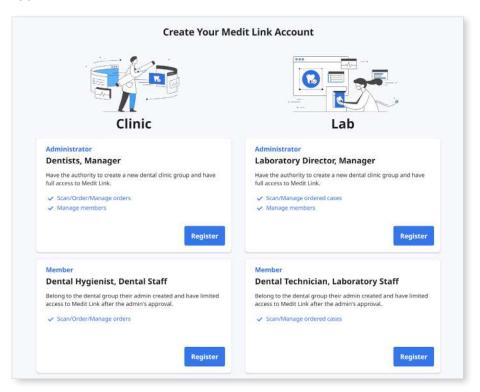

3 The following registration form appears.

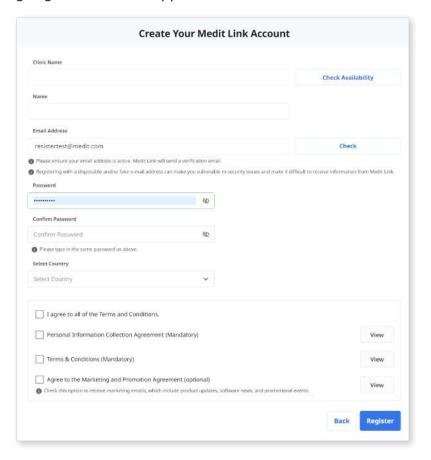

- Enter the clinic or lab name and then, depending on your desired account type, do the following:
 - If you are registering as an administrator, click "Check Availability" to see if the entered name is available.
 - If you are registering as a member, click "Search." The administrator must already register your organization before you sign up.
- ⑤ Fill out the rest of the registration form and click "Register."




If you agree to receive marketing information during registration, you will receive helpful information about the product and service from Medit.


- 6 A verification email will be sent to you.
- To complete the registration process, verify your email address using the received link.

Login and Welcome Page

 ${f @ }$ Enter your email address and password on the login page and click "Log In."

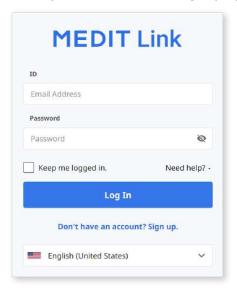

② The following pop-up will appear if you haven't filled out your business information.

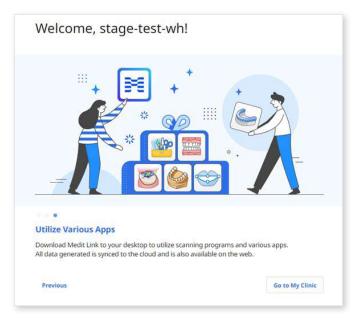

Note

The clinic or lab information should be entered using the administrator account.


- 3 Click "Go to My Clinic" or "Go to My Lab" and enter the clinic or lab info including business name, cover/logo images, services, office address, office number, etc.
- The information entered on the welcome page can be edited anytime on My Info > My Clinic or My Lab tab in Medit Link Web.

Medit Link > Medit Link Web

Dashboard

Dashboard Overview


You can view the statistics for all the work done on Medit Link in visualized graphs and charts - both in the Medit Link App and Web.

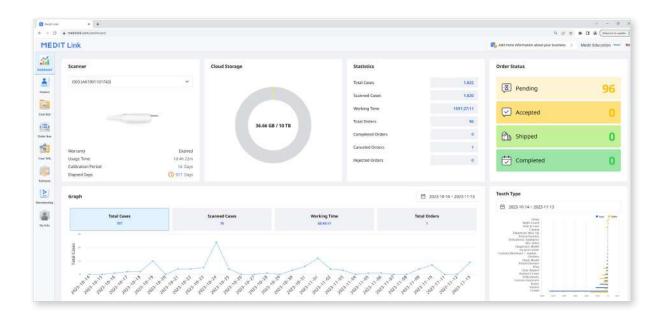

Graphs and charts in the dashboard show statistics for all the work done in Medit Link.

Scanner Scanner Provide information about the active scanners, including warranty, scanner usage time, calibration interval, and calibration elapsed time.	
Cloud Storage	Display the total capacity of the cloud and the amount in use.
Statistics	Provide statistics on cases based on job status.
Order Status	Provide statistics on cases based on order status.
Graph	Graph job information by period.
Tooth Type	Provide statistics on the number of scans and orders by treatment information.


Medit Insights

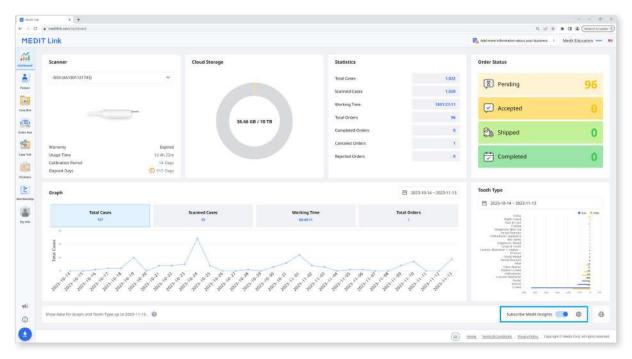

Medit Insights is a reporting service that helps you analyze your usage statistics


Subscribe Medit Insights

You can receive a report email on the second day of every month. No additional sign-up process is required. Just enable "Subscribe Medit Insights."

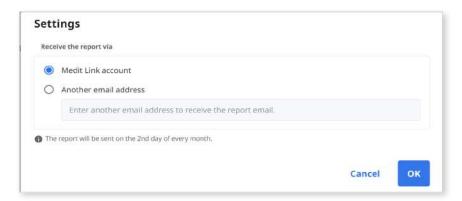

If you don't use the email address you used for the Medit Link account, you can set another email address to receive Medit Insights.

On the second of every month, you will receive an email with statistics, as shown below.

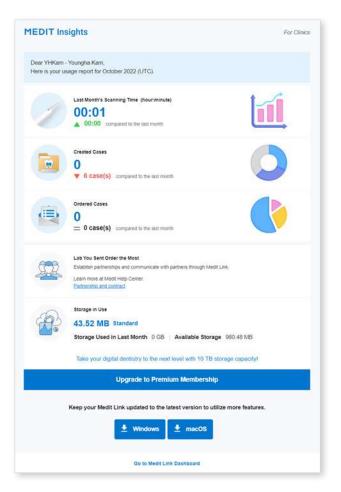

Banner	 Medit provides useful tips and information here. If you don't want to see the banner, uncheck the Marketing and Promotion Agreement check box in My Info > My Account. 	
Statistical Information The statistics information is displayed, such as scann time, created cases, most ordered lab, etc.		
Medit Link Download	The button provides the download link of the Medit Link.	
Dashboard	All statistical information is based on your Dashboard information. You can see more details in the Dashboard.	


UnSubscribe Medit Insights

Disable "Subscribe Medit Insights" in Dashboard or click "Unsubscribe" in the received email.

Patient


You can manage cases based on patient name and patient ID.

All patients registered through the Medit Link application are shown in a list, making it easier to manage patients and their cases.

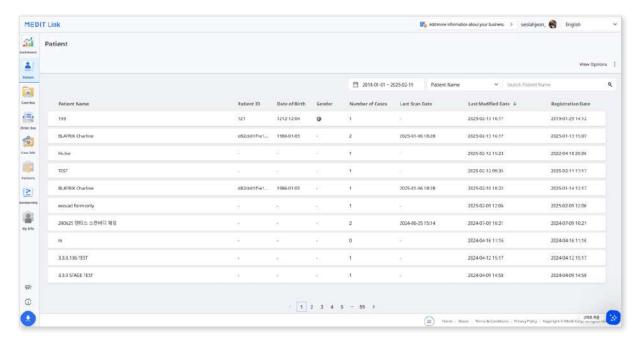

Search Patient


① Select your search criteria between Patient Name or Patient ID from the dropdown menu at the top right of the patient list.

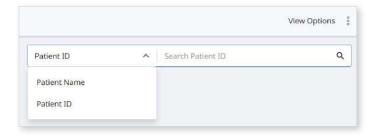

- 2 Enter the patient name or patient ID to search according to the selected search criteria.
- 3 Only patients with the entered patient name or ID are displayed in the list.

Case Box/Work Box


The Case Box (Clinic) or Work Box (Lab) shows all cases the user has created on the Medit Link application. You can see files via a web browser without installing a separate program.

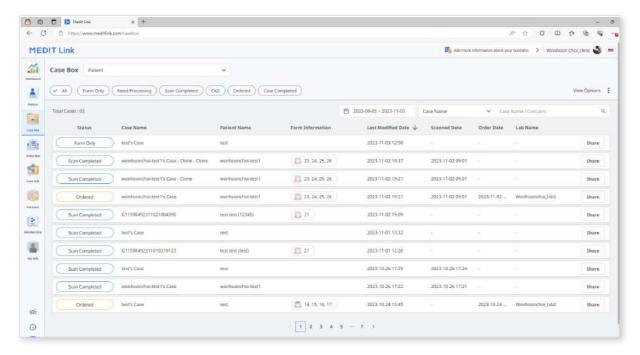

Case Filtering

Filter by patient


You can filter cases by patients. It is useful when you find cases for specific patients.

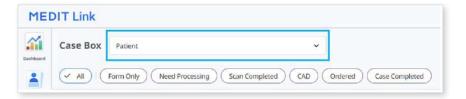


① Enter a patient name in the search field.



② Select a patient from the list of search results.

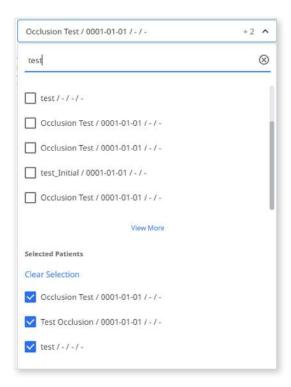


3 Only cases for the selected patients are shown in the list, making it easier to find the cases you want to work with.

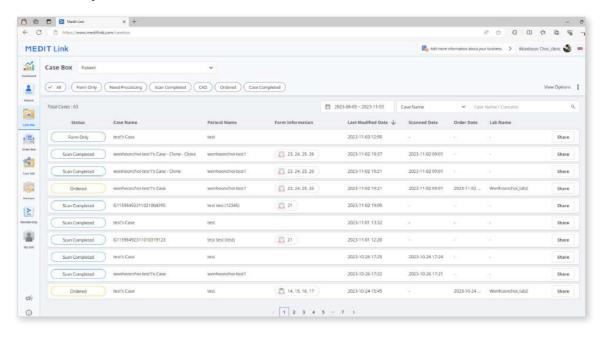

Filter by status

You can filter cases based on their status.

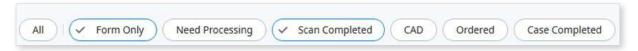

- The user can click the status they want to view. The browser will display only cases with the chosen status.
- The user can select multiple statuses at once.

The following status type is provided based on account type:


Case Box (Clinic)	All / Form Only / Need Processing / Scan Completed / CAD / Ordered / Case Completed
Work Box (Lab)	All / Form Only / Need Processing / Scan Completed / CAD / Done

Filter by duration

You can filter and display cases that were last modified within your set period.



② Select the range.

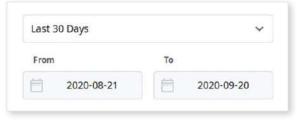

- All: Shows all cases.
- Custom Range: Specify start and end dates to display cases modified within that period.
- Today: Show cases last modified today.
- Yesterday: Show cases last modified with yesterday's date.
- Last 7 days: Show cases last modified during the last 7 days, excluding today.
- Last 30 days: Show cases last modified during the last 30 days, excluding today.
- Last 90 days: Show cases last modified during the last 90 days, excluding today.
- This Month: Show this month's cases.
- Previous Month: Show the previous month's cases.

The case list will be updated in real time with the cases created or last modified within the selected duration.

Filter by Search


You can leverage search capabilities to search for the required information easily and quickly.



Search options based on clinic/lab account:

Case Box (Clinic)	Case Name, Patient Name, Patient ID, Lab Name	
Work Box (Lab)	Case Name, Patient Name, Clinic Name, Order ID	

Case Details

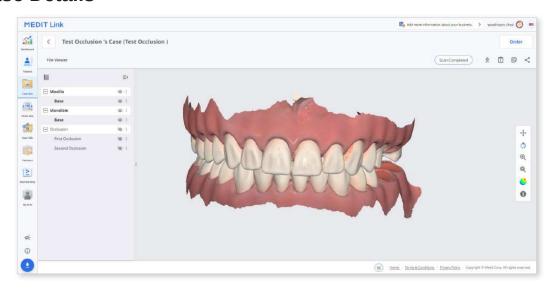


The following case management tools are provided in Case Box/Work Box.

	Case Status	Show the status of the case.	
\Rightarrow	Go to Order Box/In Box	Provide a shortcut for the same case in another box (Case Box \leftrightarrow Order Box; In Box \leftrightarrow Work Box).	
$\overline{\Phi}$	Download	Download files in the case to the local PC.	
Î	Case Information	Review case details and save them as a PDF or print them out.	
₽	Memo & Tags	Add a memo or tags when registering or ordering a case. Use them to communicate any additional or supplementary information about the case to your partners.	
<	Share	Create a link for sharing. Users with the shared link can access the 3D data in a browser such as Chrome, Firefox, or Safari without installing additional programs.	

Order Box/In Box

The Order Box (Clinic) or In Box (Lab) shows ordered cases on the Medit Link application.

Order Details

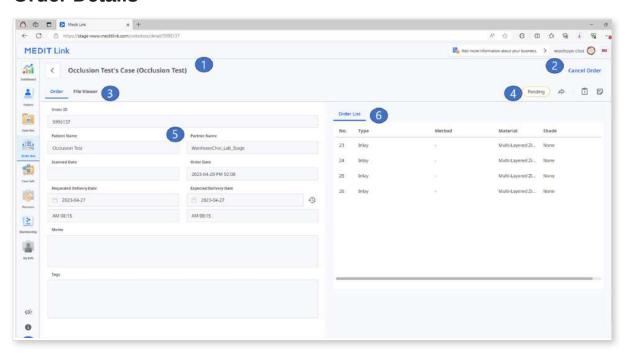

1	Show case name and patient name.
2	Provide functions for order management (accepting/canceling; delivering/receiving).
3	Show available tabs: Order, File Viewer
4	Show functions for managing cases and files: case status, shortcuts for detailed information, case information, memo & tags, etc.
5	Show the detailed information for the order including the order number, patient name, partner name, scanning date, order date, delivery date, and memo & tags.
6	Show the ordered product information and delivery status.


Tabs in Order Box/In Box

Order	File Viewer
Check the detailed information for the	View files generated by scan/CAD
order, such as order ID, patient name,	operations. The file viewer displays
account name, scanning date, order	and manages the acquired or attached
date, delivery date, memo, and tags.	3D models and 2D images.

Case Talk

Cases created in the Medit Link can be shared with specific people you specify, anyone with the shared link or registered members of Medit Link.

The user can see a list of all shared cases through "Case Talk" in Medit Link App and Medit Link Web.

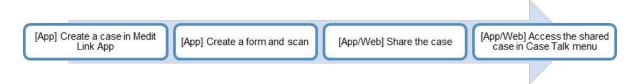


Shared cases can be viewed via a web browser without installing a separate program, enabling easy communication over the 3D data on various devices.


Creating a Case Talk

You can create a shared link to a case that was created in Case Box/Work Box.

① Find a case you want to share and click the "Share" button on the right.

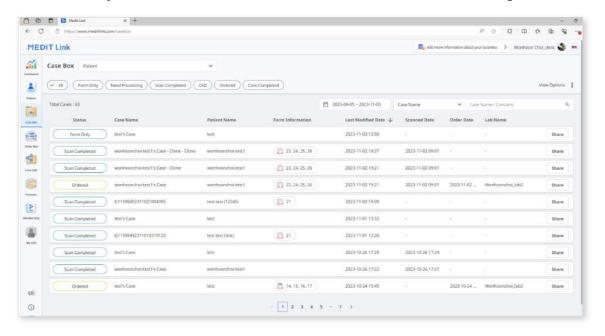


2 Select the files you want to share.

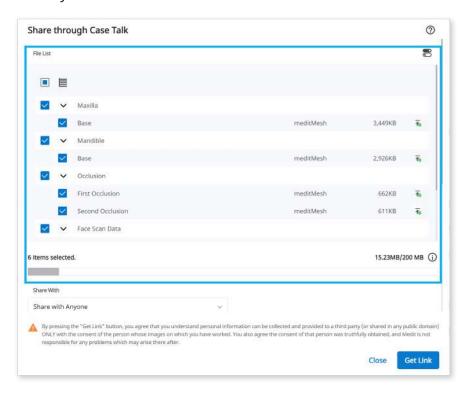

- 3 Set the sharing permission type and sharing options.
- 4 Click "Get Link" to create a shared link.


Setting the sharing permission type and sharing options

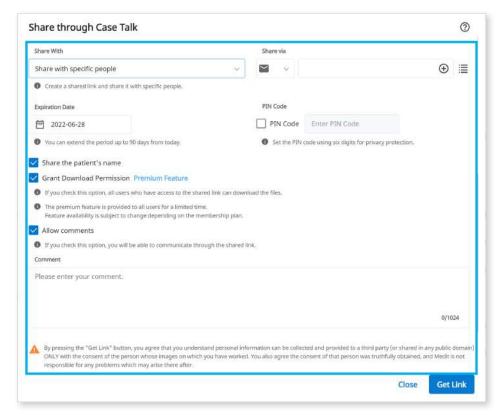

Share With	 Share with anyone Creates a shared link. You can share the files with anyone. Share with specific people Creates and sends a shared link to specific people. 	
Share via	If you select "Share with specific people," you can share the link with specific people in various ways. • Email Address • SMS • Partners	
Expiration Date	Set the expiration date for the link. After the link expires, the people you shared it with will no longer have access to files. You can extend the expiration period up to 90 days from today.	
PIN Code	Set a 6-digit access code to ensure data security.	
Share the patient's name	If you check this option, all people with the link will see the patient's name.	
Grant Download Permission	If you check this option, all users with the link can download the files.	
 If you check this option, you will be able to communicate through the shared link. If you uncheck the option, the recipient can only your comments but will not be able to add their or 		
Comment	You can leave a message in the comment section.	

Case Talk details

You can see files in the shared case. The following functions are available on the Case Talk details page.

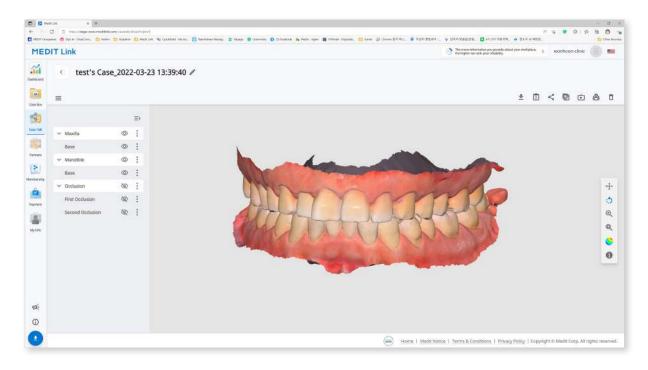


$\overline{\Phi}$	Download	Download all the files in this case to the local PC.	
	Case Information	Show detailed information about the case.	
<	Share	Modify the sharing setting.	
00	QR Code	Share the case with a QR code.	
	Copy link	Copy the shared link to the clipboard. The copied link shows the shared files on your web browser.	
Ě	Live	Share your screen with the connected users. The screen sharing is synchronized with the connected users.	
	Attendees	Show all users who are accessing the shared link.	
	Delete	Delete the shared link. The deleted shared link is no longer accessible.	

Web Viewer

You can view shared data through Case Talk in a web browser.


It is possible to communicate with the people who have access to the link using annotations and comments.

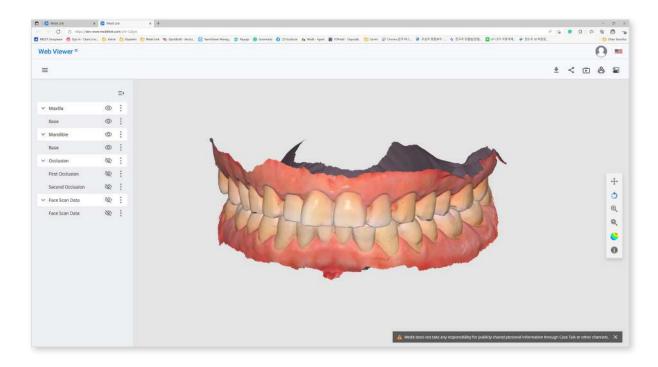


$\overline{\Phi}$	Download	Download all the files in this case to the PC.	
Î	Case Information	Show detailed information of the case.	
<	Share	Modify the share settings.	
88	QR Code	Share the case with a QR code.	
Ď	Live	Share your screen with the connected users. The screen sharing is synchronized with the connected users.	
: -	Attendees	Show all users who are connected to the shared link.	
	Tooth Labeling Change the teeth labeling system.		

Annotation


You can add annotations to a specified location on the 3D model. Anyone with the link can view the annotations you add.

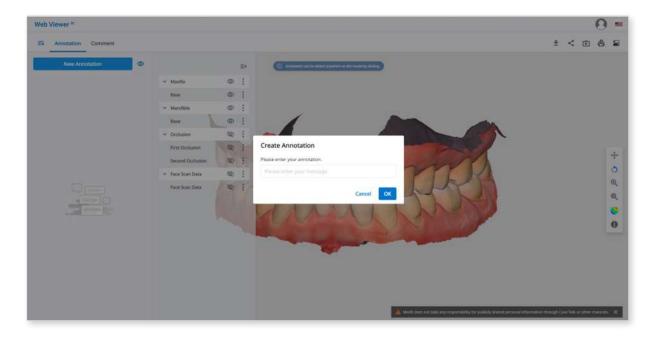

- ① Find the menu icon in the top left corner and select the "Annotation" tab. Use the "New Annotation" button to add an annotation anywhere on the data with a click.
- ② Type the message, then click "OK" to save it.

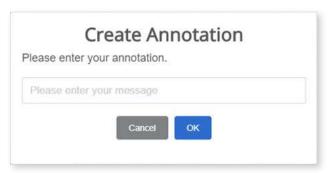

You can use annotations as a simple communication channel. Click the created annotation to enter a reply and hit "Send."

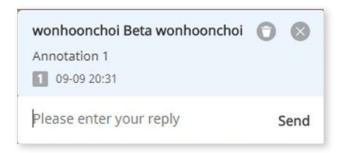

Comments

You can communicate with users who have access to the link through the "Comment" tab in the Web Viewer menu.

Real-time screen sharing

You can share your screen in real-time.

① Click "Live" to share your Web Viewer screen with the connected users.



- ② Click "OK" to start sharing your screen.
- 3 Other people who have accessed the link must accept the screen sharing.
- ④ The screen of the user who initiated screen sharing is displayed in sync with the screen of the person who received the request.

Data Tree

All data is grouped and displayed according to attributes in the data tree. You can show or hide data, adjust opacity, and so on.

Data Tree Controls

Show/Hide	Click the 'eye' icon to show or hide data from the view. Control data visibility as a group or individual file.	
Opacity	Hover the mouse over a visible element to see the opacity slide bar. Adjust the slider to change the transparency of your data.	
	preoperationmaxillary.meditMe	
	The extended menu is provided on the right side of each Data Tree element. It includes the following options:	
Extended menu	preoperationmaxillary.meditMe	
	 Show This Only: Hide all other data except for the current one. Hide: Hide the current data. 	

3D Model Controls


	Pan		Move the model.
\bigcirc	Rotate		Rotate the model.
±	Zoom In/Out		Zoom in and out on the model.
Q	Zoom Fit		Position the model in the center of the screen.
		Matte	See the data in color with matte finish (Phong rendering).
Data	Data Display		
	Mode	Monochrome	See the data in one color.

Partners

You can search for potential partners to request a partnership and view the list of current partners.


Search for Partners

You can search for Medit Link users based on their business name and/or address. In the search results, you can send a partnership request to a business you want to work with.

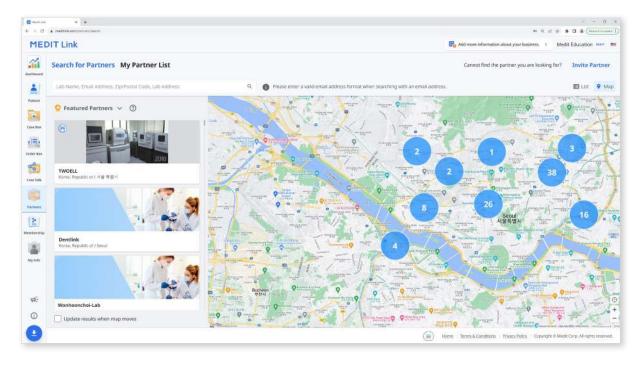

My Partner List


You can review your current partnerships with all your existing partner businesses in My Partner List.

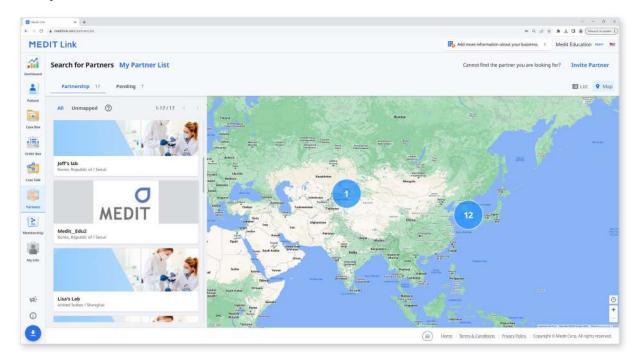

Partner Invitation


If you have a clinic or lab you want to work with, you can invite them to join Medit Link. The partnership will be automatically formed once they accept the invitation.

- ① Click "Invite Partner" in the top left of your screen.
- 2 Enter the email address of a business you want to work with and send them an invitation with a partnership request.

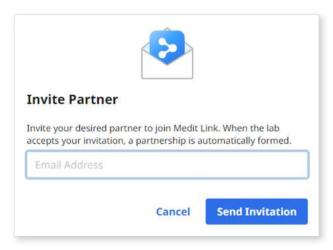


③ Once the invited organization accepts the invitation and signs up for Medit Link, the partnership will be automatically established.

Membership

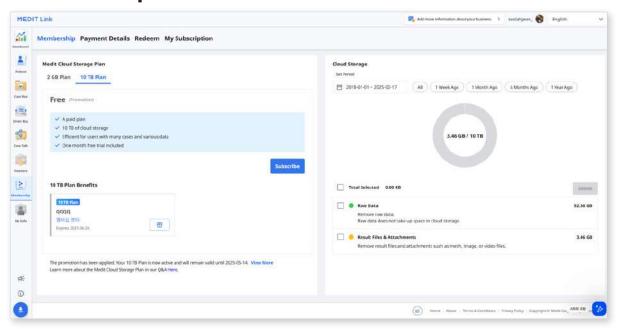

Medit Cloud Storage Plan

You can see what each plan offers, including discount coupons and vouchers, and your current subscription status. You can upgrade your plan or cancel your subscription at any time.

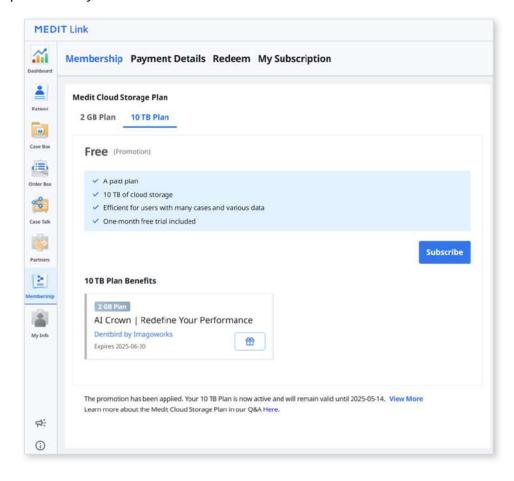

Medit Link provides the following cloud storage plans:

2 GB Plan

- Free standard plan for all signed-in users
- 2 GB cloud storage
- For new users

10 TB Plan

- Paid plan
- 10 TB cloud storage
- For users with many cases and various data
- One-month free trial


	2 GB Plan	10 TB Plan	
Cloud Storage*	2 GB	10 TB	
Monthly Fee Free of charge \$9.99/month			
* Cloud storage does not count raw data toward the used space.			

⊘ Note

The membership plans will be available on Medit Link China. You can enjoy the 10 TB Plan for free until the official release of the feature.

Cloud Storage

In the right-side section, you can manage your cloud storage.

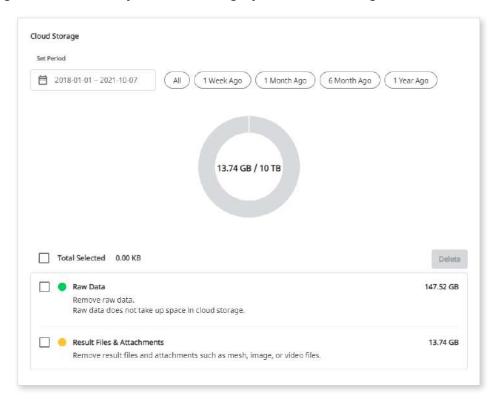


Set Period	 Set the specific period to delete data created within that time frame. All: See all files. 1 Week Ago: See the files that have passed a week after creating. 1 Month Ago: See the files that have passed a month after creating. 6 Months Ago: See the files that have passed six months after creating. 1 Year Ago: See the files that have passed a year after creating.
Raw Data	The raw data does not take up space in the cloud storage, so it will not be in the pi-chart. But you can manage the raw data from here if needed, e.g., delete it to protect the patient's private information.
Result & Attachment	The result and attachment files take up space in the cloud. If the cloud is full, you can secure more storage space by removing unnecessary files.
Delete	Remove selected files from the cloud storage to free up space.

Subscribe


If you are currently on the 2 GB Plan or the 10 TB Plan with a redeem code, you can subscribe to the 10 TB Plan.

The payment is completed through Stripe service (www.stripe.com).


① Go to Membership > Medit Cloud Storage Plan > 10 TB Plan and click the "Subscribe" button.

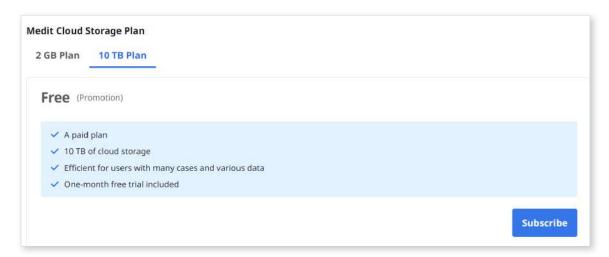


② Check the 10 TB Plan description and click "Subscribe" again.

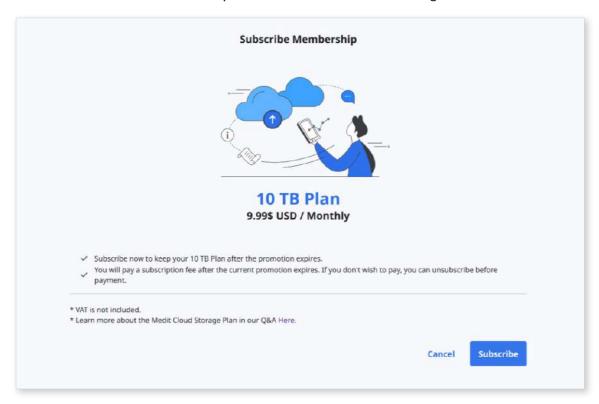

3 Check the price and conditions and input information necessary for the payment. Then, click "Subscribe."

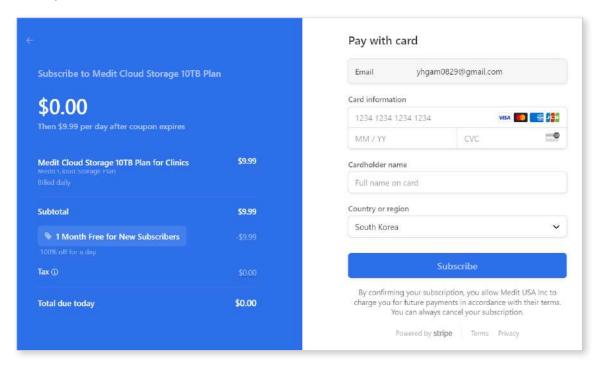


The payment may take several seconds. When done, you will see the confirmation message.

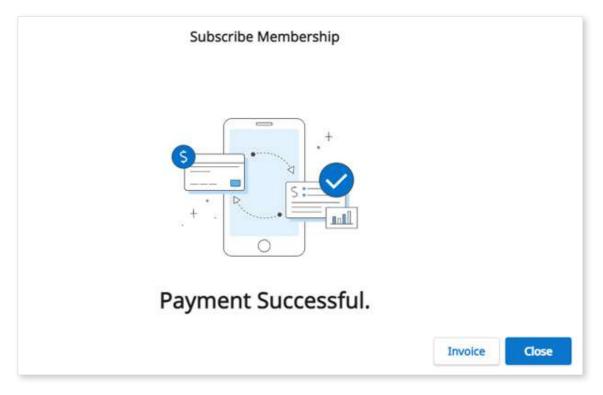


Cancel Subscription

You can cancel your subscription at any moment. After canceling, you can still use the 10 TB Plan for the remaining days of that paid month. Your plan will be changed to the 2 GB Plan from the next payment day, and the cloud storage will be limited to 2 GB. If the cloud storage capacity is exceeded, the files will not be backed up, and the order and share functions will not be available.

- Click "Unsubscribe."
- ② Click "Cancel Subscription."

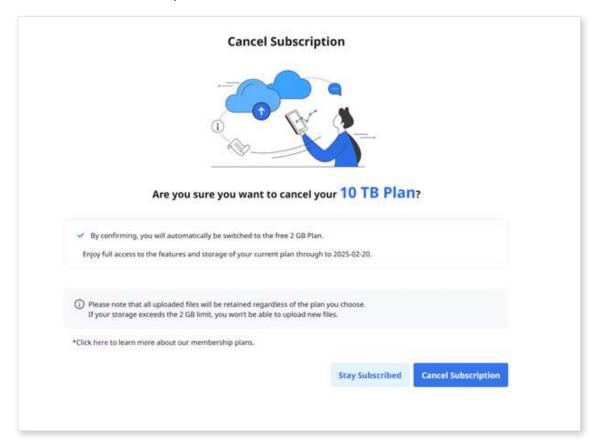

③ To confirm that you want to unsubscribe, click "Cancel Subscription" again in the new popup window.

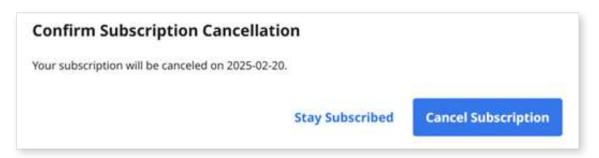


4 Your subscription has been canceled.



S Your plan will be changed to the 2 GB Plan from the next payment day, and the cloud storage will be limited to 2 GB. If the cloud storage capacity is exceeded, the files will not be backed up, and the order and share functions will not be available.

My Subscription

You can view the details of your current subscription, resubscribe, or cancel it from My Subscription. You can also change the payment method, which will be applied starting from the next scheduled payment.

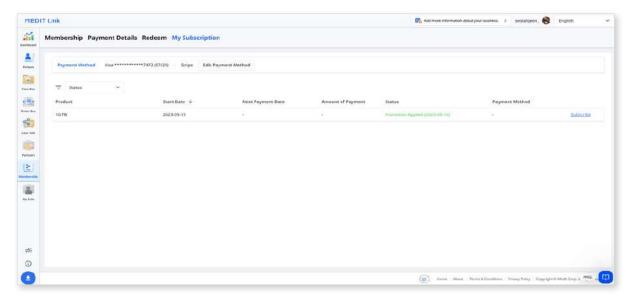

Details


Item	Description
Product	Displays the type of subscription.
Start Date	Displays when the subscription or promotion started.
Next Payment Date	Displays the next payment date.
Amount of Payment	Displays the payment amount.
Status	 Subscription Active: The subscription is active. Expiring: The subscription will not renew on the next payment date and will be canceled. Expired: The subscription has ended. Promotion Promotion Applied: The promo code is now active. Promotion Pending: The promo code is awaiting activation. Promotion Expired: The promo code has expired.
Payment Method	Displays the last four digits of the card used for payment.

Resubscribe


Item	Description
Cloud Storage Plan	Click the subscription button to resubscribe to 10 TB Plan.
App Subscription Plan	Sign in to the app to resubscribe.

How to add a payment method


① Click "Edit Payment Method."

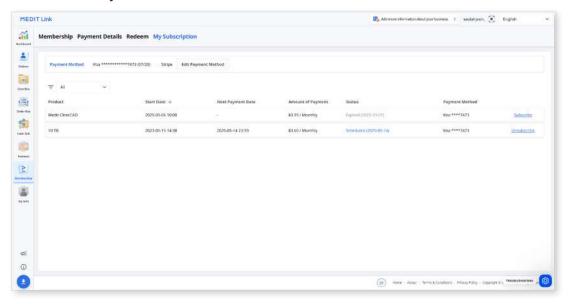


② Click "Add payment method."

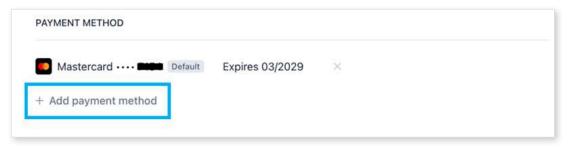

3 Input the required information and click "Add."

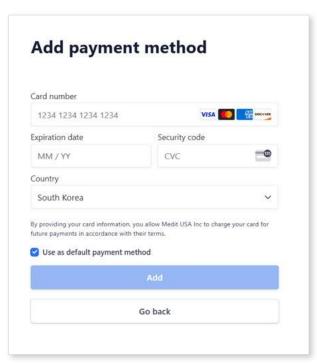


4 The newly added payment method will be set as default and used for the next payment.

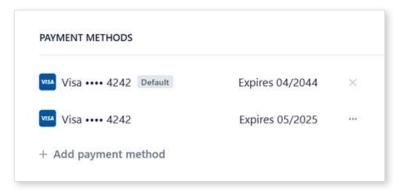


How to change a payment method

① Click the expanded menu icon next to the payment method you want to change in the billing dialog window.

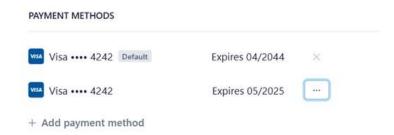


② Click "Make default" to set the selected card as the default payment method.

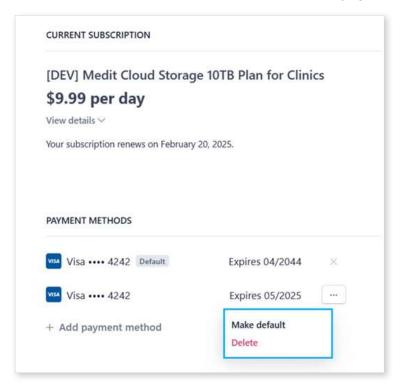

Redeem

You can redeem a promo code to access the 10 TB Plan for free.


Details

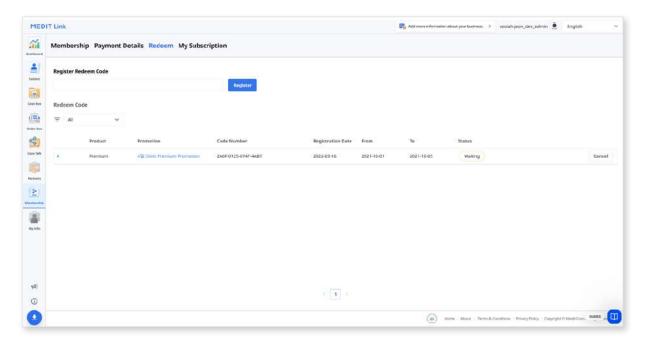

Item	Description	
Product	Displays the plan to which the promotion is applied.	
Promotion	Displays the name of the promotion.	
Code Number	Displays the code number.	
Registration Date	Displays the date the code was registered.	
From	Displays the start date of the promotion.	
То	Displays the expiration date of the promotion.	
Status	 Registered: The promo code has been added and is now available. 	
	 Waiting: The promo code is awaiting activation. 	
	 Activated: The promo code is now active. 	
	 Used: The promo code has expired. 	

Register Redeem Code

- Input the redeem code and click "Register."
- 2 Read the notice and choose "Register" or "Apply" in the pop-up dialog.
 - Register: Register the redeem code. You can activate and use the redemption code later after registration.
 - Apply: Register and activate the redemption code. The promotion discount applies immediately.

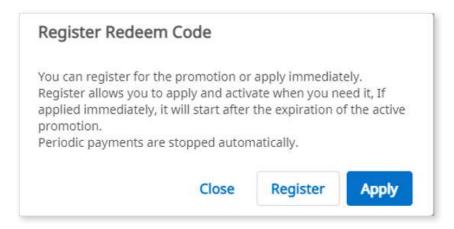

Activate Redeem Code

You can register multiple redeem codes, but you need to activate the one you want to use.


- ① Click the "Activate" button next to the code you want to use.
 - If there is no code with the "Activated" status, the one you choose will be applied immediately.
 - If an already activated code exists, the newly selected code changes to the "Waiting" status. When the currently activated redeem code expires, the new redeem code in the "Waiting" status will be automatically activated.



Only one redeem code can be activated, and only one can be in "Waiting" status.

Payment Details


You can view your subscription and payment history in the Payment Details tab.

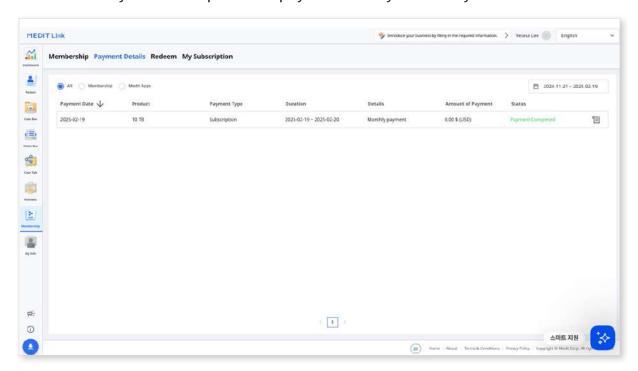


Item	Description	
Payment	Date Displays the date of payment.	
Product	Displays the purchased plan or app.	
	Displays the type of payment:	
Payment Type	Subscription	
	One-time payment	
Duration	Displays the subscription period.	
Details	Displays the detailed payment information.	
Amount of Payment	Displays the amount for a subscription or one-time purchase.	
Status	 Payment Completed: The payment has been processed. 	
	 Payment Failed: The payment was not processed. 	
	 Payment Incomplete: The payment has not been completed. Click "Pay Now" to proceed. 	
	 Waiting: The payment is pending for a subscription renewal. 	

View Bill Details

You can view your payment invoices and receipts by clicking the receipt button on the right.

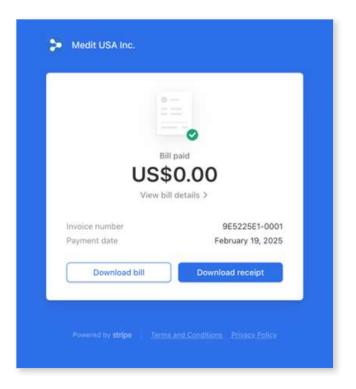


Click "View bill details" for payment details.

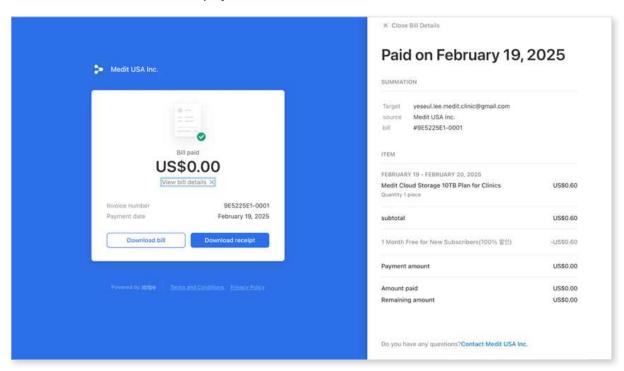

My Info


My Account

In My Info, you can review and edit the information you provided during registration and check your membership status.

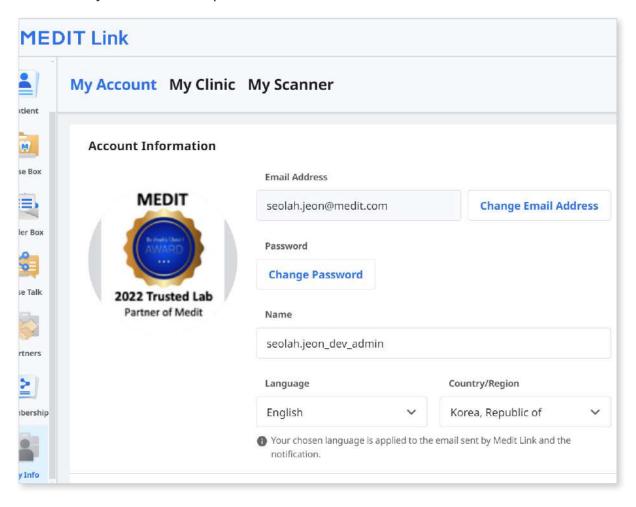


My Clinic/My Lab (Admin Account Only)

You can review and edit the information about your clinic or lab. This is only available to admin accounts.

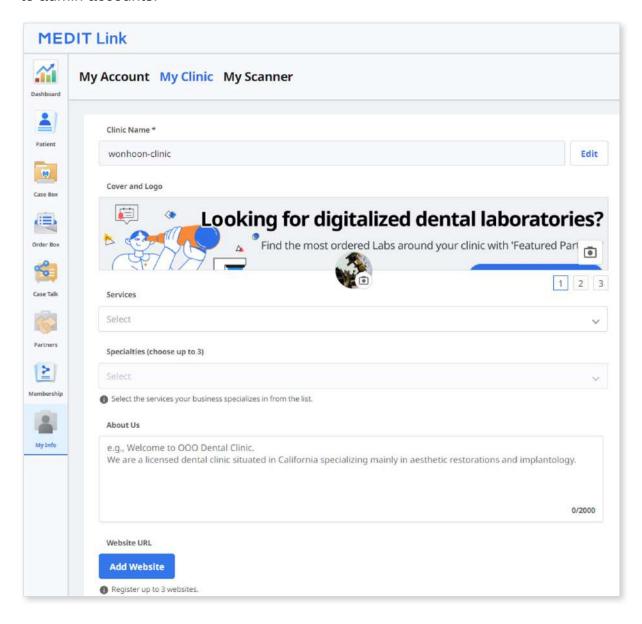

My Scanner

You can check the connected device information for the account.

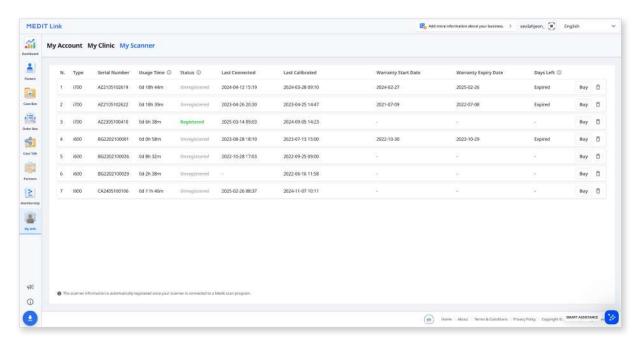


Item	Description
Туре	Displays the model name.
Serial Number	Displays the device's serial number.
Usage Time	Indicates how long the device has been used.
Status	Displays the device's registration status:
	 'Registered' indicates that the scanner has been successfully connected and registered.
	 'Unregistered' indicates that the scanner is no longer registered. If the scanner is registered with a different group, it will lose its registration in the current group.
Last Connected	Displays the date of the last connection.
Last Calibrated	Displays the date of the last calibration.
Warranty Start Date	Displays the date when the warranty was activated.
Warranty Expiry Date	Displays the date when the warranty expires.
Days Left	Displays the number of days remaining before the warranty expires.
Buy	Allows the user to purchase and extend the warranty.
Delete icon	Allows the user to remove the scanner from the list.

Other Menu


Notice & Update

You can check the important notices from Medit and program updates of Medit software.

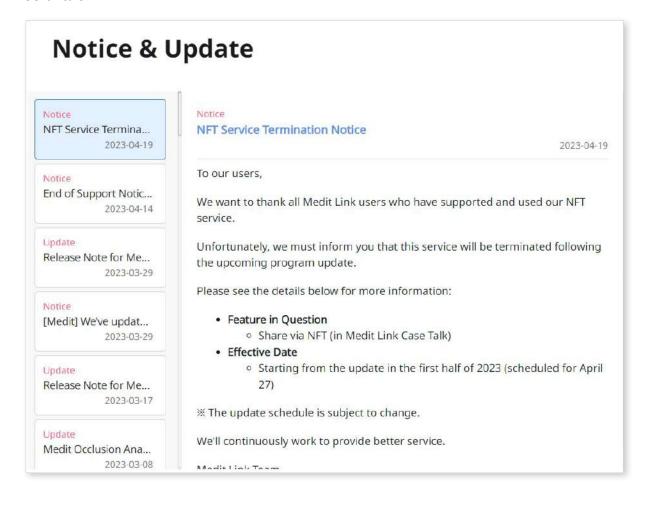


Related Websites

You can get various information about using Medit Link. Access Medit's educational and informational resources, such as Medit Academy, Medit Academy YouTube, and Medit Help Center.

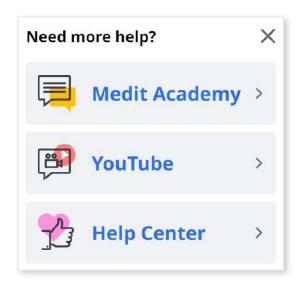


Download Medit Link App

You can download the latest version of the program installer from the web. This menu is helpful when you lose the installer provided in the product package, or you have an outdated installer.

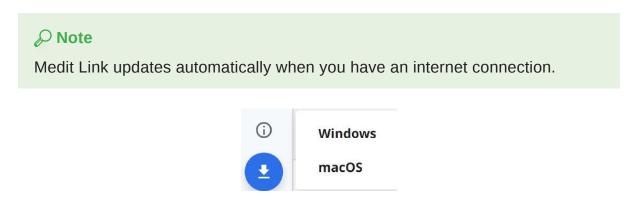

Medit Link > Medit Link App

Overview

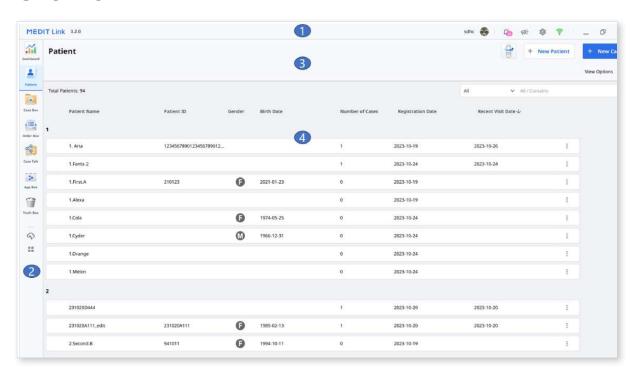

1	Title Bar	Refer to Medit Link App > Title Bar for detailed information.
2	Menu	Refer to Medit Link App > Menu for detailed information.
3	Tools and Functions	Refer to Medit Link App > File Viewer and Medit Link App > View Options for detailed information.
4	Case List	Refer to Medit Link App > Filtering and Search for detailed information.

The table below summarises what menus are available in each type of clinic and lab account.

Accoun	nt Type	Patient	Case Box	Order Box	In Box	Work Box	Case Talk	Trash Box	App Box
	Admin	0	0	0	Χ	Х	0	0	0
Clinic Accounts	Member	0	0	0	X	X	0	O * Restore feature only	0
	Admin	X	Χ	X	0	0	0	0	0
Lab Accounts	Member	X	X	Х	0	0	0	O * Restore feature only	0


Title Bar

The title bar provides quick access to features such as account information, notifications, notices and updates, settings, and network status.




My Information

My information contains basic user information such as profile image, clinic or lab name, email address, and user name.

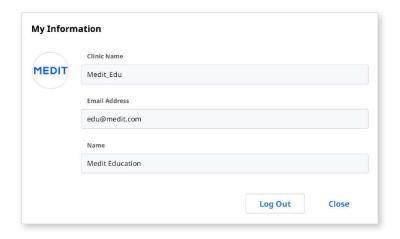

Notifications

You can see notifications related to partnerships, orders, and work processes.

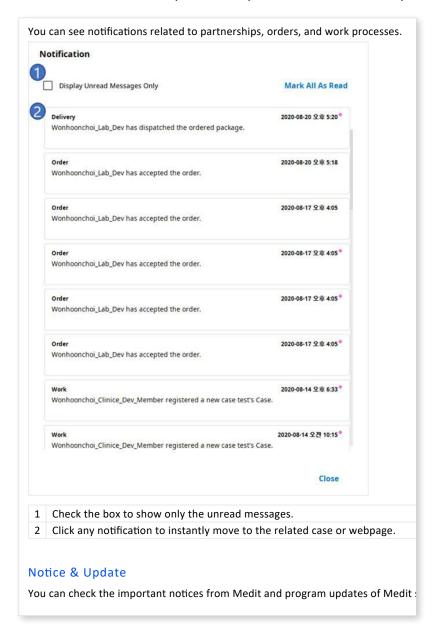


Check the box to show only the unread messages.Click any notification to instantly move to the related case or webpage.

Notice & Update

You can check the important notices from Medit and program updates of Medit software.

Settings

The Settings dialog allows users to configure the program.

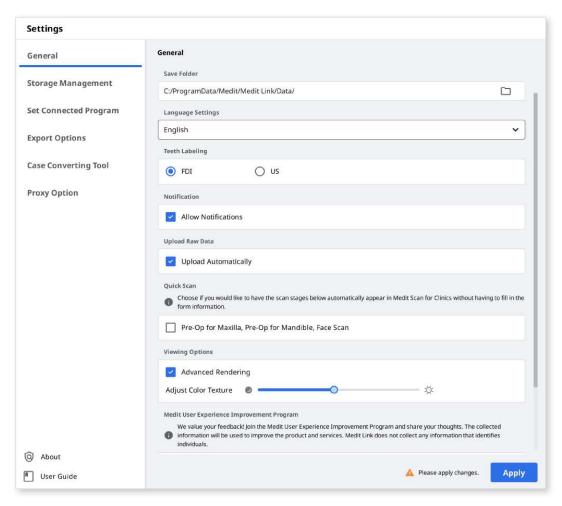


Please refer to the Medit Link App > Settings for detailed information about the setting options.

Network Status

Medit Link enables you to use all the features seamlessly when connected to a network.

To help you check the status of your network connection in real time, the network status icon is displayed in the title bar.

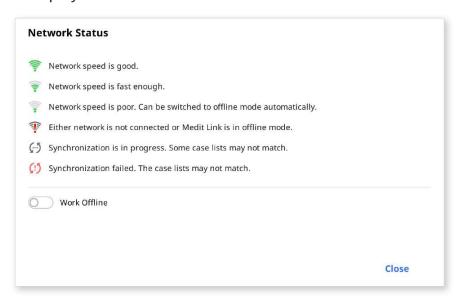

If the network connection is not good, you may experience the following restrictions:

- Some functions will be restricted, such as ordering a case, creating a case, sharing via Case Talk, and cloud synchronization.
- The overall performance of the service can be reduced due to the impact of the network environment.

In this case, you can enter the Offline Mode to perform basic tasks and then use the features that require internet once you are connected.

The network symbol indicates the response speed between the current area and AWS (Amazon Web Service). It does not mean your network speed, so sometimes it may show the status as poor or bad, even though your network speed is fast enough.


If the other process uses the same network, the speed may show as poor or bad, and the overall performance of Medit Link could be bad. In this case, please do not share the network with other processes.

For example, the "Upload Raw Data" feature is a huge part of the network process. If the symbol shows poor or bad status, go to Settings > Upload Raw Data and turn "Upload Automatically" off.

Settings

General

Provides options for overall system setup.

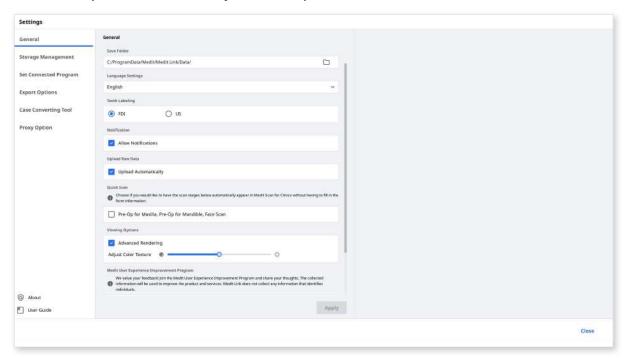


Save Folder	Specify a folder to store the data results generated in Medit Link and connected programs.
Apps Installation Folder	Specify an installation path for Medit Apps and partner apps available in the App Box.
Languages Settings	Select the language for the program.
Teeth Labeling	Select a teeth labeling system between FDI and US.
Notifications	When checked, you will receive notifications for program updates.
Upload Raw Data	When checked, the raw data is automatically uploaded to the cloud. It helps to back up the original data and perform seamless scanning when using Medit Link with the same account on another PC. The original data cloud backup only applies to data acquired in Medit Scan for Clinics.

Quick Scan	Choose if you want to have the scan stages below automatically appear in Medit Scan for Clinics without having to fill in the form information. • Pre-op for Maxilla • Pre-op for Mandible • Face Scan		
	Advanced Rendering	When checked, the 3D model is displayed in a more vivid and realistic rendering.	
Viewing		Adjust the brightness of the 3D model.	
Options	Adjust Color Texture	* The resulting colors may slightly differ from Medit Link when you open the same data with other software.	
Medit User	Join the Medit User Experience Improvement Program.		
Experience Improvement Program	 We use the collected information to improve our products and services. We do not collect any information that identifies individuals. 		
The de not concertary information that facility		ult menu to display when starting the program.	
Default Page After Login	 Clinic accounts can choose between Patient and Case Box Lab accounts can choose between In Box and Work Box 		

Storage Management

Check and manage the local and cloud storage space.

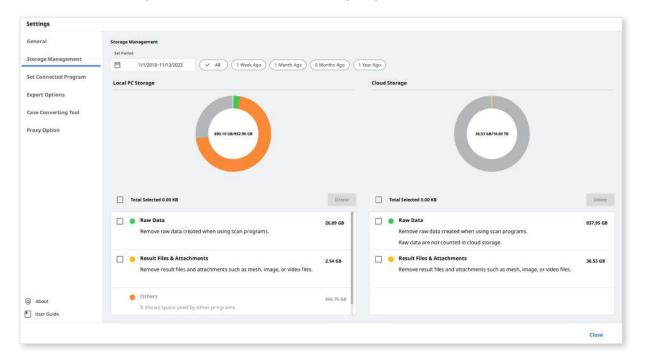

Set Period

Set the specific period to delete data created within the below time frame.

- All: See all files.
- 1 Week Ago: See the files that have passed a week after creating.
- 1 Month Ago: See the files that have passed a month after creating.
- 6 Months Ago: See the files that have passed six months after creating.
- 1 Year Ago: See the files that have passed a year after creating.

Local Storage

Manage the PC storage space.

	Remove the selected raw data from the PC.
Raw Data	If the raw data is already uploaded to the cloud, you can download and use it whenever you want.
Result &	Remove the result and attachment files from the PC, such as mesh, images, and videos.
Attachment	If these files are already uploaded to the cloud, you can download and use them whenever you want.
Others	See the files used to operate the system and other programs. It shows the storage of your PC. You cannot remove these files from Medit Link.
Delete	Remove selected files from the PC.

Cloud Storage

Manage the cloud storage space.

Raw Data	The raw data does not take up space in the cloud storage, so it will not be in the pi-chart. But you can manage the raw data from here if needed, e.g., delete it to protect the patient's private information, etc.
Result & Attachment	The result and attachment files take up space in the cloud storage. If the cloud is full, you can secure more storage space by removing unnecessary files.
Buy Storage	If you're on the 2 GB Plan, subscribe to the 10 TB Plan to expand your cloud storage.
Delete	Remove selected files from the cloud storage to free up storage space.

Set Connected Program

Manage the programs that support integration in Medit Link.



You can connect the following programs:

	Clinic Account	Lab Account
Scan Program	Medit Scan for Clinics	Medit Scan for Clinics Medit Scan for Labs




Integration of the following services is not available for Medit Link for Mac.

- Medit Scan for Labs
- exocad

Export Options

Set how to export files from Medit Link.

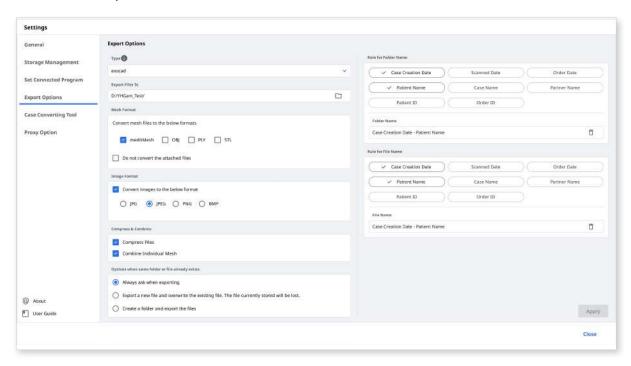


Туре	Select the CAD program you want to use for designing.
	The program will adjust the results based on your selection, allowing you to proceed with the design without any additional axis alignment.
Export Files To	Specify the path where you want to store the results.
Mesh Format	Specify the format of the final mesh file.
Image Format	Specify the format of the exported image files.
Compress & Combine	Compress Files: Compress all results into a single file.
	Combine Individual Mesh: Combine all results for the maxilla or mandible into one file. When using some programs that can't import and work with multiple files, you can merge them into one to work in those programs.
Options when the same folder or file already exists.	Choose the default action for cases when the same folder or file name exists in the target path.
	 Always ask when exporting. Export a new file and overwrite the existing files. The file currently stored will be lost. Create a folder and export the files

Define rules for creating a folder name when exporting files. The set rules will be applied to folder naming, and the exported file will be saved in the created folder. The below information can be used to set the rule. Rule for Folder Clinic account Name Case Creation Date, Scan Date, Order Date, Patient Name, Case Name, Partner Name, Patient ID, Order ID Lab account Case Creation Date, Scan Date, Order Date, Patient Name, Case Name, Partner Name, Order ID Define rules for the exported file name. The below information can be used to set the rule. Clinic account Rule for File Case Creation Date, Scan Date, Order Date, Patient Name Name, Case Name, Partner Name, Patient ID, Order ID Lab account Case Creation Date, Scan Date, Order Date, Patient Name, Case Name, Partner Name, Order ID

Case Converting Tool


You can move cases between different accounts by exporting existing cases of the account or importing cases exported from other accounts.

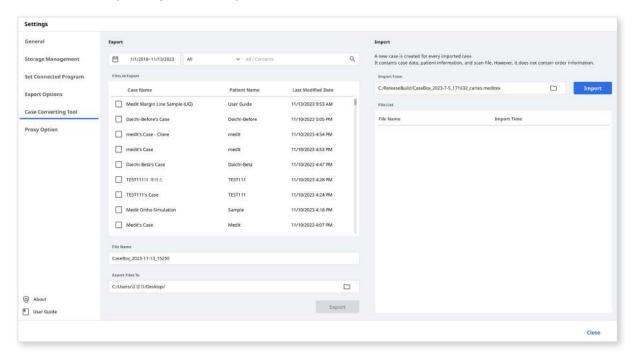


Export Case

Export cases of the currently logged-in account to the local PC.


① Open Medit Link and click the Settings icon.

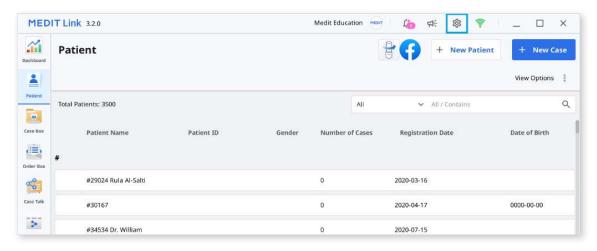

2 Choose the "Case Converting Tool" section in the left-side menu.

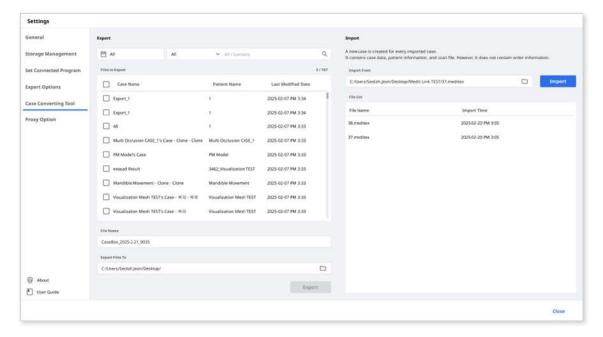


③ In the Export section, set a period or search for cases to export if necessary.

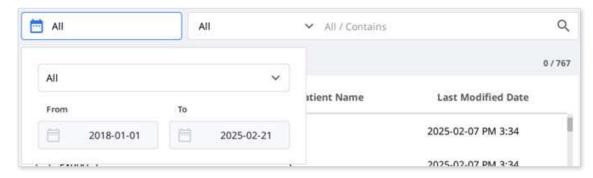

4 Choose cases you want to export from the list in the "Files to Export" section.




- 5 Set the file name by editing the case name in the File Name field.
- 6 Set the destination folder to export the selected cases.
- 7 Click "Export."
- 8 Click "Yes" to open the folder where the file is exported.

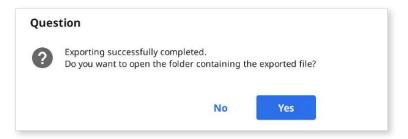


The exported cases are saved in .meditex file format.

Import Case

① Open Medit Link and click the Settings icon.

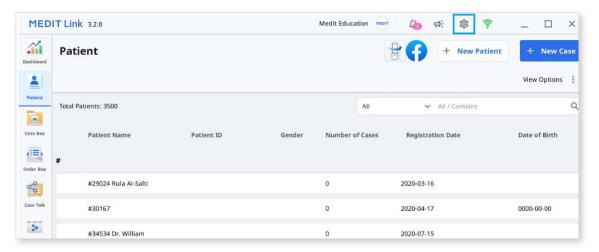


② Choose the "Case Converting Tool" section in the left-side menu.

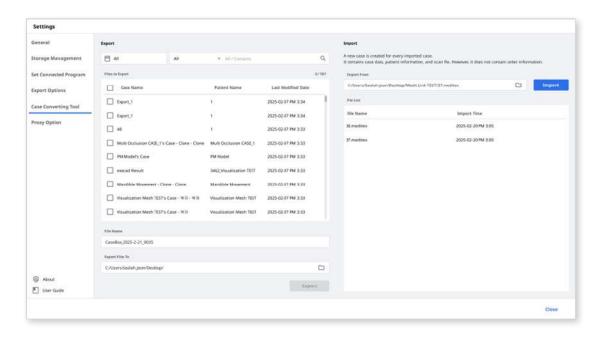

3 In the Import section, click the folder icon to browse your PC for the .meditex file you want to import.




④ Click the "Import" button and check the files to import in the list below.

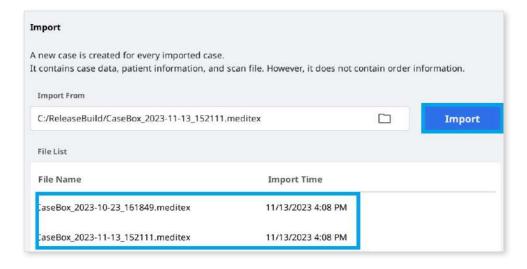

5 The imported case will also appear in your Case Box (Clinic account) or Work Box (Lab account).

Proxy Options

In some limited network environments, you can change network settings.

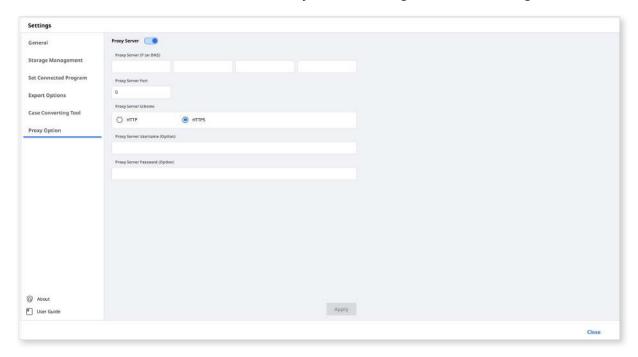

About

Show detailed information about Medit Link.



User Guide


Get the user guide for Medit Link.

Opening the user guide requires an Internet connection.

Menu

Dashboard

You can view the statistics for the entire work done on Medit Link in visualized graphs and charts – both in the Medit Link App and Web.

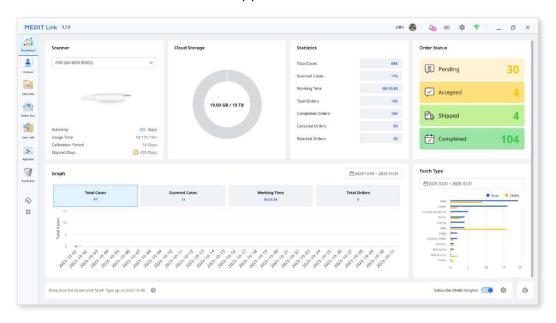


₽ Note

Please refer to the Medit Link Web > Dashboard for more information on the dashboard and Medit Insights.


Patient

You can add, edit, delete, or merge patients on the Patient menu.

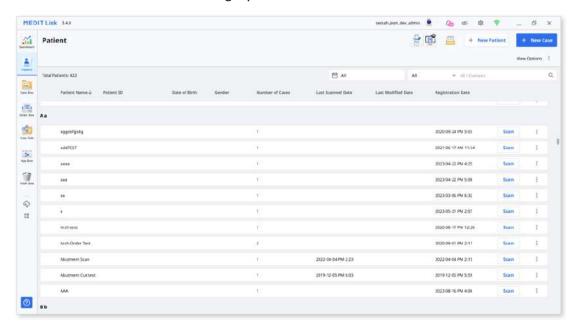

Add New Patient

① Click "New Patient."

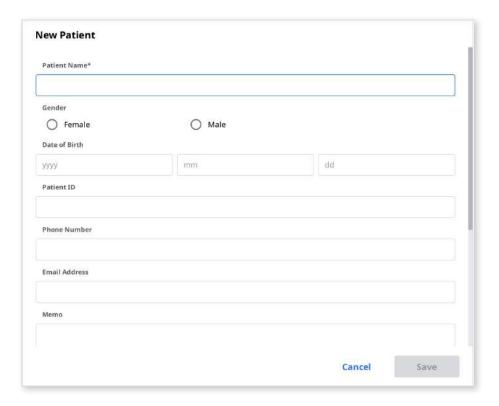

② Fill in the required information in the New Patient dialog, then click "Save."

Delete Patient



You can delete a patient only when there are no cases for the patient.

① Select the patient to delete from the list.



2 Click "Delete."


Merge Patients

You can merge all cases of multiple patients into one patient.

① Select the patients you want to merge from the list.




- ② Click "Merge Patients."
- 3 Select the target patient you would like to merge all cases into.
- 4 All cases are merged into the target patient and the patient information of the target patient maintains.

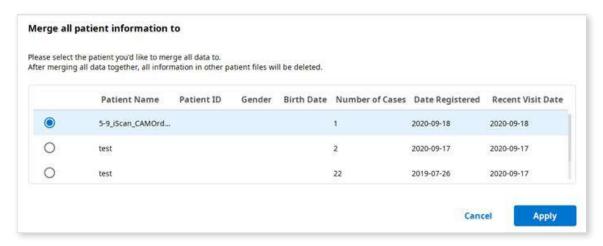

⑤ Click "Apply" to proceed with the merge.

Case Box/Work Box

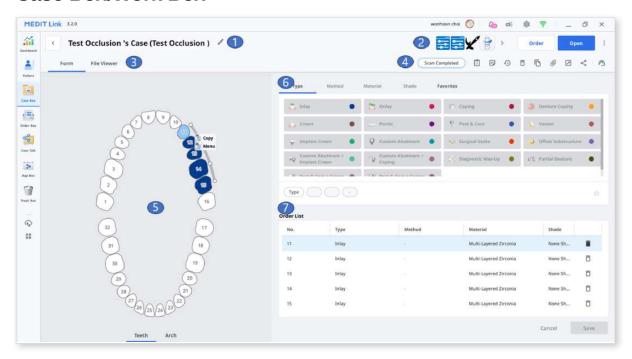

1	Show case name and patient name.
2	Provide key features available: Scan or Open button, Order button, app icons
3	Show available tabs: Form, File Viewer
4	Show functions for managing cases and files.
5	Show the form registration UI for the selected scanning program.
6	Provide the type/method/material/shade information for users to choose for the selected tooth number.
7	Show the type/method/material/shade information applied to the selected teeth.

Tabs in Case Box

Form	File Viewer
The form tab provides a UI for entering basic information for scanning work, such as type, method, material for teeth numbers, and scan options. The information you enter is passed to the connected scanning program and used as order information when placing an order with your partner.	View result files generated by scanning programs and CAD. The file viewer displays and manages the acquired or attached 3D models and 2D images.


Functions in Case Box/Work Box

The following functions are provided in Case Box/Work Box.

	Case Status	Shows th	ne status of the ca	ise.
4	Go to Order Box/In Box	Provides a shortcut for the same case in another box (Case Box \leftrightarrow Order Box; In Box $<->$ Work Box).		
î	Case Information	Review c		ve them as a PDF or print
=	Memo & Tags	Add a memo or tags when registering or ordering a case. Use them to communicate any additional or supplementary information about the case to your partners.		
		Provides	detailed informatio	n about the case history.
			History	
			Last Modified Date	3/29/2022 1:57 PM
	Case History		Scanning Date	4/12/2021 2:06 PM
+9			Creation Date	3/22/2022 1:17 PM
			Case Size	435MB
				Close
			e case.	
	Delete	You cannot delete ordered cases or cases that were created offline.		
		Create a duplicate of the current case in Case Box. All data will be copied to the cloned case.		
	Clone	This feature helps replicate treatment information for the same patient or make an additional order with the same information or files.		
0	Attach	Attach files in various formats.		
	Export	Export scan or CAD result files to the local PC. You can choose your preferred file format when exporting files.		
	·	The exported files can be used in CAD and CAM software for further work.		

≪	Share	Create a link for sharing. Users with the shared link can access the 3D data in a browser such as Chrome, Firefox, or Safari without installing additional programs.
		You can share the link freely with anybody or specific people only. A PIN code or a link expiration date can be set to secure private information.
	Submit Support Request	Land on a Medit Help Center page to submit the support request.

Order Box/In Box

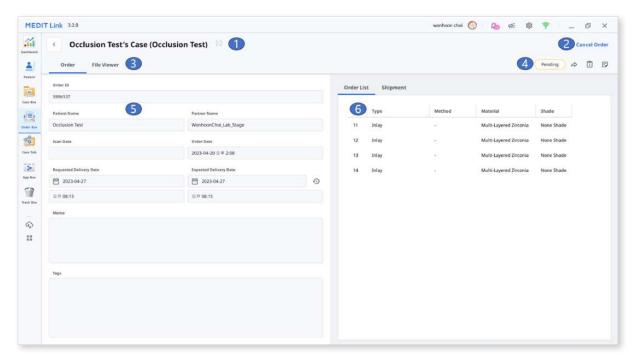

1	Show case name and patient name.
2	Provide functions for order management (accepting/canceling; delivering/receiving).
3	Show available tabs: Order, File Viewer
4	Show functions for managing cases and files: case status, shortcuts for detailed information, case information, memo & tags, etc.
5	Show the detailed information for the order including the order number, patient name, partner name, scanning date, order date, delivery date, and memo & tags.
6	Show the ordered product information and delivery status.

Tabs in Order Box/In Box

Order	File Viewer
Check the detailed information related to the order, such as order number, patient name, account name, scanning date, order date, delivery date, memo, and tags.	View files generated by scan/CAD operations. The file viewer displays and manages the acquired or attached 3D models and 2D images.


Case Talk

Cases created in the Medit Link can be shared with specific people, anyone with the shared link or registered members of Medit Link.

The user can see a list of all shared cases through Case Talk in the Medit Link App and Medit Link Web.




Case Talk service requires an internet connection.

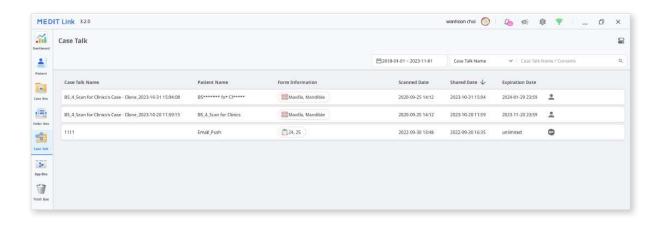

Shared cases can be viewed via a web browser without installing a separate program, enabling easy communication over the 3D data on various devices.

App Box

App Box provides various add-in programs that complement your basic scanning workflow and enhance productivity. You can download and install the provided apps to create dental appliances, analyze scan data, assist you during a consultation, or streamline your work process.analyze scan data, assist you during a consultation, or streamline your work process.

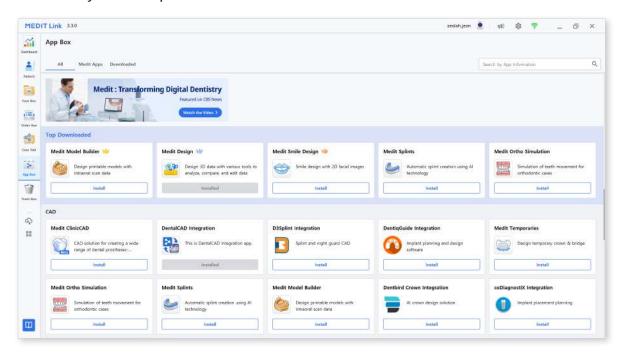


All

The "All" tab shows the list of all provided apps including Medit Apps which are Medit-developed original apps and partner apps, which are integration apps with Medit's partners from different sectors of digital dentistry, such as 3D printing, CAD/CAM software, etc

Medit Apps

The "Medit Apps" tab shows only the list of Medit-developed orignal apps.

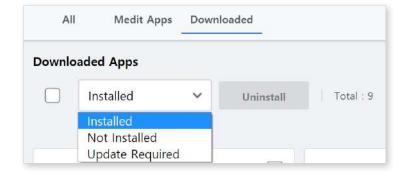

Downloaded

The "Downloaded" tab shows the list of currently installed apps on your PC and apps that have a history of being downloaded by the user account.


- Installed: The option shows only apps that are installed on the PC the user is currently signed in to.
- Not Installed: This option lets you see apps you've installed on other PCs so you can quickly install them.
- Update Required: This option only shows apps installed on the current PC that need to be updated

Trash Box

The Trash Box temporarily stores the deleted cases. Any case in the Trash Box can be either restored or permanently deleted.




The data deleted from the Trash Box will be permanently deleted from your PC and the server and cannot be recovered.

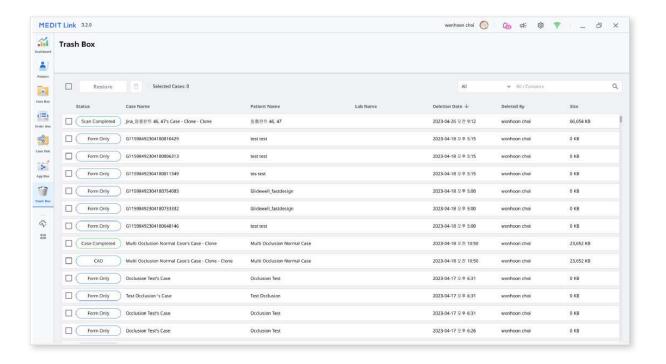


Case Synchronization Manager

The Case Synchronization Manager helps you to upload and download cases efficiently. You can upload multiple cases to the cloud or download them to your local PC. To identify cases requiring synchronization, simply search for them or apply date filters.

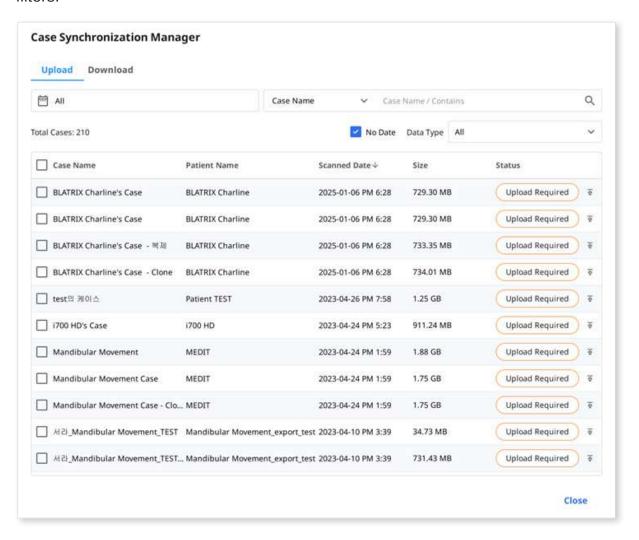

Upload

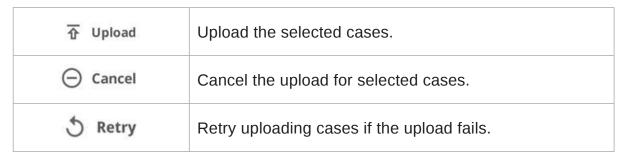

Download


◆ Download	Download the selected cases.
○ Cancel	Cancel the download for selected cases.
5 Retry	Retry downloading cases if the download fails.

Related Websites

You can go directly to menus only available on Medit Link Web, such as My Info, Partners, and Membership. Please refer to Medit Link > Medit Link Web > Partners, Membership, and My Info for detailed descriptions of the Medit Link Web menus.

You can access Medit's online store, Medit Shop, where you can purchase extended warranties for scanners and other Medit goods, such as scanner backpacks.

Also, you can get various information about using Medit Link. Access Medit's educational and informational resources, such as Medit Help Center, Medit Academy YouTube channel, and Medit Academy online education platform.

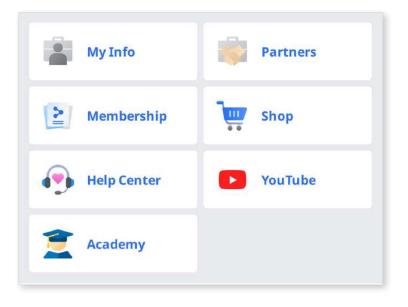

File Viewer

File Viewer groups and shows various data, including the data acquired from programs linked to Medit Link and data attached by users.

It provides basic management functions such as showing/hiding data, adjusting opacity, and deleting data.

It supports previews of files in the following formats:

• 3D data: STL, OBJ, PLY

• 2D images: PNG, JPG, JPEG, BMP, GIF, TIF, TIFF

Video files: MKV, MP4, AVI, WMV

Data Tree

You can group and display all data for the case in the data tree. You can also show/ hide data, adjust transparency, and delete data from the data tree.

1-				
(E)	Show/Hide	You can show or hide the data.		
•	Opacity	The slide bar appears when you click one of the data on the data tree. You can adjust the opacity of the shown data by moving the slider bar.		
000	More	Clicking the More icon shows additional features available. Maxilla Base Show This Only: Hide all other data but display only the corresponding data. Hide: Hide the data. Duplicate: Copy the data and add it to the attachment group. Export: Export the selected files. Rename: Change the name of the data. Delete: Delete the data.		




You can rename data when online.

Model Control Tools

3D Model Controls

\leftrightarrow	Pan		Move the model.
\bigcirc	Rotate		Rotate the model.
(±)	Zoo	m In/Out	Zoom in and out on the model.
Q	Zoom Fit		Position the model in the center of the screen.
			See the data in color with glossy finish (PBR rendering).
		Glossy	
			See the data in color with matte finish (Phong rendering).
<u>&</u>	Data Display Mode	Matte	
			See the data in one color.
		Monochrome	

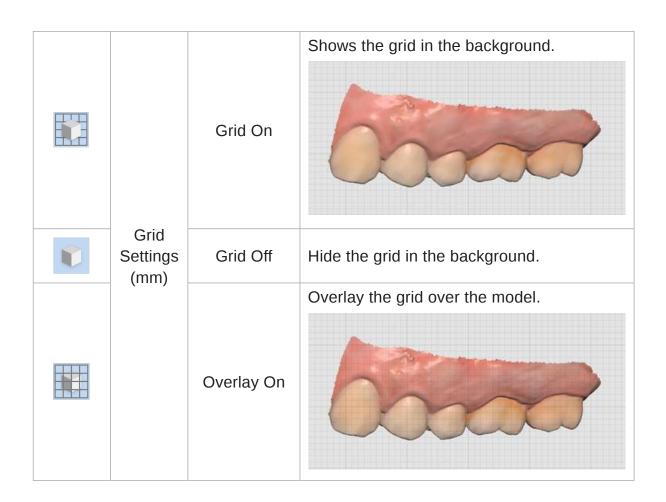

2D Image Control


< >	Next/Previous Image	Move among images.
0"	Edit	Edit the image.
	Pinch/Paste Window	You can detach the image window and adjust it to the desired position. You can resize the window.
		When the window is restored to its original location, it returns to its default size.

Function Tools

Image Capture and Memo


Capture the current screen as a 2D image, draw on it, or add notes.

You can save it as a file with your notes and send it as a reference.

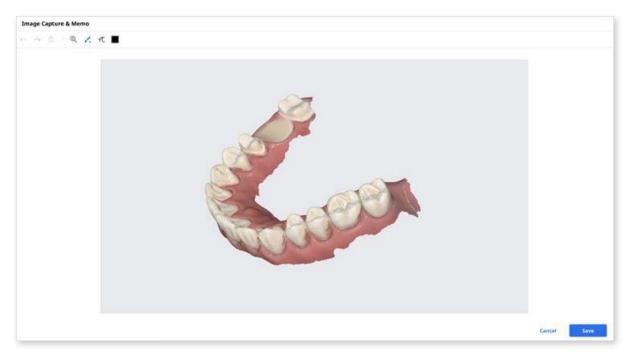

Ortho Capture


Capture the images in the current viewing direction and top/bottom/left/right from the current viewing direction.

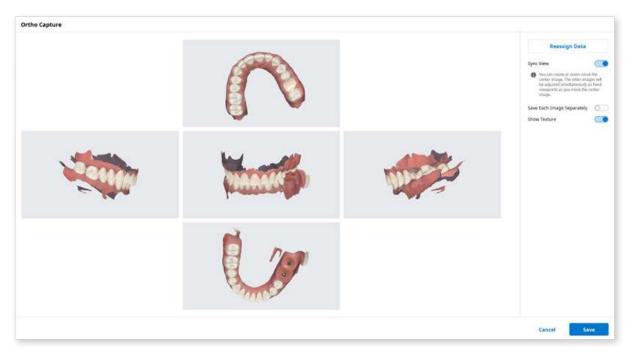

Save Each Image	ave each image in high resolution.	
⊥ Sa	ave each image in high resolution.	
Show Texture Tu	urn on/off the texture for the model.	
Da uti	hange the data for maxilla/mandible with the Reassign at a feature. You can use the changed data to create and ilize the result images of Ortho Capture. ***Scan Data: Scan data for the case are all shown. **Maxilla: The data you selected for the maxilla are displayed. **Mandible: The data you selected for the mandible are	


Fill Holes

Fill holes in 3D mesh data.

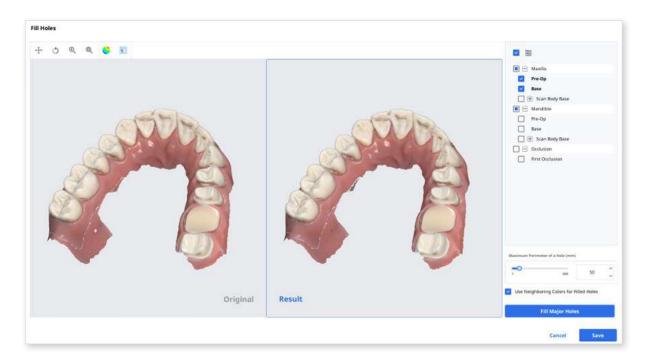

Trimming


Trimming provides various tools to edit data.

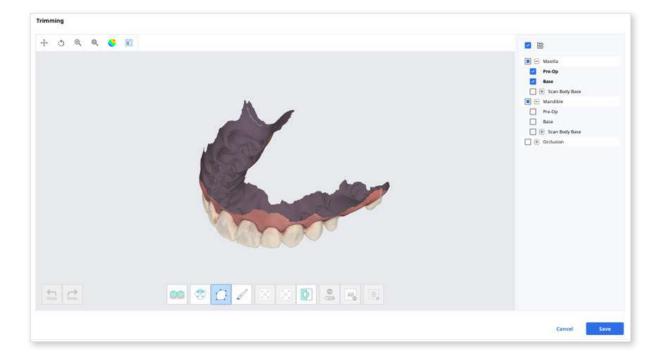

The selected 3D mesh data can be edited using different tools located at the bottom of the window.

	∞	Smart Teeth Selection	Automatically select all teeth of the arch, leaving out gingiva parts. Note This function is only available for the scan data acquired using Medit Scan for Clinics with the "Use GPU" option on.
Selection	(3)	Smart Single Tooth Selection	Automatically select the area of a single tooth with a click. You can click or drag on the tooth.
		Polyline Selection	Select all entities within a polyline shape drawn on the screen.
	8	Brush Selection	Select all entities on a freehand-drawn path on the screen. Only the front face will be selected. The brush comes in three different sizes.
	N K	Shrink Selected Area	Reduce the selected area each time you press the button.
Adjustment of Selected Area	K N	Expand Selected Area	Expand the selected area each time you press the button.
7 licu	Ф	Invert Selected Area	Invert the selection.
	•	Selection/ Deselection	When on, deselect the area using various tools.
Deselection	All	Clear All Selection	Clear all selected areas.
	٥	Delete Selected Area	Delete the data from the selected area.


① Select the area you want to trim with the area selection tools.

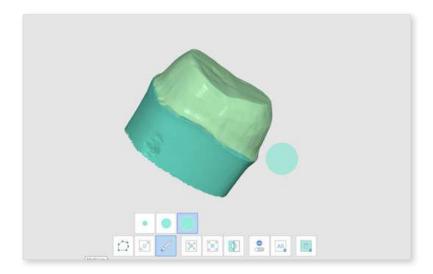


⊘ Note

The Polyline trimming tool selects all entities within a polyline shape drawn on the screen, while the Brush trimming tool selects only the front face data.

- 2 You can invert, shrink, or expand the selection with the following three tools.
 - Invert Selected Area

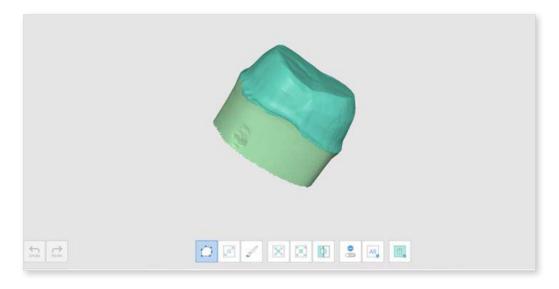

Shrink Selected Area

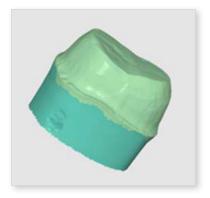


• Expand Selected Area



3 You can deselect the selected areas with the Delete Selected Area tool.

Optimize Occlusion Alignment

Optimize the occlusion alignment. You can choose the target data and adjust the occlusion alignment.

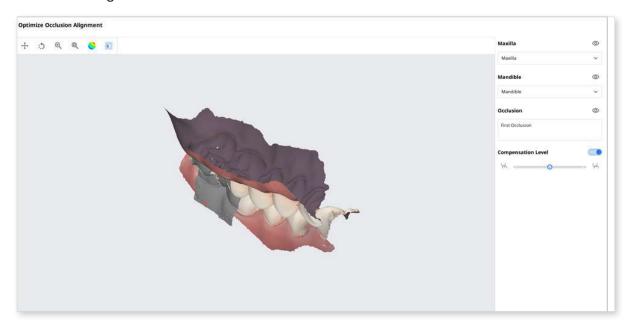

Compensation Level



Mandibular Movement

You can play the mandibular movements recorded in Medit Scan for Clinics to see the dynamic occlusal contacts between the upper and lower dentitions.



Free Movement	Play the recording of free mandibular movement.
Left Lateral Movement	Play the recording of the mandible moving to the left.
Right Lateral Movement	Play the recording of the mandible moving to the right.
Protrusive Movement	Play the recording of the mandible moving forward.

Manage Multi Occlusion Data in File Viewer

You can get multiple results of occlusion data from Medit Scan Application.

Medit Link provides the function to manage the results through the "File Viewer."

How to manage multiple results

When the user gets multiple occlusion results from Medit Scan for Clinics, Medit Link provides the option to select a result.

The list of results is listed by name, and you can select one item.

The selected model will be shown in the "Data Tree" and "Model Display Area."




You can collapse the list and change the item using the arrow icons.



View Options

You can change the View Options to customize the layouts of the case list in the Patient/Case Box/Work Box/In Box/Order Box.

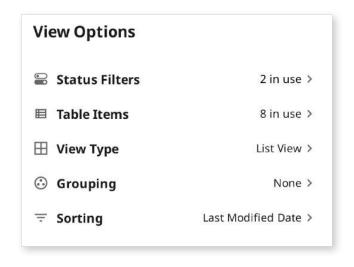

Status Filters

Show or hide each filter representing the status of the case. The hidden status filters are not displayed on the screen in the Case Box/Work Box/In Box/Order Box.

You can easily find the case you need with status filters.

- You can click the status you want to view to display only cases in that state.
- You can select multiple statuses at a time.

For each box, the following filtering options are available:

Вох	Status Filters
Case Box	All / Form Only / Need Processing / Scan Completed / CAD / Ordered / Case Completed
Order Box	All / Pending / Canceled / Rejected / Accepted / Shipped / Case Completed
In Box	All / Pending / Canceled / Rejected / Accepted / Ready / Shipped / Case Completed
Work Box	All / Form Only / Need Processing / Scan Completed / CAD / Done

Table Items

Select the items to display in the case list in the Patient/Case Box/Work Box/In Box/ Order Box. You can change the order of table items or adjust the table column width in the list view to best optimize for the user environment.

For each box, the following table items are available:

Box	Table Items
Patient	Patient Name / Patient ID / Gender / Number of Cases / Registration Date / Date of Birth / Last Scanned Date
Case Box	Status / Case Name / Date of Birth / Form Information / Last Modified Date / Scanned Date / Order Date / Lab Name
Patient	Patient Name / Patient ID / Gender / Number of Cases / Registration Date / Date of Birth
Case Box	Status / Case Name / Date of Birth / Form Information / Last Modified Date / Scanned Date / Order Date / Lab Name
Order Box	Status / Case Name / Patient Name / Scanned Date / Order Date / Lab Name / Requested Delivery Date / Last Modified Date

In Box	Status / Case Name / Order Date / Requested Delivery Date / Last Modified Date / Clinic Name
Work Box	Status / Case Name / Patient Name / Form Information / Last Modified Date / Requested Delivery Date / Scanned Date / Clinic Name / Order Date

View Type

Change the view type between the Gallery View and List View on the Case Box.

For each box, the following view type options are available:

Box	View Type Options
Patient	List View
Case Box	List View / Gallery View
Order Box	List View
In Box	List View
Work Box	List View

Grouping

Select a grouping option to classify and rearrange cases in the list by the selected option.

For each box, the following grouping options are available:

Box	Grouping Options
Case Box	None / Patient Name / Scanned Date / Order Date
Order Box	None / Scanned Date / Order Date / Requested Delivery Date
In Box	None / Order Date / Requested Delivery Date
Work Box	None / Requested Delivery Date



The grouping options are unavailable in the Patient, Case Talk, and Trash Box.

Sorting

Select a sorting option to change the order of cases in the list according to the selected option.

For each box, the following sorting options are available:

Box	Grouping	Sorting Options
Patient	-	Patient Name / Patient ID / Date of Birth / Registration Date / Number of Cases / Last Scanned Date
	None	Patient Name / Last Modified Date / Scanned Date/ Order Date / Lab Name
Coop Boy	Patient Name	Last Modified Date / Scanned Date / Order Date / Lab Name
Case Box	Scan Date	Patient Name / Last Modified Date / Order Date / Lab Name
	Order Date	Patient Name / Last Modified Date / Scanned Date / Lab Name
	None	Patient Name / Scanned Date / Order Date/ Requested Delivery Date / Lab Name
	Scan Date	Patient Name / Order Date / Requested Delivery Date / Lab Name
Order Box	Order Date	Patient Name / Scanned Date / Requested Delivery Date / Lab Name
	Requested Delivery Date	Patient Name / Scanned Date / Order Date / Lab Name
In Box	None	Patient Name / Order Date / Requested Delivery Date / Clinic Name
	Order Date	Patient Name / Scanned Date / Requested Delivery Date / Clinic Name
	Requested Delivery Date	Patient Name / Order Date / Clinic Name
Work Box	None	Patient Name / Last Modified Date / Requested Delivery Date / Clinic Name
	Requested Delivery Date	Patient Name / Last Modified Date / Clinic Name


⊘ Note

The sorting options are unavailable in the Case Talk and Trash Box.

Filtering and Search

Filter by Patient


Filter and display cases by patient name. You can create a list by filtering only the patients you need to work on.

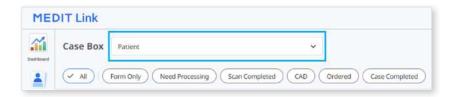


Enter a patient name.



② Select the patients among the search results below.

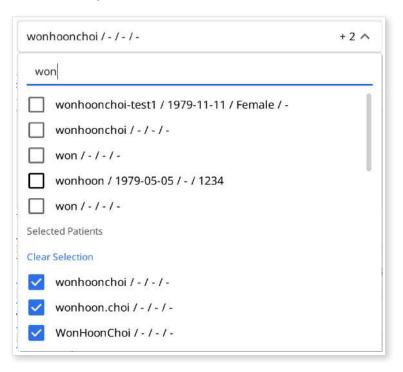

3 Only cases of the selected patients are displayed on the list.

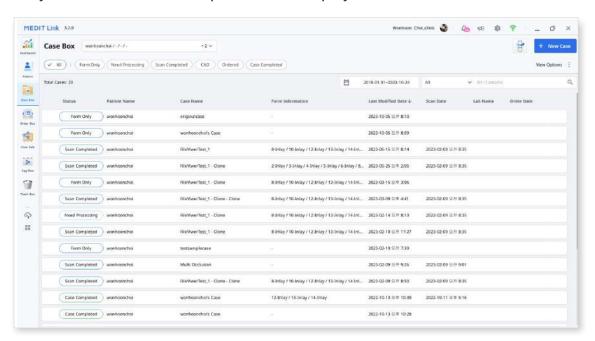

Filter by Date

Filter and display cases last modified within the set duration.

Click the duration filter.




- Select the duration from the list.
 - All: Shows all cases.
 - Custom Range: Specify the start and end dates to display cases last modified within the period.
 - Today: Show the cases last modified today.
 - Yesterday: Show the cases last modified yesterday.
 - Past 7 days: Show the cases last modified during the previous 7 days excluding today.
 - Past 30 days: Show the cases last modified during the previous 30 days excluding today.
 - This Month: Show the cases last modified during this month.
 - Previous Month: Show the cases modified during the last month.

The case list will be updated in real time with the cases created or last modified within the selected duration.

Advanced Search Capabilities


Leverage search capabilities to quickly find the information you want.

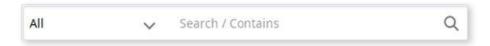

The following Searching Criteria are available for each box:

Patient	All, Patient Name, Patient ID
Case Box	All, Case Name, Patient Name, Patient ID, Form Information, Lab Name
Order Box	All, Case Name, Patient Name, Patient ID, Lab Name, Order ID
In Box	All, Case Name, Patient Name, Clinic Name, Order ID
Work Box	All, Case Name, Patient Name, Form Information, Clinic Name, Order ID
Case Talk	Case Talk Name (Contains), Patient Name (Equals)
Trash Box (Clinic)	All, Case Name, Patient Name, Lab Name, Deleted By
Trash Box (Lab)	All, Case Name, Patient Name, Clinic Name, Deleted By

You can select the Searching Criteria and choose between the Contains and Equals logic.

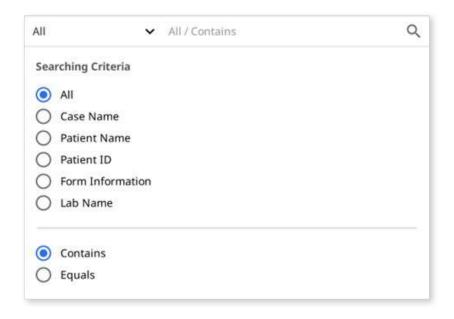


Medit Link > Workflow

Create/Edit/Clone/Delete Case

Creating Cases

Create Case in Case Box (Clinic)


① Click the "New Case" button in the upper right corner.

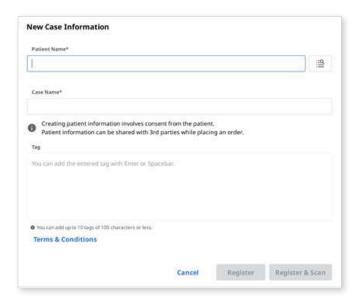

- ② Enter basic information, such as the patient's name and case name...
 - If you enter a name not on the patient list, a new patient will be added.
 - If you enter a name that is in the patient list, you can select it from the list.
 - If registering a case for an existing patient, click the Patient List icon to select an existing patient.




 In the Patent List dialog, you can register a new patient or edit the information of an existing patient.

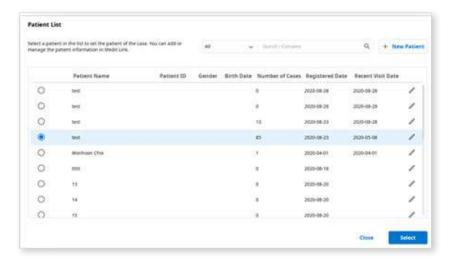

- 3 The case name will be filled in automatically, but you can still change it.
- 4 Click "Register & Scan" to complete the case registration and proceed to scan.
 - Register: You will be redirected to the main page. This is useful when registering multiple cases.
 - Register & Scan: Move to the Form tab to register form information for the case.

Create Case in Work Box (Lab)

① Click the "New Case" button in the upper right corner.

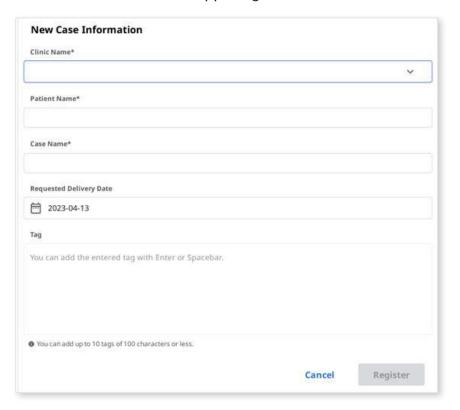


- (2) Enter or select the clinic name.
- 3 Enter the patient's name and case name. The case name will be filled in automatically, but you can still change it.
- 4 Enter the expected delivery date considering the completion date of the work.
- S After registering the case, you will be redirected to the main page.


Editing Cases

Edit Case in Case Box (Clinic)

Click the "Edit" icon next to the case name.

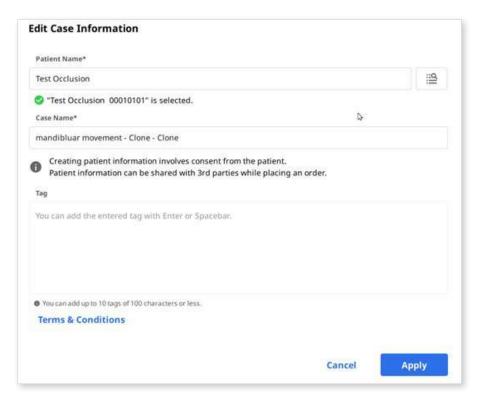


2 Click the "Search" icon to change the patient's name. Note that it cannot be edited directly.; instead, you can find a patient name you would like to replace the current one with and can edit the patient information from the list.

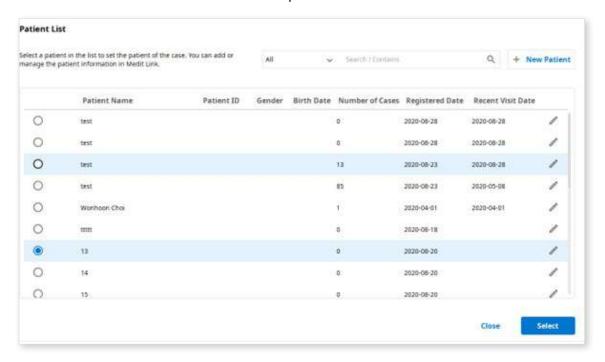

- 3 Check the patient's name you want to replace the current one with.
- 4 To create a new patient, press the "New Patient" button.
- ⑤ Use the "Edit" button to change the registered information about the patient.
- 6 Click "Apply" to finish the process.

Edit Case in Work Box (Lab)

1 Click the "Edit" icon next to the case name.

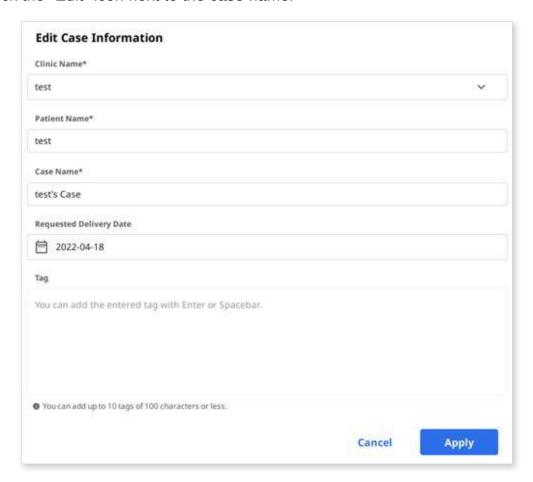

② Click "Apply" after the editing is done.



You cannot edit the information of the cases received through the order in In Box.


Cloning Cases

Clone Case in Case Box (Clinic)

① Click the "Clone" icon at the top.



2 You will be asked what kind of information you would like to clone to the new case.

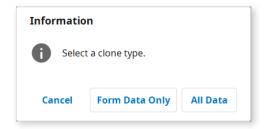

- 3 Choose whether to replicate only form information only or all data.
- 4 When cloning is complete, a case named "[Case name] Clone" will be created.

Clone Case in Work Box (Lab)

① Click the "Clone" icon at the top.



- ② Duplicate all data to create a new case.
- ③ When cloning is complete, a case named "[Case name] Clone" will be created.


Deleting Case

Delete Case (Clinic and Lab)

① Click the "Delete" icon.



② Click "OK" to delete the case.

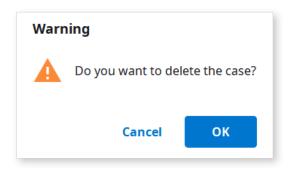

3 Deleted cases are moved to the Trash Box and can be recovered if necessary.

Form Information

You can enter the form information based on the connected program and the treatment type.

The information you enter will be used as basic information for working on scanning or CAD programs and ordering.

Registering Form Information in Case Box

Enter on the Form Tab

- ① Select whether you are going to input treatment information by teeth or by arch.
- 2 Select Type Method Material Shade.
 - If you entered information incorrectly, you go back to select it again.
 - The selected information is automatically registered to the order list.
- 3 Click "Save" to save your registration.

Registering Form Information in Work Box

Enter on the Form Tab


- ① Select the scan program you want to use.
- ② Select one of the following options that will define your scanning strategy: Teeth/ Flexible Multi-Die/Multi-Die.
- 3 Methods may differ depending on the selected scan program.

	Medit Scan for Clinics	Medit Scan for Labs
Teeth	0	0
Arch	0	0
Flexible Multi-Die	X	0
Multi-Die	X	0

- 4 Select teeth, arch, or die numbers for the scanning process.
- 5 Select Type Method Material Shade.
 - If you entered information incorrectly, you go back to select it again.
 - The selected information is automatically registered to the list.
- 6 Click "Save" to save your registration.

Connecting Teeth into Bridge

You can connect multiple teeth into a bridge when the same material is applied to multiple adjacent teeth. Click the bridge icon beside the selected teeth to connect or disconnect them.

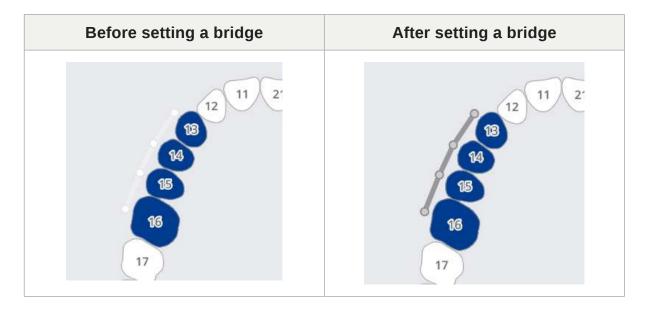

Setting Scanning Options

Depending on the program you are working with, various scan options are provided.

The selected scan options are passed to each scanning program.

Medit Scan for Labs

General options

	Adjacent teeth	Mark the prepared and adjacent teeth and pass this information to the scanning program.
1	Antagonist	Automatically mark the antagonist and passes this information to the scanning program.
	Articulator	Display options for selecting the articulator in the scanning program.
•	Movement Marker	Allow using the "Jaw Motion" scan strategy in the scanning program.

Options by teeth

ck	Scan Body	Allow setting a strategy for scanning scan bodies in the scanning program.
100 mg	Pre-Op Scan	Allow setting a strategy for scanning a pre-op model in the scanning program.
0000 0000	Wax-Up Scan	Allow setting a strategy for scanning wax-ups in the scanning program.

Adding to Favorite

You can quickly select a product by registering it as a favorite.

The list of registered favorites is managed as a template and can be shared through the export and import features.

Add to Favorites

- ① Select the tooth number.
- ② Select Type Method Material Shade.
 - Type Method Material Shades can be registered as favorites even if not all are selected.
- 3 Click the "Star" icon.



④ A combination of the selected parameters, excluding tooth number, is on your favorite search list.

Manage Favorite Templates

You can register, manage, and share different types of templates.

Add	Add a new template.
Rename	Rename the current template.
Delete	Delete the current template.
Export	Export the current template to a file.
Import	Import a template file.

Work on Scanning/CAD

Scan Process

Based on the entered information, you proceed with scanning.

- ① Enter/save treatment information and select the necessary scan options.
- 2 Click the "Scan" button in the upper-right corner of the window.

CAD Process

Medit Link provides the DentalCAD Integration app to connect Medit Link with DentalCAD.

- ① Install the DentalCAD Integration app in the App Box.
- ② Go to the case details and click on the DentalCAD Integration app icon on Quick Launcher.
- 3 Specify the path to the DentalCAD app installed on your PC in the pop-up dialog.
- ④ Enter the case details of the scanned case and click the DentalCAD Integration app icon, which will pop up a dialog to create a dentalProject file.
- (5) Register form information for DentalCAD and click "Open" to open the CAD program.
- 6 If you install the DentalDB Integration app, the CAD results will be automatically delivered to the specified save path in exocad. (Lab accounts only)



Medit Link for Mac does not support the function to link with CAD.


Utilize Add-in Apps

Add-in Quick Launcher

You directly run various add-in apps by clicking on the installed app icons in the Quick Launcher. You can change the order of app icons by dragging them.



App Launcher Management

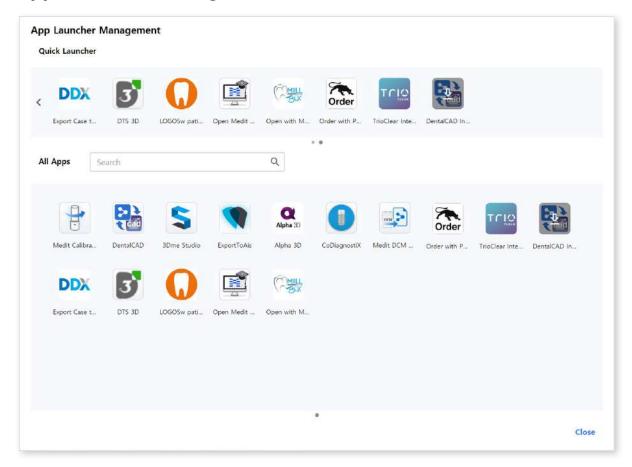

Quick Launcher

You can define which apps appear in the Quick Launcher. You can change the order of app icons by dragging them or or create folders to group similar apps together to organize them.


All Apps

All apps you have installed are shown.

Get Result Files

Export Scan Result Files


In addition to sharing results with partners through orders, Medit Link allows you to store your result files in the desired file format using the export function.

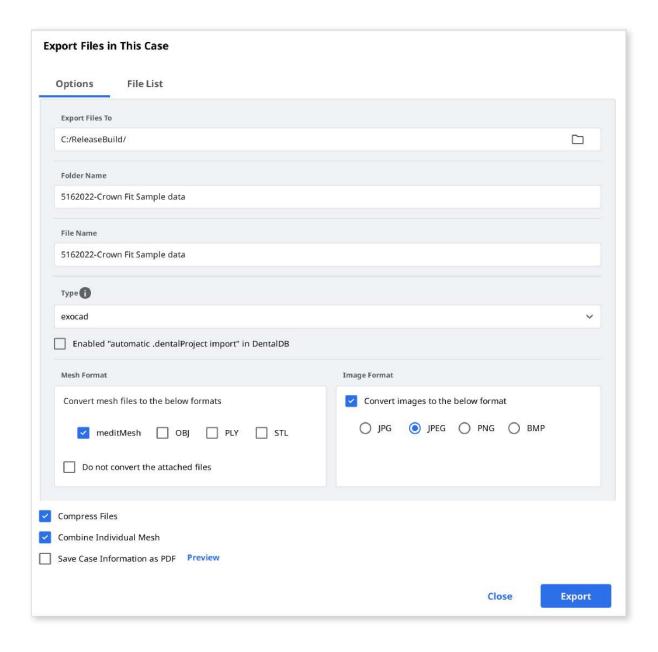


Export Files To	Specify the path where you want to store the result files.
	The system creates a new folder and stores all result files in it.
Folder Name	The default folder name is set by the naming rule. But you can change the folder name directly.
	You can define the naming rule on Settings > Export Options > Rules for Folder Name.
File Name	Set the file name for the result. The naming rule sets the default file name. But you can change the file name directly.
	You can define the naming rule through Settings > Export Options > Rules for File Name
	Choose the design program you want to use.
Туре	Medit Link will automatically adjust the files according to your selection, enabling you to proceed with the design without the need for additional axis alignment.
Mesh Format	Specify the format of the mesh file.
Image Format	Specify the format of the exported image files.
Compress Files	Compress all results into a single file.
Combine Individual Mesh	Combine the result of the maxilla or mandible with the individual tooth. It is useful when importing several files into a program is not possible.
Export All Occlusal Relationship	When multiple occlusal results are generated from the scan app, you can export all occlusal results at once.
Save Case Information as PDF	Save the case information as a PDF with the result.

Export Result Files from Case List


You can export result files of a case or multiple cases simultaneously with the "Export" option above the case list in Case Box and Work Box and In Box.

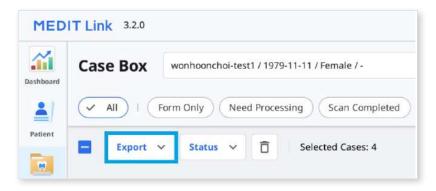


- ① Go to Case Box or Work Box.
- ② Select all cases you want to export result files from the list.

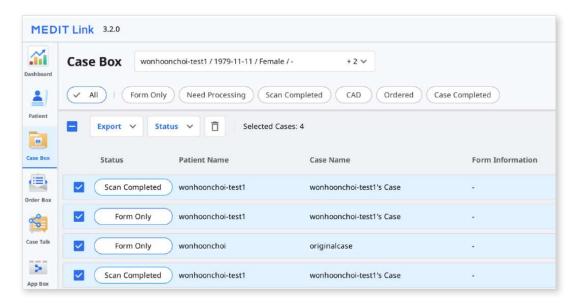

3 Click "Export" and select "Result Files."

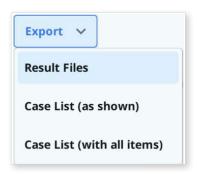

④ Set the destination folder, file type, mesh format, and image format, and select export options on the "Options" tab.

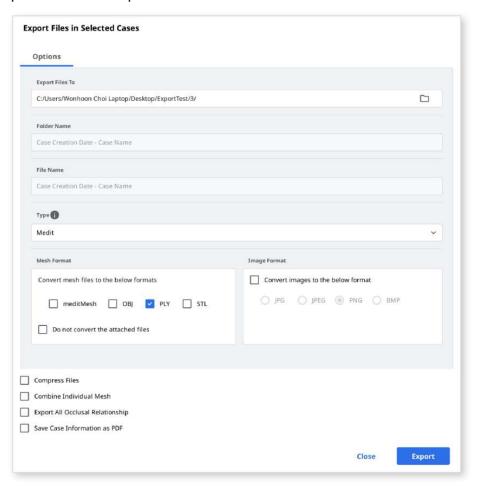

The "Folder Name" and "File Name" fields are not editable when exporting files for multiple cases.


The folder name and file name are automatically created according to the rules you set for the "Rule for Folder Name" and "Rule for File Name" options in Settings > Export Options.

- 5 Click "Export."
- 6 Click "Yes" to open the folder where the files are saved.



Export Result Files from File Viewer


You can export scan result files for a case with the "Export" icon on the top right corner of the File Viewer tab.

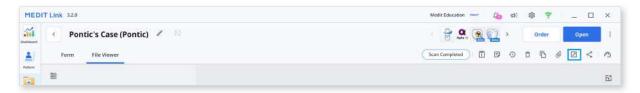


- Go to Case Box or Work Box.
- 2 Click to open a case from the list.
- 3 Click the "Export" icon on the top right corner of the File Viewer tab.



④ Set the destination folder, folder name, file name, file type, mesh format, and image format, and select export options on the "Options" tab.

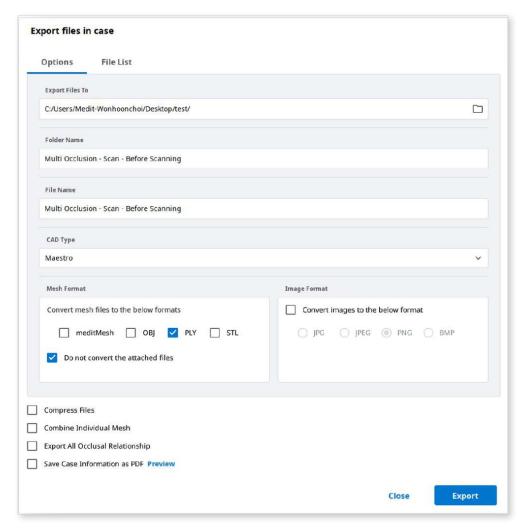

⑤ In the "File List" tab, you can select the required files for the case by deselecting unnecessary ones.

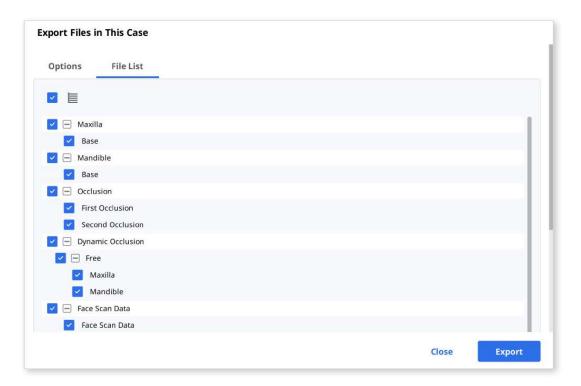


- 6 Click "Export."
- 7 Click "Yes" to open the folder where the files are saved.



Export Result Files from Data Tree


You can export a file or files in a specific data group from the expanded menu in the file viewer.

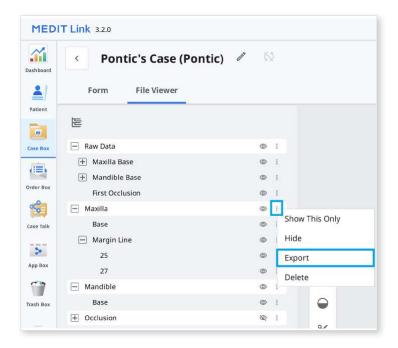


- ① Go to Case Box.
- 2 Click to open a case from the list.
- 3 Click the three dots icon next to the data group or an entity in the data tree in the File Viewer tab.

0


4 Then click "Export."

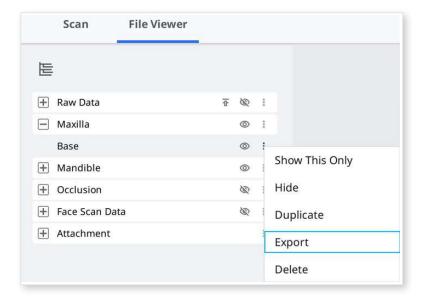

(5) Set the destination folder, folder name, file name, file type, mesh format, and image format, and select export options on the "Options" tab.

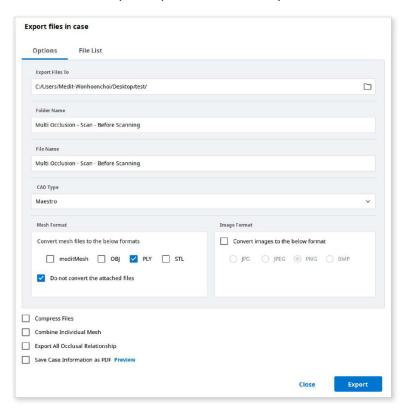

6 You can see only the files you chose in the data tree are selected in the File List.

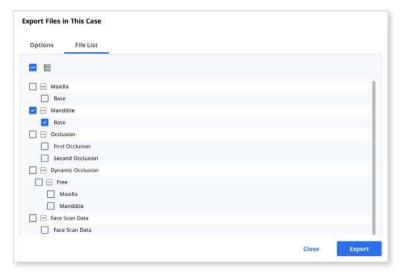

- Olick "Export."
- 8 Click "Yes" to open the folder where the files are saved.




Export Case List as CSV


Medit Link provides the function to export the case list in CSV format.

You can export the case list with basic information and manage cases easily.

① Go to Case Box, Order Box, In Box, or Work Box. Select the cases that you want to export.

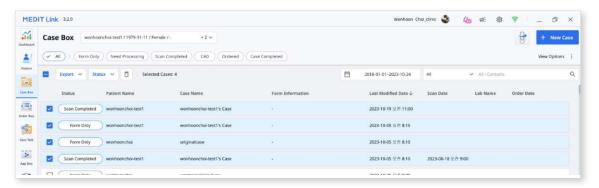

③ Click "Export" and select "Case List (as shown)" or "Case List (with all items)."

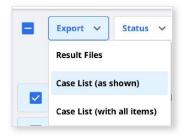

₽ Note

Case List (as shown): Export the case list with the currently displayed items on the screen. Go to View Options > Table Items to show or hide the items as you desire.

Case List (with all items): Export the case list with all items, including Status, Case Name, Date of Birth, Form Information, Last Modified date, Scanned Date, Order Date, and Lab Name.

- 4 Define the path and file name, then save the list.
- (5) The case list is saved as a CSV file.


Export Margin Line

Medit Scan for Clinics provides a function to create the margin line, and Medit Link can transfer the margin line to the CAD program.

Export Margin Line as a Color

In general, Medit Link converts the margin line as color and exports the result with maxilla or mandible together. You can create the margin line in their cad referencing the color information.

Example:

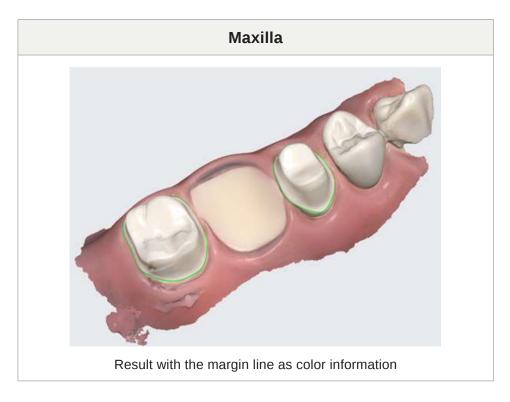

⊘ Note

This feature is available for formats that include color information, such as meditMesh, OBJ, and PLY. The margin line does not convert into STL.

Note


Type > 3shape does not support this feature.

Export Margin Line as a File

In the case of Exocad, Medit Link export the margin line to "~margin.xyz" which can be imported into exocad directly.

You can use this file in exocad without additional work.

Example:

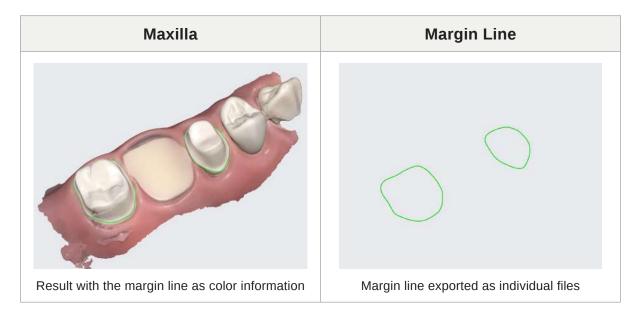

Download Files from Medit Link Web

Medit Link provides a download function.

If you want to get the result files without installing the Medit Link application, you can download the files from the Medit Link web service.




The data should be uploaded on the Medit Cloud to download the files from the web.

The download function is available in the following menu of the Medit Link web service.

- Case Box
- Work Box
- Web Viewer

Case Box/Work Box

You can find the "Download" button at the top right.

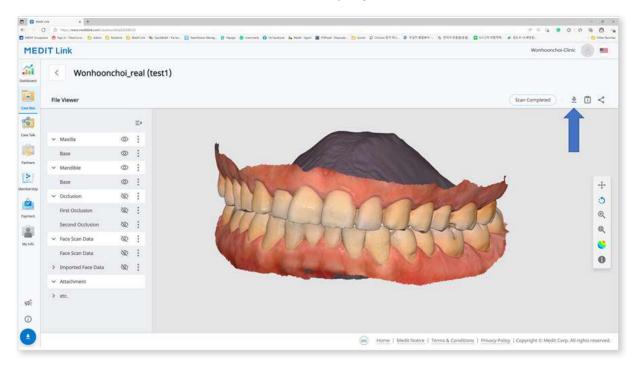


Web Viewer


When you share the case using Case Talk, you can set whether to allow downloading or not.



If you check "Grant Download Permission" and share the case, all users who connect to the shared link can download the files.

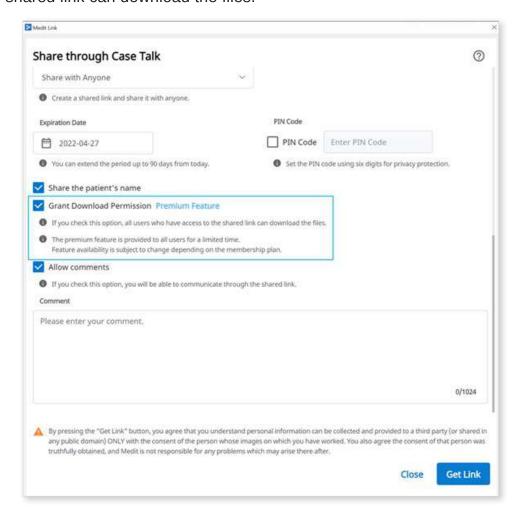


In the Web Viewer, you can find the "Download" icon at the top right.

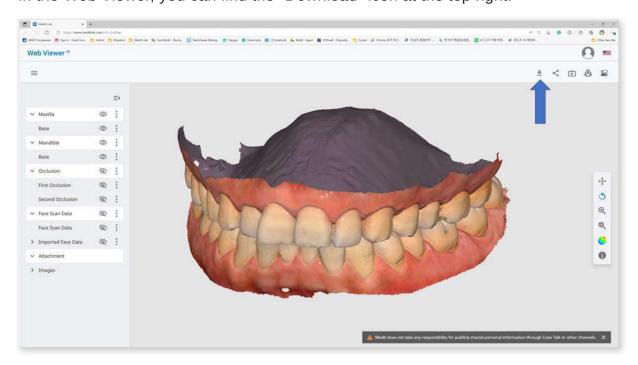

① Click "Download."



② Select the mesh format from the download dialog.

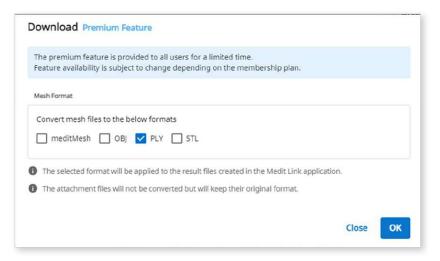

3 Then click "OK."


Order/Manage Orders

You can easily order a case with data acquired from connected programs and attached by the user.

Ordering Cases

You can place an order for your partners in Case Box.

- Click the "Order" button.
- ② Check the information required for your order.

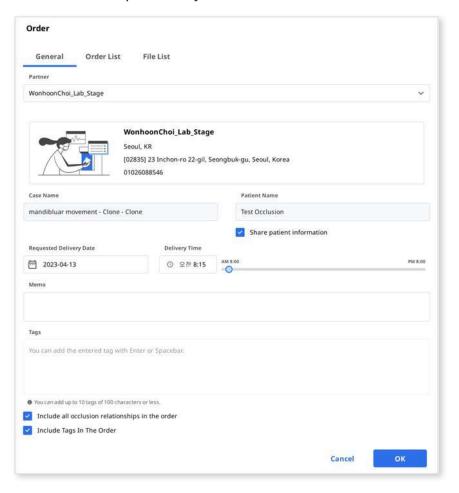

- 3 Select a partner.
- 4 Check the product information from the Order List tab.
 - You can decide whether to share the patient's name.
- (5) Check the files to order from the File List tab.
- 6 Enter your desired delivery date.
 - The default value is set to 7 days after the order date.


- 7 You can add a memo and tags to your partner to outline specific details.
- When multiple occlusal results are generated from the scan app, you can send all occlusal results at once by checking "Order All Occlusal Relationship."
- 9 Proceed with your order.

Confirm Order (In Box)

- · Accept the order upon receiving it.
- · Accepted orders will be forwarded to the Work Box.

Cancel Order (Order Box)

- You can reject a received order.
- Rejected cases can be re-ordered.

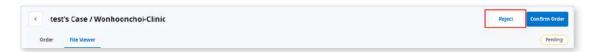


Reject Order (In Box)

- Accept the order upon receiving it.
- Accepted orders will be forwarded to the Work Box.

Checking Ordered Cases

You can check the details of the case you ordered from the Order Box in the clinic account. You can also check the case you received from the In Box in the lab account.

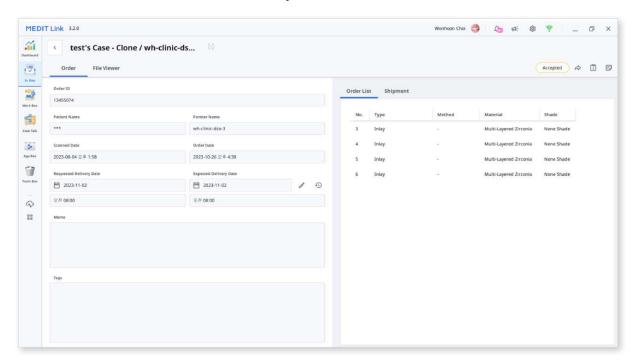


Order Info	The information that can be included in the order is as follows: order ID/patient name/account name/scan date/order date/delivery date/delivery due date/memo.
	You can check the details of the ordered product and delivery information.
Order List	The order list displays the details of the ordered product.
Shipment	Enter/check the shipping information.

Entering Shipment Information (In Box)

Enter the shipping information of the completed case and start shipping.

- ① Click the Shipment tab.
- ② Enter the shipment information and click "Register."

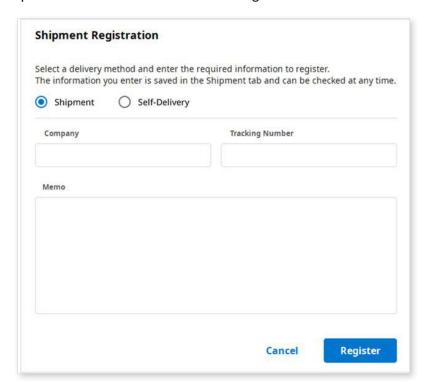


3 The entered information is sent to your partner.

Managing Delivery Date

You can set the expected delivery date when placing an order, and this information will be forwarded to your partner.

Setting Requested Delivery Date (Clinic account)


• When ordering from Case Box, you can enter the requested delivery date.

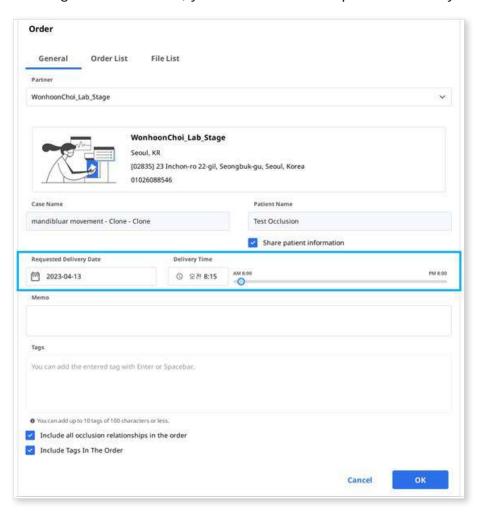

- The default value is 7 days after the date of the order.
- Enter the date/time you wish to receive the products and proceed with the order.
- You can set the time easily using the slider.

Setting Expected Delivery Date (Lab account)


• You can enter the expected delivery date for the case ordered from In Box.

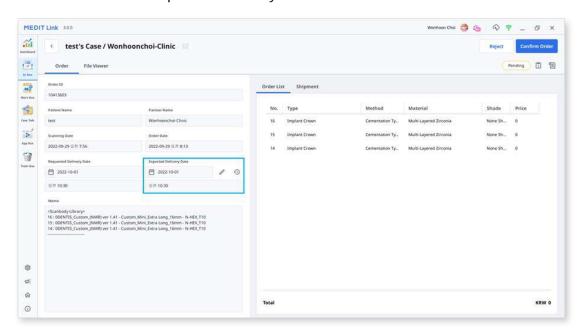

① Check the requested delivery date with the partner and reschedule using the "Edit" button.



- 2 Enter the expected delivery date considering the actual working day. If required, you can leave a comment also.
- 3 Click "Apply."

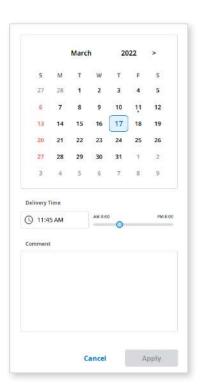

- 4 The expected delivery date and the memo will be forwarded together with the order.
 - You can check the date and memos in the Order Box.
 - You can change the expected delivery date, and the changes will be forwarded to your partner.
- 5 You can check the case history using the "History" button.



Completing Case

Complete (Work Box/In Box)


Once all tasks are done, click "Done" to complete the case.

Complete a case from the case detail

You can find the three dots icon on the right from the "Open" or "Rescan" button.


0 0 0

Complete the work by clicking the icon and selecting "Done."

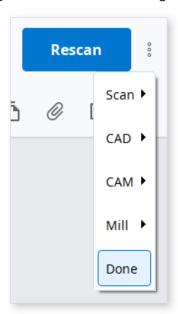

Complete the multiple cases from the case list

- Go to Work Box.
- ② Select all cases to complete.

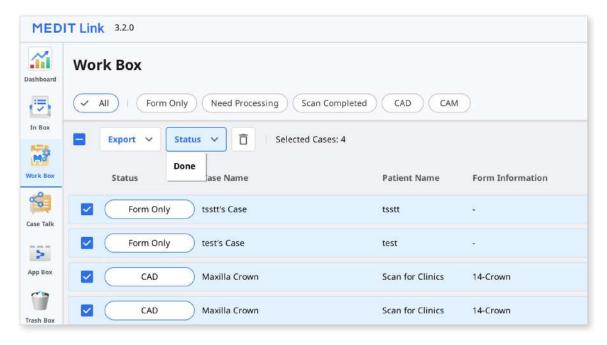


3 Click "Status" and select "Done."

Complete (Case Box/Order Box)

After the case is completed on the lab side, the clinic needs to confirm the case to complete the whole order process.

The clinic can find the "Confirm Delivery" button in the Order Box if the lab completes its work.


Click this, the whole order process completes, and the case status will be changed to "Case Completed."

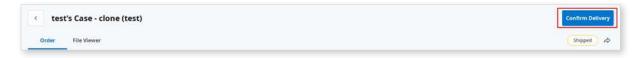

Complete a non-ordered case in the Case Box

For cases that are not ordered to the Lab, you can complete these cases from the Case Box.

- ① Go to the Case Box.
- 2 Select the case that you want to complete.

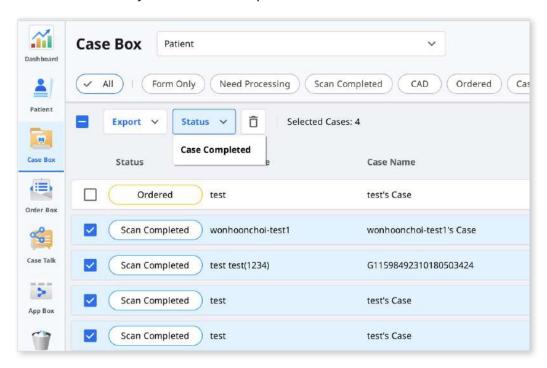


3 Click "Status" and select "Case Completed."

Communicate Using Tags


You can add additional information for your case using tags and send them to your partners.

Add Tags When Creating a New Case

You can enter tags when you create a new case or edit case information.

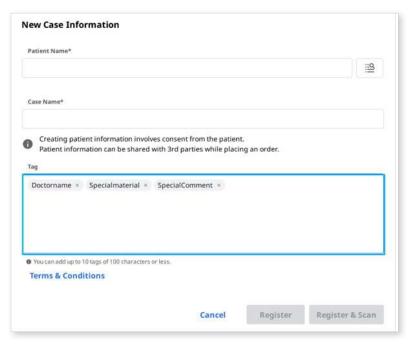


You can easily search and select tags you've used before.

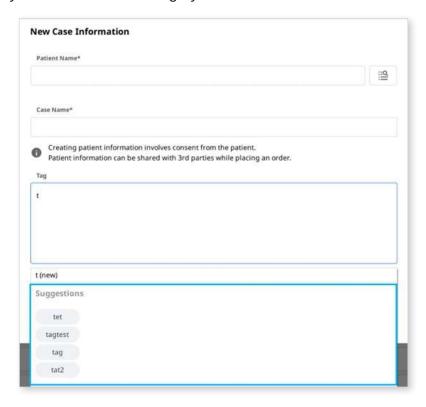

Add Tags When Ordering

You can check and edit tags in the Order dialog while ordering.

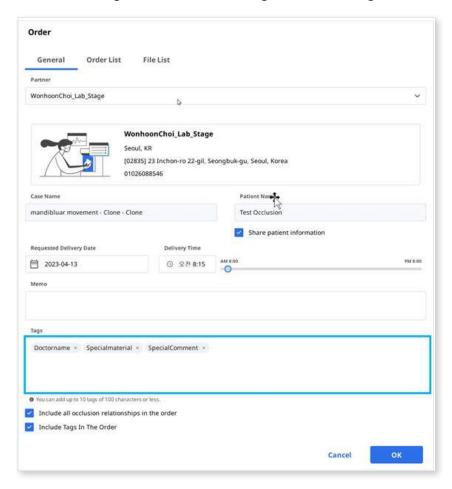

Check Tags on Case Details

