


MEDIT Link

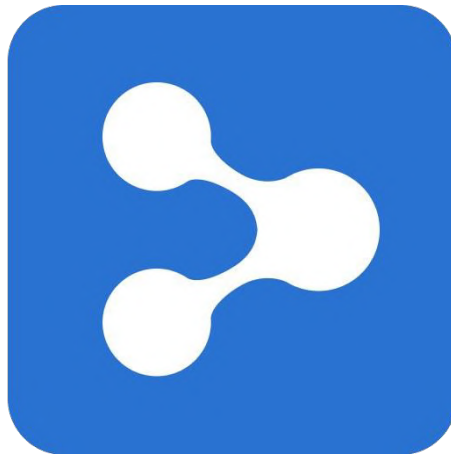

ME-UG-702H
Revision 11 (12.08.2025)
SW version 3.4.5

Table of Contents

Medit Link

Introduction	7
Medit Link	7
Service Overview	7
Cloud Service	8
Workflow Management Platform with Ordering System	9

Medit Link > Installation

System Requirements	10
System Requirements for Windows	10
System Requirements for macOS	10
System Requirements for Web Service	10
Recommended for Browsers	10
Minimum for Mobile Devices	10
Installation on Windows	11
Installation on macOS	14

Medit Link > Getting Started

Sign Up & Sign In	18
Medit Link Website	18
Sign Up	18
Login and Welcome Page	21

Medit Link > Medit Link Web

Dashboard	24
Dashboard Overview	24
Medit Insights	24
Subscribe Medit Insights	25
Unsubscribe Medit Insights	26
Patient	27
Search Patient	27
Case Box/Work Box	28
Case Filtering	28
Filter by patient	28
Filter by status	29
Filter by duration	30
Filter by Search	30
Case Details	30
Order Box/In Box	32
Order Details	32
Case Talk	33
Creating a Case Talk	33
Setting the sharing permission type and sharing options	34
Case Talk details	36
Web Viewer	37
Annotation	38
Comments	39
Real-time screen sharing	40

Data Tree	40
Data Tree Controls	40
3D Model Controls	41
Partners	43
Search for Partners	43
My Partner List	43
Partner Invitation	44
Membership	45
Membership	45
Medit Cloud Storage Plan	45
Cloud Storage	47
Subscribe	47
Cancel Subscription	49
My Subscription	50
Details	50
Resubscribe	51
How to add a payment method	51
How to change a payment method	52
Redeem	53
Details	54
Register Redeem Code	54
Activate Redeem Code	55
Payment Details	55
View Bill Details	56
My Info	58
My Account	58
My Clinic/My Lab (Admin Account Only)	58
My Scanner	59
Other Menu	61
Notice & Update	61
Related Websites	61
Download Medit Link App	62
 Medit Link > Medit Link App	
Overview	63
The table below summarises what menus are available in each type of clinic and lab account.	63
Title Bar	64
My Information	64
Notifications	64
Notice & Update	65
Settings	65
Network Status	66
Settings	68
General	68
Advanced	69
Export – Export Settings	70
Network – Proxy Server	73
Default App	73
User Experience Program	73
Storage Management – Overview & Cleanup	73
Set Period	73
Storage Management – Advanced Settings	74

Case Converting Tool	74
Export Case	74
Import Case	76
About	77
User Guide	77
Menu	79
Dashboard	79
Patient	79
Add New Patient	80
Delete Patient	80
Merge Patients	81
Case Box/Work Box	81
Tabs in Case Box/Work Box	82
Functions in Case Box/Work Box	82
Order Box/In Box	84
Tabs in Order Box/In Box	84
Case Talk	84
App Box	85
All	85
Medit Apps	86
Downloaded	86
Trash Box	86
Case Synchronization Manager	87
Upload	87
Download	87
Related Websites	88
File Viewer	89
Data Tree	89
Model Control Tools	89
3D Model Controls	90
2D Image Control	91
Function Tools	91
Image Capture and Memo	91
Ortho Capture	92
Fill Holes	93
Trimming	94
Optimize Occlusion Alignment	96
Mandibular Movement	98
Manage Multi Occlusion Data in File Viewer	99
How to manage multiple results	99
View Options	101
Status Filters	101
View Type	102
Grouping	102
Sorting	102
Filtering and Search	104
Filter by Patient	104
Filter by Date	105
Advanced Search Capabilities	106
 Medit Link > Workflow	
Create/Edit/Clone/Delete Case	107

Creating Cases	107
Create Case in Case Box (Clinic)	107
Create Case in Work Box (Lab)	109
Editing Cases	109
Edit Case in Case Box (Clinic)	109
Edit Case in Work Box (Lab)	111
Cloning Cases	112
Clone Case in Case Box (Clinic)	112
Clone Case in Work Box (Lab)	113
Deleting Case	113
Delete Case (Clinic and Lab)	113
Form Information	114
Registering Form Information in Case Box	114
Enter on the Form Tab	114
Registering Form Information in Work Box	114
Enter on the Form Tab	114
Connecting Teeth into Bridge	114
Setting Scanning Options	115
Medit Scan for Labs	115
General options	115
Options by teeth	115
Adding to Favorite	116
Add to Favorites	116
Manage Favorite Templates	116
Work on Scanning/CAD	117
Scan Process	117
CAD Process	117
Utilize Add-in Apps	117
Add-in Quick Launcher	117
App Launcher Management	117
Quick Launcher	118
All Apps	118
Get Result Files	119
Export Scan Result Files	119
Export Result Files from Case List	120
Export Result Files from File Viewer	122
Export Result Files from Data Tree	124
Export Case List as CSV	127
Export Margin Line	128
Export Margin Line as a Color	128
Export Margin Line as a File	129
Download Files from Medit Link Web	129
Case Box/Work Box	130
Web Viewer	130
Order/Manage Orders	133
Ordering Cases	133
Confirm Order (In Box)	134
Cancel Order (Order Box)	134
Reject Order (In Box)	134
Checking Ordered Cases	134
Entering Shipment Information (In Box)	135
Managing Delivery Date	135

Setting Requested Delivery Date (Clinic account)	135
Setting Expected Delivery Date (Lab account)	136
Completing Case	137
Complete (Work Box/In Box)	137
Complete a case from the case detail	137
Complete the multiple cases from the case list	138
Complete (Case Box/Order Box)	138
Complete a non-ordered case in the Case Box	138
Communicate Using Tags	140
Add Tags When Creating a New Case	140
Add Tags When Ordering	141
Check Tags on Case Details	141

Introduction

Medit Link

Medit Link is a collaborative tool designed to facilitate communication between clinics and labs to provide an optimized workflow for each party.

It offers differentiated features based on the type and permissions of each account. The program provides a user-friendly user interface for managing the workflow, from scanning with oral and model scanners, ordering cases, and designing and producing prostheses.

Medit Link consists of a web service and a computer application.

Service Overview

		Web	App
		• Register and manage the account • Search and manage partners • Share cases via Web Viewer with a sharable link • Access cases made from the App • Manage orders	• Enter form information • Scan • Manage orders • Link with CAD • Manage patients
Clinic	Admin	Search and manage partners and your clinics, and share cases. • Dashboard • Patient • Case Box • Order Box • Case Talk • Partners • Membership • My Info	Enter form information, scan, place orders, and manage patients. • Dashboard • Patient • Case Box • Order Box • Case Talk • App Box • Trash Box
	Member	• Patient • Case Box • Order Box • Case Talk • My Info	• Patient • Case Box • Case Talk • Order Box • App Box • Trash Box
Lab	Admin	Search and manage partners and your lab, and share cases. • Dashboard • In Box • Work Box • Case Talk • Partners • Membership • My Info	Enter form information, scan, and manage orders. • Dashboard • In Box • Work Box • Case Talk • App Box • Trash Box
	Member	• In Box • Work Box • Case Talk • My Info	• In Box • Work Box • Case Talk • App Box • Trash Box

Cloud Service

Medit Link supports the automatic upload/download of databases and files through AWS (Amazon Web Services).

The data stored in the cloud is automatically synchronized to the computers of multiple users in the organization so that they can access the same data from any computer at any time. Cloud-based backup and recovery keep all your data safe. It keeps your data safe even in case of data loss due to sudden computer failure.

The cloud service allows partnered dentists and labs to quickly share data without the hassle of emailing or manually transferring the scans. All data, including patient information, is securely encrypted, stored, and transmitted. Medit complies with medical laws and regulations like HIPAA and GDPR.

Workflow Management Platform with Ordering System

Medit Link offers its ordering system.

- Medit Link allows users to establish partnerships between dentists and dental laboratories and share data quickly and easily.
- You can check and manage the ordering status.

System Requirements

Note

Please refer to the system requirements for Medit Scan for Clinics and Medit Scan for Labs if you want to use the scanning software with your scanner.

System Requirements for Windows

	Laptop & Desktop
CPU	Intel Core i5 2.6 GHz or higher
RAM	16 GB or higher
Graphics	NVIDIA GeForce GTX 760 (2GB) or higher / or equivalent AMD video card
OS	Windows 10 64 Bit (unavailable in 32 Bit) or higher

System Requirements for macOS

	Mac
Processor	M1 with 8-core, 7-core GPU, 16-core Neural Engine
RAM	16 GB or higher
OS	Monterey 12 or higher

System Requirements for Web Service

Recommended for Browsers

	Browsers
PC	Chrome, Firefox, Safari, Edge (Chromium)
Android	Chrome, Firefox
macOS	Safari, Chrome, Firefox

Minimum for Mobile Devices

	Android	iOS
Device	Galaxy S7 equivalent or higher	iPhone 7 equivalent or higher
OS	Android 7.1 Nougat	iOS 11.0
RAM	3 GB or higher	2 GB or higher
AP	Above Snapdragon 710	A10 Fusion

Installation on Windows

Medit Link is installed as a package with scan acquisition programs - Medit Scan for Clinics and Medit Scan for Labs.


1. Run Medit_LinkSetup.exe.
2. Select a language for installation.
3. Select the folder to install Medit Link. Please read and agree to the license terms and conditions and click "INSTALL."



4. Make sure that the scanner cable is disconnected from the computer before proceeding to the next step. The installation may not proceed if the scanner is connected to your PC.
5. The program will be installed automatically and may take a few minutes. Do not turn off or unplug your PC until the installation is complete.



6. Please restart your PC after the installation is complete. You can proceed without restarting the PC if you do not have the scanner.

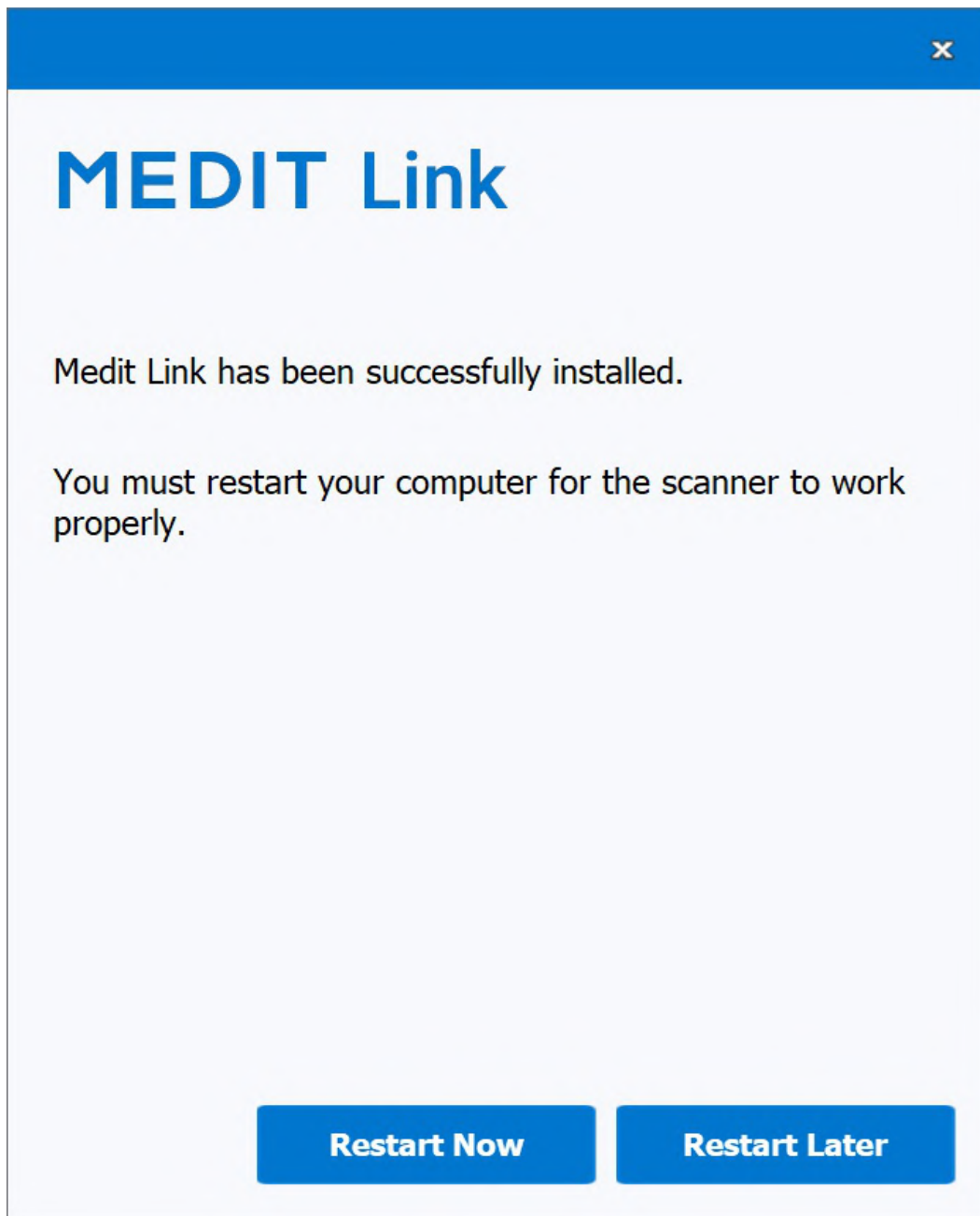


 Note

If you do not restart your PC after the installation is complete, the scanner may not function properly.


Installation on macOS

Medit Link is installed as a package with Medit Scan for Clinics.


1. Run Medit_Link_x.x.x.pkg and click "Continue."

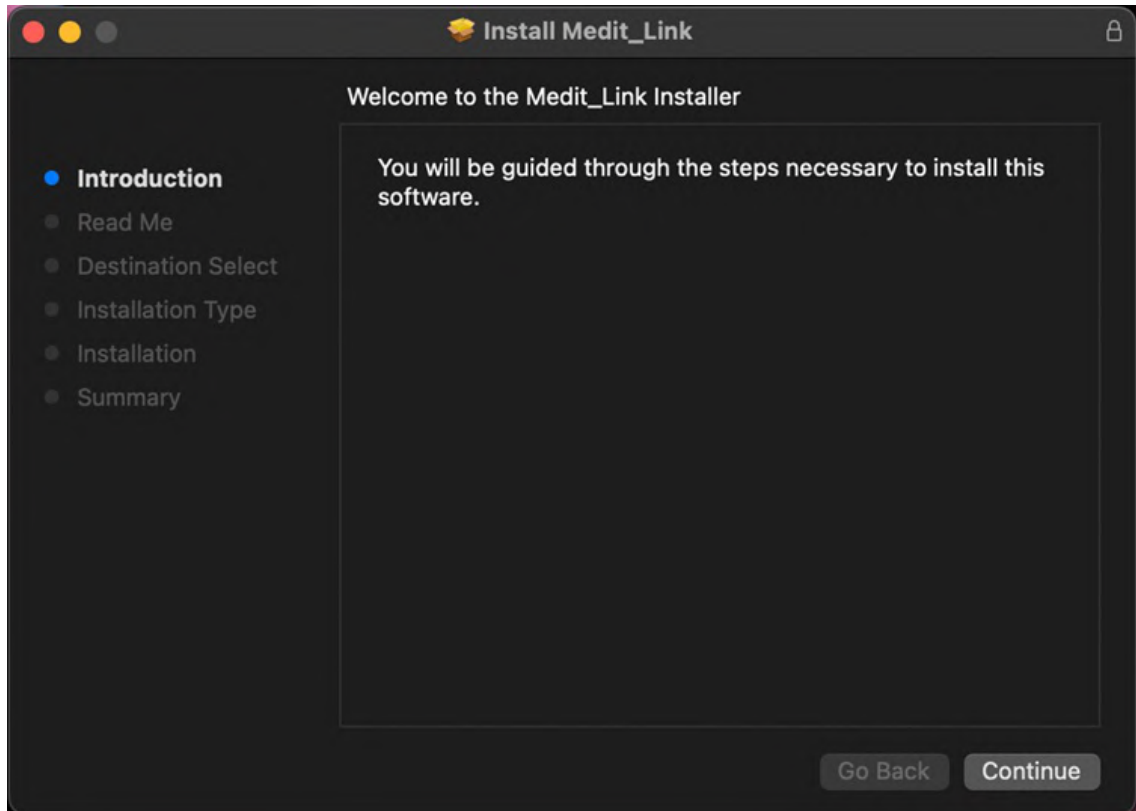


2. Read the End User License Agreement and click "Continue."

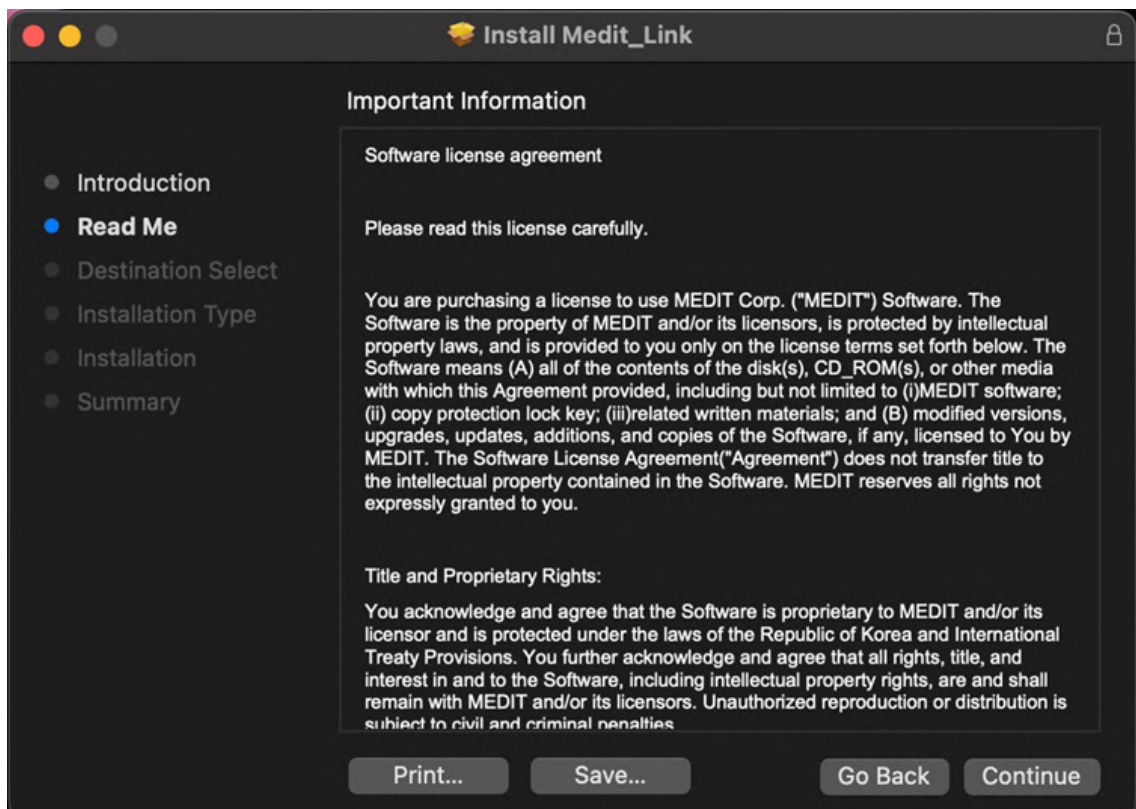


3. Select the disk to install and click "Continue."

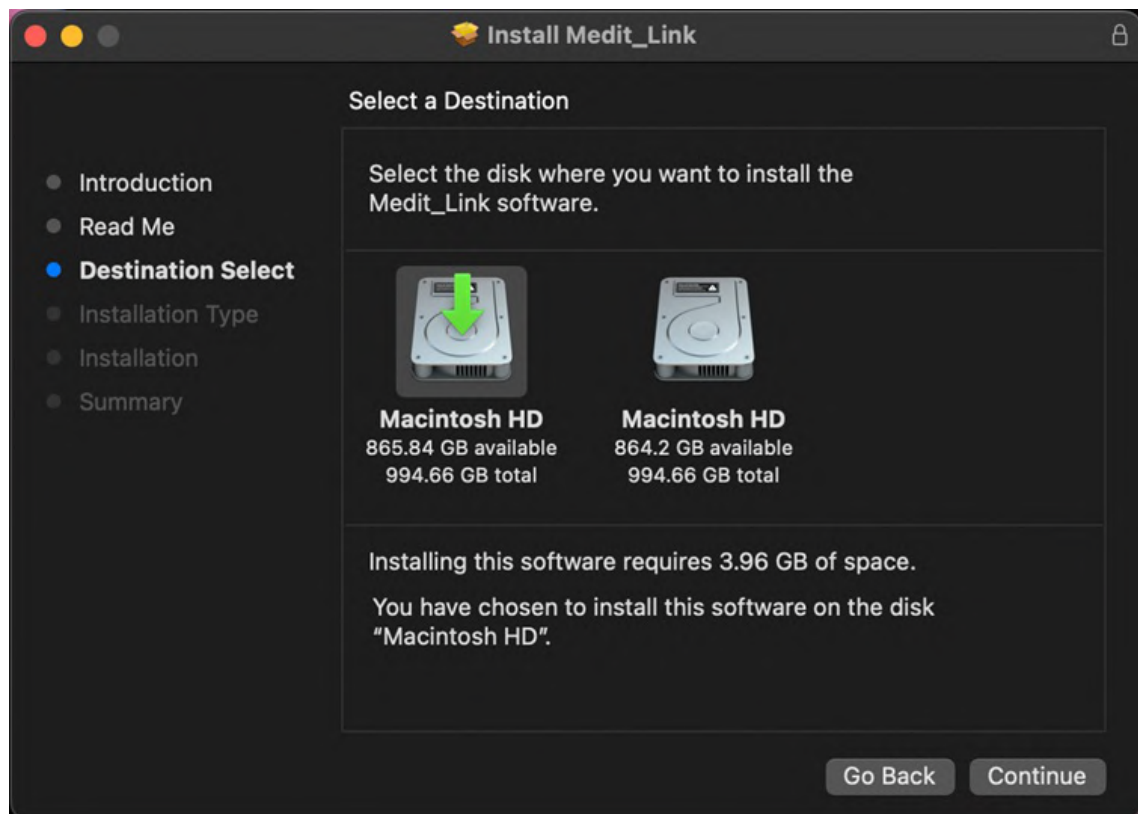


4. Click the "Change Install Location..." to change the installation destination.

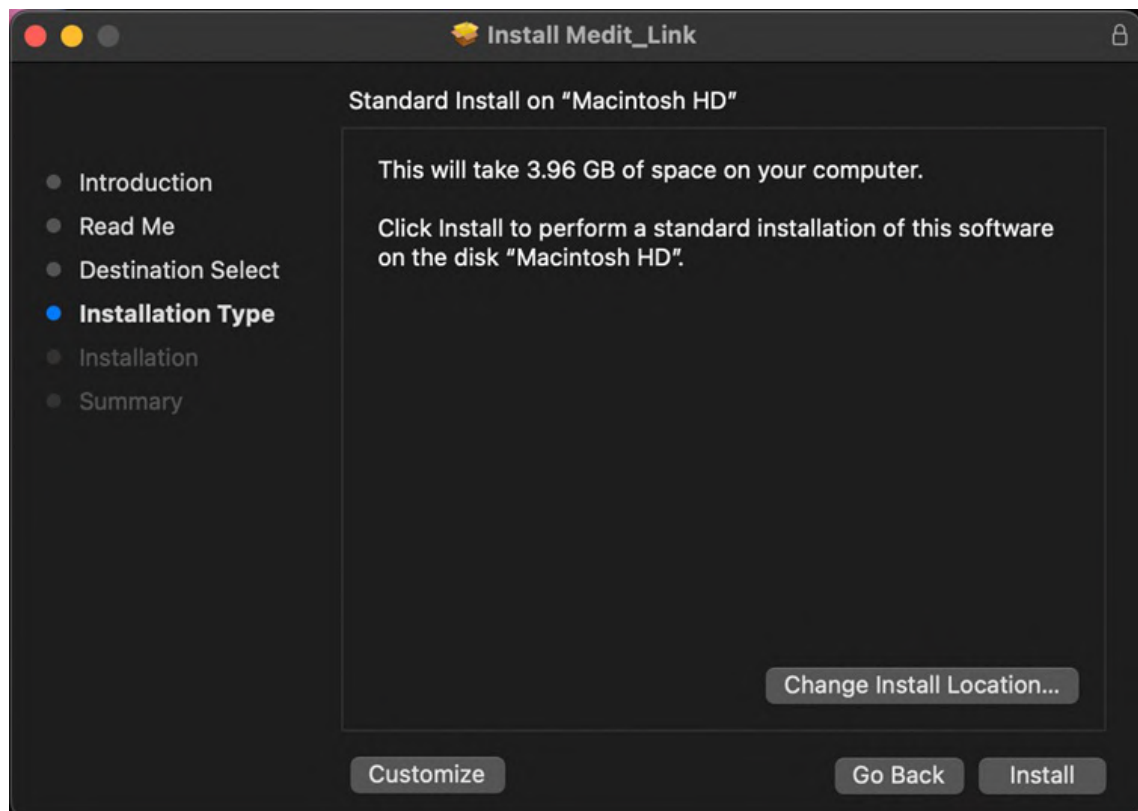


5. Click "Install" and type your Mac password to allow installation if requested.

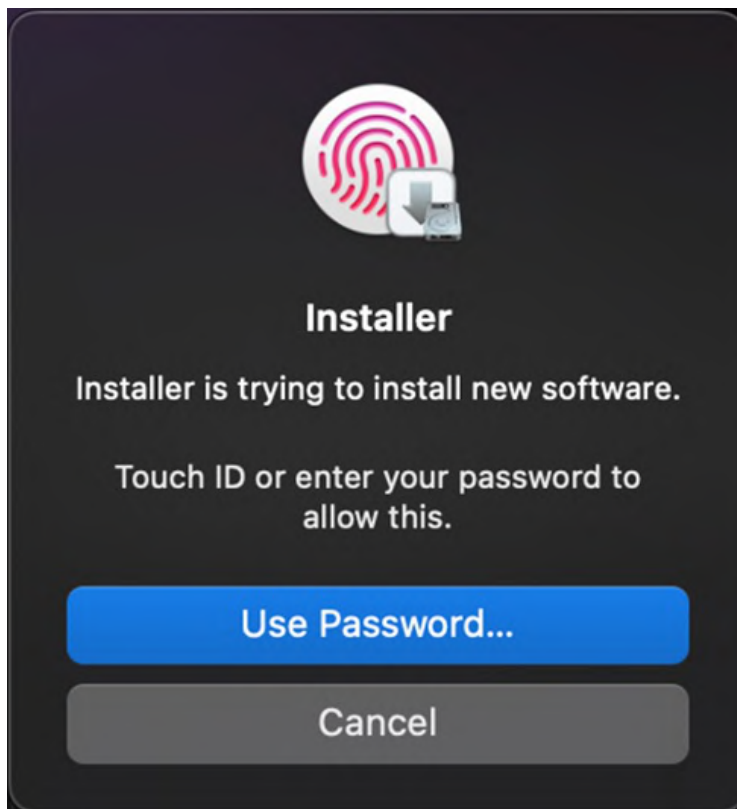

6. Wait until the installation is completed.

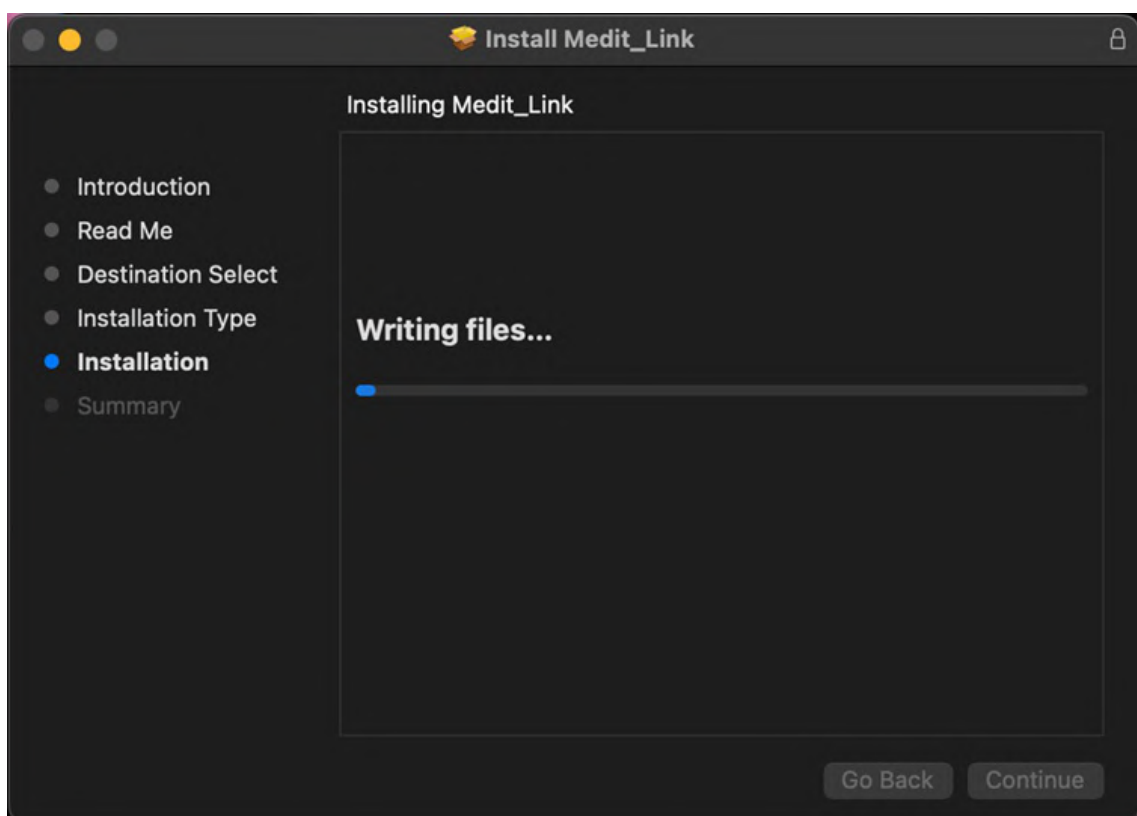


7. Click "Close" to exit the installation window.

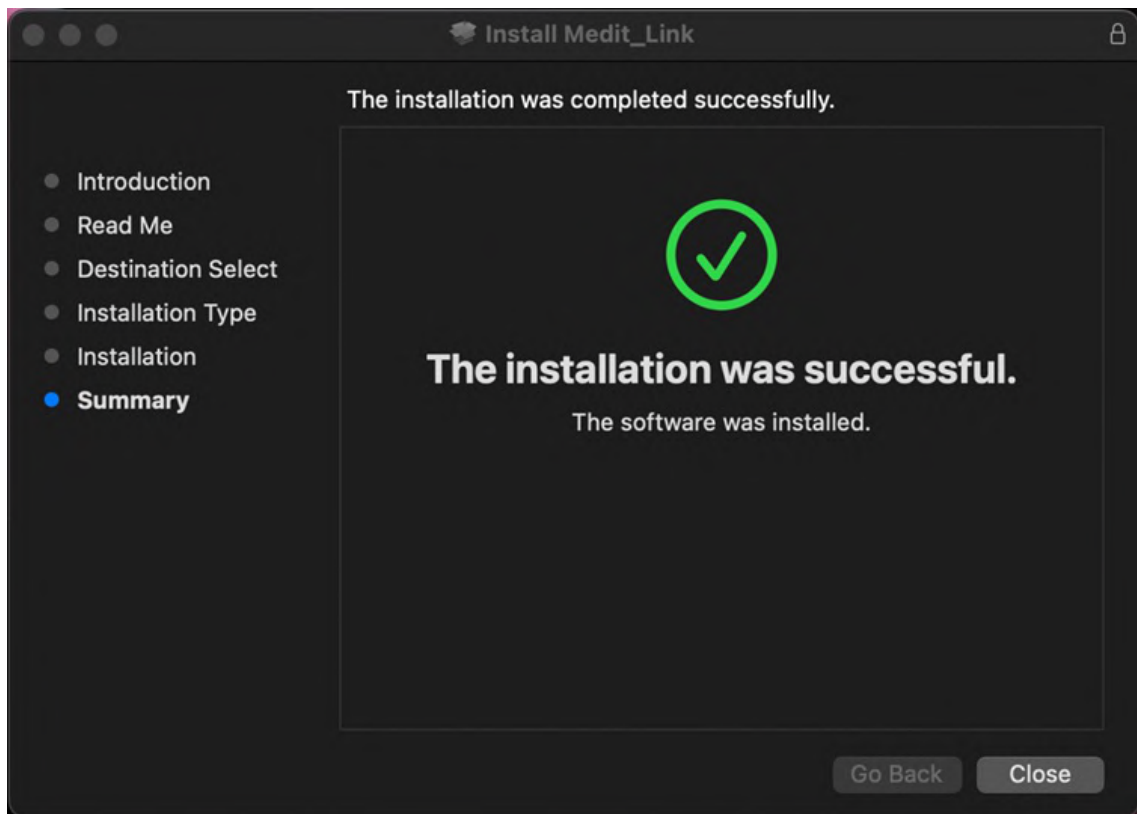

Sign Up & Sign In

Medit Link Website

Sign up for Medit Link service at www.meditlink.com.

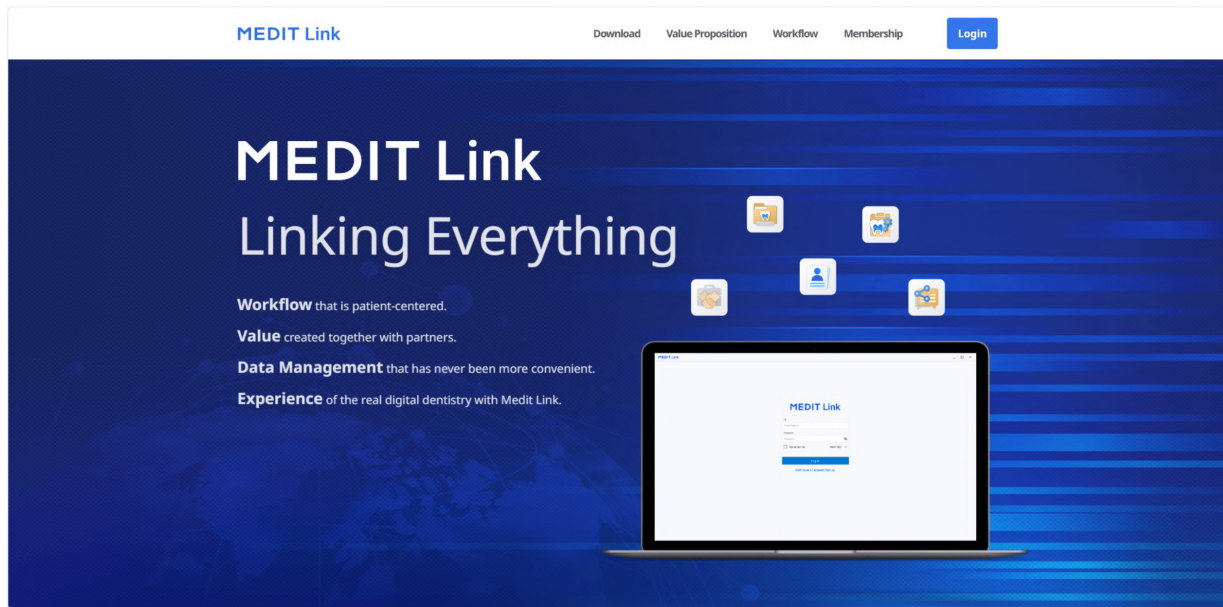

Note

Register for Medit Link China at www.meditlink.cn.

Sign Up

To use the Medit Link service, you must sign up for an account. You can utilize features provided in Medit Link after creating an account as a clinic or lab.

Users can select one of the following four account types to register for membership and use the service with full or limited features and privileges, depending on the account type.

- Clinic - Admin
- Clinic - Member
- Lab - Admin
- Lab - Member

The administrator is usually a representative of your organization and will have full access to all functions and services of Medit Link. In contrast, the member account has limited access.

To sign up for membership, follow these steps:

1. Click "Don't have an account yet? Sign up." on the login page.

MEDIT Link

ID

Password

☐

Keep me logged in.

Need help? ▾

Log In

Don't have an account? [Sign up.](#)



English (United States)



2. After reading the provided descriptions, click the "Register" button for the desired account type.

Create Your Medit Link Account



Clinic


Administrator

Dentist, Manager

Have the authority to create a new dental clinic account and have full access to Medit Link.

- ✓ Scan/Order/Manage orders
- ✓ Manage members

Register

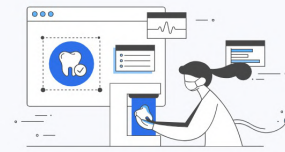

Lab

Administrator

Laboratory Director, Manager

Have the authority to create a new dental lab account and have full access to Medit Link.

- ✓ Scan/Manage ordered cases
- ✓ Manage members

Register

Member

Dental Hygienist, Dental Staff

Belong to the dental clinic their admin created and have limited access to Medit Link after the admin's approval.

- ✓ Scan/Order/Manage orders

Register

Member

Dental Technician, Laboratory Staff

Belong to the dental lab their admin created and have limited access to Medit Link after the admin's approval.

- ✓ Scan/Manage ordered cases

Register

3. The following registration form appears.

Create Your Medit Link Account

Clinic Name

Check Availability

Name

Email Address

Check

Please ensure your email address is active. Medit Link will send a verification email.
Registering with a disposable and/or fake e-mail address can make you vulnerable to security issues and make it difficult to receive information from Medit Link.

Password

Password must be 8 to 16 characters long, including a combination of at least three letters, numbers, and special characters. (Passwords are accepted in English language only.)

Confirm Password

Please type in the same password as above.

Select Country/Region

☐ I agree to all of the Terms and Conditions.
 ☐ Personal Information Collection Agreement (Mandatory)

View

☐ Terms & Conditions (Mandatory)

View

☐ Agree to the Marketing and Promotion Agreement (optional)

View

Check this option to receive marketing emails, which include product updates, software news, and promotional events.

Back

Register

- Enter the clinic or lab name and then, depending on your desired account type, do the following:
 - If you are registering as an administrator, click "Check Availability" to see if the entered name is available.
 - If you are registering as a member, click "Search." The administrator must already register your organization before you sign up.
- Fill out the rest of the registration form and click "Register."

Note

If you agree to receive marketing information during registration, you will receive helpful information about the product and service from Medit.

- A verification email will be sent to you.
- To complete the registration process, verify your email address using the received link.

Login and Welcome Page

- Enter your email address and password on the login page and click "Log In."

MEDIT Link

ID

Password



☐ Keep me logged in.

Need help? ▾

Log In

Don't have an account? [Sign up.](#)




English (United States)



2. The following pop-up will appear if you haven't filled out your business information.

Welcome, stage-test-wh!

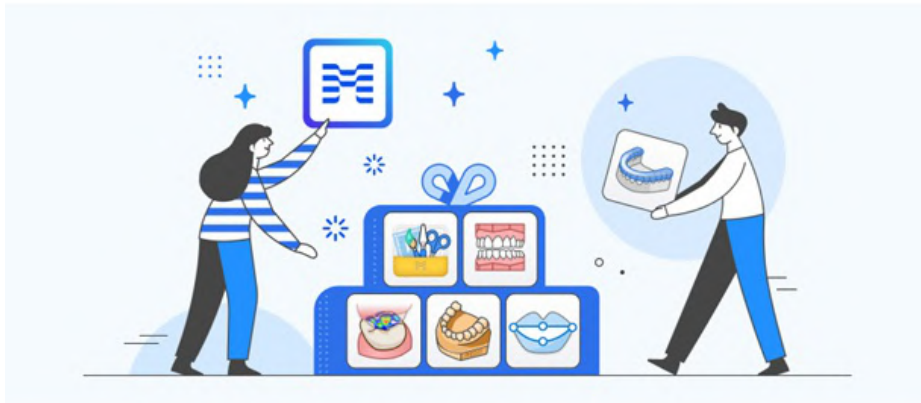

1 2 3 4

Utilize Various Apps

Download Medit Link to your desktop to utilize scanning programs and various apps. All data generated is synced to the cloud and is also available on the web.

[Previous](#)

[Go to My Clinic](#)

Note


To use Medit Link services, your clinic or lab information should be entered using the administrator account.

3. Click the "Go to My Clinic" or "Go to My Lab" button and enter the clinic or lab information, including business name, cover/logo images, services you offer, office address, office number, etc.
4. The information entered on the welcome page can be edited anytime on My Info > My Clinic or My Lab tab in Medit Link Web.

Dashboard

Dashboard Overview

You can view the statistics for all the work done on Medit Link in visualized graphs and charts - both in the Medit Link App and Web.

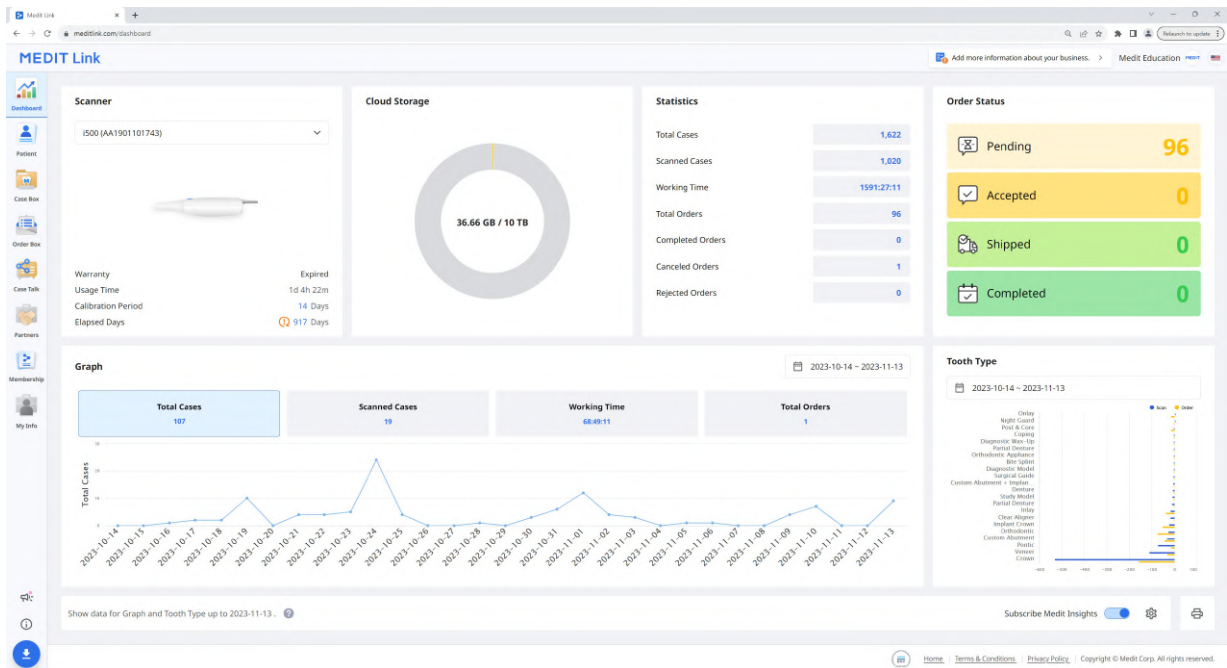

Graphs and charts in the dashboard show statistics for all the work done in Medit Link.

Scanner	Provide information about the active scanners, including warranty, scanner usage time, calibration interval, and calibration elapsed time.
Cloud Storage	Display the total capacity of the cloud and the amount in use.
Statistics	Provide statistics on cases based on job status.
Order Status	Provide statistics on cases based on order status.
Graph	Graph job information by period.
Tooth Type	Provide statistics on the number of scans and orders by treatment information.


Medit Insights

Medit Insights is a reporting service that helps you analyze your usage statistics.

Note

The following service is not available on Medit Link China.

- Medit Insights

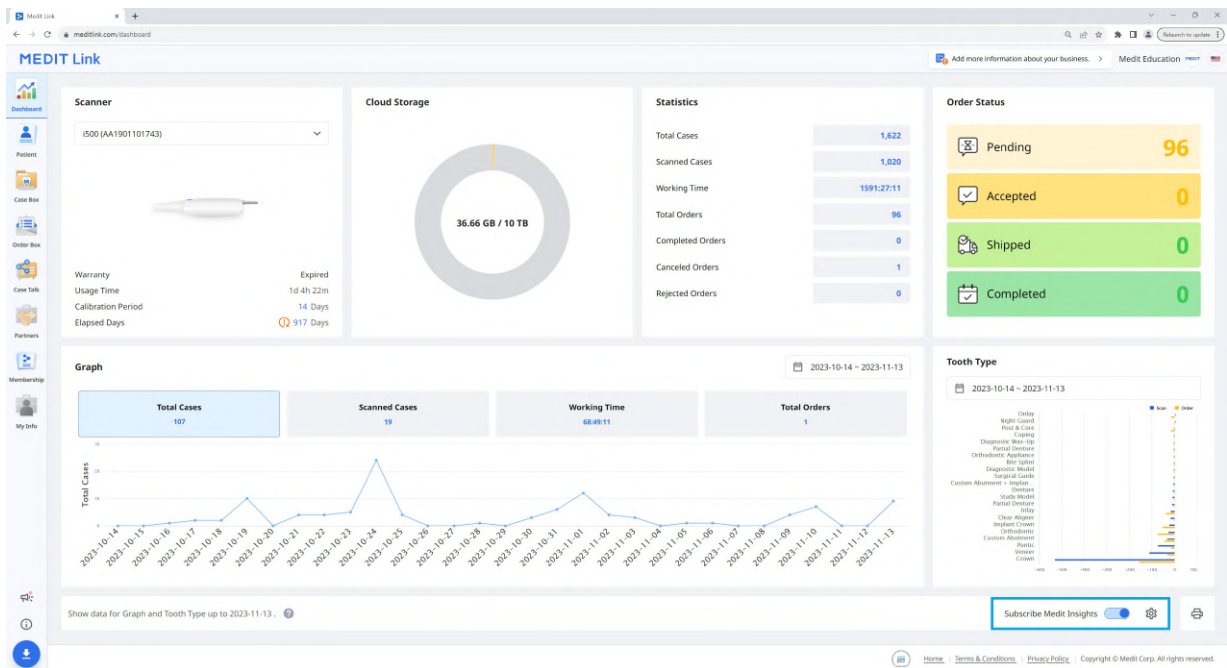

Subscribe Medit Insights

You can receive a report email on the second day of every month. No additional sign-up process is required. Just enable "Subscribe Medit Insights."

If you don't use the email address you used for the Medit Link account, you can set another email address to receive Medit Insights.

Settings

Receive the report via

☒ Medit Link account


☐ Another email address

Enter another email address to receive the report email.

The report will be sent on the 2nd day of every month.

Cancel
OK

On the second of every month, you will receive an email with statistics, as shown below.

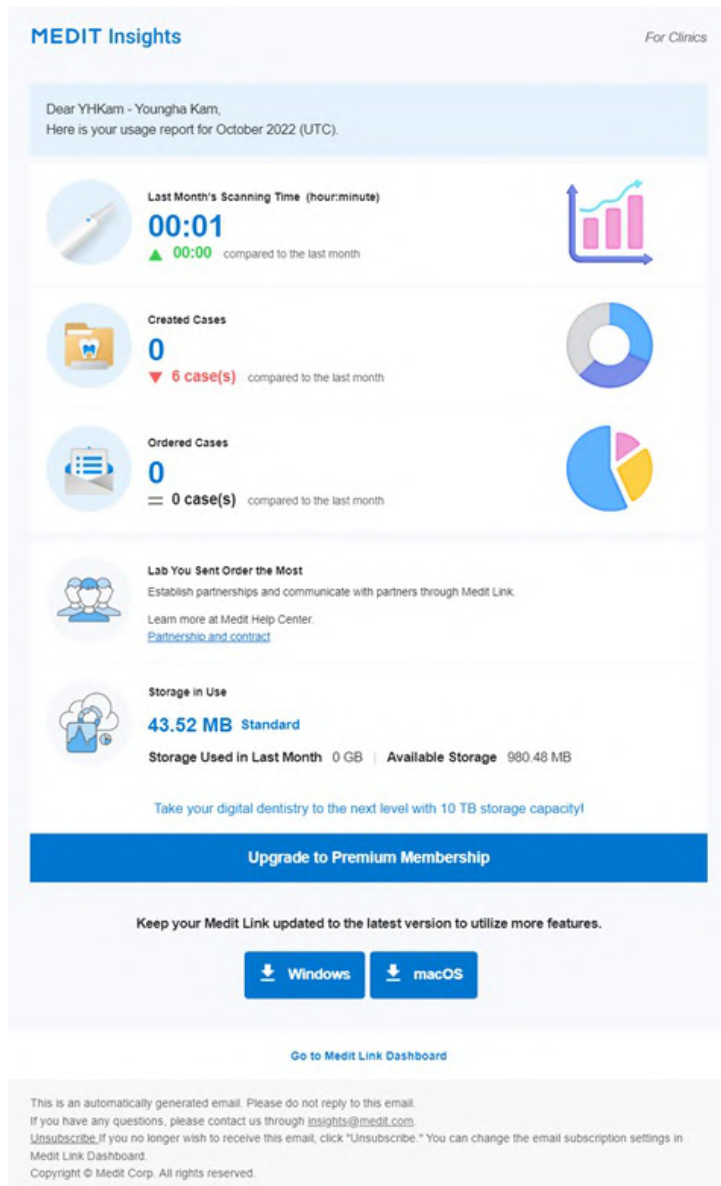

Banner	Medit provides useful tips and information here. * If you don't want to see the banner, uncheck the Marketing and Promotion Agreement check box in My Info > My Account.
Statistical Information	The statistics information is displayed, such as scanning time, created cases, most ordered lab, etc.
Medit Link Download	The button provides the download link of the Medit Link.
Dashboard	All statistical information is based on your Dashboard information. You can see more details in the Dashboard.


Unsubscribe Medit Insights

Disable "Subscribe Medit Insights" in Dashboard or click "Unsubscribe" in the received email.

Patient

You can manage cases based on patient name and patient ID.


All patients registered through the Medit Link application are shown in a list, making it easier to manage patients and their cases.

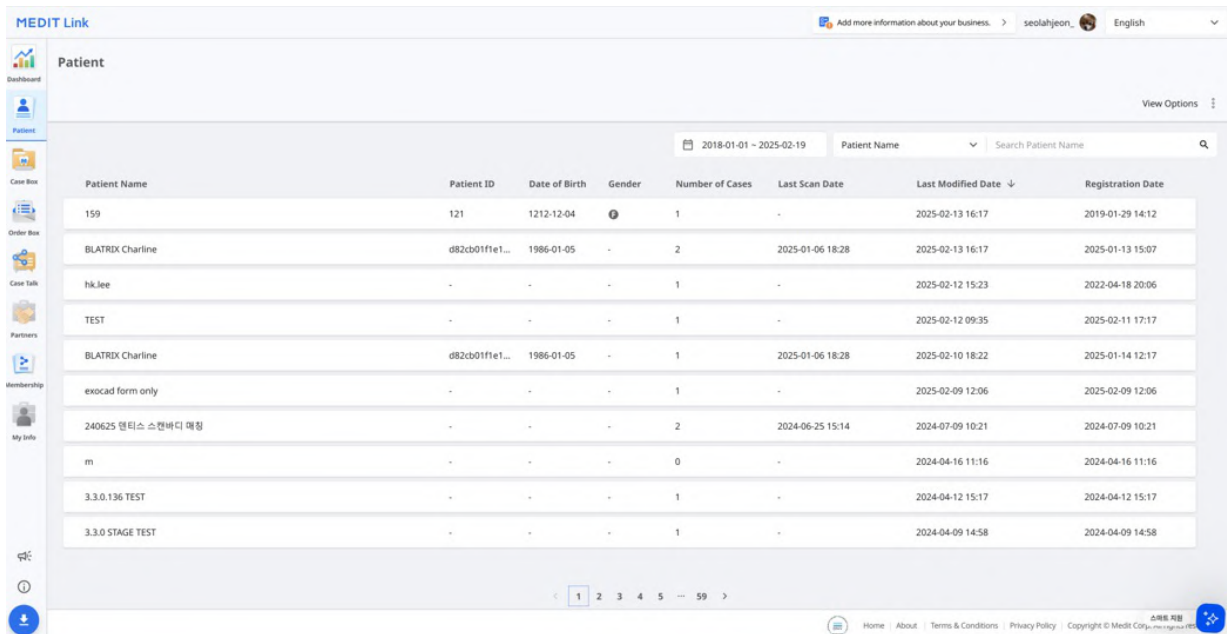

Patient Name	Patient ID	Date of Birth	Gender	Number of Cases	Last Scan Date	Last Modified Date	Registration Date
159	121	1212-12-04		1	-	2025-02-13 16:17	2019-01-29 14:12
BLATRIX Charline	d82cb01f1e1...	1986-01-05	-	2	2025-01-06 18:28	2025-02-13 16:17	2025-01-13 15:07
hik.lee	-	-	-	1	-	2025-02-12 15:23	2022-04-18 20:06
TEST	-	-	-	1	-	2025-02-11 09:35	2025-02-11 17:17
BLATRIX Charline	d82cb01f1e1...	1986-01-05	-	1	2025-01-06 18:28	2025-02-10 18:22	2025-01-14 12:17
exocad form only	-	-	-	1	-	2025-02-09 12:06	2025-02-09 12:06
240625 덴티스 스캔배디 매칭	-	-	-	2	2024-06-25 15:14	2024-07-09 10:21	2024-07-09 10:21
m	-	-	-	0	-	2024-04-16 11:16	2024-04-16 11:16
3.3.0.136 TEST	-	-	-	1	-	2024-04-12 15:17	2024-04-12 15:17
3.3.0 STAGE TEST	-	-	-	1	-	2024-04-09 14:58	2024-04-09 14:58

Search Patient

1. Select your search criteria between Patient Name or Patient ID from the drop-down menu at the top right of the patient list.

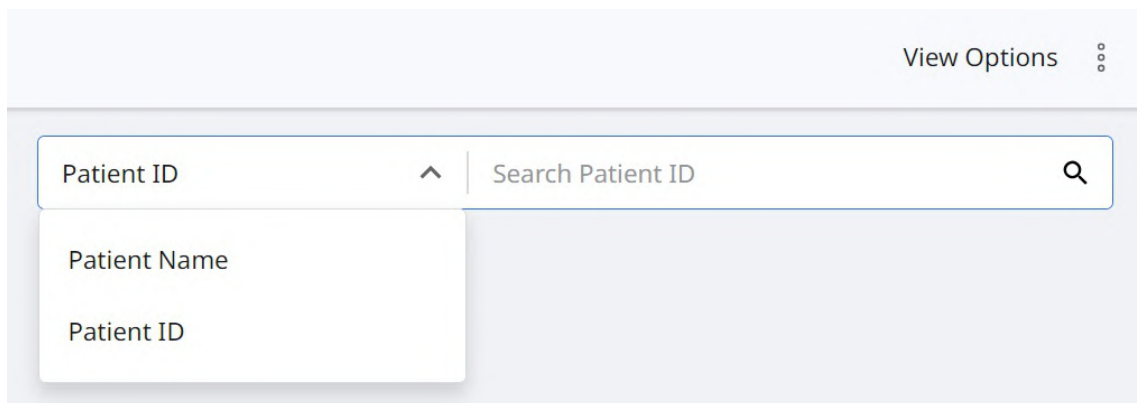

View Options

Patient ID ^ Search Patient ID


Patient Name

Patient ID

2. Enter the patient name or patient ID to search according to the selected search criteria.
3. Only patients with the entered patient name or ID are displayed in the list.

Case Box/Work Box

The Case Box (Clinic) or Work Box (Lab) shows all cases the user has created on the Medit Link application. You can see files via a web browser without installing a separate program.

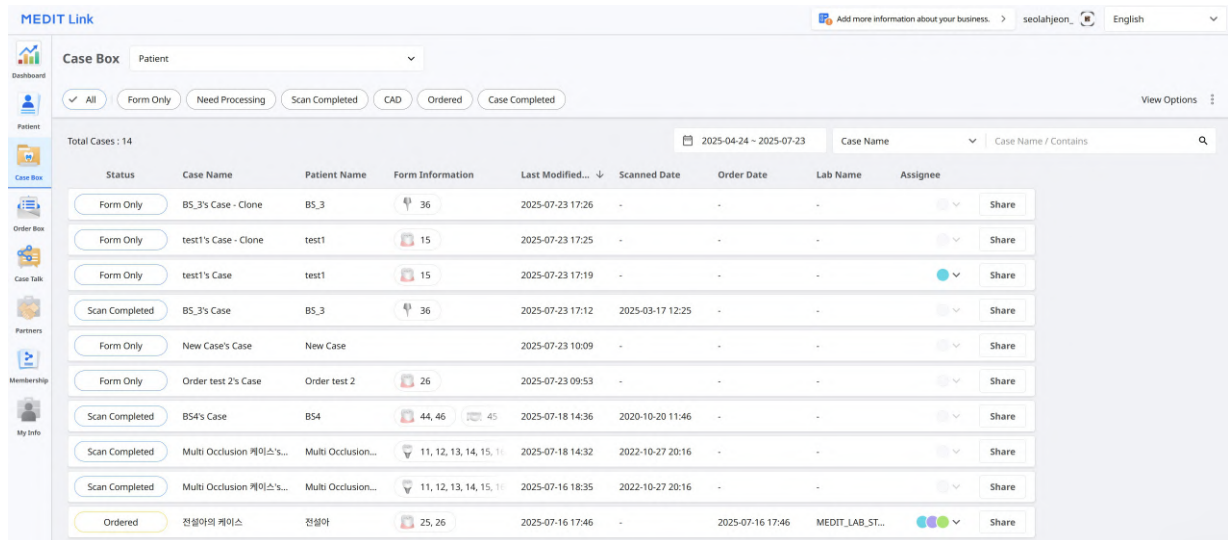


The screenshot shows the Medit Link Case Box interface. At the top, there's a header with the Medit Link logo, a user profile, and language settings. Below the header, there's a sidebar with navigation icons for Dashboard, Patient, Case Box, Order Box, Case Talk, Partners, Membership, and My Info. The main area is titled 'Case Box' and has a dropdown menu set to 'Patient'. Below this, there are filter buttons: All, Form Only, Need Processing, Scan Completed, CAD, Ordered, and Case Completed. A date range filter shows '2025-04-24 ~ 2025-07-23'. A search bar is labeled 'Case Name / Contains'. The table below lists 14 cases with columns for Status, Case Name, Patient Name, Form Information, Last Modified, Scanned Date, Order Date, Lab Name, Assignee, and a Share button.

Status	Case Name	Patient Name	Form Information	Last Modified...	Scanned Date	Order Date	Lab Name	Assignee	Share
Form Only	BS_3's Case - Clone	BS_3	36	2025-07-23 17:26	-	-	-		Share
Form Only	test1's Case - Clone	test1	15	2025-07-23 17:25	-	-	-		Share
Form Only	test1's Case	test1	15	2025-07-23 17:19	-	-	-		Share
Scan Completed	BS_3's Case	BS_3	36	2025-07-23 17:12	2025-03-17 12:25	-	-		Share
Form Only	New Case's Case	New Case		2025-07-23 10:09	-	-	-		Share
Form Only	Order test 2's Case	Order test 2	26	2025-07-23 09:53	-	-	-		Share
Scan Completed	BS4's Case	BS4	44, 46, 45	2025-07-18 14:36	2020-10-20 11:46	-	-		Share
Scan Completed	Multi Occlusion 케이스's...	Multi Occlusion...	11, 12, 13, 14, 15, 16	2025-07-18 14:32	2022-10-27 20:16	-	-		Share
Scan Completed	Multi Occlusion 케이스's...	Multi Occlusion...	11, 12, 13, 14, 15, 16	2025-07-16 18:35	2022-10-27 20:16	-	-		Share
Ordered	전생아의 케이스	전생아	25, 26	2025-07-16 17:46	-	2025-07-16 17:46	MEDIT_LAB_ST...		Share


Case Filtering

Filter by patient

You can filter cases by patients. It is useful when you find cases for specific patients.

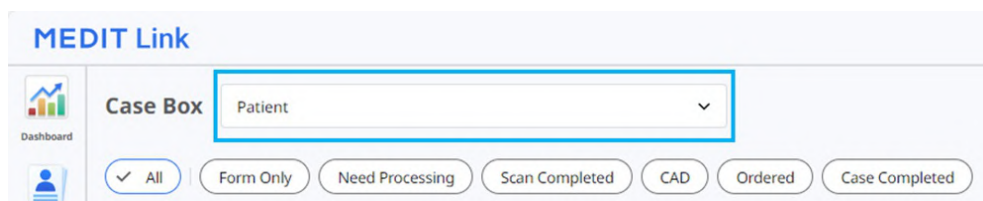

1. Enter a patient name in the search field.

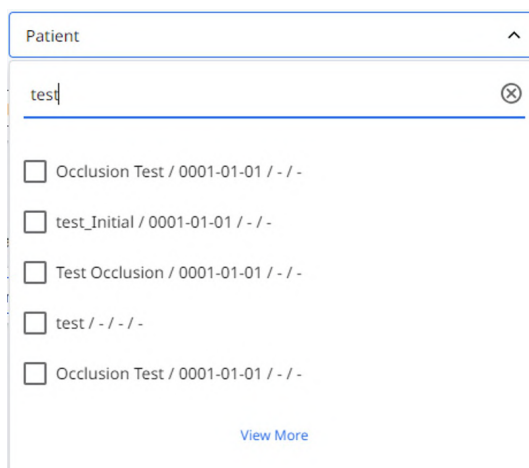

The screenshot shows a search dropdown menu for patients. The search field contains 'test'. Below the search field, there is a list of search results, each with a checkbox and a text label. The results are:

- ☐ Occlusion Test / 0001-01-01 / - / -
- ☐ test_Initial / 0001-01-01 / - / -
- ☐ Test Occlusion / 0001-01-01 / - / -
- ☐ test / - / - / -
- ☐ Occlusion Test / 0001-01-01 / - / -

At the bottom of the dropdown, there is a 'View More' link.

2. Select a patient from the list of search results.

test

☐ test / - / - / -

☐ Occlusion Test / 0001-01-01 / - / -

☐ Occlusion Test / 0001-01-01 / - / -

☐ test_Initial / 0001-01-01 / - / -

☐ Occlusion Test / 0001-01-01 / - / -

[View More](#)

Selected Patients

[Clear Selection](#)

☒ Occlusion Test / 0001-01-01 / - / -

☒ Test Occlusion / 0001-01-01 / - / -

☒ test / - / - / -

3. Only cases for the selected patients are shown in the list, making it easier to find the cases you want to work with.

MEDIT Link

Case Box test / - / - / - + 3

☒ All ☐ Form Only ☐ Need Processing ☐ Scan Completed ☐ CAD ☐ Ordered ☐ Case Completed

Total Cases : 2

Status	Case Name	Patient Name	Form Information	Last Modified...	Scanned Date	Order Date	Lab Name	Assignee	Share
Form Only	test's Case - Clone	test1	15	2025-07-23 17:25	-	-	-		Share
Form Only	test's Case	test1	15	2025-07-23 17:19	-	-	-		Share

Filter by status

You can filter cases based on their status.

All ☒ Form Only ☐ Need Processing ☒ Scan Completed ☐ CAD ☐ Ordered ☐ Case Completed

- The user can click the status they want to view. The browser will display only cases with the chosen status.
- The user can select multiple statuses at once.

The following status types are provided based on account type:

Case Box (Clinic)	All / Form Only / Need Processing / Scan Completed / CAD / Ordered / Case Completed
Work Box (Lab)	All / Form Only / Need Processing / Scan Completed / CAD / Done


Filter by duration

You can filter and display cases that were last modified within your set period.

1. Click the duration filter.



2. Select the range.

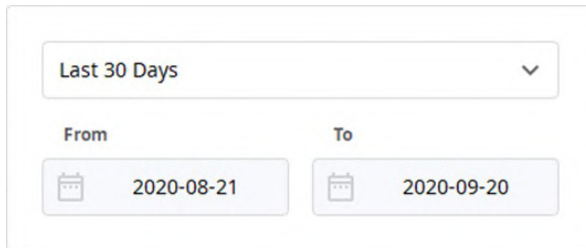


- All: Show all files created since January 1, 2018, regardless of the selected date range.
- Custom Range: Select a start date (no earlier than Jan 1, 2018) and an end date to filter files.
- Today: Files created or modified on the current day only.
- Yesterday: Files created or modified one day before today.
- Last 7 Days: Files created or modified in the past 7 days, including today.
- Last 30 Days: Files created or modified in the past 30 days, including today.
- Current Month: Files created or modified within the current calendar month.
- Previous Month: Files created or modified within the previous calendar month.

The case list will be updated in real time with the cases created or last modified within the selected duration.

Filter by Search


You can leverage search capabilities to search for the required information easily and quickly.

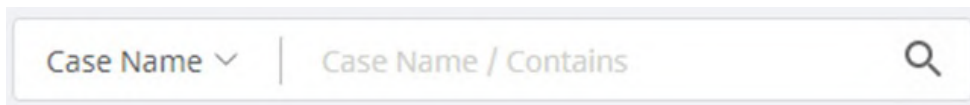

The following search options are provided based on account type:

Case Box (Clinic)	Case Name, Patient Name, Patient ID, Lab Name
Work Box (Lab)	Case Name, Patient Name, Clinic Name, Order ID

Case Details

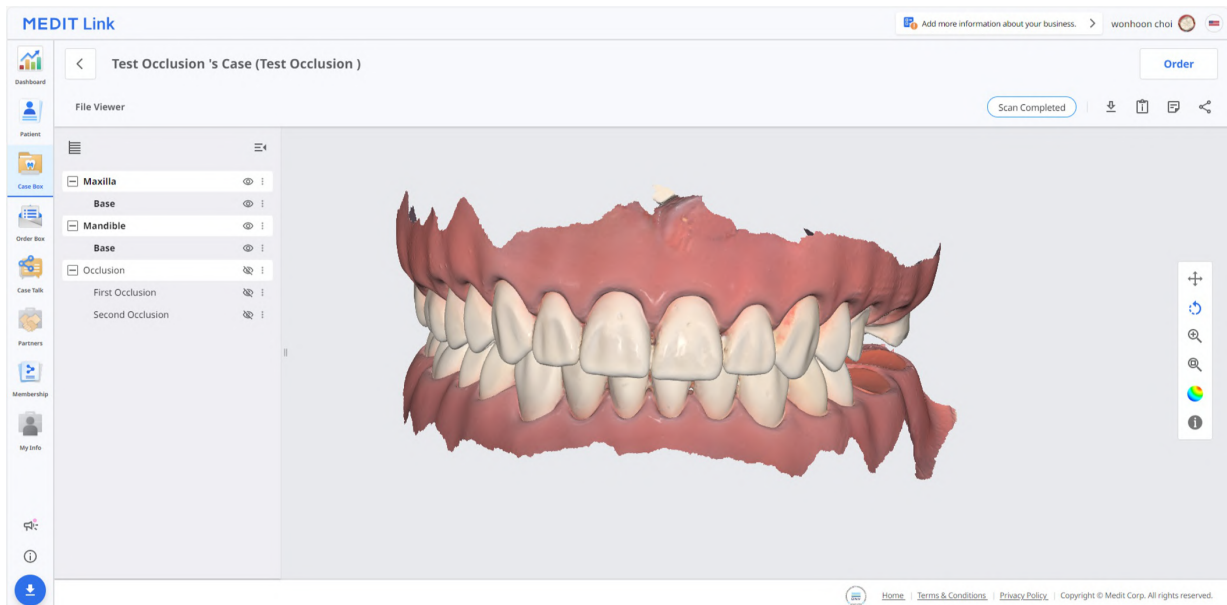

The following case management tools are provided in Case Box/Work Box.

	Case Status	Show the status of the case.
	Go to Order Box/In Box	Provide a shortcut for the same case in another box (Case Box ↔ Order Box; In Box ↔ Work Box).
	Download	Download files in the case to the local PC.
	Case Information	Review case details and save them as a PDF or print them out.
	Memo & Tags	Add a memo or tags when registering or ordering a case. Use them to communicate any additional or supplementary information about the case to your partners.
	Share	Create a link for sharing. Users with the shared link can access the 3D data in a browser such as Chrome, Firefox, or Safari without installing additional programs.

Order Box/In Box

The Order Box (Clinic) or In Box (Lab) shows ordered cases on the Medit Link application.

Order Details


Order Box							
✓ All Pending Canceled Rejected Accepted Shipped Case Completed 1							
Total Cases : 7 2 3 2025-04-29 ~ 2025-07-28 Case Name Case Name / Contains							
Status	Case Name	Patient Name	Order Date	Requested Deliv...	Lab Name	Order ID	Assignee
Canceled	test1's Case	test1	2025-07-23 17:17	2025-07-30 17:00	Elysee Dental Test	6004858	
Canceled	BS_3's Case	BS_3	2025-07-23 17:10	2025-07-30 17:10	Elysee Dental Test	6004857	
Pending	전성미의 케이스	전성미	2025-07-16 17:46	2025-07-17 15:00	MEDIT_LAB_STAGE	6004719	
Pending	Multi Occlusion 케이스's Case - Clone - CL...	Multi Occlusion 케이스	2025-07-15 11:54	2025-07-16 15:00	MEDIT_LAB_STAGE	6004696	
Pending	오프라만의 케이스 - Clone	오프라만	2025-07-14 13:57	2025-07-22 12:00	MEDIT_LAB_STAGE	6004684	
Canceled	오프라만의 케이스 - Clone	오프라만	2025-03-16 17:24	2025-03-17 15:00	MEDIT_LAB_STAGE	6002667	
Canceled	31050 - 복제	seolah	2023-03-22 17:02	2023-03-22 16:00	MEDIT_LAB_STAGE	5995671	

- 1 Filter cases by status such as Pending, Accepted, Shipped, Case Completed, Rejected, or Canceled.
- 2 See key details at a glance, including patient name, order date, delivery request date, lab name, and assigned technician.
- 3 Narrow down the list by case name or a specific date range.

Case Talk

Cases created in the Medit Link can be shared with specific people you specify, anyone with the shared link or registered members of Medit Link.

The user can see a list of all shared cases through "Case Talk" in Medit Link App and Medit Link Web.

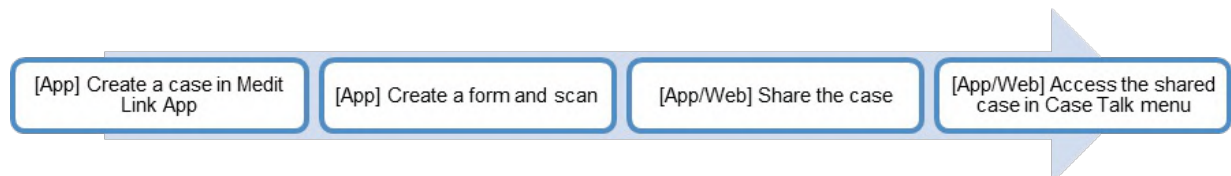


Shared cases can be viewed via a web browser without installing a separate program, enabling easy communication over the 3D data on various devices.


Creating a Case Talk

You can create a shared link to a case that was created in Case Box/Work Box.

1. Find a case you want to share and click the "Share" button on the right.

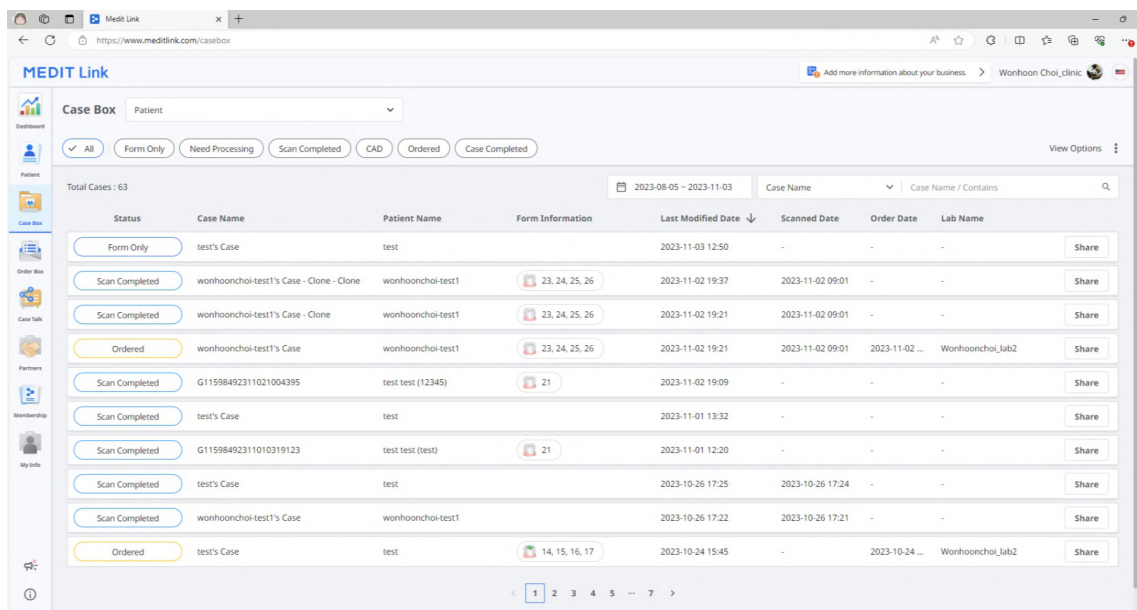

2. Select the files you want to share.

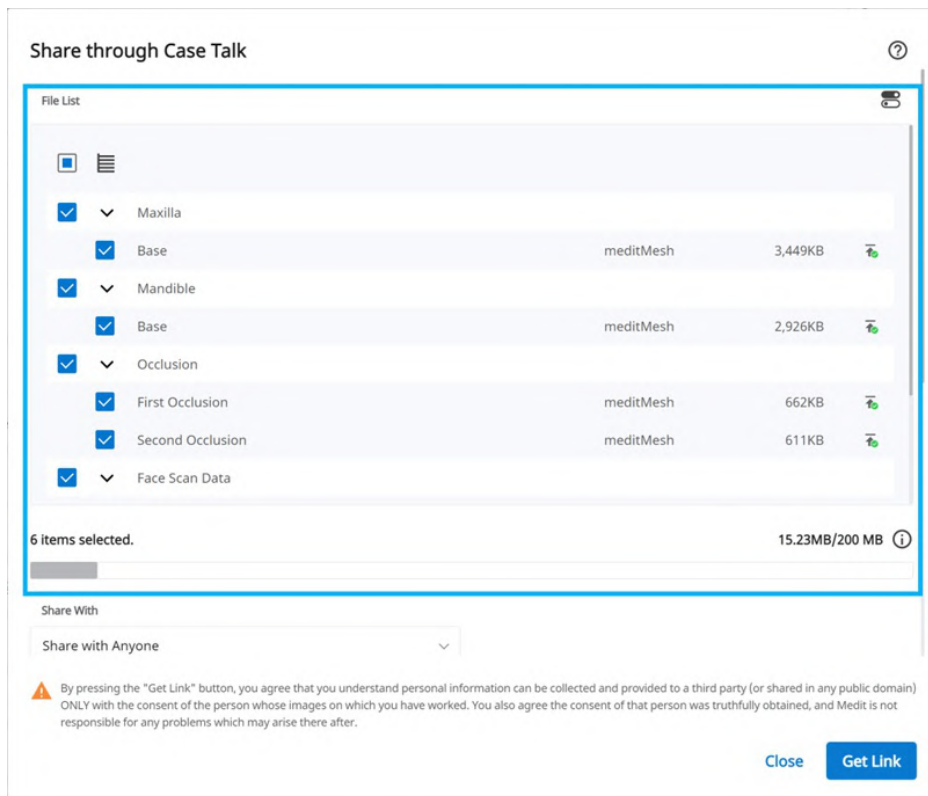

3. Set the sharing permission type and sharing options.
4. Click "Get Link" to create a shared link.

Setting the sharing permission type and sharing options

Share through Case Talk



Share With

Share with specific people

Create a shared link and share it with specific people.

Expiration Date

2022-06-28

You can extend the period up to 90 days from today.

Share via



PIN Code



PIN Code

Enter PIN Code

Set the PIN code using six digits for privacy protection.

☒ Share the patient's name

☒ Grant Download Permission [Premium Feature](#)

If you check this option, all users who have access to the shared link can download the files.

The premium feature is provided to all users for a limited time.
Feature availability is subject to change depending on the membership plan.

☒ Allow comments

If you check this option, you will be able to communicate through the shared link.

Comment

Please enter your comment.

0/1024



By pressing the "Get Link" button, you agree that you understand personal information can be collected and provided to a third party (or shared in any public domain) ONLY with the consent of the person whose images on which you have worked. You also agree the consent of that person was truthfully obtained, and Medit is not responsible for any problems which may arise there after.


Close

Get Link

Share With	• Share with anyone Creates a shared link. You can share the files with anyone. • Share with specific people Creates and sends a shared link to specific people.
Share via	If you select "Share with specific people," you can share the link with specific people in various ways. • Email Address • SMS • Partners
Expiration Date	Set the expiration date for the link. After the link expires, the people you shared it with will no longer have access to files. You can extend the expiration period up to 90 days from today.
PIN Code	Set a 6-digit access code to ensure data security.
Share the patient's name	If you check this option, all people with the link will see the patient's name.
Grant Download Permission	If you check this option, all users with the link can download the files.
Allow comments	• If you check this option, you will be able to communicate through the shared link. • If you uncheck the option, the recipient can only view your comments but will not be able to add their own.
Comment	You can leave a message in the comment section.

Case Talk details

You can see files in the shared case. The following functions are available on the Case Talk details page.

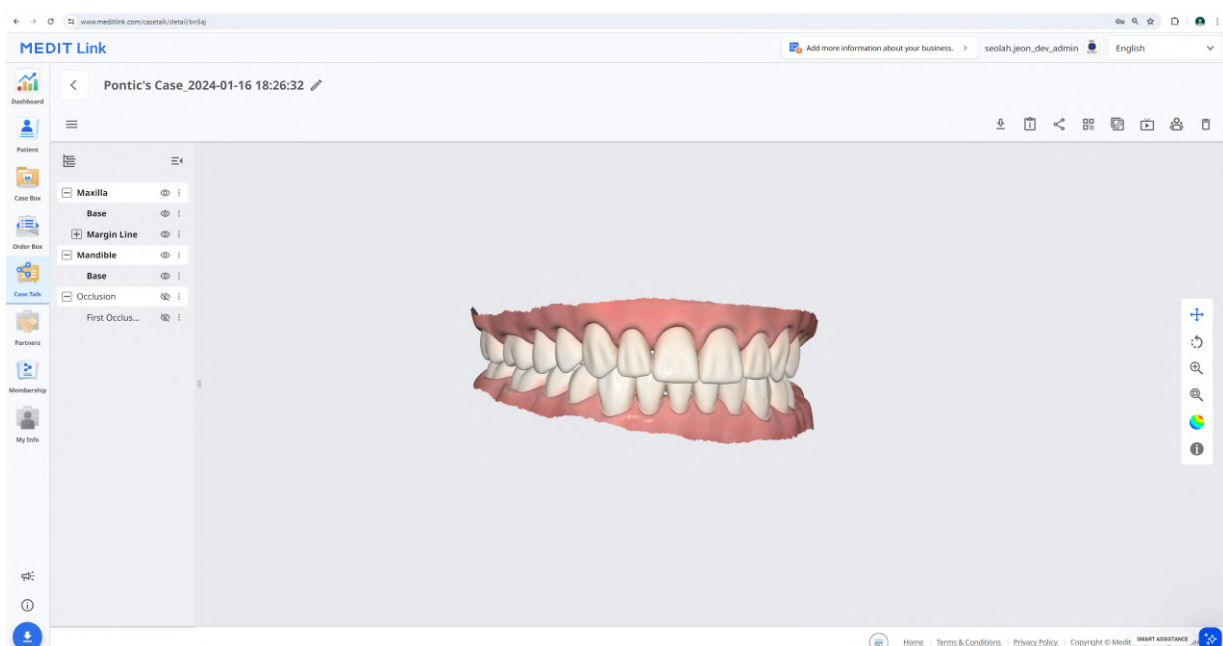

	Download	Download all the files in the case to the local PC.
	Case Information	Show detailed information about the case.
	Share	Modify the share setting.
	QR Code	Share the case with a QR code.
	Copy Link	Copy the shared link to the clipboard. The copied link shows the shared files on your web browser.
	Live	Share your screen with the connected users. The screen sharing is synchronized with the connected users.
	Attendees	Show all users who are accessing the shared link.
	Delete	Delete the shared link. The deleted shared link is no longer accessible.

Note


The following services are not available on Medit Link China.

- Live
- Attendees

Web Viewer

You can view shared data through Case Talk in a web browser.

It is possible to communicate with the people who have access to the link using annotations and comments.

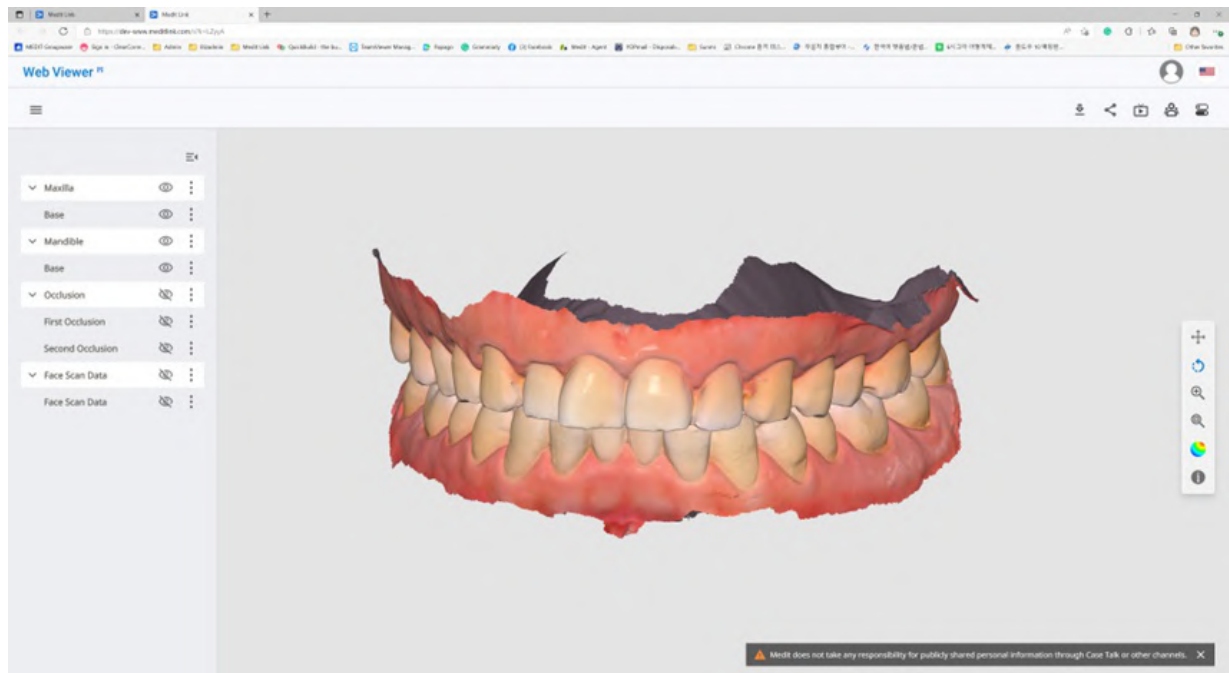

	Download	Download all the files in this case to the PC.
	Case Information	Show detailed information about the case.
	Share	Modify the share settings.
	QR Code	Share the case with a QR code.
	Live	Share your screen with the connected users. The screen sharing is synchronized with the connected users.
	Attendees	Show all users who are connected to the shared link.
	Tooth Labeling	Change the teeth labeling system.


Note

The following services are not available on Medit Link China.

- Live
- Attendees

Annotation

You can add annotations to a specified location on the 3D model. Anyone with the link can view the annotations you add.

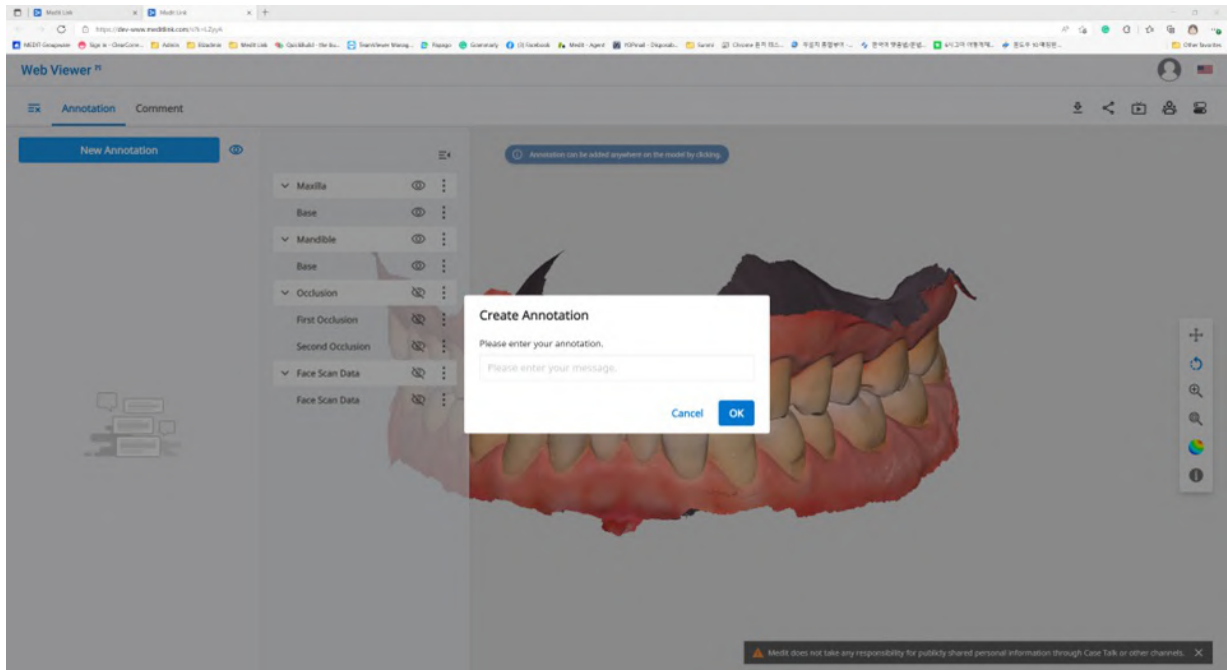

Note

The following service is not available on Medit Link China.

- Annotation

1. Find the menu icon in the top left corner and select the "Annotation" tab. Use the "New Annotation" button to add an annotation anywhere on the data with a click.
2. Type the message, then click "OK" to save it.

A screenshot of the 'Create Annotation' dialog box. It has a title 'Create Annotation' and a prompt 'Please enter your annotation.' Below this is a text input field with the placeholder text 'Please enter your message'. At the bottom are two buttons: 'Cancel' and 'OK'.

3. You can use annotations as a simple communication channel. Click the created annotation to enter a reply and hit "Send."

A screenshot of an annotation detail view. It shows the username 'wonhoonchoi Beta wonhoonchoi' with delete and close icons. Below is the text 'Annotation 1' and a timestamp '1 09-09 20:31'. At the bottom is a text input field with the placeholder 'Please enter your reply' and a 'Send' button.

Comments

You can communicate with users who have access to the link through the "Comment" tab in the Web Viewer menu.

Note

The following service is not available on Medit Link China.

- Comment

Real-time screen sharing

You can share your screen in real-time.

Note

The following service is not available on Medit Link China.

- Live

1. Click "Live" to share your Web Viewer screen with the connected users.




2. Click "OK" to start sharing your screen.
3. Other people who have accessed the link must accept the screen sharing.
4. The screen of the user who initiated screen sharing is displayed in sync with the screen of the person who received the request.


Data Tree

All data is grouped and displayed according to attributes in the data tree. You can show or hide data, adjust opacity, and so on.

Data Tree Controls

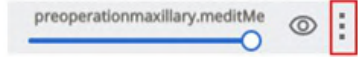Show/Hide	Click the 'eye' icon to show or hide data from the view. Control data visibility as a group or individual file.
Opacity	Hover the mouse over a visible element to see the opacity slide bar. Adjust the slider to change the transparency of your data. 
Extended Menu	The extended menu is provided on the right side of each Data Tree element. It includes the following options:  • Show This Only: Hide all other data except for the current one. • Hide: Hide the current data.

3D Model Controls

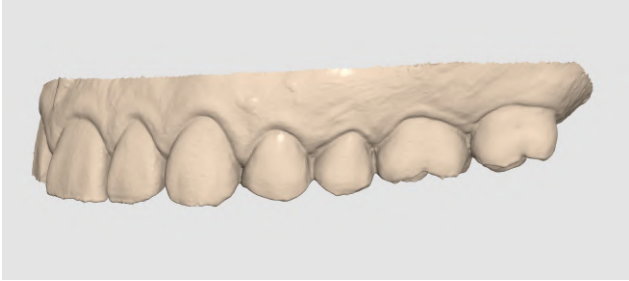	Pan	Move the model.
	Rotate	Rotate the model.
	Zoom In/Out	Zoom in and out on the model.
	Zoom Fit	Position the model in the center of the screen.
	Data Display Mode	Matte See the data in color with matte finish (Phong rendering). 
		Monochrome See the data in one color. 

Partners

You can search for potential partners to request a partnership and view the list of current partners.




Services using Google Maps are not available on Medit Link China.

Search for Partners


You can search for Medit Link users based on their business name and/or address.

In the search results, you can send a partnership request to a business you want to work with.

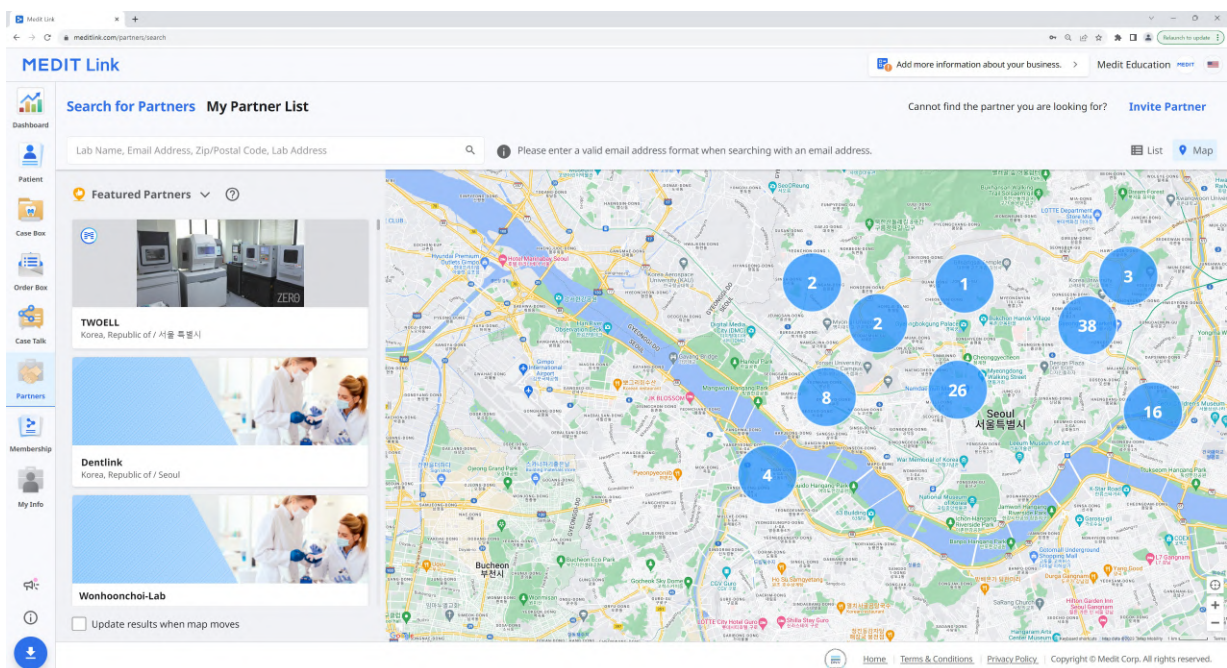

My Partner List

You can review your current partnerships with all your existing partner businesses in My Partner List.

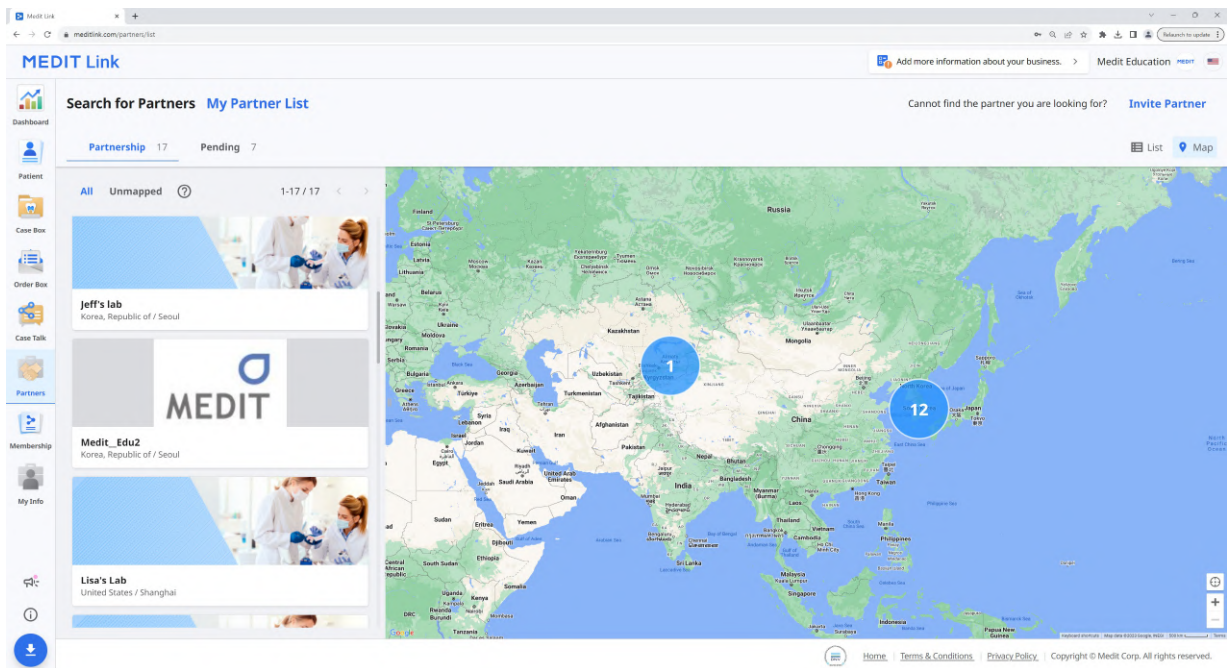

Partner Invitation

If you have a clinic or lab you want to work with, you can invite them to join Medit Link. The partnership will be automatically formed once they accept the invitation.

1. Click "Invite Partner" in the top left of your screen.
2. Enter the email address of a business you want to work with and send them an invitation with a partnership request.



Invite Partner

Invite your desired partner to join Medit Link. When the lab accepts your invitation, a partnership is automatically formed.

Cancel

Send Invitation

3. Once the invited organization accepts the invitation and signs up for Medit Link, the partnership will be automatically established.


Membership

Membership

The screenshot displays the MEDIT Link Membership page. The top navigation bar includes 'MEDIT Link', a link to 'Add more information about your business', the user 'seolahjeon_', and the language 'English'. The main navigation tabs are 'Membership', 'Payment Details', 'Redeem', and 'My Subscription'. The left sidebar contains icons for Dashboard, Patient, Case Box, Order Box, Case Talk, Partners, Membership, and My Info. The 'Membership' section is active, showing the 'Medit Cloud Storage Plan' with options for '2 GB Plan' and '10 TB Plan'. The '10 TB Plan' is selected and highlighted as a 'Free (Promotion)'. It lists benefits: 'A paid plan', '10 TB of cloud storage', 'Efficient for users with many cases and various data', and 'One-month free trial included'. A 'Subscribe' button is present. Below, the '10 TB Plan Benefits' section shows a '10 TB Plan' with a coupon code 'qqqq' and an expiration date of '2025-06-26'. A note states: 'The promotion has been applied. Your 10 TB Plan is now active and will remain valid until 2025-05-14. View More. Learn more about the Medit Cloud Storage Plan in our Q&A Here.' The 'Cloud Storage' section on the right shows a 'Set Period' from '2018-01-01' to '2025-02-17' with filters for 'All', '1 Week Ago', '1 Month Ago', '6 Months Ago', and '1 Year Ago'. A donut chart indicates '3.46 GB / 10 TB' usage. Below the chart, a table lists storage items: 'Total Selected' (0.00 KB), 'Raw Data' (52.36 GB), and 'Result Files & Attachments' (3.46 GB). Each item has a checkbox and a 'Delete' button. The footer contains links for 'Home', 'About', 'Terms & Conditions', 'Privacy Policy', 'Copyright © Medit Co.', and a Korean notice.

Medit Cloud Storage Plan

You can see what each plan offers, including discount coupons and vouchers, and your current subscription status. You can upgrade your plan or cancel your subscription at any time.

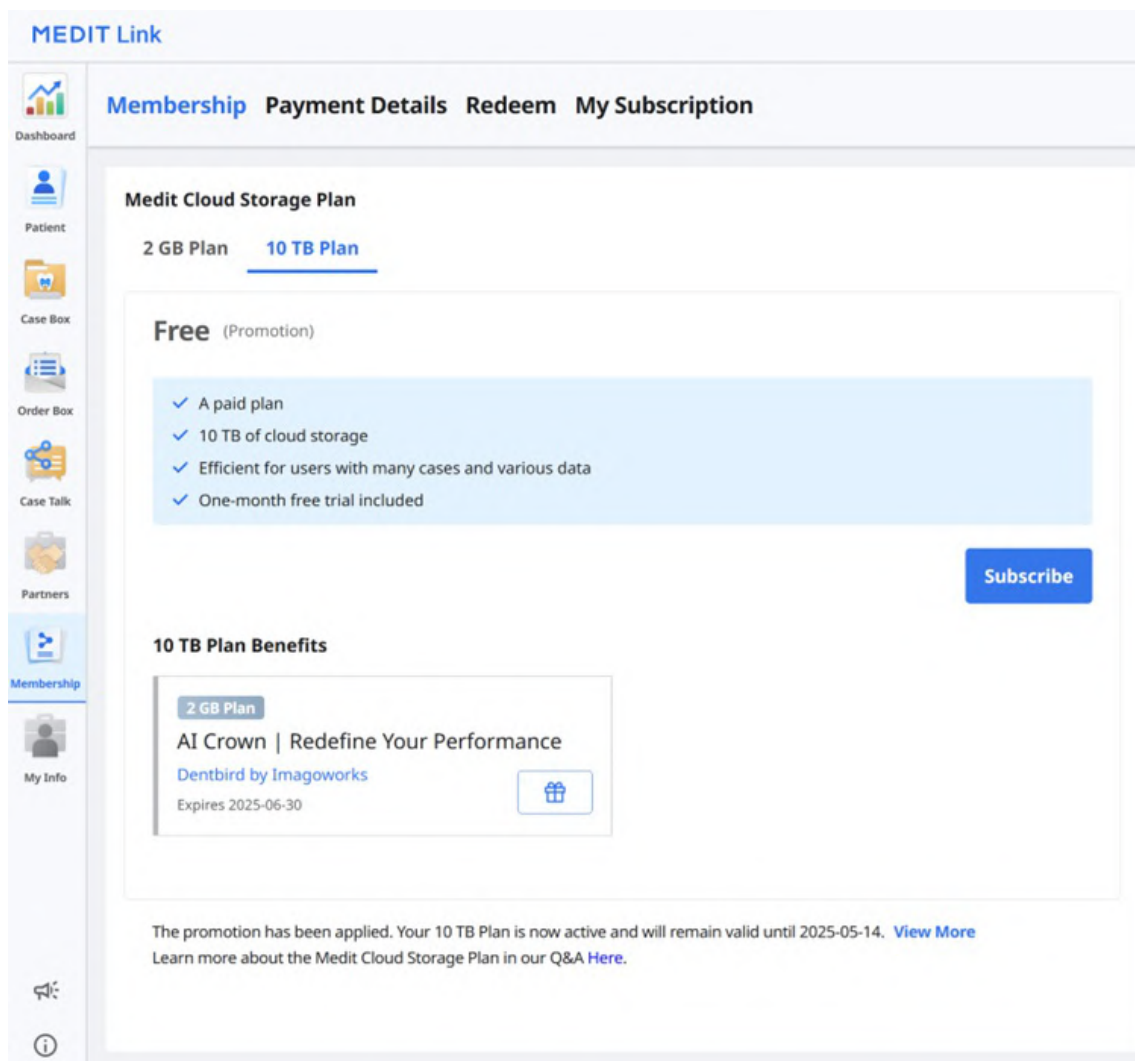

Medit Link provides the following cloud storage plans:

2 GB Plan

- Free standard plan for all signed-in users
- 2 GB cloud storage
- For new users

10 TB Plan

- Paid plan
- 10 TB cloud storage
- For users with many cases and various data
- One-month free trial


	2 GB Plan	10 TB Plan
Cloud Storage	2 GB	10 TB
Monthly Fee	Free of charge	\$9.99/month
* Cloud storage does not count raw data toward the used space.		

Note

The membership plans will be available on Medit Link China. You can enjoy the 10 TB Plan for free until the official release of the feature.

Cloud Storage

In the right-side section, you can manage your cloud storage.

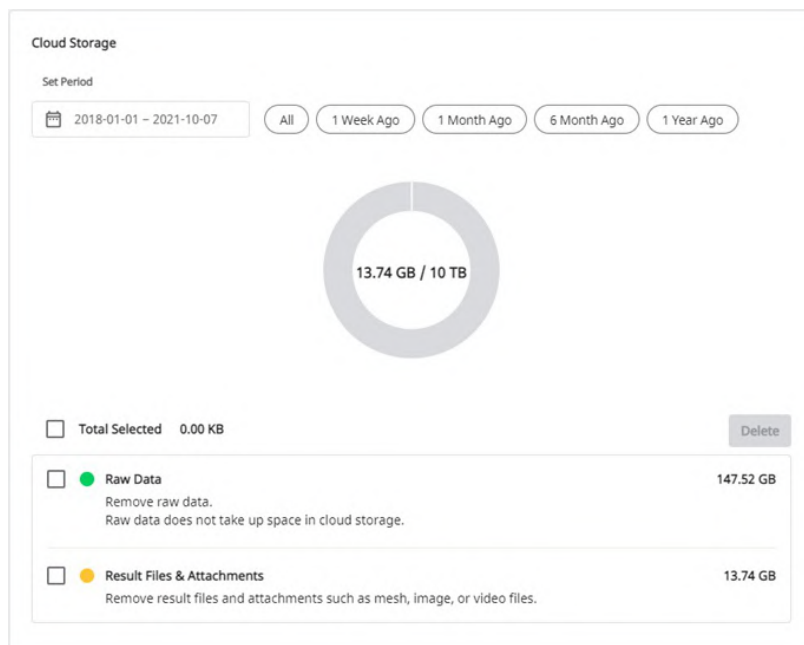

Set Period	Set the specific period to delete data created within that time frame. • All: See all files. • 1 Week Ago: See the files that have passed a week after creating. • 1 Month Ago: See the files that have passed a month after creating. • 6 Months Ago: See the files that have passed six months after creating. • 1 Year Ago: See the files that have passed a year after creating.
Raw Data	The raw data does not take up space in the cloud storage, so it will not be in the pi-chart. But you can manage the raw data from here if needed, e.g., delete it to protect the patient's private information.
Result & Attachment	The result and attachment files take up space in the cloud. If the cloud is full, you can secure more storage space by removing unnecessary files.
Delete	Remove selected files from the cloud storage to free up space.

Subscribe

If you are currently on the 2 GB Plan or the 10 TB Plan with a redeem code, you can subscribe to the 10 TB Plan.

The payment is completed through Stripe service (www.stripe.com).

1. Go to Membership > Medit Cloud Storage Plan > 10 TB Plan and click the "Subscribe" button.

Medit Cloud Storage Plan

2 GB Plan 10 TB Plan


Free (Promotion)

- ✓ A paid plan
- ✓ 10 TB of cloud storage
- ✓ Efficient for users with many cases and various data
- ✓ One-month free trial included

Subscribe

2. Check the 10 TB Plan description and click "Subscribe" again.

Subscribe Membership

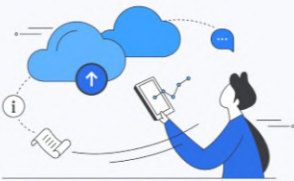

10 TB Plan
9.99\$ USD / Monthly

- ✓ Subscribe now to keep your 10 TB Plan after the promotion expires.
- ✓ You will pay a subscription fee after the current promotion expires. If you don't wish to pay, you can unsubscribe before payment.

* VAT is not included.
* Learn more about the Medit Cloud Storage Plan in our Q&A [Here](#).

Cancel **Subscribe**

3. Check the price and conditions and input information necessary for the payment. Then, click "Subscribe."

←

Subscribe to Medit Cloud Storage 10TB Plan

\$0.00
Then \$9.99 per day after coupon expires

Medit Cloud Storage 10TB Plan for Clinics Medit Cloud Storage Plan Billed daily	\$9.99
Subtotal	\$9.99
1 Month Free for New Subscribers 100% off for a day	-\$9.99
Tax ⓘ	\$0.00
Total due today	\$0.00

Pay with card

Email

Card information

Cardholder name

Country or region

▼

Subscribe

By confirming your subscription, you allow Medit USA Inc to charge you for future payments in accordance with their terms. You can always cancel your subscription.

Powered by [Terms](#) [Privacy](#)

4. The payment may take several seconds. When done, you will see the confirmation message.

Subscribe Membership



Payment Successful.

[Invoice](#)


[Close](#)

Cancel Subscription

You can cancel your subscription at any moment. After canceling, you can still use the 10 TB Plan for the remaining days of that paid month. Your plan will be changed to the 2 GB Plan from the next payment day, and the cloud storage will be limited to 2 GB. If the cloud storage capacity is exceeded, the files will not be backed up, and the order and share functions will not be available.

1. Click "Unsubscribe."
2. Click "Cancel Subscription."

Cancel Subscription

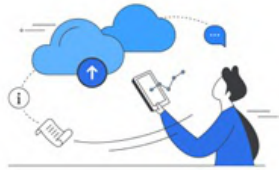

Are you sure you want to cancel your **10 TB Plan**?

✓ By confirming, you will automatically be switched to the free 2 GB Plan.
Enjoy full access to the features and storage of your current plan through to 2025-02-20.

ⓘ Please note that all uploaded files will be retained regardless of the plan you choose.
If your storage exceeds the 2 GB limit, you won't be able to upload new files.

*Click [here](#) to learn more about our membership plans.

[Stay Subscribed](#)

[Cancel Subscription](#)

3. To confirm that you want to unsubscribe, click "Cancel Subscription" again in the new pop-up window.

Confirm Subscription Cancellation

Your subscription will be canceled on 2025-02-20.

[Stay Subscribed](#)

[Cancel Subscription](#)

4. Your subscription has been canceled.

Confirm Subscription Cancellation

Your subscription has been canceled.

You can use your current plan until 2025-02-20. After that, the cloud capacity will be limited to 2 GB.

If the used space exceeds the limit, data will not be backed up, and functions such as Order and Share will not be available.

[Close](#)

5. Your plan will be changed to 2 GB Plan from the next payment day, and the cloud storage will be limited to 2 GB. If the cloud storage capacity is exceeded, the files will not be backed up, and the order and share functions will not be available.

My Subscription

You can view the details of your current subscription, resubscribe, or cancel it from My Subscription. You can also change the payment method, which will be applied starting from the next scheduled payment.

MEDIT Link

Add more information about your business. > seolahjeon_ English

Membership Payment Details Redeem My Subscription

Payment Method Visa *****7473 (07/29) Stripe Edit Payment Method

Status

Product	Start Date	Next Payment Date	Amount of Payment	Status	Payment Method
10 TB	2023-05-15	-	-	Promotion Applied (2025-05-14)	Subscribe

Home About Terms & Conditions Privacy Policy Copyright © Medit Corp. A. PMS

Details

Item	Description
Product	Displays the type of subscription.
Start Date	Displays when the subscription or promotion started.
Next Payment Date	Displays the next payment date.
Amount of Payment	Displays the payment amount.
Status	Subscription • Active: The subscription is active. • Expiring: The subscription will not renew on the next payment date and will be canceled. • Expired: The subscription has ended. Promotion • Promotion Applied: The promo code is now active. • Promotion Pending: The promo code is awaiting activation. • Promotion Expired: The promo code has expired.
Payment Method	Displays the last four digits of the card used for payment.

Resubscribe

Item	Description
Cloud Storage Plan	Click the subscription button to resubscribe to 10 TB Plan.
App Subscription Plan	Sign in to the app to resubscribe.

How to add a payment method

1. Click "Edit Payment Method."


The screenshot shows the 'My Subscription' page in the MEDIT Link interface. The page has a sidebar with navigation icons and a main content area. The main content area includes a 'Payment Method' section at the top, which shows 'Visa ****7473 (07/29)' and a 'Stripe' button. Below this is a table of subscriptions. The table has columns for Product, Start Date, Next Payment Date, Amount of Payment, Status, and Payment Method. There are two rows of data: one for 'Medit ClinicCAD' and one for '10 TB'. The '10 TB' row has a 'Subscribe' button next to it.

Product	Start Date	Next Payment Date	Amount of Payment	Status	Payment Method
Medit ClinicCAD	2025-03-06 10:00	-	\$0.99 / Monthly	Expired (2025-03-07)	Visa ****7473
10 TB	2023-05-15 14:38	2025-05-14 23:59	\$0.60 / Monthly	Scheduled (2025-05-14)	Visa ****7473

2. Click "Add payment method."

PAYMENT METHOD

 Mastercard ••••  Default Expires 03/2029 

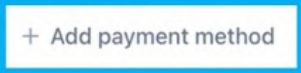

3. Input the required information and click "Add."

Add payment method

Card number

1234 1234 1234 1234   

Expiration date

MM / YY

Security code

CVC 

Country

South Korea 

By providing your card information, you allow Medit USA Inc to charge your card for future payments in accordance with their terms.

☒ Use as default payment method

Add

Go back

4. The newly added payment method will be set as default and used for the next payment.

PAYMENT METHODS

 Visa •••• 4242 Default Expires 04/2044 

 Visa •••• 4242 Expires 05/2025 



How to change a payment method

1. Click the expanded menu icon next to the payment method you want to change in the billing dialog window.

PAYMENT METHODS

 Visa •••• 4242 Default Expires 04/2044 ×

 Visa •••• 4242 Expires 05/2025 ...

+ Add payment method

2. Click “Make default” to set the selected card as the default payment method.

CURRENT SUBSCRIPTION

[DEV] Medit Cloud Storage 10TB Plan for Clinics
\$9.99 per day
View details ▼
Your subscription renews on February 20, 2025.

PAYMENT METHODS

 Visa •••• 4242 Default Expires 04/2044 ×

 Visa •••• 4242 Expires 05/2025 ...

+ Add payment method

Make default

Delete

Redeem

You can redeem a promo code to access the 10 TB Plan for free.

Details

MEDIT Link Add more information about your business. seolah.jeon_dev_admin English

Membership Payment Details Redeem My Subscription

Register Redeem Code

Register

Redeem Code

All

Product	Promotion	Code Number	Registration Date	From	To	Status	
4	Premium	4 Clinic Premium Promotion	2A6F-0125-6747-4A87	2022-03-16	2021-10-01	2021-10-05	Waiting

Cancel

1

Home About Terms & Conditions Privacy Policy Copyright © Medit Corp.

Item	Description
Product	Displays the plan to which the promotion is applied.
Promotion	Displays the name of the promotion.
Code Number	Displays the code number.
Registration Date	Displays the date the code was registered.
From	Displays the start date of the promotion.
To	Displays the expiration date of the promotion.
Status	- Registered: The promo code has been added and is now available. - Waiting: The promo code is awaiting activation. - Activated: The promo code is now active. - Used: The promo code has expired.

Register Redeem Code

1. Input the redeem code and click "Register."
2. Read the notice and choose "Register" or "Apply" in the pop-up dialog.
 - **Register:** Register the redeem code. You can activate and use the redemption code later after registration.
 - **Apply:** Register and activate the redemption code. The promotion discount applies immediately.

Register Redeem Code

You can register for the promotion or apply immediately.
 Register allows you to apply and activate when you need it, If
 applied immediately, it will start after the expiration of the active
 promotion.
 Periodic payments are stopped automatically.

Close
Register
Apply

Activate Redeem Code

You can register multiple redeem codes, but you need to activate the one you want to use.

1. Click the "Activate" button next to the code you want to use.
- If there is no code with the "Activated" status, the one you choose will be applied immediately.

If an already activated code exists, the newly selected code changes to the "Waiting" status. When the currently activated redeem code expires, the new redeem code in the "Waiting" status will be automatically activated.

Note

Only one redeem code can be activated, and only one can be in "Waiting" status.

Payment Details

You can view your subscription and payment history in the Payment Details tab.

MEDIT Link

Introduce your business by filling in the required information.

Yeseul Lee

English

Membership

Payment Details

Redeem

My Subscription

Dashboard

Patient

Case Box

Order Box

Case Talk

Partners

Membership

My Info

All

Membership

Medit Apps

2024-11-21 ~ 2025-02-19

Payment Date	Product	Payment Type	Duration	Details	Amount of Payment	Status
2025-02-19	10 TB	Subscription	2025-02-19 ~ 2025-02-20	Monthly payment	0.00 \$ (USD)	Payment Completed

<

1

>

스마트 지원

Home

About

Terms & Conditions


Privacy Policy

Copyright © MEDIT Corp. All rights reserved.

Item	Description
Payment Date	Displays the date of payment.
Product	Displays the purchased plan or app.
Payment Type	Displays the type of payment: • Subscription • One-time payment
Duration	Displays the subscription period.
Details	Displays the detailed payment information.
Amount of Payment	Displays the amount for a subscription or one-time purchase.
Status	• Payment Completed: The payment has been processed. • Payment Failed: The payment was not processed. • Payment Incomplete: The payment has not been completed. Click "Pay Now" to proceed. • Waiting: The payment is pending for a subscription renewal.

View Bill Details

You can view your payment invoices and receipts by clicking the receipt icon on the right.

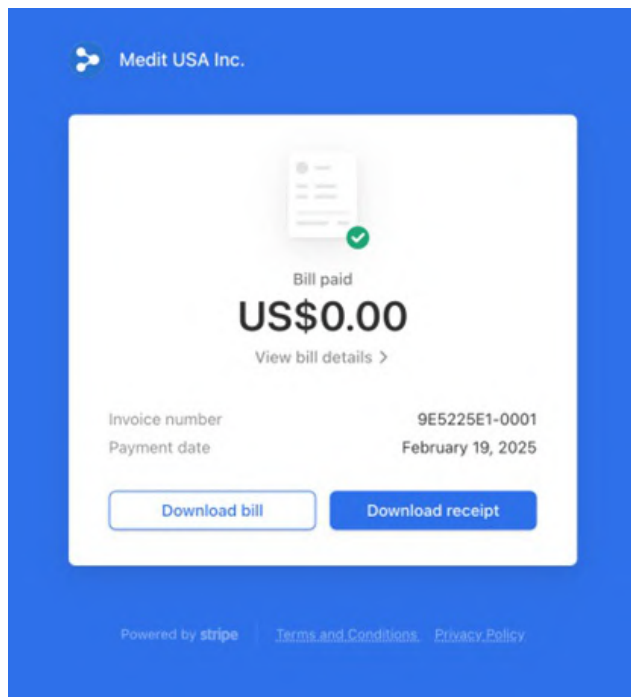

Click "View bill details" for payment details.

Medit USA Inc.

9E5225E1-0001

February 19, 2025

Bill paid

US\$0.00

View bill details X

Invoice number

9E5225E1-0001

Payment date

February 19, 2025

Download bill

Download receipt

Powered by stripe

Terms and Conditions

Privacy Policy

X Close Bill Details

Paid on February 19, 2025

SUMMATION				
Target	yeseul.lee.medit.clinic@gmail.com			
source	Medit USA Inc.			
bill	#9E5225E1-0001			
ITEM				
FEBRUARY 19 - FEBRUARY 20, 2025				
Medit Cloud Storage 10TB Plan for Clinics		US\$0.60		
Quantity 1 piece				
subtotal		US\$0.60		
1 Month Free for New Subscribers(100% 할인)				
		-US\$0.60		
Payment amount		US\$0.00		
Amount paid		US\$0.00		
Remaining amount		US\$0.00		

Do you have any questions?Contact Medit USA Inc.

57

My Info

My Account

In My Info, you can review and edit the information you provided during registration and check your membership status.

My Clinic/My Lab (Admin Account Only)

You can review and edit the information about your clinic or lab. This section is only available to admin accounts.

MEDIT Link

Dashboard

Patient

Case Box

Order Box

Case Talk

Partners

Membership

My Info

My Account

My Clinic

My Scanner

Clinic Name *

wonhoon-clinic

Edit

Cover and Logo

Looking for digitalized dental laboratories?

Find the most ordered Labs around your clinic with 'Featured Par

1

2

3

Services

Select

Specialties (choose up to 3)

Select

Select the services your business specializes in from the list.

About Us

e.g., Welcome to OOO Dental Clinic.

We are a licensed dental clinic situated in California specializing mainly in aesthetic restorations and implantology.

0/2000

Website URL

Add Website

Register up to 3 websites.

My Scanner

You can check the connected device information for the account.

MEDIT Link

Add more information about your business. > seolahjeon_ English

My Account

My Clinic

My Scanner

N.	Type	Serial Number	Usage Time	Status	Last Connected	Last Calibrated	Warranty Start Date	Warranty Expiry Date	Days Left	
1	i700	AZ2105102619	0d 18h 44m	Unregistered	2024-04-12 15:19	2024-03-28 09:10	2024-02-27	2025-02-26	Expired	Buy
2	i700	AZ2105102622	0d 18h 39m	Unregistered	2023-04-26 20:30	2023-04-25 14:47	2021-07-09	2022-07-08	Expired	Buy
3	i700	AZ2305100410	0d 6h 38m	Registered	2025-03-14 09:03	2024-09-05 14:23	-	-	-	Buy
4	i600	BG2202100001	0d 0h 58m	Unregistered	2023-08-28 18:10	2023-07-13 15:00	2022-10-30	2023-10-29	Expired	Buy
5	i600	BG2202100026	0d 8h 32m	Unregistered	2022-10-28 17:03	2022-09-25 09:00	-	-	-	Buy
6	i600	BG2202100029	0d 2h 38m	Unregistered	-	2022-06-16 11:58	-	-	-	Buy
7	i900	CA2405100106	0d 11h 46m	Unregistered	2025-02-26 08:37	2024-11-07 10:11	-	-	-	Buy

The scanner information is automatically registered once your scanner is connected to a Medit scan program.

Home


About

Terms & Conditions

Privacy Policy

Copyright

SMART ASSISTANCE

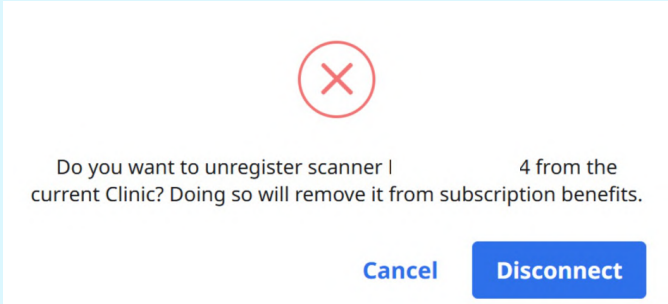Item	Description
Type	Displays the model name.
Serial Number	Displays the device's serial number.
Usage Time	Indicates how long the device has been used.
Status	Displays the device's registration status: • 'Registered' indicates that the scanner has been successfully connected and registered. • 'Unregistered' indicates that the scanner is no longer registered. If the scanner is registered with a different group, it will lose its registration in the current group.  Note To unregister your scanner: 1. Click the Registered button next to the scanner you want to unregister. 2. When the following pop-up appears, click Disconnect to confirm: 
Last Connected	Displays the date of the last connection.
Last Calibrated	Displays the date of the last calibration.
Warranty Start Date	Displays the date when the warranty was activated.
Warranty Expiry Date	Displays the date when the warranty expires.
Days Left	Displays the number of days remaining before the warranty expires.
Buy	Allows the user to purchase and extend the warranty.
Delete icon	Allows the user to remove the scanner from the list.

Other Menu

Notice & Update

You can check the important notices from Medit and program updates of Medit software.

Notice & Update

Notice
NFT Service Termina...
2023-04-19

Notice
End of Support Notic...
2023-04-14

Update
Release Note for Me...
2023-03-29

Notice
[Medit] We've updat...
2023-03-29

Update
Release Note for Me...
2023-03-17

Update
Medit Occlusion Ana...
2023-03-08

Notice
NFT Service Termination Notice
2023-04-19

To our users,

We want to thank all Medit Link users who have supported and used our NFT service.

Unfortunately, we must inform you that this service will be terminated following the upcoming program update.

Please see the details below for more information:

- **Feature in Question**
 - Share via NFT (in Medit Link Case Talk)
- **Effective Date**
 - Starting from the update in the first half of 2023 (scheduled for April 27)

※ The update schedule is subject to change.

We'll continuously work to provide better service.

Medit Link Team

Related Websites

You can get various information about using Medit Link. Access Medit's educational and informational resources, such as Medit Academy, Medit Academy YouTube, and Medit Help Center.

61

Need more help?



Medit Academy >



YouTube >



Help Center >

Download Medit Link App

You can download the latest version of the program installer from the web. This menu is helpful when you lose the installer provided in the product package, or you have an outdated installer.



Note

Medit Link updates automatically when you have an internet connection.



Windows

macOS

Overview

MEDIT Link 3.4.5

Patient

Total Patients: 620

Patient Name ↓	Patient ID	Gender	Date of Birth	Number of Cases	Registration Date
0511 test				1	2021-05-11 PM 3:30
19.12.09 test				0	2019-12-09 PM 1:58
150				2	2022-10-19 PM 7:54
123454321				0	2019-12-19 PM 2:05
1231456		F	1231-12-04	4	2018-09-18 PM 3:13
123				2	2019-07-12 PM 2:41
111				1	2021-04-23 AM 11:03
1				1	2025-03-13 AM 9:08
1				2	2025-03-13 AM 9:14
1				1	2025-03-13 AM 9:14
1				1	2025-03-13 AM 9:14

1	Title Bar	Refer to Medit Link App > Title Bar for detailed information.
2	Menu	Refer to Medit Link App > Menu for detailed information.
3	Tools and Functions	Refer to Medit Link App > File Viewer and Medit Link App > View Options for detailed information.
4	Case List	Refer to Medit Link App > Filtering and Search for detailed information.

The table below summarises what menus are available in each type of clinic and lab account.

Account Type		Patient	Case Box	Order Box	In Box	Work Box	Case Talk	Trash Box	App Box
Clinic Accounts	Admin	O	O	O	X	X	O	O	O
	Member	O	O	O	X	X	O	O * Restore feature only	O
Lab Accounts	Admin	X	X	X	O	O	O	O	O
	Member	X	X	X	O	O	O	O * Restore feature only	O

Title Bar

The title bar provides quick access to features such as account information, notifications, notices and updates, settings, and network status.



My Information

My information contains basic user information such as profile image, clinic or lab name, email address, and user name.

My Information



Clinic Name

Medit_Edu

Email Address

edu@medit.com

Name

Medit Education

Log Out

Close

Notifications

You can see notifications related to partnerships, orders, and work processes.

Notification

1

☐ Display Unread Messages Only

Mark All As Read

2

Delivery

Wonhoonchoi_Lab_Dev has dispatched the ordered package.

2020-08-20 오후 5:20

Order

Wonhoonchoi_Lab_Dev has accepted the order.

2020-08-20 오후 5:18

Order

Wonhoonchoi_Lab_Dev has accepted the order.

2020-08-17 오후 4:05

Order

Wonhoonchoi_Lab_Dev has accepted the order.

2020-08-17 오후 4:05

Order

Wonhoonchoi_Lab_Dev has accepted the order.

2020-08-17 오후 4:05

Order

Wonhoonchoi_Lab_Dev has accepted the order.

2020-08-17 오후 4:05

Work

Wonhoonchoi_Clinice_Dev_Member registered a new case test's Case.

2020-08-14 오후 6:33

Work

Wonhoonchoi_Clinice_Dev_Member registered a new case test's Case.

2020-08-14 오전 10:15

Close


1	Check the box to show only the unread messages.
2	Click any notification to instantly move to the related case or webpage.

Notice & Update

You can check the important notices from Medit and program updates of Medit software.

Settings

The Settings dialog allows users to configure the program.

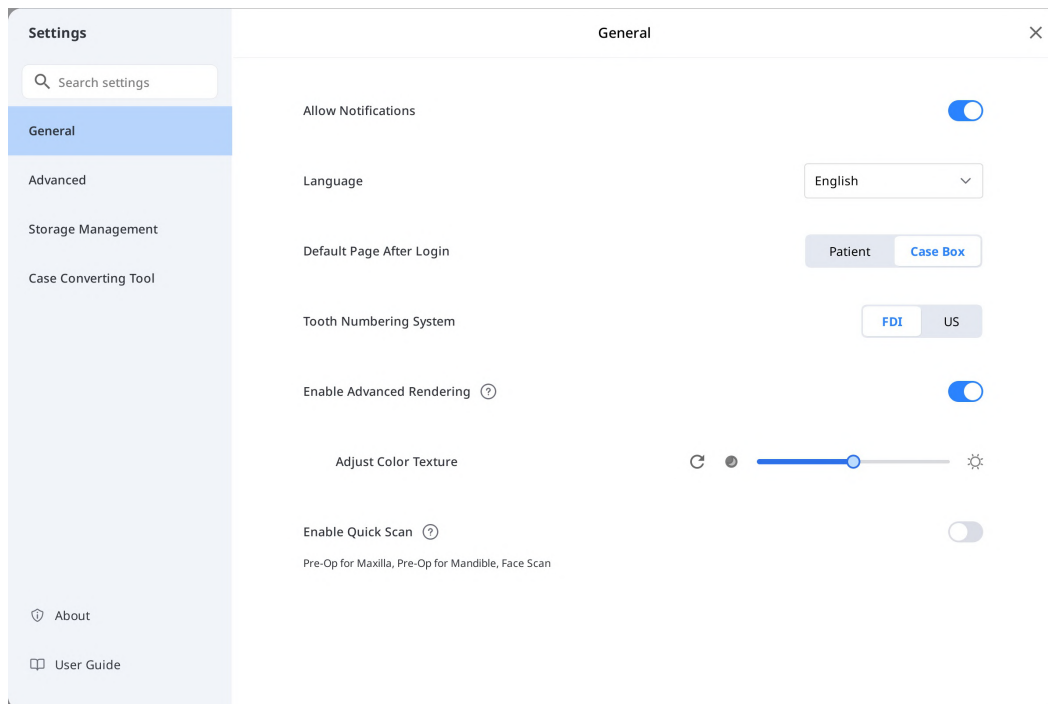


Please refer to the [Medit Link App > Settings](#) for detailed information about the setting options.

Network Status

Medit Link enables you to use all the features seamlessly when connected to a network.

To help you check the status of your network connection in real time, the network status icon is displayed in the title bar.

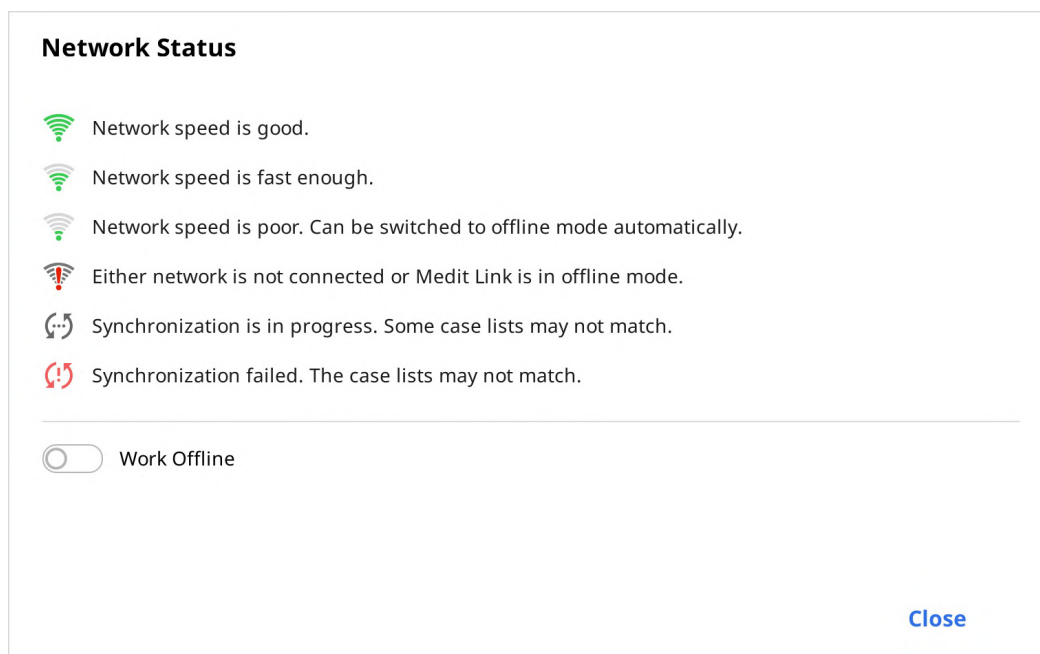

If the network connection is not good, you may experience the following restrictions:

- Some functions will be restricted, such as ordering a case, creating a case, sharing via Case Talk, and cloud synchronization.
- The overall performance of the service can be reduced due to the impact of the network environment.

In this case, you can enter the Offline Mode to perform basic tasks and then use the features that require internet once you are connected.

 Note

The network symbol indicates the response speed between the current area and AWS (Amazon Web Service). It does not mean your network speed, so sometimes it may show the status as poor or bad, even though your network speed is fast enough.

If the other process uses the same network, the speed may show as poor or bad, and the overall performance of Medit Link could be bad. In this case, please do not share the network with other processes.

Settings

General

Provides options for overall system setup.

Settings

General

Advanced

Storage Management

Case Converting Tool

About

User Guide

General

Allow Notifications

Language

Default Page After Login

Tooth Numbering System

Enable Advanced Rendering ?

Adjust Color Texture

Enable Quick Scan ?

Pre-Op for Maxilla, Pre-Op for Mandible, Face Scan

Patient

Case Box

FDI

US

Language	Select the language for the program.	
Tooth Numbering System	Select a teeth labeling system between FDI and US.	
Allow Notifications	When checked, you will receive notifications for program updates.	
Enable Quick Scan	Choose if you want to have the scan stages below automatically appear in Medit Scan for Clinics without having to fill in the form information. • Pre-op for Maxilla • Pre-op for Mandible • Face Scan	
Enable Advanced Rendering	Advanced Rendering	When checked, the 3D model is displayed in a more vivid and realistic rendering.
	Adjust Color Texture	Adjust the brightness of the 3D model. * The resulting colors may slightly differ from Medit Link when you open the same data with other software.
Default Page After Login	Select the default menu to display when starting the program. • Clinic accounts can choose between Patient and Case Box • Lab accounts can choose between In Box and Work Box	

Advanced

Configure advanced options such as network proxy, default app, and experience sharing settings.

Settings

Search settings

General

Advanced

Storage Management

Case Converting Tool

About

User Guide

Advanced

Export

Export Settings

Network

Proxy Server

Proxy Server IP (or DNS)

Proxy Server Port

0

Proxy Server Scheme

HTTP

HTTPS

Proxy Server Username (Option)

Proxy Server Password (Option)

Connect

Default App

☒

Medit Scan for Clinics

C:/Program Files/Medit/Medit Link/Medit Link/Medit_iScan.exe

User Experience Program

Join Medit User Experience Improvement Program

Export – Export Settings

Set how to export files from Medit Link.

Settings

Search settings

General

Advanced

Storage Management

Case Converting Tool

About

User Guide

Search settings

General

Advanced

Storage Management

Case Converting Tool

About

User Guide

<

Export Settings

×

Type

Medit

Export Files To

C:/Users/medit/Downloads/

If duplicate files exist ?

Ask every time

Mesh Format ?

MEDITMesh

OBJ

PLY

STL

☒ Do not convert the attached files

Image Format ?

No image

Compress Files

Combine Individual Mesh

Rule for Folder Name

Case Creation Date

Scanned Date

Order Date

Patient Name

Case Name

Order ID

Partner Name

Patient ID

Assignee

Preview Folder Name

Case Creation Date - Patient Name

Rule for File Name

Case Creation Date

Scanned Date

Order Date

Patient Name

Case Name

Order ID

Partner Name

Patient ID

Assignee

Preview File Name

Type	Select the CAD program you want to use for designing. The program will adjust the results based on your selection, allowing you to proceed with the design without any additional axis alignment.
Export Files To	Specify the path where you want to store the results.
If duplicate files exist	Choose the default action for cases when the same folder or file name exists in the target path. • Always ask when exporting. • Export a new file and overwrite the existing files. The file currently stored will be lost. • Create a folder and export the files
Mesh Format	Specify the format of the final mesh file.
Do not convert the attached files	When enabled, attached files will be exported in their original formats without conversion.
Image Format	Specify the format of the exported image files.
Compress & Combine	Compress Files: Compress all results into a single file. Combine Individual Mesh: Combine all results for the maxilla or mandible into one file. When using some programs that can't import and work with multiple files, you can merge them into one to work in those programs.
Rule for Folder Name	Define rules for creating a folder name when exporting files. The set rules will be applied to folder naming, and the exported file will be saved in the created folder. The below information can be used to set the rule. • Clinic account Case Creation Date, Scan Date, Order Date, Patient Name, Case Name, Partner Name, Patient ID, Order ID, Assignee • Lab account Case Creation Date, Scan Date, Order Date, Patient Name, Case Name, Partner Name, Order ID, Assignee
Rule for File Name	Define rules for the exported file name. The below information can be used to set the rule. • Clinic account Case Creation Date, Scan Date, Order Date, Patient Name, Case Name, Partner Name, Patient ID, Order ID, Assignee • Lab account Case Creation Date, Scan Date, Order Date, Patient Name, Case Name, Partner Name, Order ID, Assignee

Network – Proxy Server

Set a proxy server for Medit Link using a custom IP address, port number, and optional credentials.

Default App

Manage the programs that support integration in Medit Link.

You can connect the following programs:

	Clinic Account	Lab Account
Scan Program	Medit Scan for Clinics	Medit Scan for Clinics Medit Scan for Labs

Note

Integration of the following services is not available for Medit Link for Mac.

- Medit Scan for Labs

User Experience Program

Enable this option to help improve Medit products by sharing anonymous usage data.

Storage Management – Overview & Cleanup

This section provides a visual breakdown of your cloud and local storage usage. It helps you understand how your storage is being used and which file types consume the most space. Click "Free Up Space" to delete unnecessary files from cloud or local storage within the selected date range.

Settings

Search settings

General

Advanced

Storage Management

Case Converting Tool

Storage Management

Storage Overview & Cleanup

All

My Cloud Storage 2.87 GB / 10.00 TB

Result Files & Attachments 2.87 GB

Local Storage 257.85 GB / 475.95 GB

Raw Data 4.12 KB

Result Files & Attachments 26.76 MB

Others 257.82 GB

Free Up Space

Set Period

Set the specific period to delete data created within the time frame below:

- All: Show all files created since January 1, 2018, regardless of the selected date range.
- Custom Range: Select a start date (no earlier than Jan 1, 2018) and an end date to filter files.
- Today: Files created or modified on the current day only.
- Yesterday: Files created or modified one day before today.
- Last 7 Days: Files created or modified in the past 7 days, including today.
- Last 30 Days: Files created or modified in the past 30 days, including today.
- Current Month: Files created or modified within the current calendar month.
- Previous Month: Files created or modified within the previous calendar month.

Storage Management – Advanced Settings

Folder Path	Specify a folder to store the data results generated in Medit Link and connected programs.
Apps Installation Folder	Specify an installation path for Medit Apps and partner apps available in the App Box.
Auto Backup	Enable this option to automatically back up your scanned data to the cloud without using your cloud storage quota.


Case Converting Tool

You can move cases between different accounts by exporting existing cases of the account or importing cases exported from other accounts.

Export Case

Export cases of the currently logged-in account to the local PC.

1. Open Medit Link and click the Settings icon.

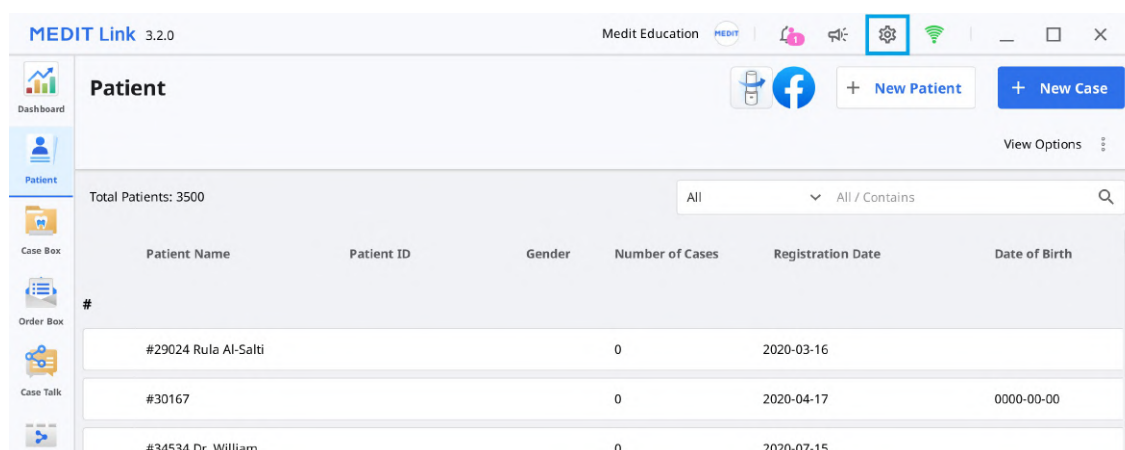

2. Choose the "Case Converting Tool" section in the left-side menu.

Settings

Search settings

General

Advanced

Storage Management

Case Converting Tool

About

User Guide

Case Converting Tool

Case Export

All

All / Contains

Files to Export

Total: 0 / 932

☐	Case Name	Patient Name	Last Modified Date
☐	0511 test's Case	0511 test	2025-07-25 PM 6:02
☐	marginline_의 케이스	marginline_	2025-07-24 PM 2:34
☐	BS_3's Case - Clone	BS_3	2025-07-24 AM 9:14
☐	test1's Case - Clone	test1	2025-07-23 PM 5:25
☐	test1's Case	test1	2025-07-23 PM 5:19
☐	BS_3's Case	BS_3	2025-07-23 PM 5:12
☐	New Case's Case	New Case	2025-07-23 AM 10:0

Export

- In the Case Export section, set a period or search for cases to export if necessary.

All

All / Contains

0 / 767

All

From

To

2018-01-01

2025-02-21

Patient Name	Last Modified Date
	2025-02-07 PM 3:34
	2025-02-07 PM 3:34

- Choose cases you want to export from the list in the "Files to Export" section.

Case Export

All

All / Contains

Files to Export

Total: 932 / 932

☒	Case Name	Patient Name	Last Modified Date
☒	0511 test's Case	0511 test	2025-07-25 PM 6:02
☒	marginline_의 케이스	marginline_	2025-07-24 PM 2:34

- Set the file name by editing the case name in the File Name field.
- Set the destination folder to export the selected cases.
- Click "Export."
- Click "Yes" to open the folder where the file is exported.

Question



Exporting successfully completed.
Do you want to open the folder containing the exported file?


No

Yes


9. The exported cases are saved in .meditex file format.

Import Case

1. Open Medit Link and click the Settings icon.

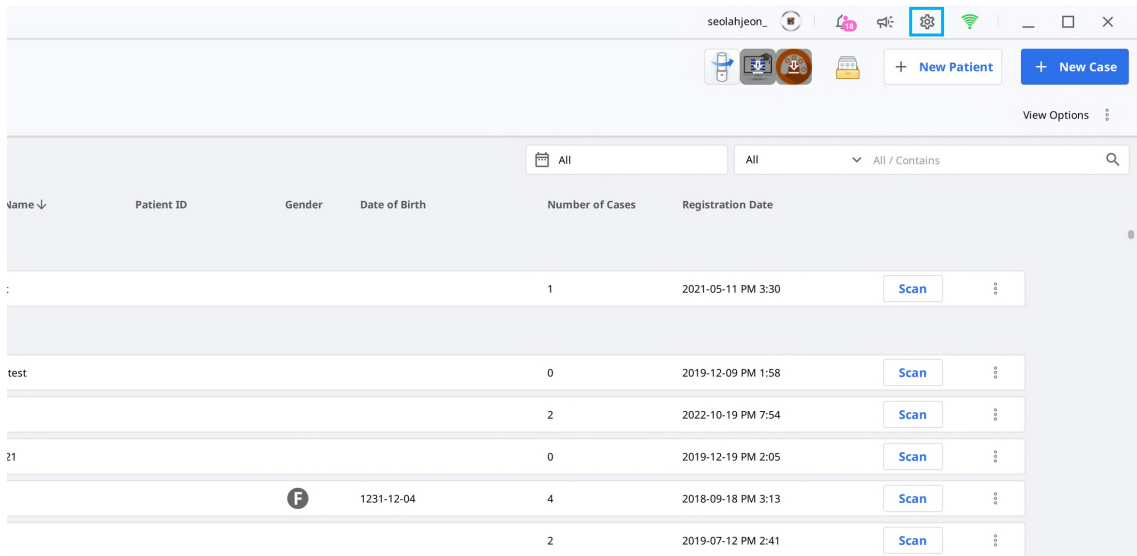

2. Choose the "Case Converting Tool" section in the left-side menu.

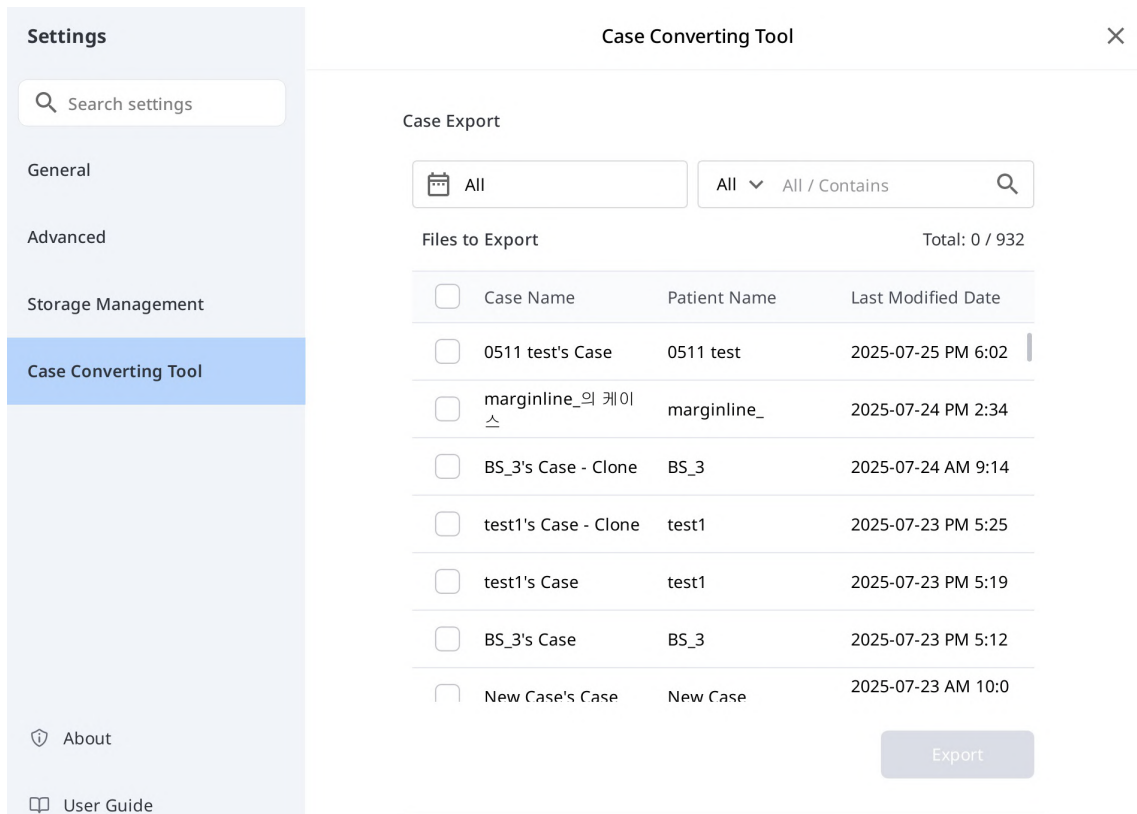

3. In the Case Import section, click the folder icon to browse your PC for the .meditex file you want to import.

Case Import

A new case is created for every imported case. It contains case data, patient information, and scan file. However, it does not contain order information.

C:/Users/medit/Downloads

4. Click the "Import" button and check the files to import in the list.
5. The imported case will also appear in your Case Box (Clinic account) or Work Box (Lab account).

About

Show detailed information about Medit Link.

About

MEDIT Link

Product Name	Medit Link
Version	3.4.5
Build Number	367
Server	Stage Korea
UDI DI	(01)8800026700449
UDI PI	(10)3.4.5

Medit Corp.

품목명	1등급 치과 영상 전송장치소프트웨어
모델명	ML-STD001
품목별제조신고번호	제신 25-112호
사용방법 및 사용시 주의사항	사용설명서 참조
부작용 보고 관련 문의처	한국의료기기안전정보원, 080-080-4183
제조자의 상호:	(주)메디트
제조자의 주소:	서울특별시 영등포구 양평로 25길 8 9층, 10층
제조자의 연락처:	+82-2-2193-9600

본 제품은 「디지털의료기기소프트웨어」임

 2025.03

[Terms & Conditions](#)

[Open Source License](#)

☒ Send Anonymous Usage Statistics

Close

User Guide


Get the user guide for Medit Link.

Opening the user guide requires an Internet connection.

Menu

Dashboard

You can view the statistics for the entire work done on Medit Link in visualized graphs and charts – both in the Medit Link App and Web.

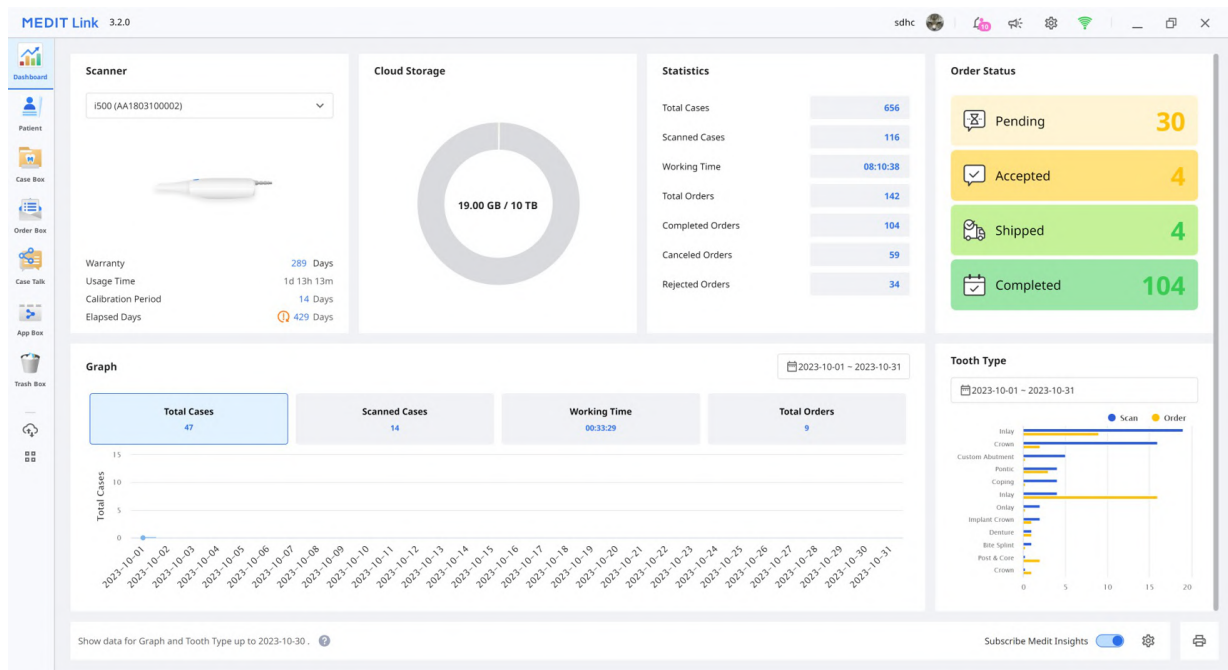

Note

Please refer to the [Medit Link Web > Dashboard](#) for more information on the dashboard and Medit Insights.

Patient

You can add, edit, delete, or merge patients on the Patient menu.

Patient Name ↓	Patient ID	Date of Birth	Gender	Number of Cases	Last Scanned Date	Last Modified Date	Registration Date
aggsfgsdg				1		2020-09-24 PM 5:03	Scan
addTEST				1		2021-06-17 AM 11:54	Scan
aaaa				1		2023-04-22 PM 4:25	Scan
aaa				1		2023-04-22 PM 5:08	Scan
aa				1		2023-03-06 PM 6:32	Scan
a				1		2023-05-31 PM 2:07	Scan
Arch test				1		2020-08-17 PM 12:26	Scan
Arch Order Test				2		2020-09-01 PM 2:17	Scan
Abutment Scan				1	2022-04-04 PM 2:23	2022-04-04 PM 2:15	Scan
Abutment Cut test				1	2019-12-05 PM 6:03	2019-12-05 PM 5:59	Scan
AAA				1		2023-08-16 PM 4:06	Scan

Add New Patient

1. Click "New Patient."

New Patient

Patient Name*

Gender

☐ Female ☐ Male

Date of Birth

YYYY mm dd

Patient ID

Phone Number

Email Address

Memo

Cancel Save


2. Fill in the required information in the New Patient dialog, then click "Save."

Delete Patient

Note

You can delete a patient only when there are no cases for the patient.

1. Select the patient to delete from the list.

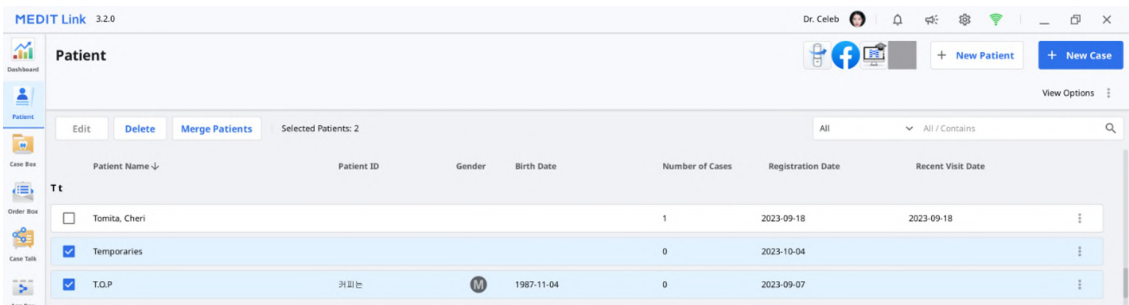


2. Click "Delete."

Merge Patients

You can merge all cases of multiple patients into one patient.

1. Select the patients you want to merge from the list.

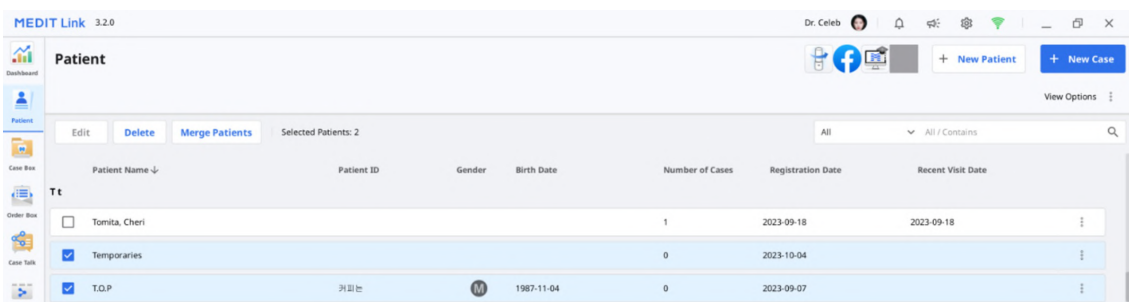

2. Click "Merge Patients."
3. Select the target patient you would like to merge all cases into.
4. All cases are merged into the target patient and the patient information of the target patient maintains.

Merge all patient information to


Please select the patient you'd like to merge all data to.
After merging all data together, all information in other patient files will be deleted.

	Patient Name	Patient ID	Gender	Birth Date	Number of Cases	Date Registered	Recent Visit Date
☒	5-9_iScan_CAMOrd...				1	2020-09-18	2020-09-18
☐	test				2	2020-09-17	2020-09-17
☐	test				22	2019-07-26	2020-09-17

Cancel
Apply

5. Click "Apply" to proceed with the merge.

Case Box/Work Box

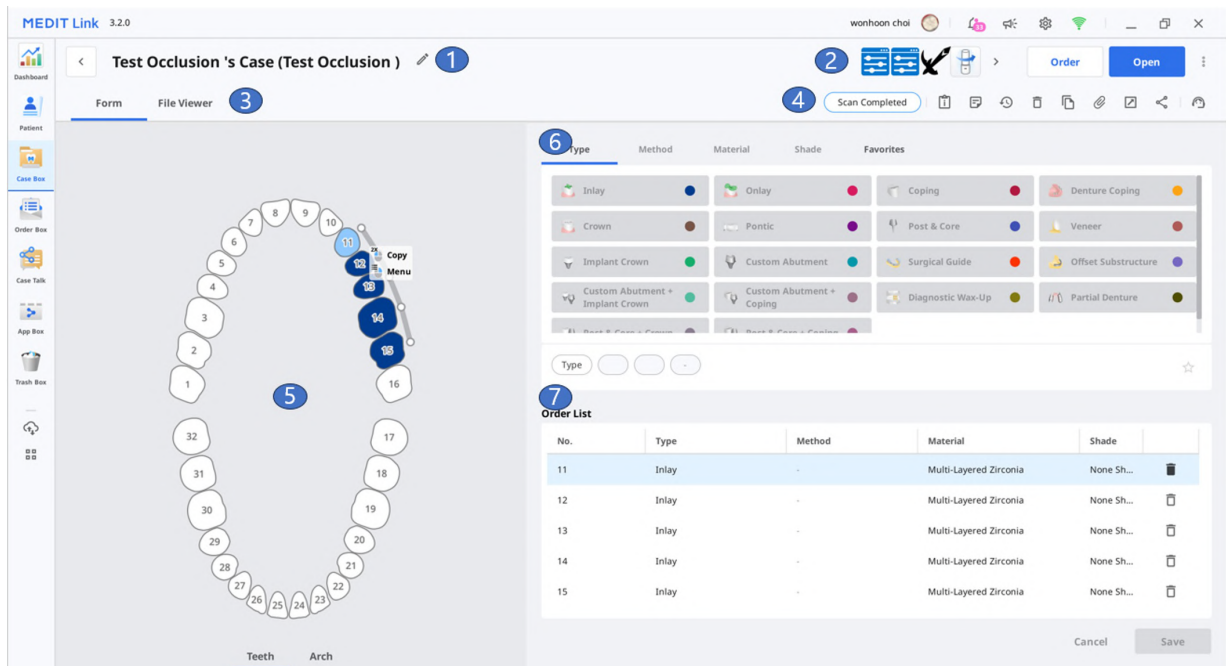

1	Show case name and patient name.
2	Provide key features available: Scan or Open button, Order button, app icons.
3	Show available tabs: Form, File Viewer.
4	Show functions for managing cases and files.
5	Show the form registration UI for the selected scanning program.
6	Provide the type/method/material/shade information for users to choose for the selected tooth number.
7	Show the type/method/material/shade information applied to the selected teeth.

Tabs in Case Box/Work Box

Form	File Viewer
The form tab provides a UI for entering basic information for scanning work, such as type, method, material for teeth numbers, and scan options. The information you enter is passed to the connected scanning program and used as order information when placing an order with your partner.	View result files generated by scanning programs and CAD. The file viewer displays and manages the acquired or attached 3D models and 2D images.

Functions in Case Box/Work Box

The following functions are provided in Case Box/Work Box.

	Case Status	Shows the status of the case.								
	Go to Order Box/In Box	Provides a shortcut for the same case in another box (Case Box ↔ Order Box; In Box ↔ Work Box).								
	Case Information	Review case details and save them as a PDF or print them out.								
	Memo & Tags	Add a memo or tags when registering or ordering a case. Use them to communicate any additional or supplementary information about the case to your partners.								
	Case History	Provides detailed information about the case history. History <table><tr><td>Last Modified Date</td><td>3/29/2022 1:57 PM</td></tr><tr><td>Scanning Date</td><td>4/12/2021 2:06 PM</td></tr><tr><td>Creation Date</td><td>3/22/2022 1:17 PM</td></tr></table> <tr><td>Case Size</td><td>435MB</td></tr>	Last Modified Date	3/29/2022 1:57 PM	Scanning Date	4/12/2021 2:06 PM	Creation Date	3/22/2022 1:17 PM	Case Size	435MB
Last Modified Date	3/29/2022 1:57 PM									
Scanning Date	4/12/2021 2:06 PM									
Creation Date	3/22/2022 1:17 PM									
Case Size	435MB									

[Close](#)

	Delete	Delete the case. You cannot delete ordered cases or cases that were created offline.
	Clone	Create a duplicate of the current case in Case Box. All data will be copied to the cloned case. This feature helps replicate treatment information for the same patient or make an additional order with the same information or files.
	Attach	Attach files in various formats.
	Export	Export scan or CAD result files to the local PC. You can choose your preferred file format when exporting files. The exported files can be used in CAD and CAM software for further work.
	Share	Create a link for sharing. Users with the shared link can access the 3D data in a browser such as Chrome, Firefox, or Safari without installing additional programs. You can share the link freely with anybody or specific people only. A PIN code or a link expiration date can be set to secure private information.
	Submit Support Request	Land on a Medit Help Center page to submit the support request.

Order Box/In Box

The screenshot shows the MEDIT Link web application. The top header includes the user's name 'seolahjeon_' and the language 'English'. The sidebar on the left contains navigation icons for Dashboard, Patient, Case Box, Order Box, Case Talk, Partners, Membership, and My Info. The main content area is titled '오프라인의 케이스 - Clone (오프라인)' and has a 'Cancel Order' button. The 'Order Box' section is active, showing a form with fields for Order ID, Patient Name, Scanned Date, Requested Delivery Date, Partner Name, Order Date, Expected Delivery Date, Memo, and Tags. A table titled 'Order List' displays the following data:

No.	Type	Method	Material	Shade
23	Crown	Anatomic	Multi-Layered Zi...	None

1	Show case name and patient name.
2	Provide functions for order management (accepting/canceling; delivering/receiving).
3	Show available tabs: Order, File Viewer
4	Show functions for managing cases and files: case status, shortcuts for detailed information, case information, memo & tags, etc.
5	Show the detailed information for the order including the order number, patient name, partner name, scanning date, order date, delivery date, and memo & tags.
6	Show the ordered product information and delivery status.
7	Show the assignee for the case.


Tabs in Order Box/In Box

Order	File Viewer
Check the detailed information related to the order, such as order number, assignee, patient name, account name, scanning date, order date, delivery date, memo, and tags.	View files generated by scan/CAD operations. The file viewer displays and manages the acquired or attached 3D models and 2D images.


Case Talk

Cases created in the Medit Link can be shared with specific people, anyone with the shared link or registered members of Medit Link.

The user can see a list of all shared cases through Case Talk in the Medit Link App and Medit Link Web.

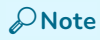


Case Talk service requires an internet connection.

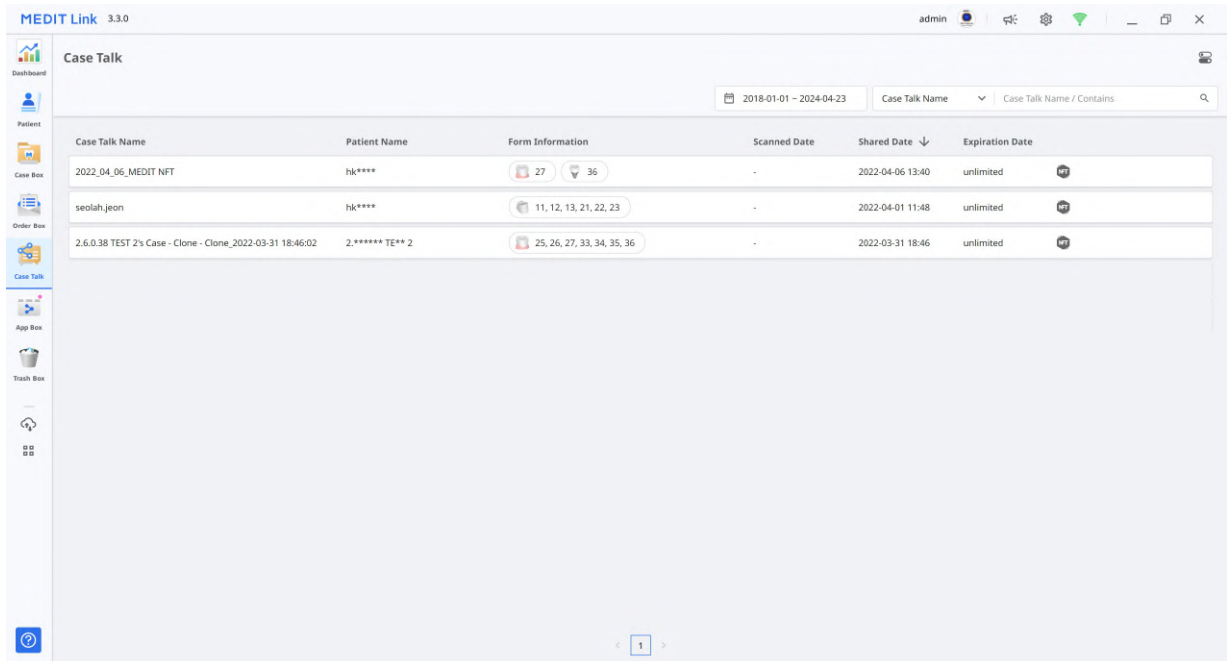

Shared cases can be viewed via a web browser without installing a separate program, enabling easy communication over the 3D data on various devices.

App Box

App Box provides various add-in programs that complement your basic scanning workflow and enhance productivity. You can download and install the provided apps to create dental appliances, analyze scan data, assist you during a consultation, or streamline your work process.

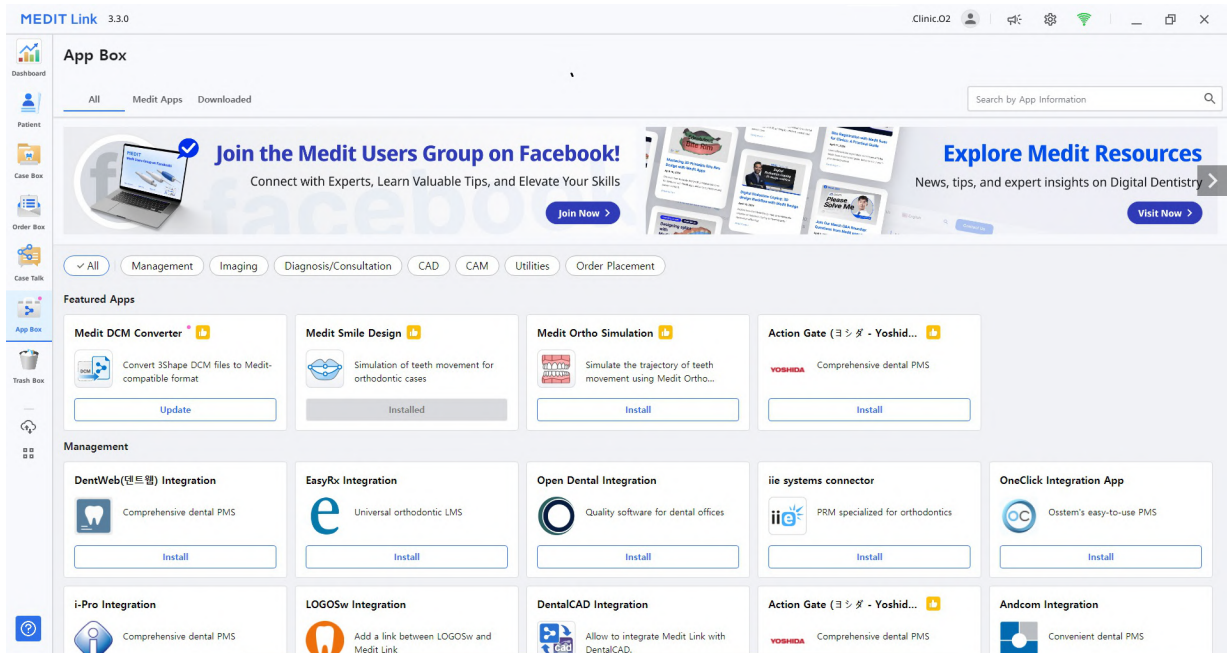


The "All" tab shows the list of all provided apps including Medit-developed original apps (Medit Apps) and partner apps, which are integration apps with Medit's partners from different sectors of digital dentistry, such as 3D printing, CAD/CAM software, etc.

Medit Apps

The "Medit Apps" tab shows only the list of Medit-developed original apps.

Downloaded

The "Downloaded" tab shows the list of currently installed apps on your PC and apps that have a history of being downloaded by the user account.

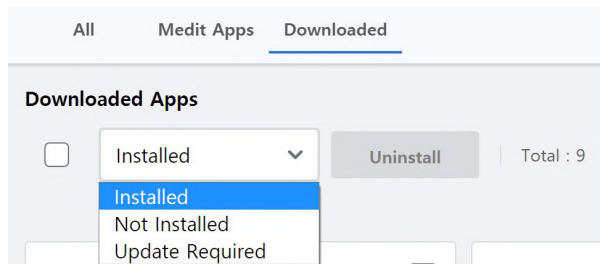

- **Installed:** The option shows only apps that are installed on the PC the user is currently signed in to.
- **Not Installed:** This option lets you see apps you've installed on other PCs so you can quickly install them.
- **Update Required:** This option only shows apps installed on the current PC that need to be updated.

Trash Box

The Trash Box temporarily stores the deleted cases. Any case in the Trash Box can be either restored or permanently deleted.

Caution

The data deleted from the Trash Box will be permanently deleted from your PC and the server and cannot be recovered.

MEDIT Link 3.2.0 wonhoon choi

Trash Box

☐ Restore ☐ Selected Cases: 0 All / Contains

	Status	Case Name	Patient Name	Lab Name	Deletion Date ↓	Deleted By	Size
☐	Scan Completed	Jira_임플란트 46, 47's Case - Clone - Clone	임플란트 46, 47		2023-04-26 오전 9:12	wonhoon choi	66,654 KB
☐	Form Only	G11598492304180810429	test test		2023-04-18 오후 5:15	wonhoon choi	0 KB
☐	Form Only	G11598492304180806313	test test		2023-04-18 오후 5:15	wonhoon choi	0 KB
☐	Form Only	G11598492304180811349	tes test		2023-04-18 오후 5:15	wonhoon choi	0 KB
☐	Form Only	G11598492304180754083	Glidewell_fastdesign		2023-04-18 오후 5:00	wonhoon choi	0 KB
☐	Form Only	G11598492304180733332	Glidewell_fastdesign		2023-04-18 오후 5:00	wonhoon choi	0 KB
☐	Form Only	G11598492304180648146	test test		2023-04-18 오후 5:00	wonhoon choi	0 KB
☐	Case Completed	Multi Occlusion Normal Case's Case - Clone	Multi Occlusion Normal Case		2023-04-18 오전 10:50	wonhoon choi	23,652 KB
☐	CAD	Multi Occlusion Normal Case's Case - Clone - Clone	Multi Occlusion Normal Case		2023-04-18 오전 10:50	wonhoon choi	23,652 KB
☐	Form Only	Occlusion Test's Case	Occlusion Test		2023-04-17 오후 6:31	wonhoon choi	0 KB
☐	Form Only	Test Occlusion 's Case	Test Occlusion		2023-04-17 오후 6:31	wonhoon choi	0 KB
☐	Form Only	Occlusion Test's Case	Occlusion Test		2023-04-17 오후 6:31	wonhoon choi	0 KB
☐	Form Only	Occlusion Test's Case	Occlusion Test		2023-04-17 오후 6:26	wonhoon choi	0 KB

Case Synchronization Manager

The Case Synchronization Manager helps you to upload and download cases efficiently. You can upload multiple cases to the cloud or download them to your local PC. To identify cases requiring synchronization, simply search for them or apply date filters.

Case Synchronization Manager

Upload
Download

All

Case Name

Case Name / Contains

Total Cases: 210

☒ No Date
 Data Type

All

☐ Case Name	Patient Name	Scanned Date↓	Size	Status
☐ BLATRIX Charline's Case	BLATRIX Charline	2025-01-06 PM 6:28	729.30 MB	Upload Required
☐ BLATRIX Charline's Case	BLATRIX Charline	2025-01-06 PM 6:28	729.30 MB	Upload Required
☐ BLATRIX Charline's Case - 복제	BLATRIX Charline	2025-01-06 PM 6:28	733.35 MB	Upload Required
☐ BLATRIX Charline's Case - Clone	BLATRIX Charline	2025-01-06 PM 6:28	734.01 MB	Upload Required
☐ test의 케이스	Patient TEST	2023-04-26 PM 7:58	1.25 GB	Upload Required
☐ i700 HD's Case	i700 HD	2023-04-24 PM 5:23	911.24 MB	Upload Required
☐ Mandibular Movement	MEDIT	2023-04-24 PM 1:59	1.88 GB	Upload Required
☐ Mandibular Movement Case	MEDIT	2023-04-24 PM 1:59	1.75 GB	Upload Required
☐ Mandibular Movement Case - Clo...	MEDIT	2023-04-24 PM 1:59	1.75 GB	Upload Required
☐ 서라_Mandibular Movement_TEST	Mandibular Movement_export_test	2023-04-10 PM 3:39	34.73 MB	Upload Required
☐ 서라_Mandibular Movement_TEST...	Mandibular Movement_export_test	2023-04-10 PM 3:39	731.43 MB	Upload Required

Close

Upload

 Upload	Upload the selected cases.
 Cancel	Cancel the upload for selected cases.
 Retry	Retry uploading cases if the upload fails.

Download


 Download	Download the selected cases.
 Cancel	Cancel the download for selected cases.
 Retry	Retry downloading cases if the download fails.

Related Websites

You can go directly to menus only available on Medit Link Web, such as My Info, Partners, and Membership. Please refer to Medit Link > Medit Link Web > [Partners](#), [Membership](#), and [My Info](#) for detailed descriptions of the Medit Link Web menus.

You can access Medit's online store, Medit Shop, where you can purchase extended warranties for scanners and other Medit goods, such as scanner backpacks.

Also, you can get various information about using Medit Link. Access Medit's educational and informational resources, such as Medit Help Center, Medit Academy YouTube channel, and Medit Academy online education platform.

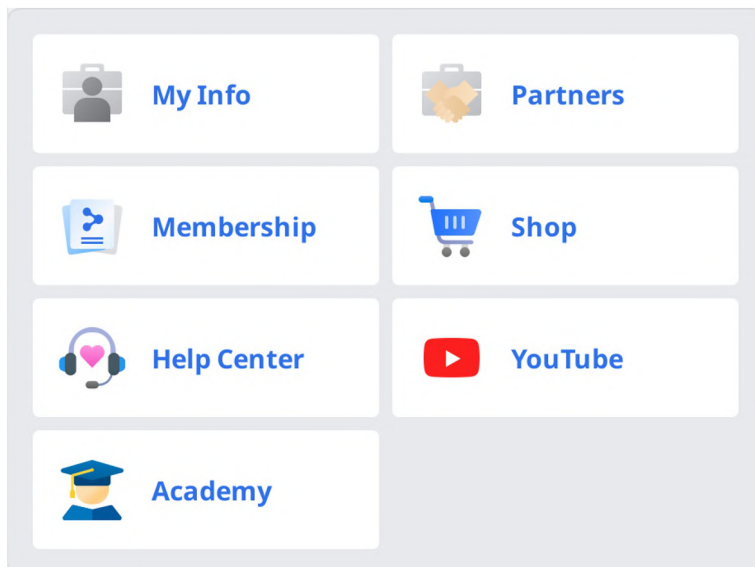

File Viewer

File Viewer groups and shows various data, including the data acquired from programs linked to Medit Link and data attached by users.


It provides basic management functions such as showing/hiding data, adjusting opacity, and deleting data.

It supports previews of files in the following formats:

- 3D data: STL, OBJ, PLY
- 2D images: PNG, JPG, JPEG, BMP, GIF, TIF, TIFF
- Video files: MKV, MP4, AVI, WMV

Data Tree

You can group and display all data for the case in the data tree. You can also show/hide data, adjust transparency, and delete data from the data tree.

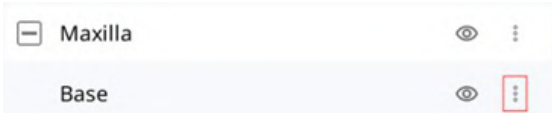
	Show/Hide	You can show or hide the data.
	Opacity	The slide bar appears when you click one of the data on the data tree. You can adjust the opacity of the shown data by moving the slider bar. 
	More	Clicking the More icon shows additional features available.  • Show This Only: Hide all other data but display only the corresponding data. • Hide: Hide the data. • Duplicate: Copy the data and add it to the attachment group. • Export: Export the selected files. • Rename: Change the name of the data. • Delete: Delete the data.



Note

You can rename data when online.

Model Control Tools

3D Model Controls

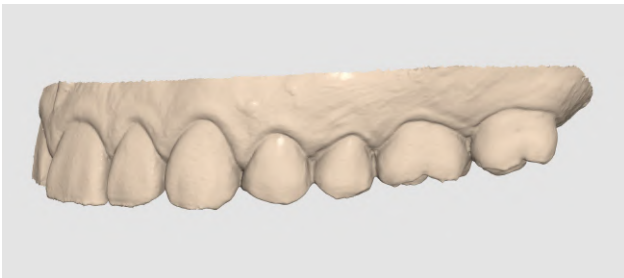	Pan	Move the model.
	Rotate	Rotate the model.
	Zoom In/Out	Zoom in and out on the model.
	Zoom Fit	Position the model in the center of the screen.
	Data Display Mode	See the data in color with glossy finish (PBR rendering). 
		See the data in color with matte finish (Phong rendering). 
		See the data in one color. 

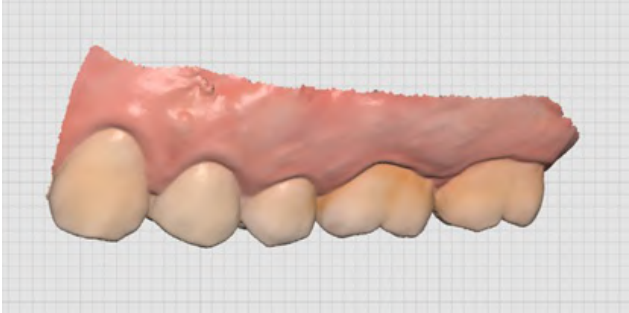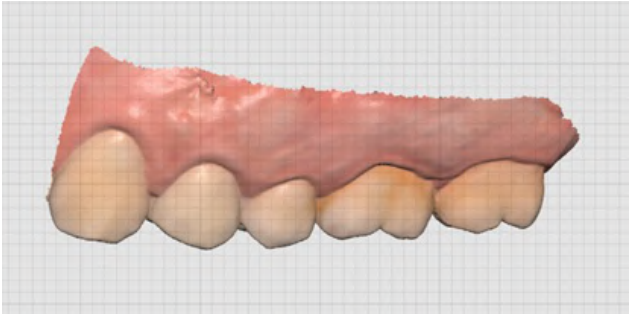	Grid Settings (mm)	Grid On	Shows the grid in the background. 
		Grid Off	Hide the grid in the background.
		Overlay On	Overlay the grid over the model. 

2D Image Control


	Next/Previous Image	Move among images.
	Edit	Edit the image.
 	Pinch/Paste Window	You can detach the image window and adjust it to the desired position. You can resize the window. When the window is restored to its original location, it returns to its default size.

Function Tools

Image Capture and Memo


Capture the current screen as a 2D image, draw on it, or add notes.


You can save it as a file with your notes and send it as a reference.

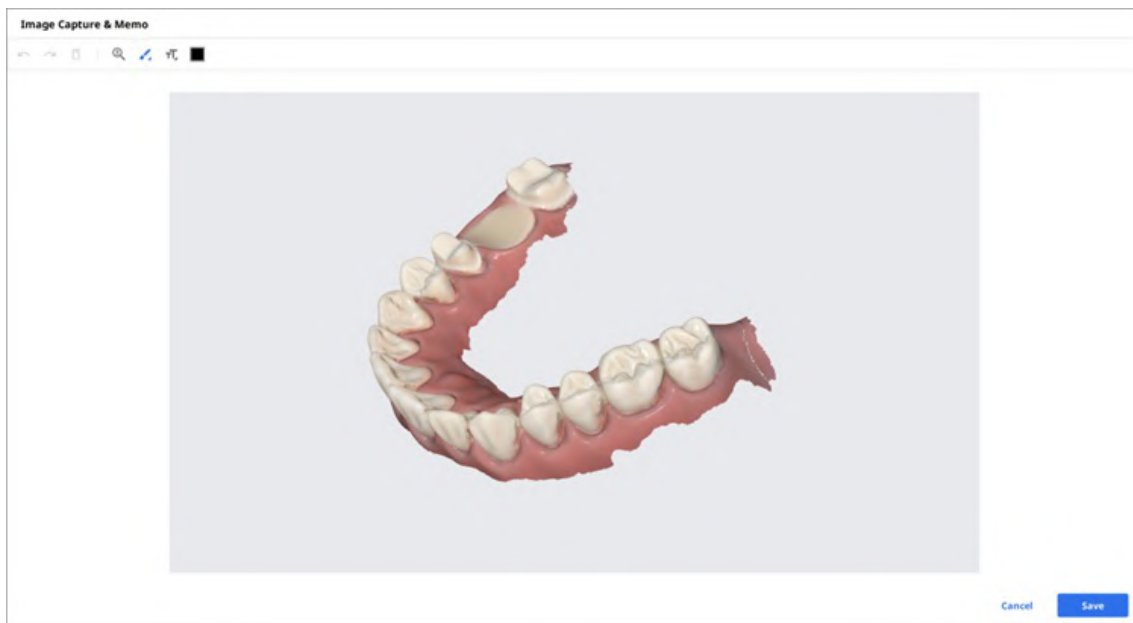

Ortho Capture

Capture the images in the current viewing direction and top/bottom/left/right from the current viewing direction.

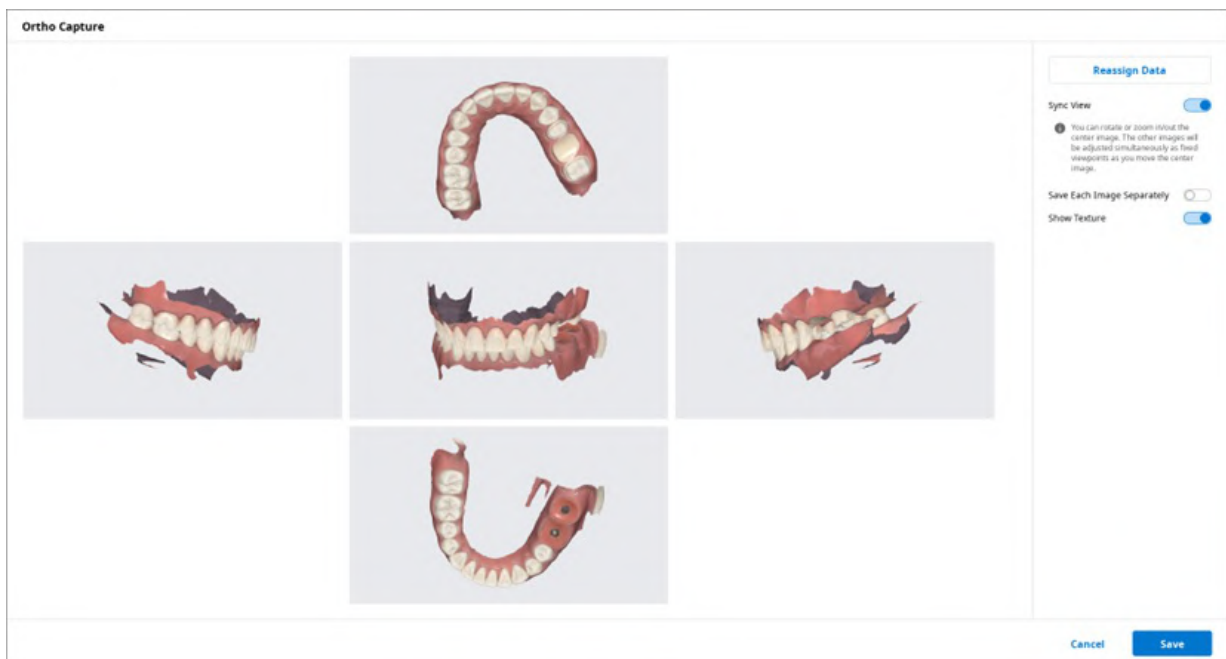

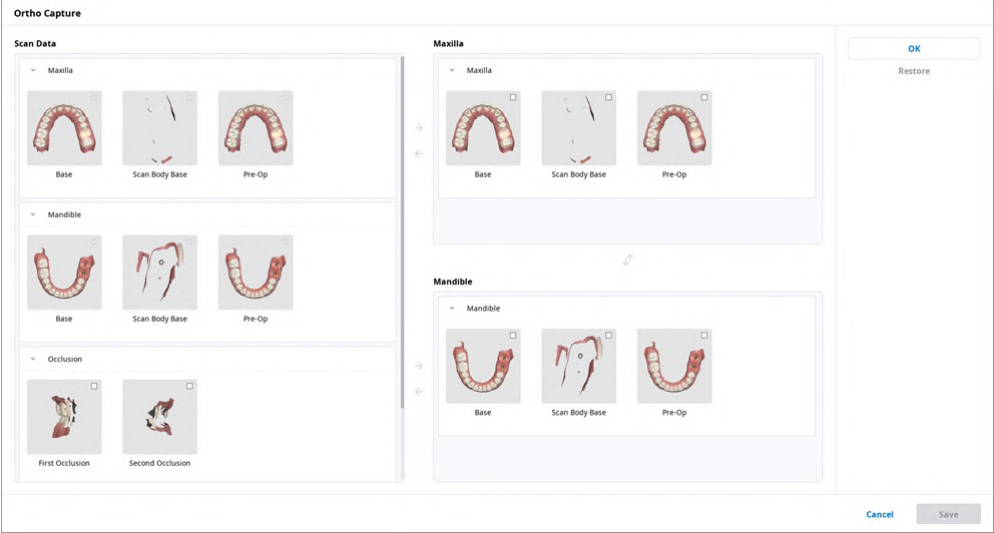Sync View	When on, you can rotate or zoom in/out the center image. The other images will be adjusted simultaneously as fixed viewpoints as you move the center image. When off, you can pan, rotate, or zoom in/out all images individually.
Save Each Image Separately	Save each image in high resolution.
Show Texture	Turn on/off the texture for the model.
Reassign Data	Change the data for maxilla/mandible with the Reassign Data feature. You can use the changed data to create and utilize the result images of Ortho Capture.  • Scan Data: Scan data for the case are all shown. • Maxilla: The data you selected for the maxilla are displayed. • Mandible: The data you selected for the mandible are displayed.


Fill Holes

Fill holes in 3D mesh data.



Trimming


Trimming provides various tools to edit data.

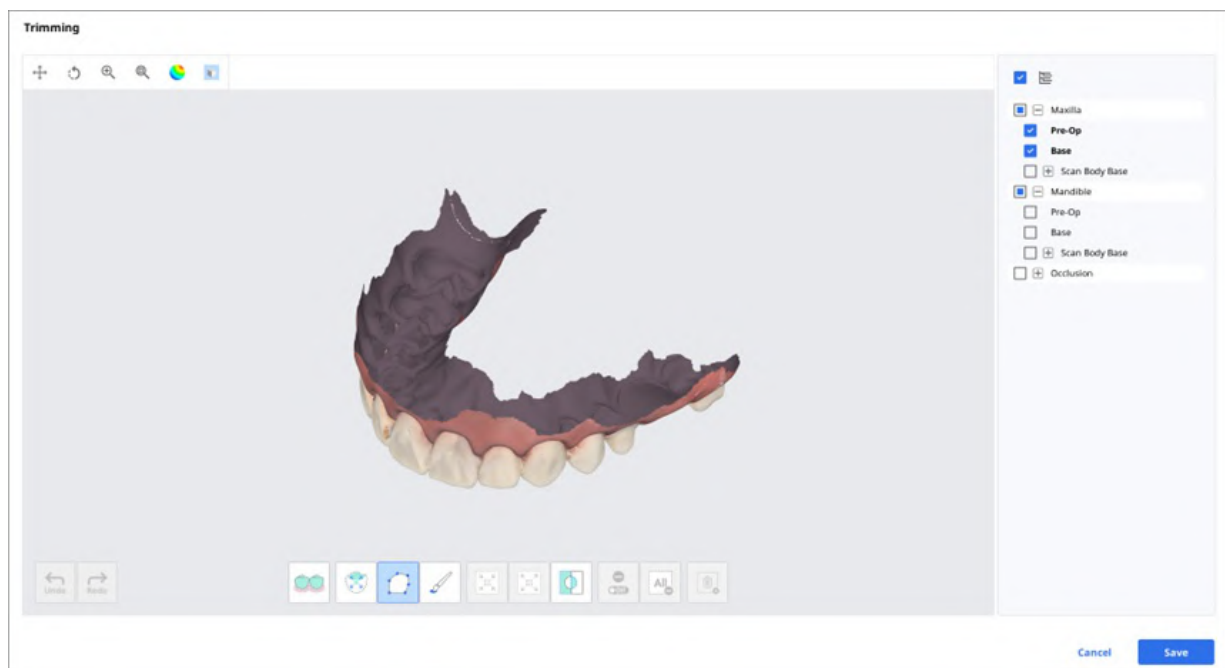

The selected 3D mesh data can be edited using different tools located at the bottom of the window.

Selection		Smart Teeth Selection	Automatically select all teeth of the arch, leaving out gingiva parts.  Note This function is only available for the scan data acquired using Medit Scan for Clinics with the "Use GPU" option on.
		Smart Single Tooth Selection	Automatically select the area of a single tooth with a click. You can click or drag on the tooth.
		Polyline Selection	Select all entities within a polyline shape drawn on the screen.
		Brush Selection	Select all entities on a freehand-drawn path on the screen. Only the front face will be selected. The brush comes in three different sizes.
Adjustment of Selected Area		Shrink Selected Area	Reduce the selected area each time you press the button.
		Expand Selected Area	Expand the selected area each time you press the button.
		Invert Selected Area	Invert the selection.
Deselection		Selection/Deselection	When on, deselect the area using various tools.
		Clear All Selection	Clear all selected areas.
		Delete Selected Area	Delete the data from the selected area.

1. Select the area you want to trim with the area selection tools.

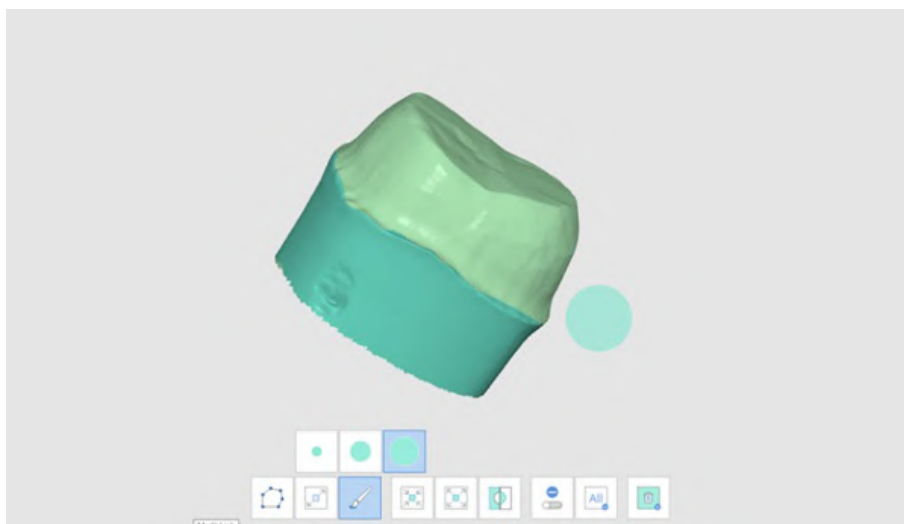


Note


The polyline trimming tool selects all entities within a polyline shape drawn on the screen, while the brush trimming tool selects only the front face data.

2. You can invert, shrink, or expand the selection with the following three tools.


- Invert Selected Area

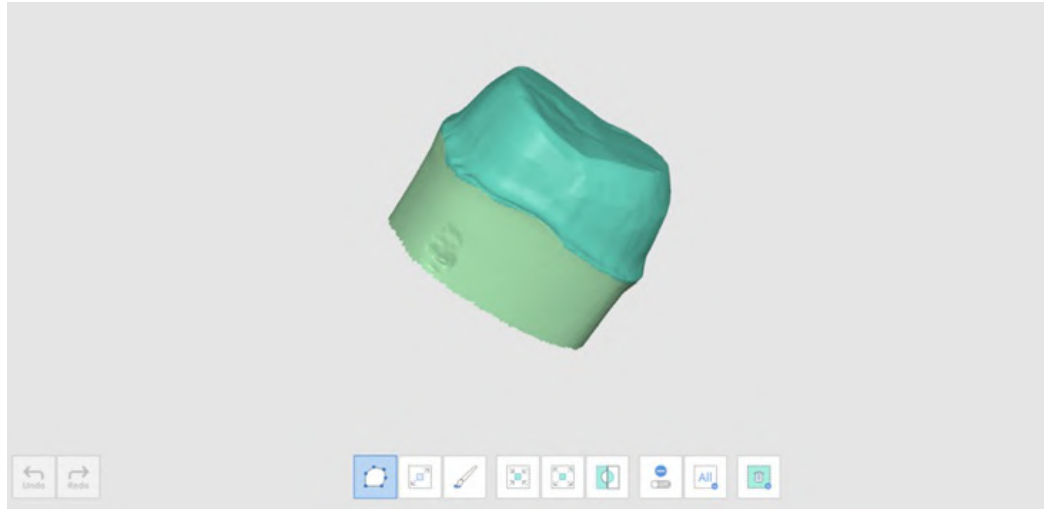

- Shrink Selected Area

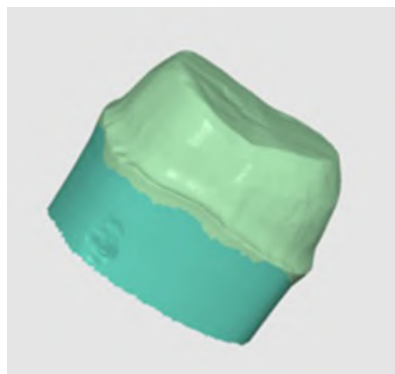


- Expand Selected Area

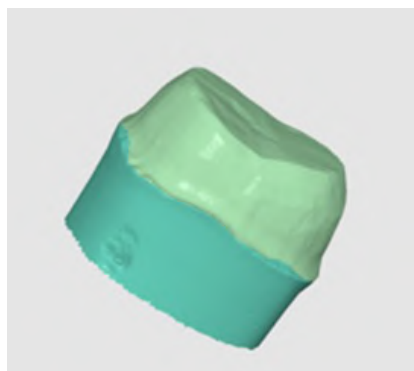






3. You can deselect the selected areas with the Delete Selected Area tool.

Optimize Occlusion Alignment

Optimize the occlusion alignment. You can choose the target data and adjust the occlusion alignment.

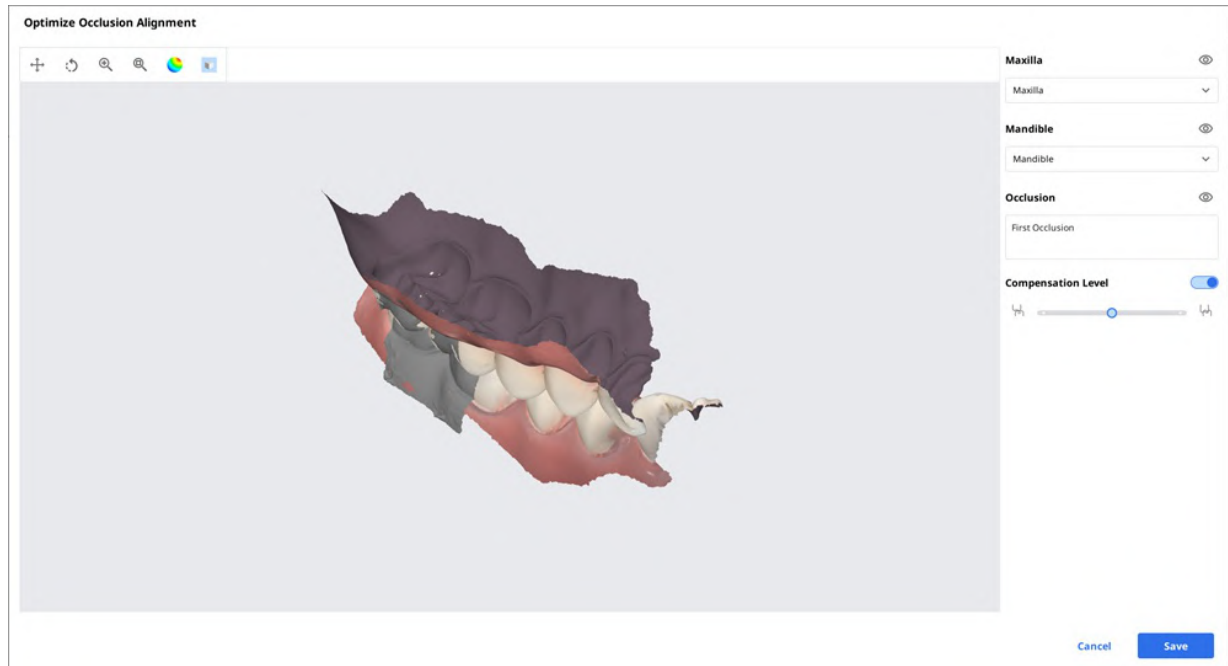


Compensation Level

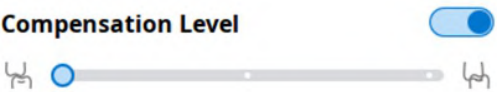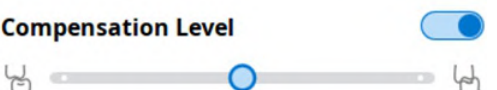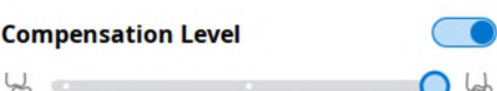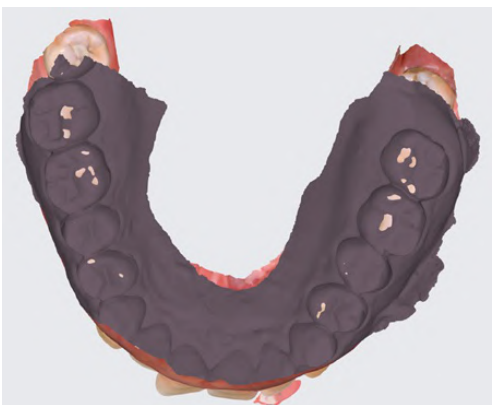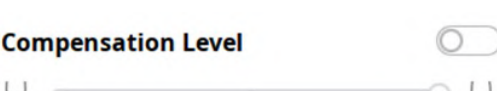Low	Compensation Level 	
Mid	Compensation Level 	
High	Compensation Level 	
Off	Compensation Level 	

Mandibular Movement

You can play the mandibular movements recorded in Medit Scan for Clinics to see the dynamic occlusal contacts between the upper and lower dentitions.

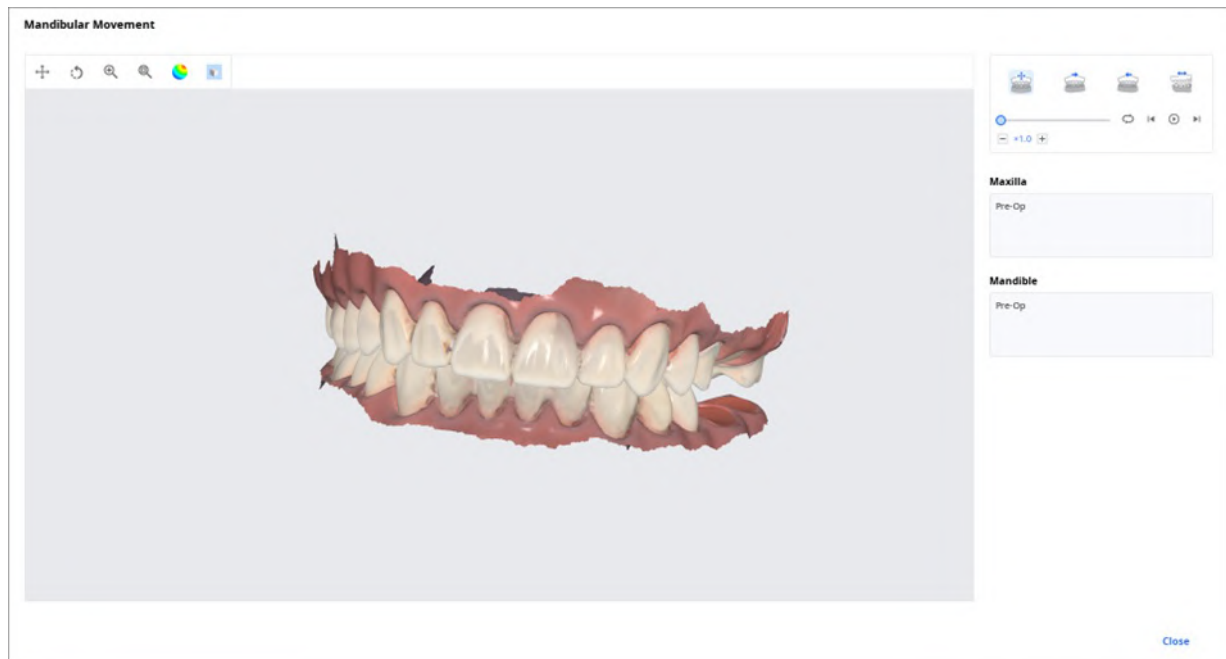

	Free Movement	Play the recording of free mandibular movement.
	Left Lateral Movement	Play the recording of the mandible moving to the left.
	Right Lateral Movement	Play the recording of the mandible moving to the right.
	Protrusive Movement	Play the recording of the mandible moving forward.

Manage Multi Occlusion Data in File Viewer

You can get multiple results of occlusion data from Medit Scan Application.

Medit Link provides the function to manage the results through the "File Viewer."

How to manage multiple results


When the user gets multiple occlusion results from Medit Scan for Clinics, Medit Link provides the option to select a result.

The list of results is listed by name, and you can select one item.

The selected model will be shown in the "Data Tree" and "Model Display Area."



You can collapse the list and change the item using the arrow icons.

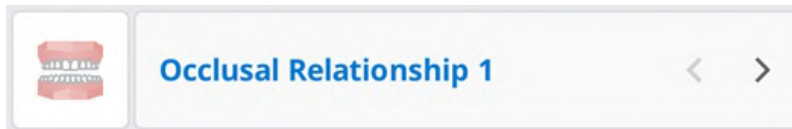

View Options

You can change the View Options to customize the layouts of the case list in the Patient/Case Box/Work Box/In Box/Order Box.

View Options

 Status Filters	2 in use >
 Table Items	8 in use >
 View Type	List View >
 Grouping	None >
 Sorting	Last Modified Date >

Status Filters

Show or hide each filter representing the status of the case. The hidden status filters are not displayed on the screen in the Case Box/Work Box/In Box/Order Box.

You can easily find the case you need with status filters.

- You can click the status you want to view to display only cases in that state.
- You can select multiple statuses at a time.

For each box, the following filtering options are available:

Box	Status Filters
Case Box	All / Form Only / Need Processing / Scan Completed / CAD / Ordered / Case Completed
Order Box	All / Pending / Canceled / Rejected / Accepted / Shipped / Case Completed
In Box	All / Pending / Canceled / Rejected / Accepted / Ready / Shipped / Case Completed
Work Box	All / Form Only / Need Processing / Scan Completed / CAD / Done

Table Items

Select the items to display in the case list in the Patient/Case Box/Work Box/In Box/Order Box. You can change the order of table items or adjust the table column width in the list view to best optimize for the user environment.

For each box, the following table items are available:

Box	Table Items
Patient	Patient Name / Patient ID / Gender / Number of Cases / Registration Date / Date of Birth / Last Scanned Date
Case Box	Status / Case Name / Date of Birth / Form Information / Last Modified Date / Scanned Date / Order Date / Lab Name
Order Box	Status / Case Name / Patient Name / Scanned Date / Order Date / Lab Name / Requested Delivery Date / Last Modified Date
In Box	Status / Case Name / Order Date / Requested Delivery Date / Last Modified Date / Clinic Name
Work Box	Status / Case Name / Patient Name / Form Information / Last Modified Date / Requested Delivery Date / Scanned Date / Clinic Name / Order Date

View Type

Change the view type between the Gallery View and List View on the Case Box.

For each box, the following view type options are available:

Box	View Type Options
Patient	List View
Case Box	List View / Gallery View
Order Box	List View
In Box	List View
Work Box	List View

Grouping

Select a grouping option to classify and rearrange cases in the list by the selected option.

For each box, the following grouping options are available:

Box	Grouping Options
Case Box	None / Patient Name / Scanned Date / Order Date
Order Box	None / Scanned Date / Order Date / Requested Delivery Date
In Box	None / Order Date / Requested Delivery Date
Work Box	None / Requested Delivery Date

Note

The grouping options are unavailable in the Patient, Case Talk, and Trash Box.

Sorting

Select a sorting option to change the order of cases in the list according to the selected option.

For each box, the following sorting options are available:

Box	Grouping	Sorting Options
Patient	-	Patient Name / Patient ID / Date of Birth / Registration Date / Number of Cases / Last Scanned Date
Case Box	None	Patient Name / Last Modified Date / Scanned Date/ Order Date / Lab Name
	Patient Name	Last Modified Date / Scanned Date / Order Date / Lab Name
	Scan Date	Patient Name / Last Modified Date / Order Date / Lab Name
	Order Date	Patient Name / Last Modified Date / Scanned Date / Lab Name
Order Box	None	Patient Name / Scanned Date / Order Date/ Requested Delivery Date / Lab Name
	Scan Date	Patient Name / Order Date / Requested Delivery Date / Lab Name
	Order Date	Patient Name / Scanned Date / Requested Delivery Date / Lab Name
	Requested Delivery Date	Patient Name / Scanned Date / Order Date / Lab Name
In Box	None	Patient Name / Order Date / Requested Delivery Date / Clinic Name
	Order Date	Patient Name / Scanned Date / Requested Delivery Date / Clinic Name
	Requested Delivery Date	Patient Name / Order Date / Clinic Name
Work Box	None	Patient Name / Last Modified Date / Requested Delivery Date / Clinic Name
	Requested Delivery Date	Patient Name / Last Modified Date / Clinic Name

Note

The sorting options are unavailable in the Case Talk and Trash Box.

Filtering and Search

Filter by Patient

Filter and display cases by patient name. You can create a list by filtering only the patients you need to work on.

MEDIT Link

 **Case Box**

Patient

✓ All

Form Only

Need Processing

Scan Completed

CAD

Ordered

Case Completed

1. Enter a patient name.

Patient

won|

☐ wonhoonchoi-test1 / 1979-11-11 / Female / -

☐ wonhoonchoi / - / - / -

☐ wonhoonchoi / - / - / -

☐ wonhoon.choi / - / - / -

☐ won / - / - / -

2. Select the patients among the search results below.

wonhoonchoi / - / - / -

+ 2 ^

won|

☐ wonhoonchoi-test1 / 1979-11-11 / Female / -

☐ wonhoonchoi / - / - / -

☐ won / - / - / -

☐ wonhoon / 1979-05-05 / - / 1234

☐ won / - / - / -

Selected Patients

Clear Selection

☒ wonhoonchoi / - / - / -

☒ wonhoon.choi / - / - / -

☒ WonHoonChoi / - / - / -

3. Only cases of the selected patients are displayed on the list.

MEDIT Link 3.2.0

Wonhoon Choi_clinic

Case Box

wonhoonchoi / - / - / -

+ 2 ^

✓ All

Form Only

Need Processing

Scan Completed

CAD

Ordered

Case Completed

2018-01-01-2023-10-24

All / Contains

Total Cases: 20

Status	Patient Name	Case Name	Form Information	Last Modified Date ↓	Scan Date	Lab Name	Order Date
Form Only	wonhoonchoi	originalCase	-	2023-10-05 오전 8:10			
Form Only	wonhoonchoi	wonhoonchoi's Case	-	2023-10-05 오전 8:09			
Scan Completed	wonhoonchoi	FileViwerTest_1	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-06-15 오전 8:14	2023-02-09 오전 8:35		
Scan Completed	wonhoonchoi	FileViwerTest_1 - Clone	2-Inlay / 3-Inlay / 4-Inlay / 5-Inlay / 6-Inlay / 8...	2023-05-25 오후 2:05	2023-02-09 오전 8:35		
Form Only	wonhoonchoi	FileViwerTest_1 - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-03-15 오후 3:06			
Scan Completed	wonhoonchoi	FileViwerTest_1 - Clone - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-03-09 오후 4:41	2023-02-09 오전 8:35		
Need Processing	wonhoonchoi	FileViwerTest_1 - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-02-14 오전 8:13	2023-02-09 오전 8:35		
Scan Completed	wonhoonchoi	FileViwerTest_1 - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-02-10 오전 11:27	2023-02-09 오전 8:35		
Form Only	wonhoonchoi	testsamplecase	-	2023-02-10 오전 7:39			
Scan Completed	wonhoonchoi	Multi Occlusion	-	2023-02-09 오전 9:26	2023-02-09 오전 9:01		
Scan Completed	wonhoonchoi	FileViwerTest_1 - Clone - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-02-09 오전 8:50	2023-02-09 오전 8:35		
Case Completed	wonhoonchoi	wonhoonchoi's Case	12-Inlay / 13-Inlay / 14-Inlay	2022-10-13 오후 10:30	2022-10-11 오후 5:16		
Case Completed	wonhoonchoi	wonhoonchoi's Case	-	2022-10-13 오후 10:28			

Filter by Date

Filter and display cases last modified within the set duration.

1. Click the duration filter.

2020-08-10 ~ 2020-09-09


2. Select the duration from the list.

- All: Shows all cases.
- Custom Range: Specify the start and end dates to display cases last modified within the period.
- Today: Show the cases last modified today.
- Yesterday: Show the cases last modified yesterday.
- Past 7 days: Show the cases last modified during the previous 7 days excluding today.
- Past 30 days: Show the cases last modified during the previous 30 days excluding today.
- This Month: Show the cases last modified during this month.
- Previous Month: Show the cases modified during the last month.

The case list will be updated in real time with the cases created or last modified within the selected duration.

Advanced Search Capabilities


Leverage search capabilities to quickly find the information you want.

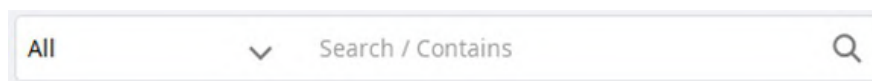A search bar with a dropdown menu currently showing 'All'. To the right of the dropdown is a search icon (magnifying glass). The text 'Search / Contains' is visible inside the bar.

The following Searching Criteria are available for each box:

Patient	All, Patient Name, Patient ID
Case Box	All, Case Name, Patient Name, Patient ID, Form Information, Lab Name
Order Box	All, Case Name, Patient Name, Patient ID, Lab Name, Order ID
In Box	All, Case Name, Patient Name, Clinic Name, Order ID
Work Box	All, Case Name, Patient Name, Form Information, Clinic Name, Order ID
Case Talk	Case Talk Name (Contains), Patient Name (Equals)
Trash Box (Clinic)	All, Case Name, Patient Name, Lab Name, Deleted By
Trash Box (Lab)	All, Case Name, Patient Name, Clinic Name, Deleted By

You can select the Searching Criteria and choose between the Contains and Equals logic.

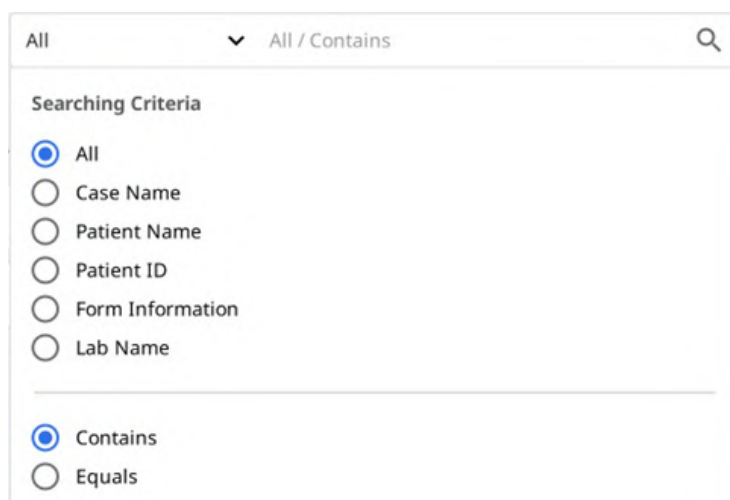A dropdown menu titled 'Searching Criteria'. It contains two sections. The first section lists search criteria: 'All' (selected with a blue radio button), 'Case Name', 'Patient Name', 'Patient ID', 'Form Information', and 'Lab Name'. The second section lists search logic: 'Contains' (selected with a blue radio button) and 'Equals'.

Create/Edit/Clone/Delete Case

Creating Cases

Create Case in Case Box (Clinic)

1. Click the "New Case" button in the upper right corner.

New Case Information

Patient Name*



Case Name*

 Creating patient information involves consent from the patient.
Patient information can be shared with 3rd parties while placing an order.

Assignee

Select assignees

▼



 You can select up to 5 assignees.

Tags

You can add the entered tag with Enter or Spacebar.

 You can add up to 10 tags of 100 characters or less.

[Terms & Conditions](#)

[Cancel](#)

Register

Register & Scan

2. Enter basic information, such as the patient's name and case name.
 - If you enter a name not on the patient list, a new patient will be added.
 - If you enter a name that is in the patient list, you can select it from the list.
 - If registering a case for an existing patient, click the Patient List icon to select an existing patient.



- In the Patient List dialog, you can register a new patient or edit the information of an existing patient.

Patient Name	Patient ID	Gender	Birth Date	Number of Cases	Registered Date	Recent Visit Date
test				0	2020-08-28	2020-08-28
test				0	2020-08-28	2020-08-28
test				13	2020-08-23	2020-08-28
test				85	2020-08-23	2020-05-08
Wonhoon Choi				1	2020-04-01	2020-04-01
test				0	2020-08-18	
13				0	2020-08-20	
14				0	2020-08-20	
15				0	2020-08-20	

3. The case name will be filled in automatically, but you can still change it.

4. Select or add assignees for the case in the Assignee field.

- You can search for and select up to 5 assignees per case using the dropdown list.

- To add a new assignee:

- Click the Assignee Management icon in the top-right corner of the dropdown.

- In the window that opens, click Add Assignee.

Assignee Management
Add and manage assignees per case.

Total 3 ■ Dentist 1 ■ Dental Technician 1 ■ Dental Hygienist 1

Name	Role
☐ rana.jeon	■ Dental Hygienist
☐ kisoo.jeon	■ Dental Technician
☐ seolah.jeon_2	■ Dentist

[+ Add Assignee](#)

- Enter a name (up to 50 characters) and select a role for the new assignee. Each assignee can have only one role, chosen from: Dentist, Dental Technician, or Dental Hygienist.

Create Assignee
Enter the name and select a role for the new assignee.

Name*
Enter assignee name 0/50

Role*
☒ Dentist
☐ Dental Technician
☐ Dental Hygienist

5. Click "Register & Scan" to complete the case registration and proceed to scan.

- **Register:** You will be redirected to the main page. This is useful when registering multiple cases.
- **Register & Scan:** Move to the Form tab to register form information for the case.

Create Case in Work Box (Lab)

1. Click the "New Case" button in the upper right corner.

New Case Information

Clinic Name*

▼

Patient Name*

Case Name*

Assignee*

Add assignees

▼

+

! You can select up to 5 assignees.

Requested Delivery Date

2025-07-30

Tags

You can add the entered tag with Enter or Spacebar.

! You can add up to 10 tags of 100 characters or less.

Cancel

Register

2. Enter or select the clinic name.
3. Enter the patient's name and case name. The case name will be filled in automatically, but you can still change it.
4. Choose an assignee for this case.
5. Enter the expected delivery date considering the completion date of the work.
6. After registering the case, you will be redirected to the main page.

Editing Cases

Edit Case in Case Box (Clinic)

1. Click the "Edit" icon next to the case name.

Edit Case Information

Patient Name*

BS_3 

✔ "BS_3" is selected.

Case Name*

BS_3's Case

 Creating patient information involves consent from the patient.
Patient information can be shared with 3rd parties while placing an order.

Assignee

Select assignees  

 You can select up to 5 assignees.

Tags

You can add the entered tag with Enter or Spacebar.

 You can add up to 10 tags of 100 characters or less.

[Terms & Conditions](#)

[Cancel](#) [Apply](#)

- Click the "Search" icon to change the patient's name. Note that it cannot be edited directly.; instead, you can find a patient name you would like to replace the current one with and can edit the patient information from the list.

Patient List

Select a patient in the list to set the patient of the case. You can add or manage the patient information in Medit Link.

All  Search / Contains  [+ New Patient](#)

	Patient Name	Patient ID	Gender	Birth Date	Number of Cases	Registered Date	Recent Visit Date	
☐	test				0	2020-08-28	2020-08-28	
☐	test				0	2020-08-28	2020-08-28	
☒	test				13	2020-08-23	2020-08-28	
☐	test				85	2020-08-23	2020-05-08	
☐	Wonhoon Choi				1	2020-04-01	2020-04-01	
☐	ttttt				0	2020-08-18		
☒	13				0	2020-08-20		
☐	14				0	2020-08-20		
☐	15				0	2020-08-20		

[Close](#) [Select](#)

- Check the patient's name you want to replace the current one with.
- To create a new patient, press the "New Patient" button.
- Use the "Edit" button to change the registered information about the patient.
- To modify an existing assignee:
 - In the Assignee Management modal, click the three-dot icon (⋮) next to the assignee you want to edit.

Assignee Management
Add and manage assignees per case.

Total 3 ■ Dentist 1 ■ Dental Technician 1 ■ Dental Hygienist 1 + Add Assignee

Name	Role
☐ rana.jeon	■ Dental Hygienist
☐ kissoo.jeon	■ Dental Technician
☒ seolah.jeon_2	■ Dentist

- You can update both the name and the role.

Modify Assignee
Update the name and role of the assignee.

Name*

13/50

Role*

■ Dentist

■ Dentist

■ Dental Technician

■ Dental Hygienist

Note

Updates to assignee information are automatically applied to all existing cases. However, these changes won't be reflected in cases that have already been ordered.

If you remove an assignee, they will also be removed from any cases to which they were assigned.

7. Click "Apply" to finish the process.

Edit Case in Work Box (Lab)

1. Click the "Edit" icon next to the case name.

Edit Case Information

Clinic Name*

seolah

Patient Name*

3.4.2_ScanBody_Labs

Case Name*

Scan for Labs 3.4.2_ScanBody_Labs's Case

Assignee*

Add assignees

+

📘 You can select up to 5 assignees.

Requested Delivery Date

📅

2025-04-10

Tags

You can add the entered tag with Enter or Spacebar.

📘 You can add up to 10 tags of 100 characters or less.

Cancel

Apply

- Click "Apply" after the editing is done.

Note

You cannot edit the information of the cases received through the order in In Box.

You can add or edit assignees for ordered cases.


Cloning Cases

Clone Case in Case Box (Clinic)

- Click the "Clone" icon at the top.



- You will be asked what kind of information you would like to clone to the new case.

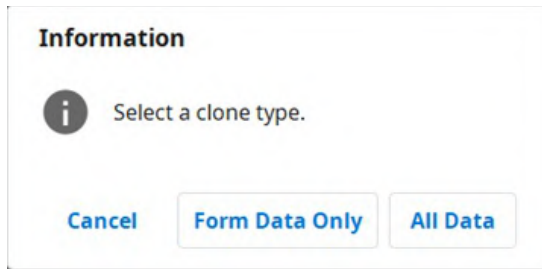

3. Choose whether to replicate only form information only or all data.
4. When cloning is complete, a case named "[Case name] – Clone" will be created.

Clone Case in Work Box (Lab)

1. Click the "Clone" icon at the top.



2. Duplicate all data to create a new case.
3. When cloning is complete, a case named "[Case name] – Clone" will be created.


Deleting Case

Delete Case (Clinic and Lab)

1. Click the "Delete" icon.



2. Click "OK" to delete the case.

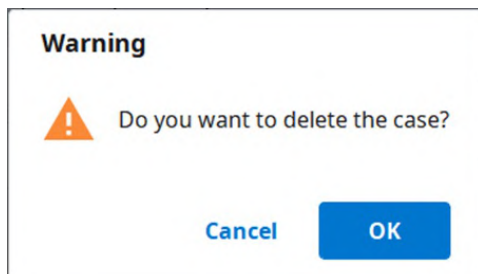

3. Deleted cases are moved to the Trash Box and can be recovered if necessary.

Form Information

You can enter the form information based on the connected program and the treatment type.

The information you enter will be used as basic information for working on scanning or CAD programs and ordering.

Registering Form Information in Case Box

Enter on the Form Tab

1. Select whether you are going to input treatment information by teeth or by arch.
2. Select Type – Method – Material – Shade.
 - If you entered information incorrectly, you go back to select it again.
 - The selected information is automatically registered to the order list.
3. Click "Save" to save your registration.

Registering Form Information in Work Box

Enter on the Form Tab


1. Select the scan program you want to use.
2. Select one of the following options that will define your scanning strategy: Teeth/Flexible Multi-Die/Multi-Die.
3. Methods may differ depending on the selected scan program.

	Medit Scan for Clinics	Medit Scan for Labs
Teeth	☐	☐
Arch	☐	☐
Flexible Multi-Die	☒	☐
Multi-Die	☒	☐

4. Select teeth, arch, or die numbers for the scanning process.
5. Select Type – Method – Material – Shade.
 - If you entered information incorrectly, you go back to select it again.
 - The selected information is automatically registered to the list.
6. Click "Save" to save your registration.

Connecting Teeth into Bridge

You can connect multiple teeth into a bridge when the same material is applied to multiple adjacent teeth. Click the bridge icon beside the selected teeth to connect or disconnect them.

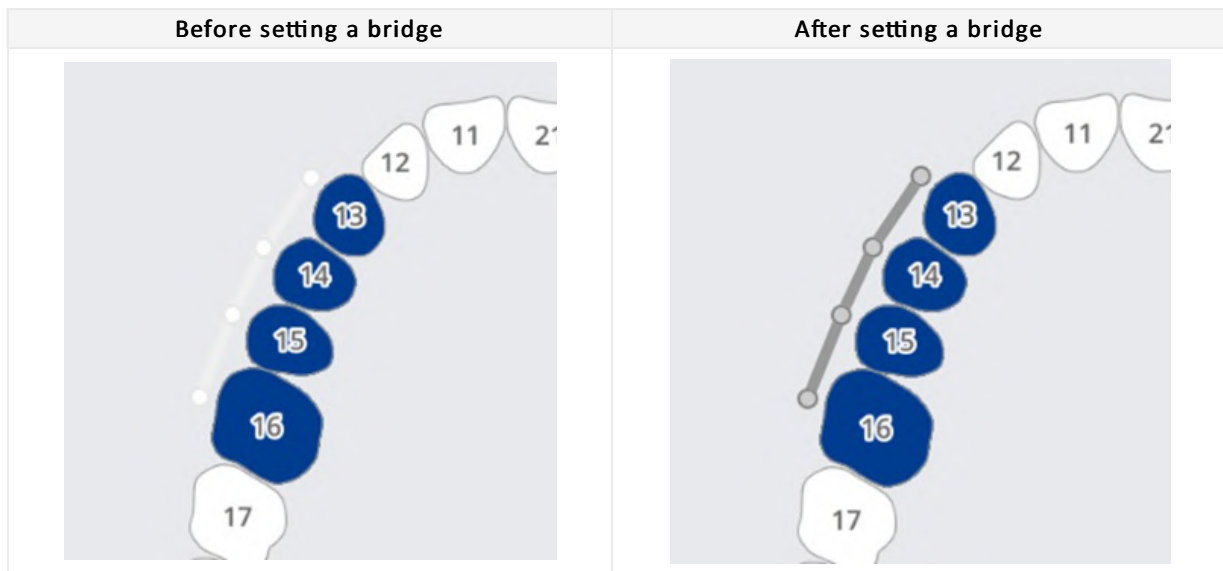

Setting Scanning Options

Depending on the program you are working with, various scan options are provided.

The selected scan options are passed to each scanning program.

Medit Scan for Labs

General options

	Adjacent Teeth	Mark the prepared and adjacent teeth and pass this information to the scanning program.
	Antagonist	Automatically mark the antagonist and passes this information to the scanning program.
	Articulator	Display options for selecting the articulator in the scanning program.
	Movement Marker	Allow using the "Jaw Motion" scan strategy in the scanning program.

Options by teeth

	Scan Body	Allow setting a strategy for scanning scan bodies in the scanning program.
	Pre-Op Scan	Allow setting a strategy for scanning a pre-op model in the scanning program.
	Wax-Up Scan	Allow setting a strategy for scanning wax-ups in the scanning program.

Adding to Favorite

You can quickly select a product by registering it as a favorite.

The list of registered favorites is managed as a template and can be shared through the export and import features.

Add to Favorites

1. Select the tooth number.
2. Select Type – Method – Material – Shade.
 - Type – Method – Material – Shades can be registered as favorites even if not all are selected.
3. Click the "Star" icon.



4. A combination of the selected parameters, excluding tooth number, is on your favorite search list.

Manage Favorite Templates

You can register, manage, and share different types of templates.

Add	Add a new template.
Rename	Rename the current template.
Delete	Delete the current template.
Export	Export the current template to a file.
Import	Import a template file.

Work on Scanning/CAD

Scan Process

Based on the entered information, you proceed with scanning.

1. Enter/save treatment information and select the necessary scan options.
2. Click the "Scan" button in the upper-right corner of the window.

CAD Process


Medit Link provides the DentalCAD integration app to connect Medit Link with DentalCAD.

1. Install the DentalCAD Integration app in the App Box.
2. Go to the case details and click on the DentalCAD Integration app icon on Quick Launcher.
3. Specify the path to the DentalCAD app installed on your PC in the pop-up dialog.
4. Enter the case details of the scanned case and click the DentalCAD Integration app icon, which will pop up a dialog to create a dentalProject file.
5. Register form information for DentalCAD and click "Open" to open the CAD program.
6. If you install the DentalDB Integration app, the CAD results will be automatically delivered to the specified save path in exocad. (Lab accounts only)


Utilize Add-in Apps

Add-in Quick Launcher

You can directly run various add-in apps by clicking on the installed app icons in the Quick Launcher. You can change the order of app icons by dragging them.

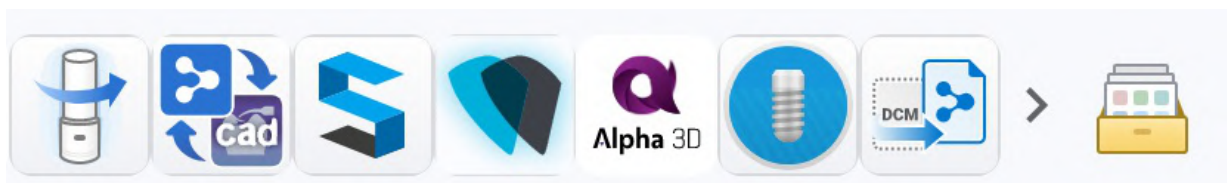

App Launcher Management

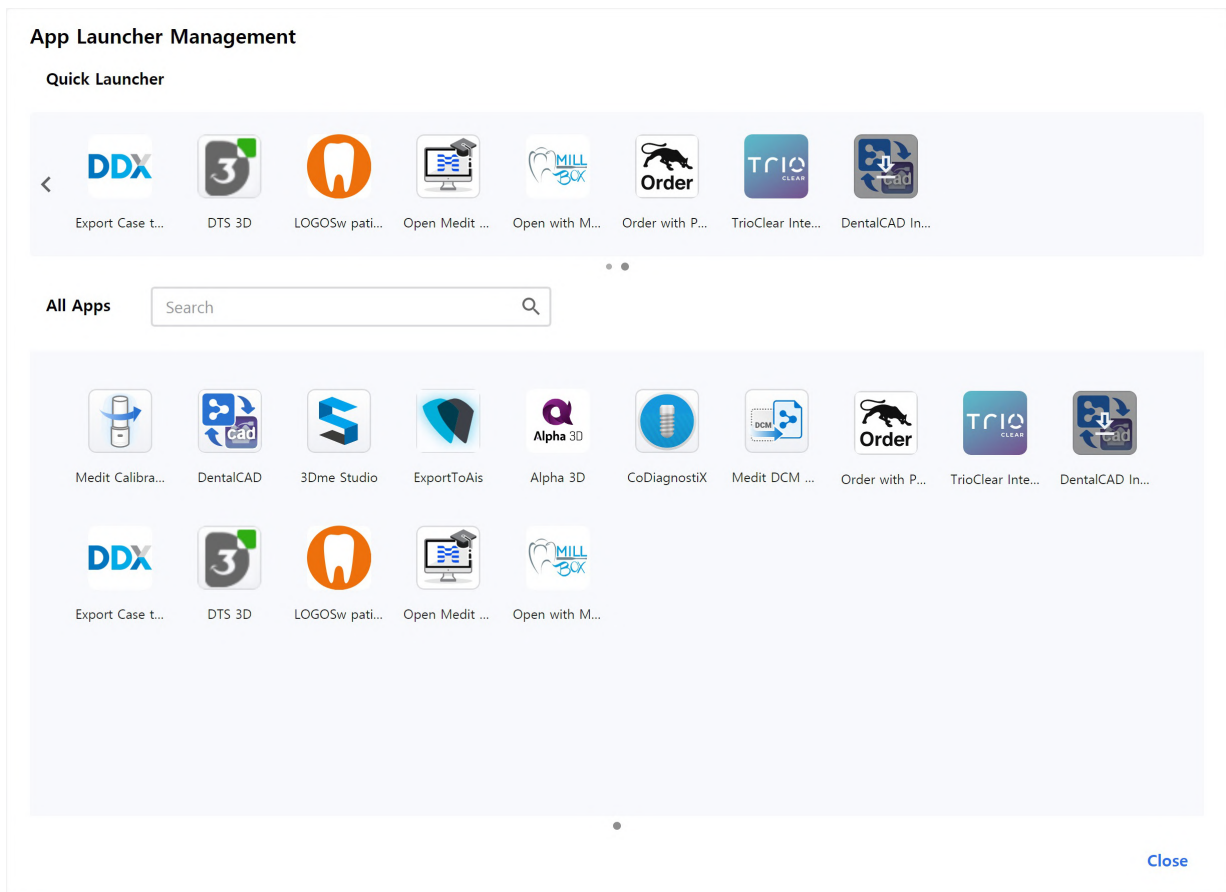

Quick Launcher

You can define which apps appear in the Quick Launcher. You can change the order of app icons by dragging them, or you can create folders to group similar apps together to organize them.

All Apps

All apps you have installed are shown. You can right-click on an app icon to add the app to the Quick Launcher. From the right-click the context menu, you can also select to run, install, or uninstall the app.

Get Result Files

Export Scan Result Files

In addition to sharing results with partners through orders, Medit Link allows you to store your result files in the desired file format using the export function.

Export Files in This Case

Options

File List

Export Files To

C:/ReleaseBuild/

Folder Name

5162022-Crown Fit Sample data

File Name

5162022-Crown Fit Sample data

Type 

exocad

☐ Enabled "automatic .dentalProject import" in DentalDB

Mesh Format

Convert mesh files to the below formats

☒ meditMesh

☐ OBJ

☐ PLY

☐ STL

☐ Do not convert the attached files

Image Format

☒ Convert images to the below format

☐ JPG

☒ JPEG

☐ PNG

☐ BMP

☒ Compress Files

☒ Combine Individual Mesh

☐ Save Case Information as PDF

[Preview](#)

Close


Export

119

Export Files To	Specify the path where you want to store the result files.
Folder Name	The system creates a new folder and stores all result files in it. The default folder name is set by the naming rule. But you can change the folder name directly. You can define the naming rule on Settings > Export Options > Rules for Folder Name.
File Name	Set the file name for the result. The naming rule sets the default file name. But you can change the file name directly. You can define the naming rule through Settings > Export Options > Rules for File Name
Type	Choose the design program you want to use. Medit Link will automatically adjust the files according to your selection, enabling you to proceed with the design without the need for additional axis alignment.
Mesh Format	Specify the format of the mesh file.
Image Format	Specify the format of the exported image files.
Compress Files	Compress all results into a single file.
Combine Individual Mesh	Combine the result of the maxilla or mandible with the individual tooth. It is useful when importing several files into a program is not possible.
Export All Occlusal Relationship	When multiple occlusal results are generated from the scan app, you can export all occlusal results at once.
Save Case Information as PDF	Save the case information as a PDF with the result.

Export Result Files from Case List

You can export result files of a case or multiple cases simultaneously with the "Export" option above the case list in Case Box and Work Box.

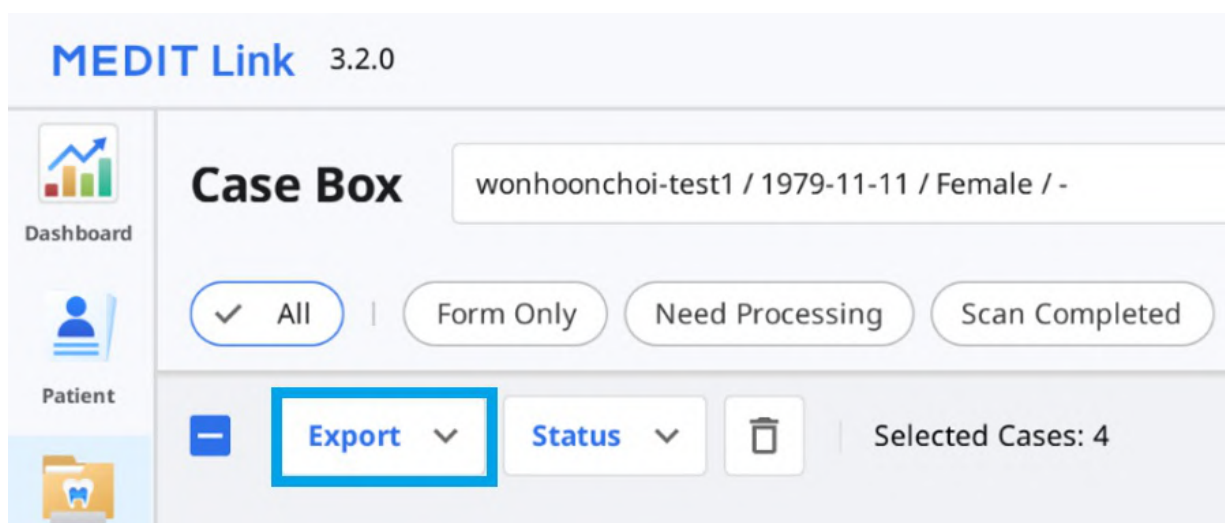

1. Go to Case Box or Work Box.
2. Select all cases you want to export result files from the list.

MEDIT Link 3.2.0

Case Box wonhoonchoi-test1 / 1979-11-11 / Female / - + 2 ▾

☒ All |
 ☐ Form Only |
 ☐ Need Processing |
 ☐ Scan Completed |
 ☐ CAD |
 ☐ Ordered |
 ☐ Case Completed

☒ **Export** ▾ |
 Status ▾ |
 |
 Selected Cases: 4

	Status	Patient Name	Case Name	Form Information
☒	Scan Completed	wonhoonchoi-test1	wonhoonchoi-test1's Case	-
☒	Form Only	wonhoonchoi-test1	wonhoonchoi-test1's Case	-
☒	Form Only	wonhoonchoi	originalcase	-
☒	Scan Completed	wonhoonchoi-test1	wonhoonchoi-test1's Case	-

3. Click "Export" and select "Result Files."

Export ▾

Result Files

Case List (as shown)

Case List (with all items)

4. Set the destination folder, file type, mesh format, and image format, and select export options on the "Options" tab.

Export Files in Selected Cases

Options

Export Files To

C:/Users/Wonhoon Choi Laptop/Desktop/ExportTest/3/

Folder Name

Case Creation Date - Case Name

File Name

Case Creation Date - Case Name

Type

Medit

Mesh Format

Convert mesh files to the below formats

☐ meditMesh
 ☐ OBJ
 ☒ PLY
 ☐ STL

☐ Do not convert the attached files

Image Format

☐ Convert images to the below format

☐ JPG
 ☐ JPEG
 ☒ PNG
 ☐ BMP

☐ Compress Files

☐ Combine Individual Mesh

☐ Export All Occlusal Relationship

☐ Save Case Information as PDF

Close

Export

Note

The "Folder Name" and "File Name" fields are not editable when exporting files for multiple cases. The folder name and file name are automatically created according to the rules you set for the "Rule for Folder Name" and "Rule for File Name" options in Settings > Export Options.

- Click "Export."
- Click "Yes" to open the folder where the files are saved.

Question




Exporting successfully completed.
Do you want to open the folder containing the exported file?

No

Yes

Export Result Files from File Viewer

You can export scan result files for a case with the "Export" icon on the top right corner of the File Viewer tab.

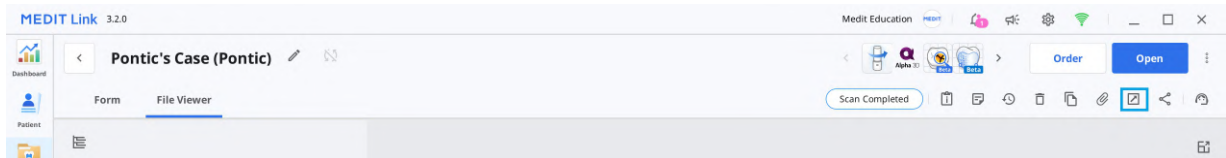

1. Go to Case Box or Work Box.
2. Click to open a case from the list.
3. Click the "Export" icon on the top right corner of the File Viewer tab.



4. Set the destination folder, folder name, file name, file type, mesh format, and image format, and select export options on the "Options" tab.

Export files in case

Options **File List**

Export Files To
C:/Users/Medit-Wonhoonchoi/Desktop/test/

Folder Name
Multi Occlusion - Scan - Before Scanning

File Name
Multi Occlusion - Scan - Before Scanning

CAD Type
Maestro


Mesh Format
Convert mesh files to the below formats
☐ meditMesh ☐ OBJ ☒ PLY ☐ STL
☒ Do not convert the attached files

Image Format
☐ Convert images to the below format
☐ JPG ☐ JPEG ☒ PNG ☐ BMP


☐ Compress Files
☐ Combine Individual Mesh
☐ Export All Occlusal Relationship
☐ Save Case Information as PDF [Preview](#)

[Close](#) [Export](#)

5. In the "File List" tab, you can select the required files for the case by deselecting unnecessary ones.

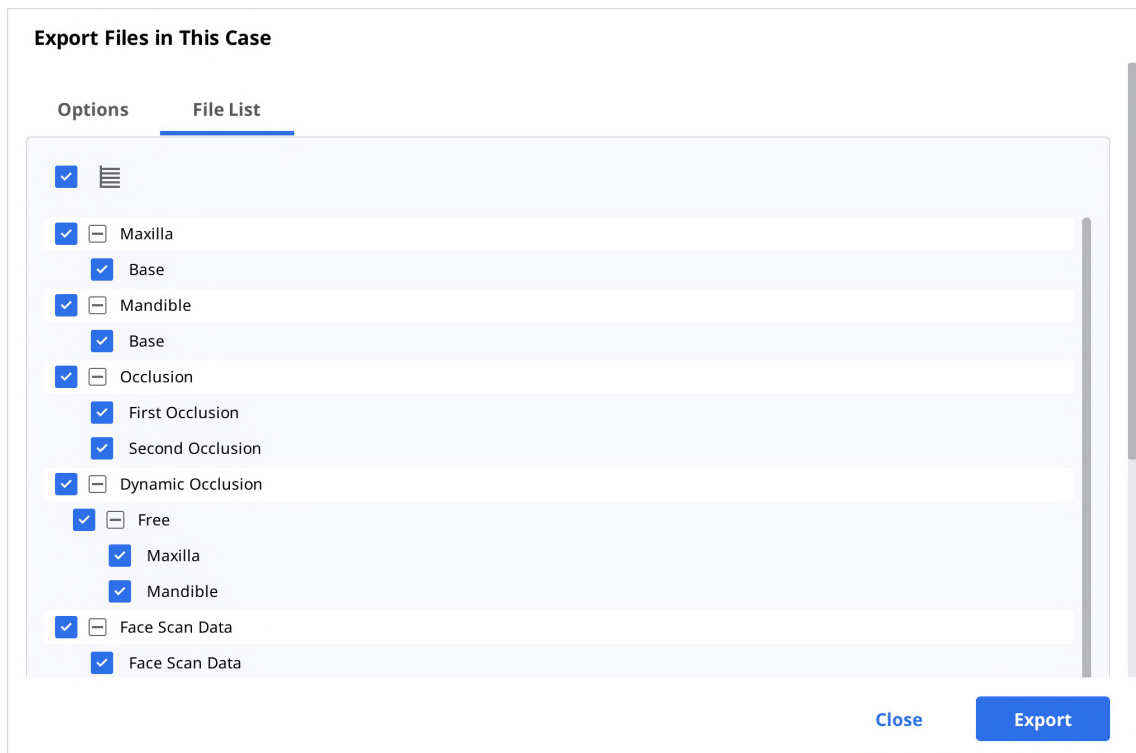


6. Click "Export."
7. Click "Yes" to open the folder where the files are saved.

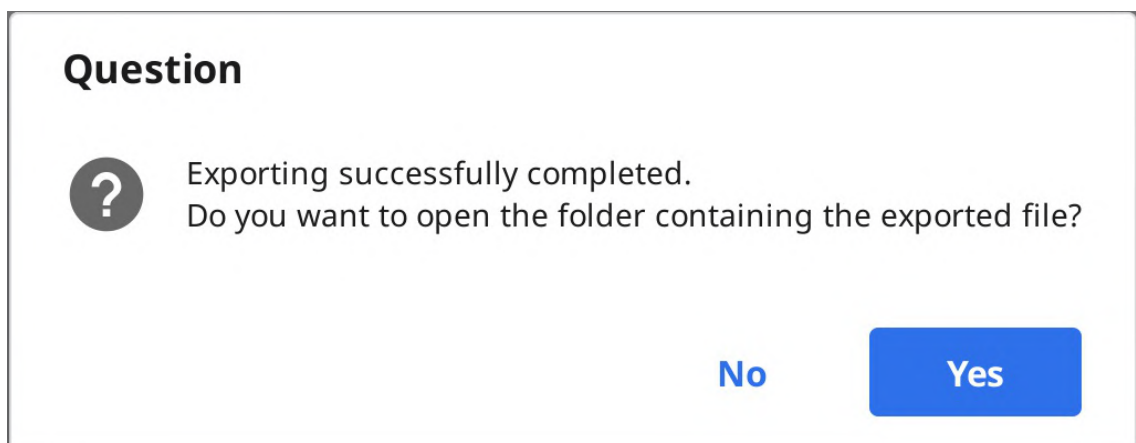

Export Result Files from Data Tree


You can export a file or files in a specific data group from the expanded menu in the file viewer.

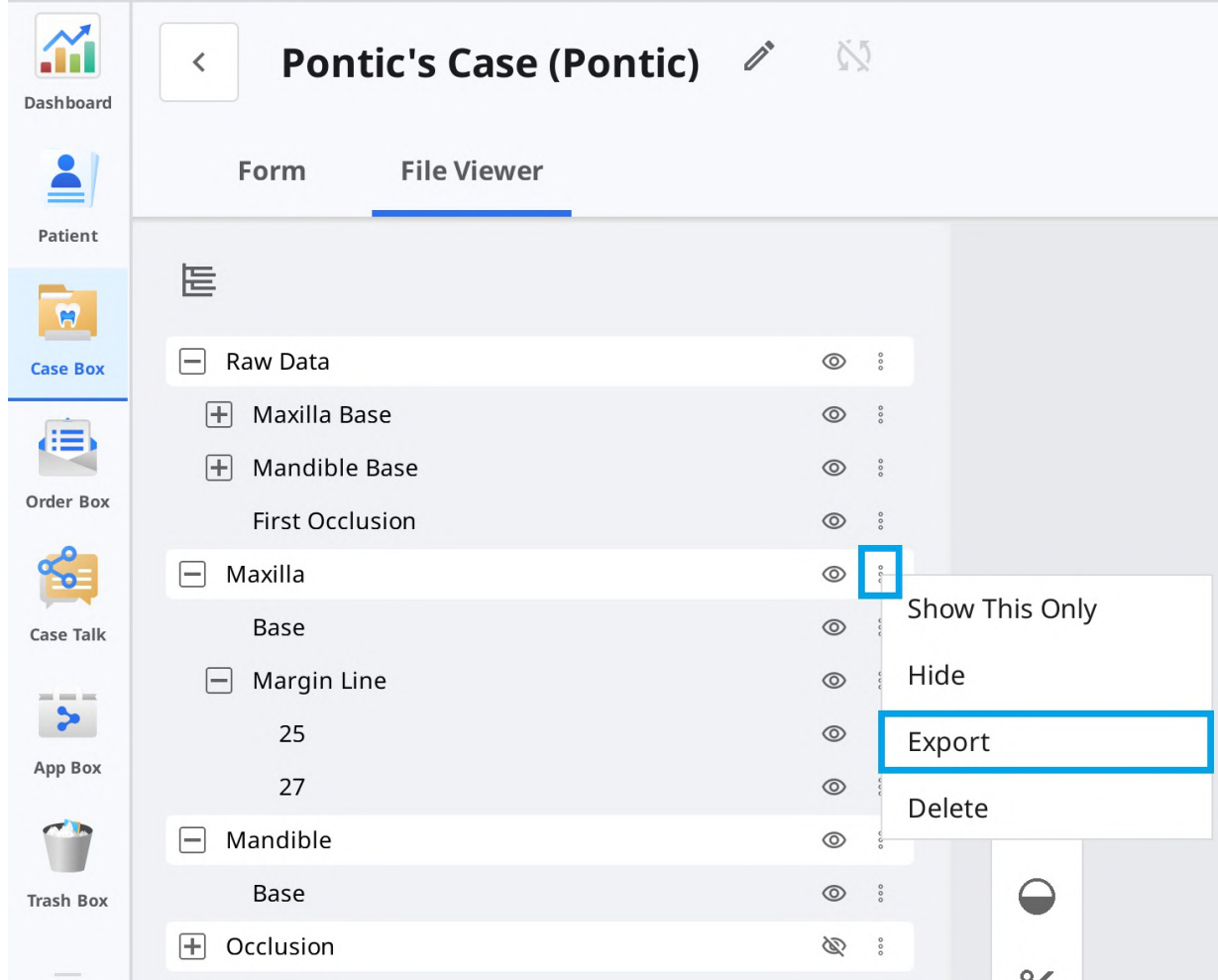

1. Go to Case Box or Work Box.
2. Click to open a case from the list.
3. Click the three dots icon next to the data group or an entity in the data tree in the File Viewer tab.



4. Then click "Export."

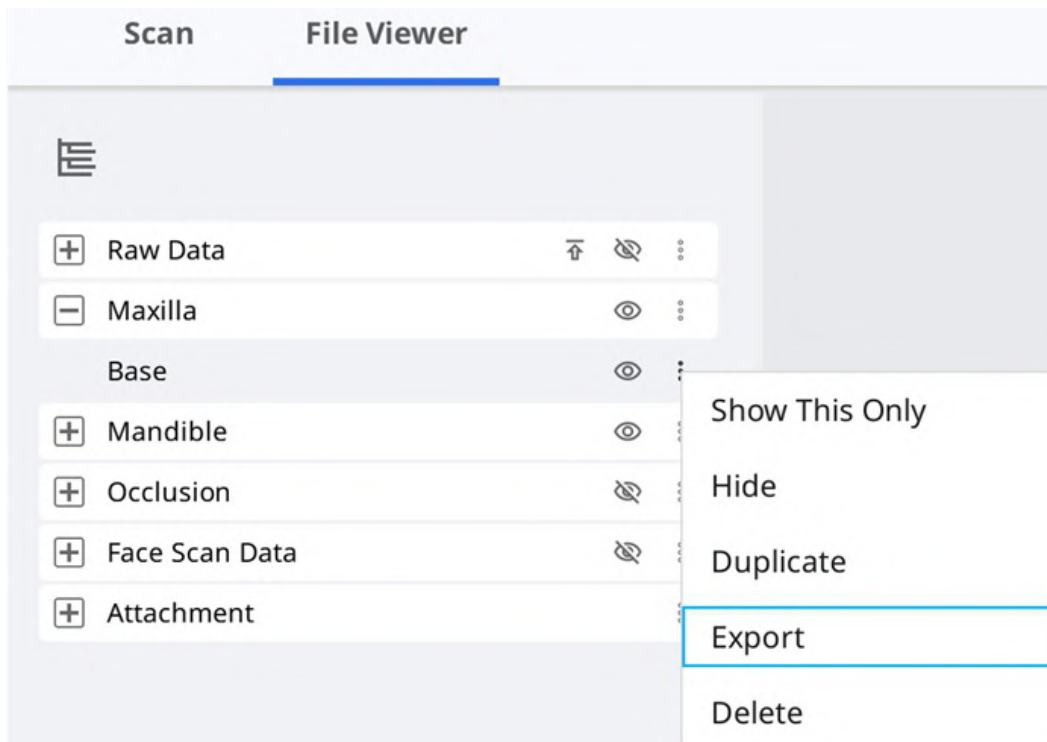

- Set the destination folder, folder name, file name, file type, mesh format, and image format, and select export options on the "Options" tab.

Export files in case

Options
File List

Export Files To

C:/Users/Medit-Wonhoonchoi/Desktop/test/

Folder Name

Multi Occlusion - Scan - Before Scanning

File Name

Multi Occlusion - Scan - Before Scanning

CAD Type

Maestro

Mesh Format

Convert mesh files to the below formats

☐ meditMesh
☐ OBJ
☒ PLY
☐ STL

☒ Do not convert the attached files

Image Format

☐ Convert images to the below format

☐ JPG
☐ JPEG
☒ PNG
☐ BMP

☐ Compress Files
☐ Combine Individual Mesh
☐ Export All Occlusal Relationship
☐ Save Case Information as PDF [Preview](#)

Close
Export

- You can see only the files you chose in the data tree are selected in the File List.

Export Files in This Case

Options

File List

☐

Maxilla

☐

Base

☒

Mandible

☒

Base

☐

Occlusion

☐

First Occlusion

☐

Second Occlusion

☐

Dynamic Occlusion

☐

Free

☐

Maxilla

☐

Mandible

☐

Face Scan Data

☐

Face Scan Data

Close

Export

- Click "Export."
- Click "Yes" to open the folder where the files are saved.

Question

?

Exporting successfully completed.

Do you want to open the folder containing the exported file?

No

Yes

Export Case List as CSV

Medit Link provides the function to export the case list in CSV format.

You can export the case list with basic information and manage cases easily.

- Go to Case Box, Order Box, In Box, or Work Box.
- Select the cases that you want to export.

MEDIT Link 3.2.0

Wonhoon Choi Clinic

Dashboard

Case Box

Order Box

Case Talk

App Box

Case Box

wonhoonchoi-test1 / 1979-11-11 / Female / -

+ 2

✓ All

Form Only

Need Processing

Scan Completed

CAD

Ordered

Case Completed

Export

Status

Selected Cases: 4


2018-01-01-2023-10-24

All

Contains

Status	Patient Name	Case Name	Form Information	Last Modified Date	Scan Date	Lab Name	Order Date
☒ Scan Completed	wonhoonchoi-test1	wonhoonchoi-test1's Case	-	2023-10-19 오전 11:00			
☒ Form Only	wonhoonchoi-test1	wonhoonchoi-test1's Case	-	2023-10-05 오전 8:15			
☒ Form Only	wonhoonchoi	originalcase	-	2023-10-05 오전 8:10			
☒ Scan Completed	wonhoonchoi-test1	wonhoonchoi-test1's Case	-	2023-10-05 오전 8:10	2023-08-18 오전 9:00		

3. Click "Export" and select "Case List (as shown)" or "Case List (with all items)."

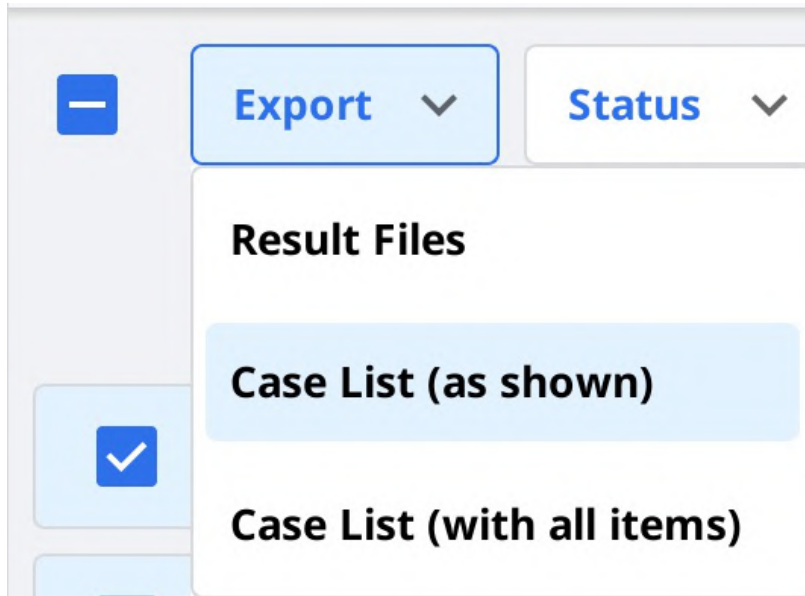

Note

Case List (as shown): Export the case list with the currently displayed items on the screen. Go to View Options > Table Items to show or hide the items as you desire.

Case List (with all items): Export the case list with all items, including Status, Case Name, Date of Birth, Form Information, Last Modified date, Scanned Date, Order Date, and Lab Name.

4. Define the path and file name, then save the list.
5. The case list is saved as a CSV file.


Export Margin Line

Medit Scan for Clinics provides a function to create the margin line, and Medit Link can transfer the margin line to the CAD program.

Export Margin Line as a Color

In general, Medit Link converts the margin line as color and exports the result with maxilla or mandible together. You can create the margin line in their cad referencing the color information.

Example:

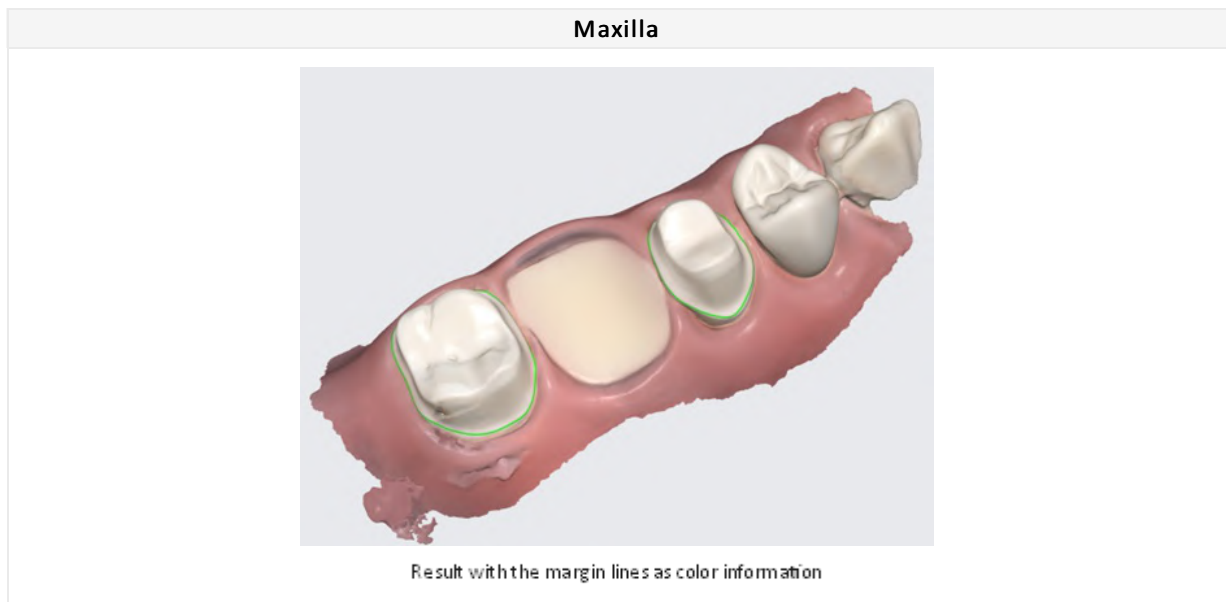

Note

This feature is available for formats that include color information, such as meditMesh, OBJ, and PLY. The margin line does not convert into STL.

Note


Type > 3Shape does not support this feature.

Export Margin Line as a File

In the case of Exocad, Medit Link export the margin line to "~margin.xyz" which can be imported into exocad directly.

You can use this file in exocad without additional work.

Example:

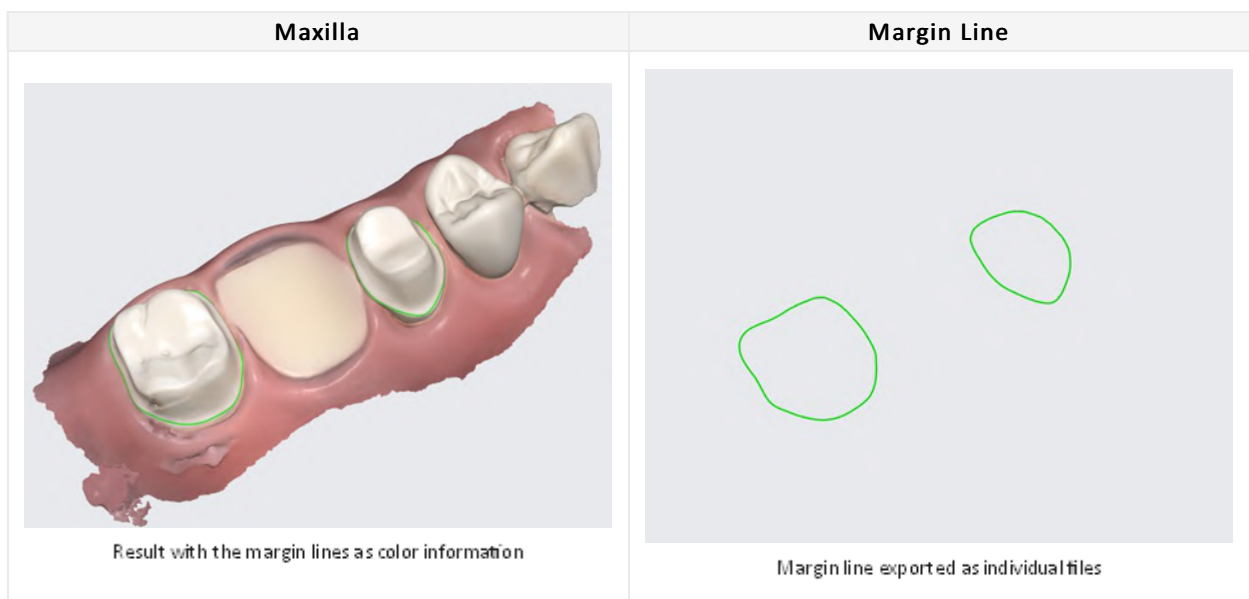

Download Files from Medit Link Web

Medit Link provides a download function.

If you want to get the result files without installing the Medit Link application, you can download the files from the Medit Link web service.

Note


The data should be uploaded on the Medit Cloud to download the files from the web.

The download function is available in the following menu of the Medit Link web service.

- Case Box
- Work Box
- Web Viewer

Case Box/Work Box


You can find the "Download" button at the top right.

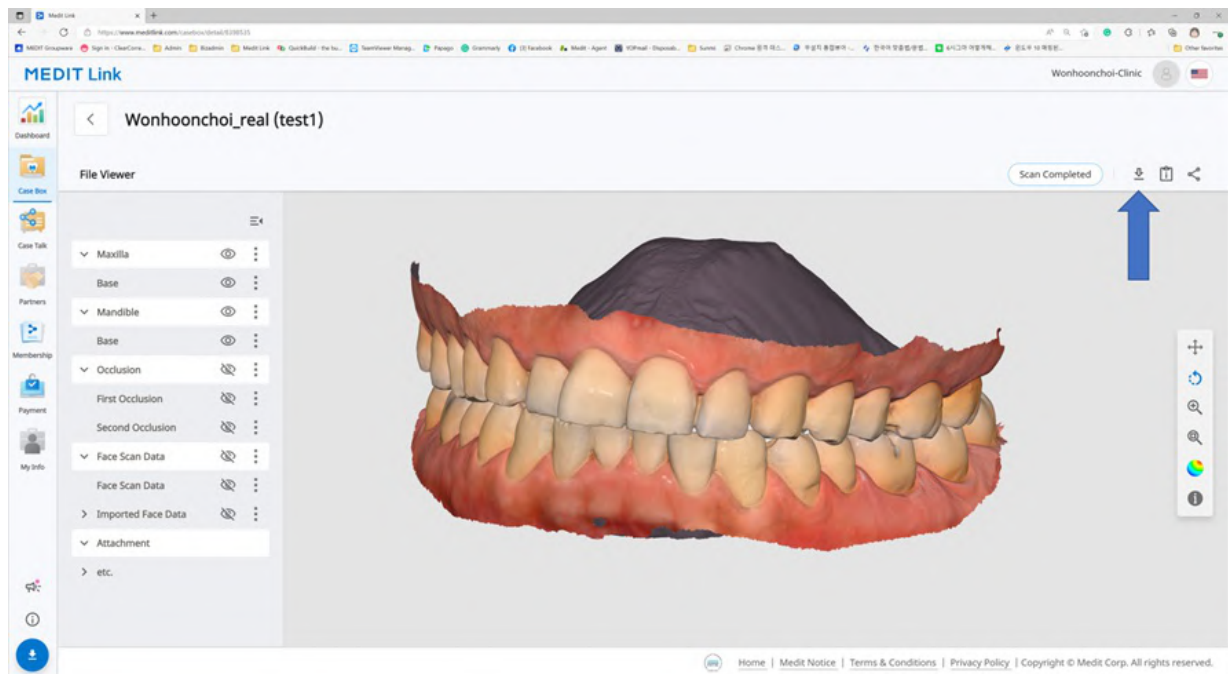


Web Viewer

When you share the case using Case Talk, you can set whether to allow downloading or not.

If you check "Grant Download Permission" and share the case, all users who connect to the shared link can download the files.

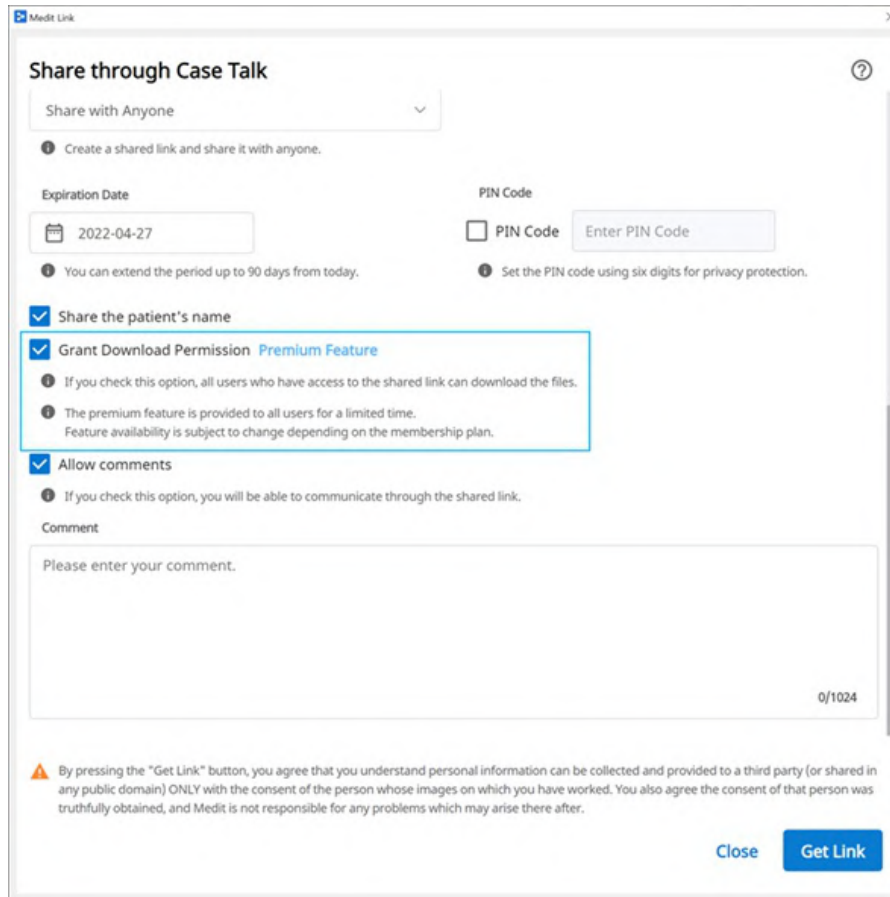

In the Web Viewer, you can find the "Download" icon at the top right.

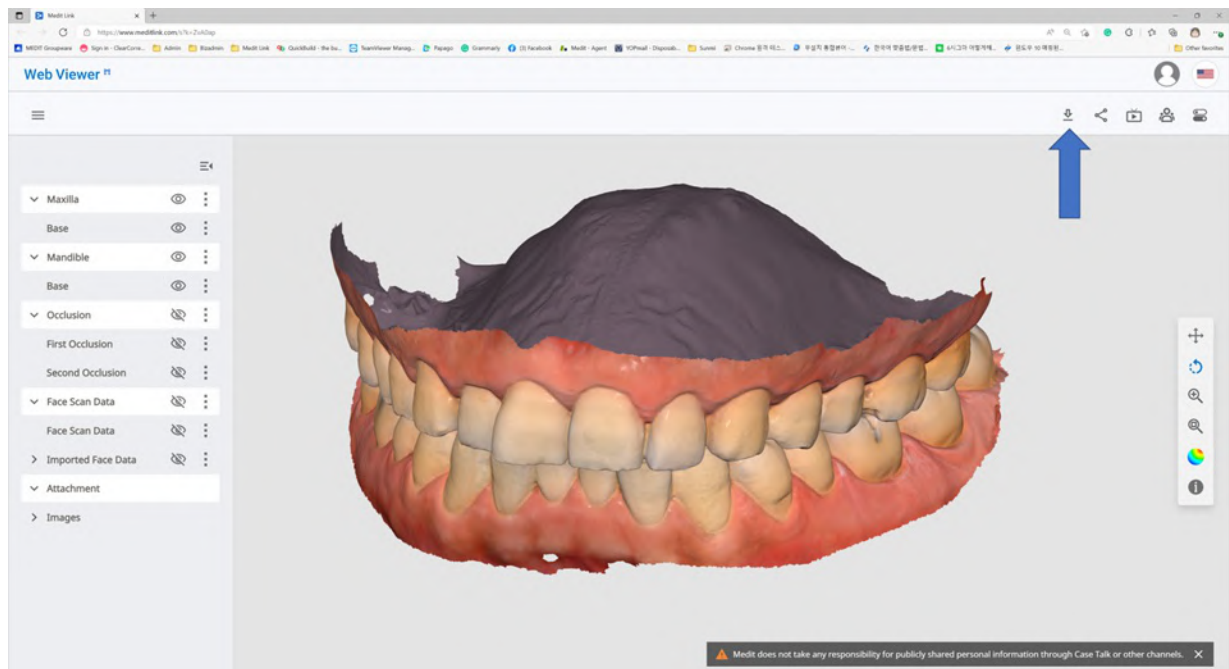

1. Click "Download."



2. Select the mesh format from the download dialog.

Download Premium Feature

The premium feature is provided to all users for a limited time.
Feature availability is subject to change depending on the membership plan.

Mesh Format

Convert mesh files to the below formats

☐ meditMesh ☐ OBJ ☒ PLY ☐ STL

i The selected format will be applied to the result files created in the Medit Link application.

i The attachment files will not be converted but will keep their original format.

Close **OK**

3. Then click "OK."

132

Order/Manage Orders

You can easily order a case with data acquired from connected programs and attached by the user.

Ordering Cases

You can place an order for your partners in Case Box.

1. Click the "Order" button.
2. Check the information required for your order.

Order

General

Order List

File List

Partner

Elysee Dental Test



Elysee Dental Test
Alphen aan den Rijn, NL
[2408 AV] Alphen
172653300

Case Name

BS_3's Case

Patient Name

BS_3

☒ Share patient information

Requested Delivery Date

2025-07-30

Delivery Time

오후 4:15

AM 8:00

PM 8:00

Assignee

Select assignees

☒ Share assignee information

Memo

<Scanbody Library>

46/2048

Tags

☒ Include tags in the order

Close

OK

3. Select a partner from the dropdown list.
4. Check the product information from the Order List tab.
 - You can decide whether to share the patient's name.
5. Check the files to order from the File List tab.
6. Enter your desired delivery date.
 - The default value is set to 7 days after the order date.
7. Choose an assignee for this case. If you don't want to share the assignee information with the lab, clear the "Share assignee information" checkbox.
8. You can add a memo and tags to your partner to outline specific details.

9. When multiple occlusal results are generated from the scan app, you can send all occlusal results at once by checking "Order All Occlusal Relationship."
10. Proceed with your order.

Confirm Order (In Box)

- Accept the order upon receiving it.
- Accepted orders will be forwarded to the Work Box.

Cancel Order (Order Box)

- You can cancel an order before your partner accepts it.
- Canceled cases can be re-ordered.

Reject Order (In Box)

- You can reject a received order.
- Rejected cases can be re-ordered.

Checking Ordered Cases

You can check the details of the case you ordered from the Order Box in the clinic account. You can also check the case you received from the In Box in the lab account.

No.	Type	Method	Material	Shad
15	Crown	Anatomic	Zirconia	A4

Order Info	The information that can be included in the order is as follows: order ID/assignee/patient name/account name/scan date/order date/delivery date/delivery due date/memo. You can check the details of the ordered product and delivery information.
Order List	The order list displays the details of the ordered product.
Shipment	Enter/check the shipping information.

Entering Shipment Information (In Box)

Enter the shipping information of the completed case and start shipping.

1. Click the Shipment tab.
2. Enter the shipment information and click "Register."

Shipment Registration

Select a delivery method and enter the required information to register.
The information you enter is saved in the Shipment tab and can be checked at any time.

☒ Shipment
 ☐ Self-Delivery

Company

Tracking Number

Memo

Cancel

Register

3. The entered information is sent to your partner.

Managing Delivery Date

You can set the expected delivery date when placing an order, and this information will be forwarded to your partner.

Setting Requested Delivery Date (Clinic account)

- When ordering from Case Box, you can enter the requested delivery date.

Order

General Order List File List

Partner
WonhoonChoi_Lab_Stage

 **WonhoonChoi_Lab_Stage**
Seoul, KR
[02835] 23 Incheon-ro 22-gil, Seongbuk-gu, Seoul, Korea
01026088546

Case Name
mandibular movement - Clone - Clone

Patient Name
Test Occlusion

☒ Share patient information

Requested Delivery Date
2023-04-13

Delivery Time
오전 8:15 AM 8:00 PM 8:00

Memo

Tags
You can add the entered tag with Enter or Spacebar.

☒ You can add up to 10 tags of 100 characters or less.

☒ Include all occlusion relationships in the order

☒ Include Tags In The Order

Cancel OK

- The default value is 7 days after the date of the order.
- Enter the date/time you wish to receive the products and proceed with the order.
- You can set the time easily using the slider.

Setting Expected Delivery Date (Lab account)

- You can enter the expected delivery date for the case ordered from In Box.

Dashboard < 전설아의 케이스 / Meditcompany Reject Confirm Order

Order File Viewer Pending

Order ID
6004719

Assignee
No one has been assigned.

Patient Name

Partner Name
Meditcompany

Scanned Date

Order Date
2025-07-16 PM 5:46

Requested Delivery Date
2025-07-17 PM 03:00

Expected Delivery Date
2025-07-17 PM 03:00

Memo

Tags
tag

Order List Shipment

No.	Type	Method	Material	Shad
25	Crown	Anatomic	Multi-Layered Zirconia	A3.5
26	Crown	Anatomic	Multi-Layered Zirconia	A3.5

1. Check the requested delivery date with the partner and reschedule using the "Edit" button.



2. Enter the expected delivery date considering the actual working day. If required, you can leave a comment also.
3. Click "Apply."

March 2022

S	M	T	W	T	F	S
27	28	1	2	3	4	5
6	7	8	9	10	11	12
13	14	15	16	17	18	19
20	21	22	23	24	25	26
27	28	29	30	31	1	2
3	4	5	6	7	8	9

Delivery Time

11:45 AM AM 8:00 PM 8:00

Comment

Cancel Apply

4. The expected delivery date and the memo will be forwarded together with the order.
 - You can check the date and memos in the Order Box.
 - You can change the expected delivery date, and the changes will be forwarded to your partner.
5. You can check the case history using the "History" button.



Completing Case

Complete (Work Box/In Box)


Once all tasks are done, click "Done" to complete the case.

Complete a case from the case detail

You can find the three dots icon on the right from the "Open" or "Rescan" button.



Complete the work by clicking the icon and selecting "Done."

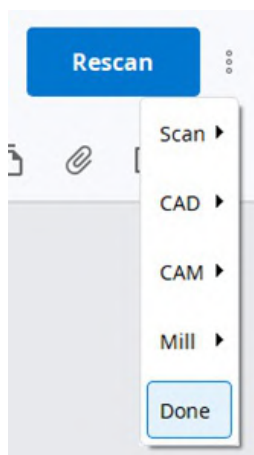

Complete the multiple cases from the case list

1. Go to Work Box.
2. Select all cases to complete.

MEDIT Link 3.2.0

Work Box

✓ All | Form Only | Need Processing | Scan Completed | CAD | CAM

Export Status Selected Cases: 4

Done

Status	Case Name	Patient Name	Form Information
✓ Form Only	tsstt's Case	tsstt	-
✓ Form Only	test's Case	test	-
✓ CAD	Maxilla Crown	Scan for Clinics	14-Crown
✓ CAD	Maxilla Crown	Scan for Clinics	14-Crown

3. Click "Status" and select "Done."

Complete (Case Box/Order Box)

After the case is completed on the lab side, the clinic needs to confirm the case to complete the whole order process.

The clinic can find the "Confirm Delivery" button in the Order Box if the lab completes its work.

Click this, the whole order process completes, and the case status will be changed to "Case Completed."

< test's Case - clone (test)

Order File Viewer

Confirm Delivery

Shipped

Complete a non-ordered case in the Case Box

For cases that are not ordered to the Lab, you can complete these cases from the Case Box.

1. Go to the Case Box.
2. Select the case that you want to complete.

Dashboard

Patient

Case Box

Order Box

Case Talk

App Box

Case Box

Patient

✓ All

Form Only

Need Processing

Scan Completed

CAD

Ordered

Cas

Export

Status

Selected Cases: 4

Case Completed

Status	Case Name
☐ Ordered test	test's Case
☒ Scan Completed wonhoonchoi-test1	wonhoonchoi-test1's Case
☒ Scan Completed test test(1234)	G11598492310180503424
☒ Scan Completed test	test's Case
☒ Scan Completed test	test's Case

3. Click "Status" and select "Case Completed."

Communicate Using Tags

You can add additional information for your case using tags and send them to your partners.

Add Tags When Creating a New Case

You can enter tags when you create a new case or edit case information.

New Case Information

Patient Name*

Case Name*

Creating patient information involves consent from the patient.
Patient information can be shared with 3rd parties while placing an order.

Tag

Doctorname × Specialmaterial × SpecialComment ×

You can add up to 10 tags of 100 characters or less.

[Terms & Conditions](#)

Cancel

Register

Register & Scan

You can easily search and select tags you've used before.

New Case Information

Patient Name*

Case Name*

i Creating patient information involves consent from the patient.
Patient information can be shared with 3rd parties while placing an order.

Tag

t

t (new)

Suggestions

- tet
- tagtest
- tag
- tat2

Add Tags When Ordering

You can check and edit tags in the Order dialog while ordering.

Order

General Order List File List

Partner

WonhoonChoi_Lab_Stage

WonhoonChoi_Lab_Stage
Seoul, KR
[02835] 23 Incheon-ro 22-gil, Seongbuk-gu, Seoul, Korea
01026088546

Case Name

mandibluar movement - Clone - Clone

Patient Name

Test Occlusion

☒ Share patient information

Requested Delivery Date

2023-04-13

Delivery Time

AM 8:00 오전 8:15 PM 8:00

Memo

Tags

Doctorname x Specialmaterial x SpecialComment x

i You can add up to 10 tags of 100 characters or less.

☒ Include all occlusion relationships in the order

☒ Include Tags In The Order

Cancel OK

Check Tags on Case Details

