USER GUIDE

Medit Link (Revision 10) 2025.03.18

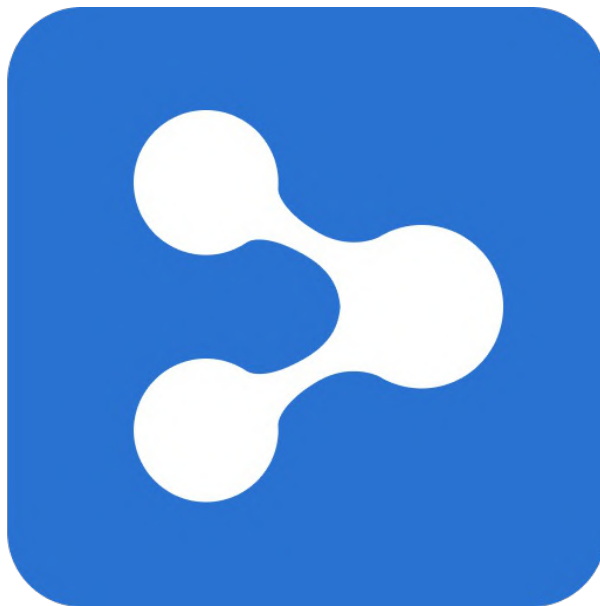
Table of Contents

Medit Link

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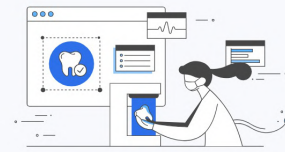
Administrator

Dentist, Manager

Have the authority to create a new dental clinic account and have full access to Medit Link.

- ✓ Scan/Order/Manage orders
- ✓ Manage members

Register

Lab

Administrator

Laboratory Director, Manager

Have the authority to create a new dental lab account and have full access to Medit Link.

- ✓ Scan/Manage ordered cases
- ✓ Manage members

Register

Member

Dental Hygienist, Dental Staff

Belong to the dental clinic their admin created and have limited access to Medit Link after the admin's approval.

- ✓ Scan/Order/Manage orders

Register

Member

Dental Technician, Laboratory Staff

Belong to the dental lab their admin created and have limited access to Medit Link after the admin's approval.

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Email Address

Check

Please ensure your email address is active. Medit Link will send a verification email.

Registering with a disposable and/or fake e-mail address can make you vulnerable to security issues and make it difficult to receive information from Medit Link.

Password

Password must be 8 to 16 characters long, including a combination of at least three letters, numbers, and special characters. (Passwords are accepted in English language only.)

Confirm Password

Please type in the same password as above.

Select Country/Region

☐ I agree to all of the Terms and Conditions.

☐ Personal Information Collection Agreement (Mandatory)

☐ Terms & Conditions (Mandatory)

☐ Agree to the Marketing and Promotion Agreement (optional)

Check this option to receive marketing emails, which include product updates, software news, and promotional events.

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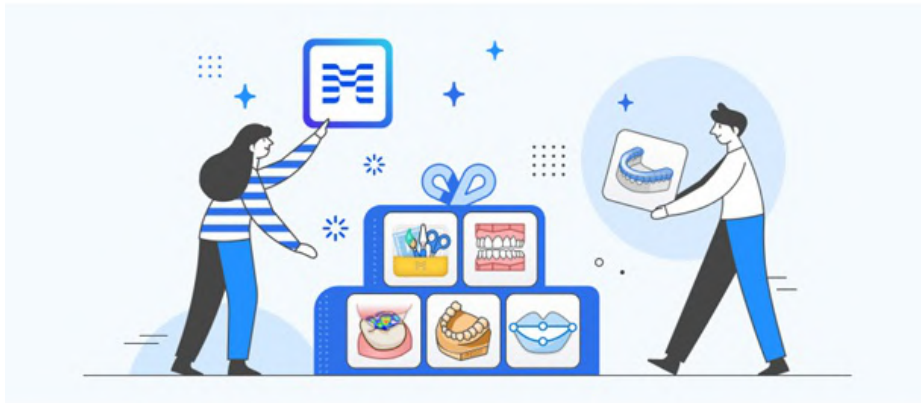
1 2 3 4

Utilize Various Apps

Download Medit Link to your desktop to utilize scanning programs and various apps.
All data generated is synced to the cloud and is also available on the web.

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
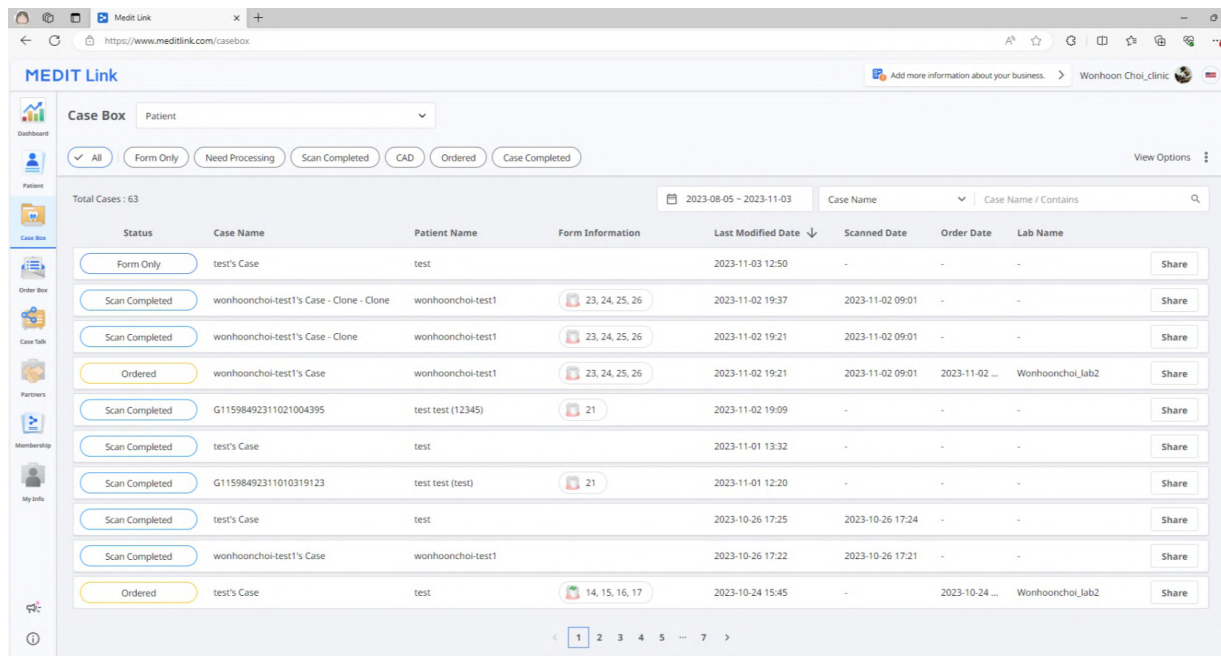
The screenshot shows the Medit Link Case Box interface. At the top, there's a header with the Medit Link logo and a user profile. Below the header, there's a sidebar with navigation icons. The main area is titled 'Case Box' and features a dropdown menu set to 'Patient'. Below this, there are filter buttons: 'All', 'Form Only', 'Need Processing', 'Scan Completed', 'CAD', 'Ordered', and 'Case Completed'. A 'View Options' link is also present. The table below shows a list of cases with columns for Status, Case Name, Patient Name, Form Information, Last Modified Date, Scanned Date, Order Date, and Lab Name. The table is paginated, showing cases 1 through 7.

Status	Case Name	Patient Name	Form Information	Last Modified Date	Scanned Date	Order Date	Lab Name
Form Only	test's Case	test		2023-11-03 12:50	-	-	-
Scan Completed	wonhoonchoi-test1's Case - Clone - Clone	wonhoonchoi-test1	23, 24, 25, 26	2023-11-02 19:37	2023-11-02 09:01	-	-
Scan Completed	wonhoonchoi-test1's Case - Clone	wonhoonchoi-test1	23, 24, 25, 26	2023-11-02 19:21	2023-11-02 09:01	-	-
Ordered	wonhoonchoi-test1's Case	wonhoonchoi-test1	23, 24, 25, 26	2023-11-02 19:21	2023-11-02 09:01	2023-11-02 ...	Wonhoonchoi_lab2
Scan Completed	G11598492311021004395	test test (12345)	21	2023-11-02 19:09	-	-	-
Scan Completed	test's Case	test		2023-11-01 13:32	-	-	-
Scan Completed	G11598492311010319123	test test (test)	21	2023-11-01 12:20	-	-	-
Scan Completed	test's Case	test		2023-10-26 17:25	2023-10-26 17:24	-	-
Scan Completed	wonhoonchoi-test1's Case	wonhoonchoi-test1		2023-10-26 17:22	2023-10-26 17:21	-	-
Ordered	test's Case	test	14, 15, 16, 17	2023-10-24 15:45	-	2023-10-24 ...	Wonhoonchoi_lab2

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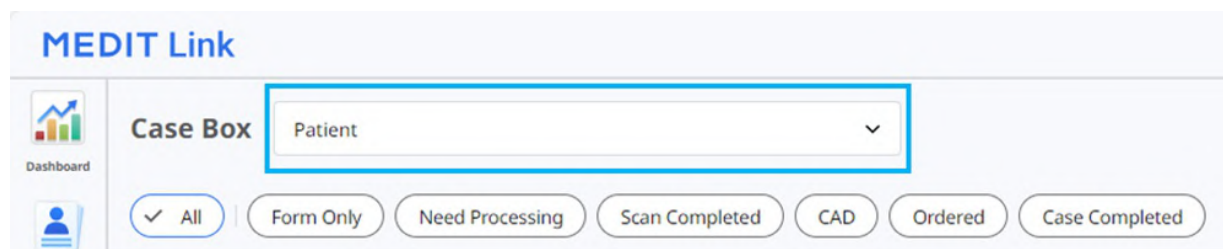The screenshot shows the Medit Link Case Box interface. The 'Case Box' dropdown menu is highlighted with a red box, and the 'Patient' option is selected. Below the dropdown, there are filter buttons: 'All', 'Form Only', 'Need Processing', 'Scan Completed', 'CAD', 'Ordered', and 'Case Completed'.

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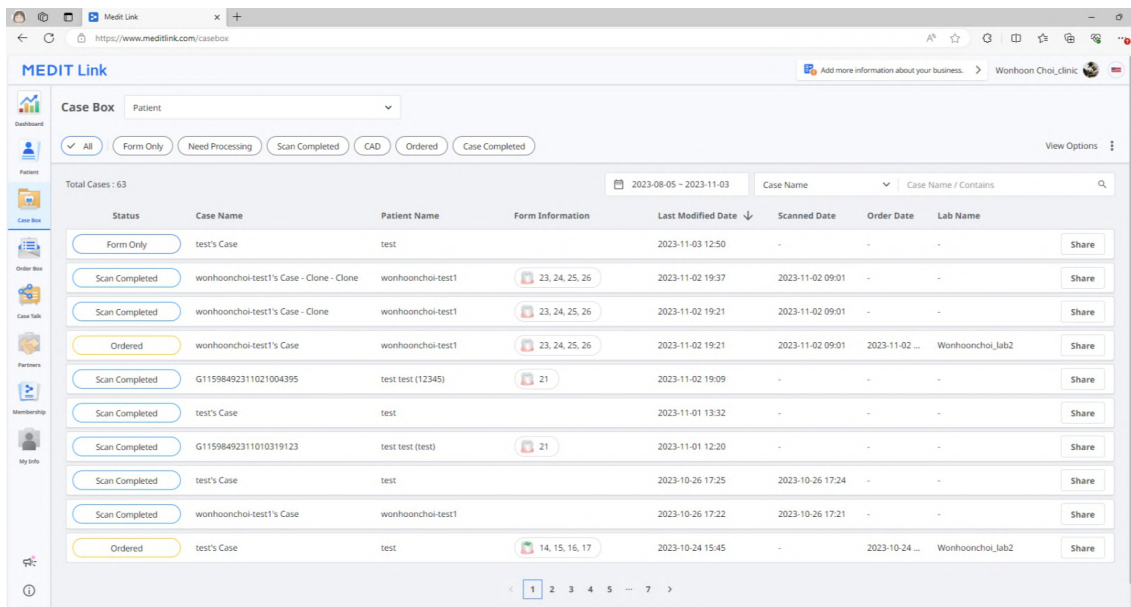
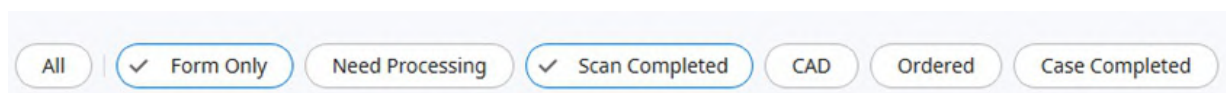
Case Box(歯科医院)	All / Form Only / Need Processing / Scan Completed / CAD / Ordered / Case Completed
Work Box(技工所)	All / Form Only / Need Processing / Scan Completed / CAD / Done

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The screenshot displays the 'Occlusion Test's Case (Occlusion Test)' page in the Medit Link application. The interface includes a sidebar with navigation icons, a top header with the case name and a 'Cancel Order' button, and a main content area with order details and an 'Order List' table. Numbered callouts 1 through 6 highlight specific features: 1. Case name, 2. Cancel Order button, 3. Order tab, 4. Pending status, 5. Order details form, 6. Order List table.

No.	Type	Method	Material	Shade
23	Inlay	-	Multi-Layered Zi...	None
24	Inlay	-	Multi-Layered Zi...	None
25	Inlay	-	Multi-Layered Zi...	None
26	Inlay	-	Multi-Layered Zi...	None

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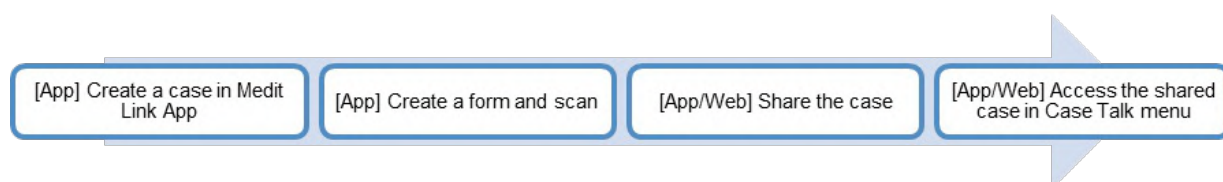
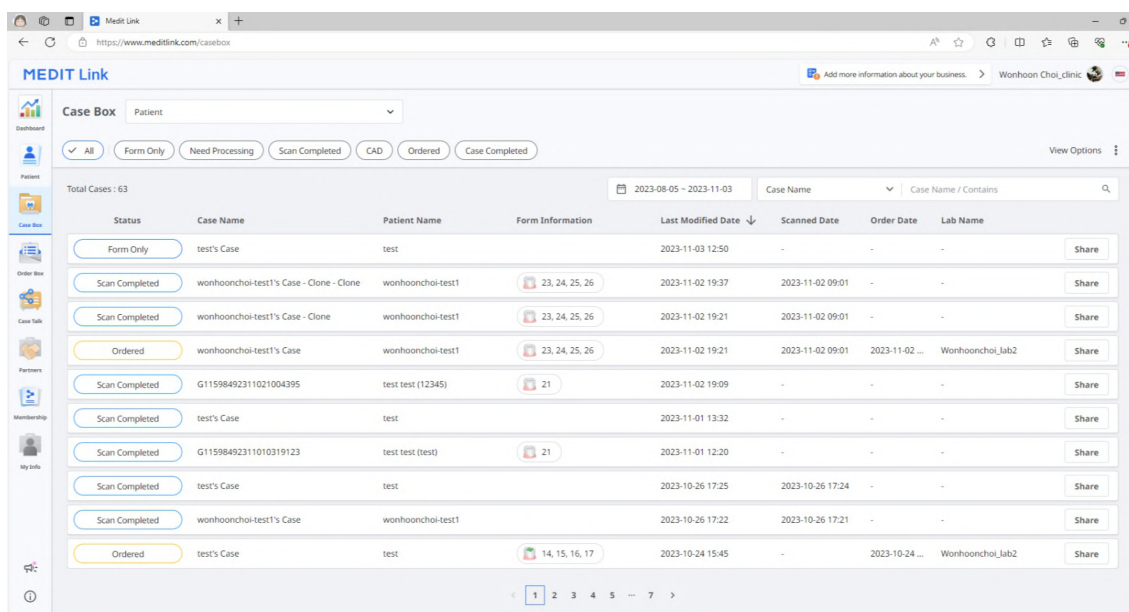
+

☰

❗ Create a shared link and share it with specific people.

Expiration Date

📅 2022-06-28

❗ You can extend the period up to 90 days from today.

PIN Code

☐ PIN Code

Enter PIN Code

❗ Set the PIN code using six digits for privacy protection.

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❗ If you check this option, all users who have access to the shared link can download the files.

❗ The premium feature is provided to all users for a limited time.
Feature availability is subject to change depending on the membership plan.

☒ Allow comments

❗ If you check this option, you will be able to communicate through the shared link.

Comment

Please enter your comment.

0/1024

⚠ By pressing the "Get Link" button, you agree that you understand personal information can be collected and provided to a third party (or shared in any public domain) ONLY with the consent of the person whose images on which you have worked. You also agree the consent of that person was truthfully obtained, and Medit is not responsible for any problems which may arise there after.

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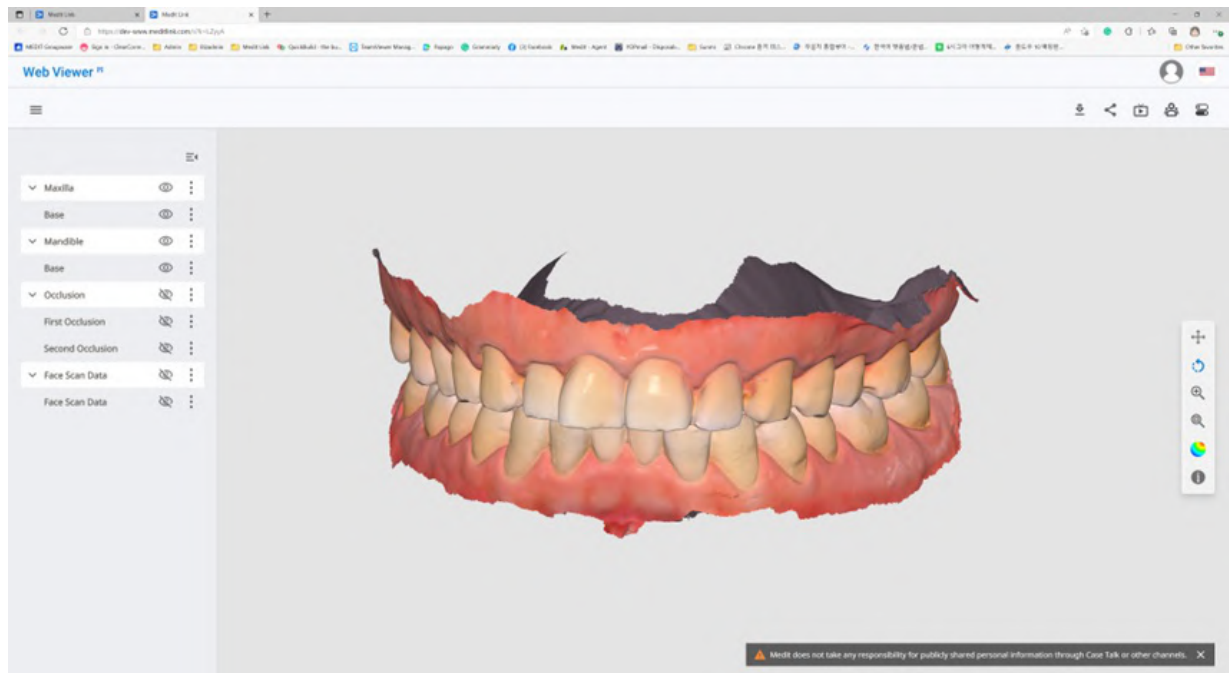
This is a detailed view of the 'Create Annotation' dialog box. It has a title 'Create Annotation' and a prompt 'Please enter your annotation.' Below this is a text input field with the placeholder text 'Please enter your message'. At the bottom, there are two buttons: 'Cancel' and 'OK'.

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This screenshot shows the detail view of an annotation. At the top, it displays the user's name 'wonhoonchoi Beta wonhoonchoi' with delete and close icons. Below is the annotation title 'Annotation 1' and a timestamp '09-09 20:31'. At the bottom, there is a text input field with the placeholder 'Please enter your reply' and a 'Send' button.

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Invite Partner

Invite your desired partner to join Medit Link. When the lab accepts your invitation, a partnership is automatically formed.

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Membership

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The screenshot displays the MEDIT Link web interface. The top navigation bar includes 'MEDIT Link', a user profile 'seolahjeon_', and a language dropdown set to 'English'. The main navigation menu on the left lists 'Dashboard', 'Patient', 'Case Box', 'Order Box', 'Case Talk', 'Partners', 'Membership' (selected), and 'My Info'. The 'Membership' section is active, showing tabs for 'Membership', 'Payment Details', 'Redeem', and 'My Subscription'. The 'Medit Cloud Storage Plan' is highlighted, showing a '2 GB Plan' and a '10 TB Plan' (selected). The '10 TB Plan' is marked as 'Free (Promotion)' and lists benefits: 'A paid plan', '10 TB of cloud storage', 'Efficient for users with many cases and various data', and 'One-month free trial included'. A 'Subscribe' button is visible. Below, '10 TB Plan Benefits' are listed, including a '10 TB Plan' with a code 'qqqq' and an expiration date of '2025-06-26'. A note states: 'The promotion has been applied. Your 10 TB Plan is now active and will remain valid until 2025-05-14. View More'. The 'Cloud Storage' section shows a 'Set Period' from '2018-01-01' to '2025-02-17' and a circular progress indicator showing '3.46 GB / 10 TB'. Below this, a table lists storage items: 'Total Selected' (0.00 KB), 'Raw Data' (52.36 GB), and 'Result Files & Attachments' (3.46 GB). The footer contains links for 'Home', 'About', 'Terms & Conditions', 'Privacy Policy', and 'Copyright © MEDIT Co., Ltd.'.

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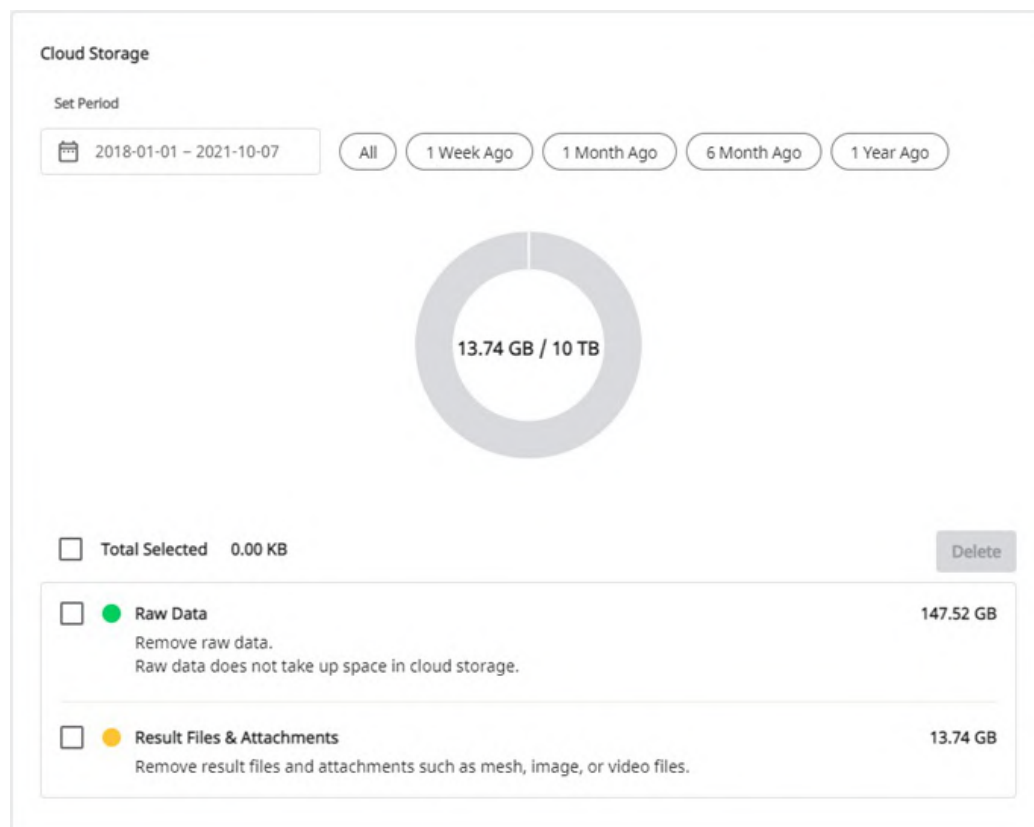
2 GB Plan 10 TB Plan

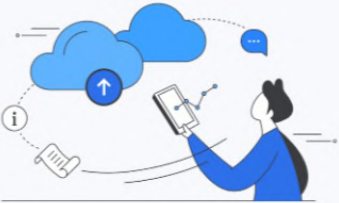
Free (Promotion)

- ✓ A paid plan
- ✓ 10 TB of cloud storage
- ✓ Efficient for users with many cases and various data
- ✓ One-month free trial included

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10 TB Plan

9.99\$ USD / Monthly

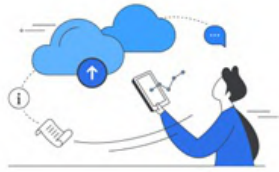
- ✓ Subscribe now to keep your 10 TB Plan after the promotion expires.
- ✓ You will pay a subscription fee after the current promotion expires. If you don't wish to pay, you can unsubscribe before payment.

* VAT is not included.
* Learn more about the Medit Cloud Storage Plan in our Q&A [Here](#).

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Cancel Subscription

Are you sure you want to cancel your **10 TB Plan**?

✓ By confirming, you will automatically be switched to the free 2 GB Plan.
Enjoy full access to the features and storage of your current plan through to 2025-02-20.

ⓘ Please note that all uploaded files will be retained regardless of the plan you choose.
If your storage exceeds the 2 GB limit, you won't be able to upload new files.

*Click [here](#) to learn more about our membership plans.

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Confirm Subscription Cancellation

Your subscription will be canceled on 2025-02-20.

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Confirm Subscription Cancellation

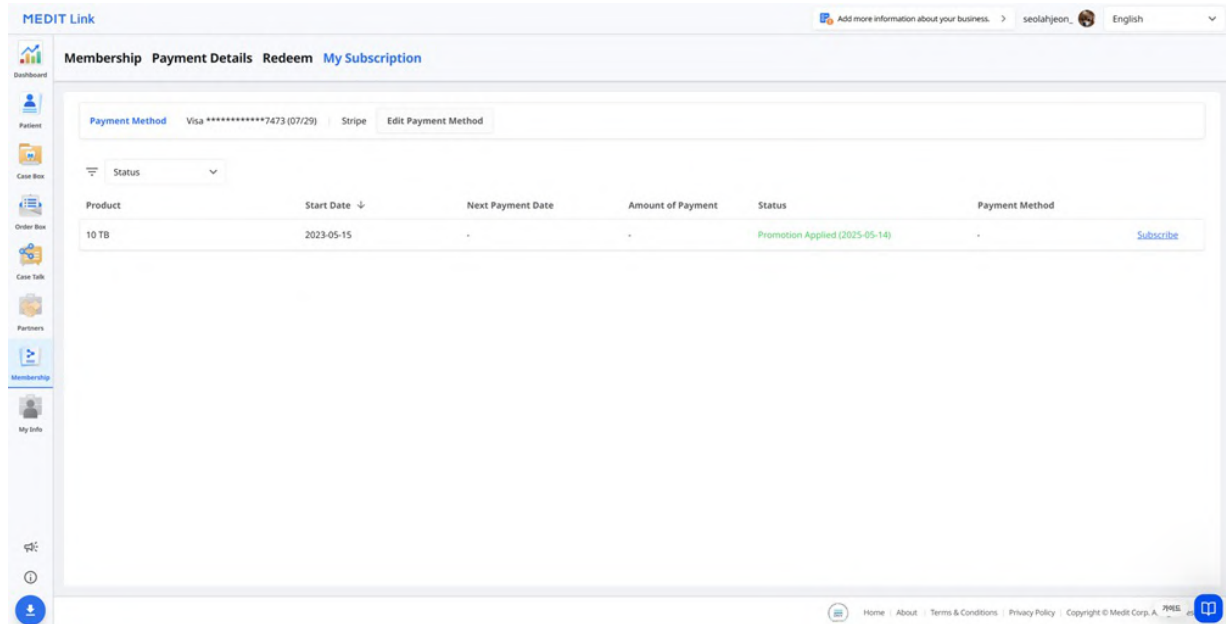
Your subscription has been canceled.
You can use your current plan until 2025-02-20. After that, the cloud capacity will be limited to 2 GB.
If the used space exceeds the limit, data will not be backed up, and functions such as Order and Share will not be available.

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Country

South Korea

By providing your card information, you allow Medit USA Inc to charge your card for future payments in accordance with their terms.

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CURRENT SUBSCRIPTION

[DEV] Medit Cloud Storage 10TB Plan for Clinics
\$9.99 per day

[View details](#) ▼

Your subscription renews on February 20, 2025.

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My Info

MEDIT Link

Add more information about your business. > seolah.jeon_dev_admin English

Membership Payment Details Redeem My Subscription

Register Redeem Code

Register

Redeem Code

All

	Product	Promotion	Code Number	Registration Date	From	To	Status	
4	Premium	4 Clinic Premium Promotion	2A6F-0125-6747-4AB7	2022-03-16	2021-10-01	2021-10-05	Waiting	Cancel

1

Home About Terms & Conditions Privacy Policy Copyright © Medit Corp. GUIDES

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Register Redeem Code

You can register for the promotion or apply immediately.
Register allows you to apply and activate when you need it, If applied immediately, it will start after the expiration of the active promotion.
Periodic payments are stopped automatically.

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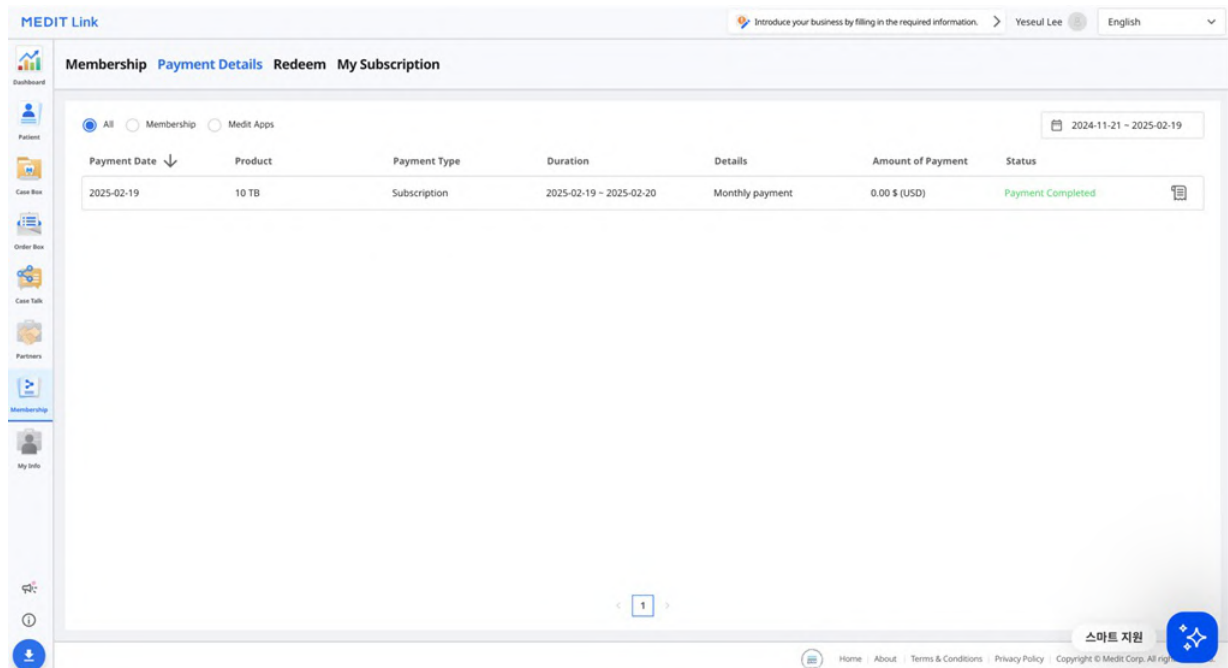
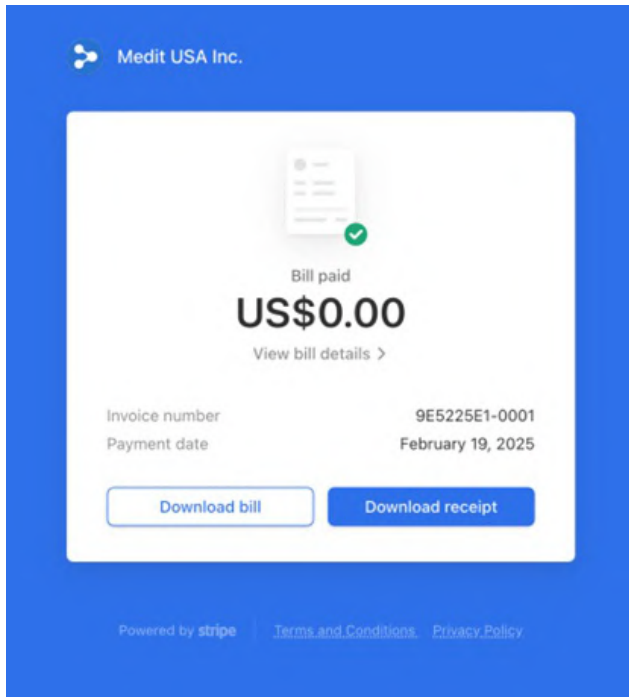
ITEM

FEBRUARY 19 - FEBRUARY 20, 2025	
Medit Cloud Storage 10TB Plan for Clinics	US\$0.60
Quantity 1 piece	
subtotal	US\$0.60
1 Month Free for New Subscribers(100% 할인)	
	-US\$0.60
Payment amount	US\$0.00
Amount paid	US\$0.00
Remaining amount	US\$0.00

Do you have any questions? [Contact Medit USA Inc.](#)

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Country/Region

Korea, Republic of ▼

📘 Your chosen language is applied to the email sent by Medit Link and the notification.

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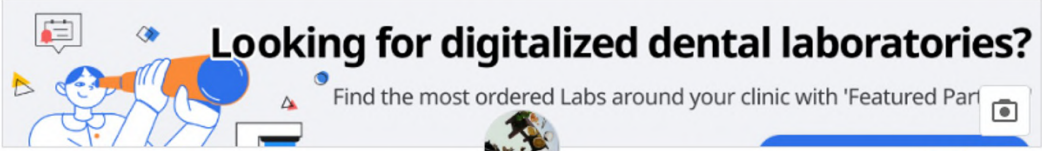
Edit

Cover and Logo

Looking for digitalized dental laboratories?
Find the most ordered Labs around your clinic with 'Featured Part

123

Services

Select

Specialties (choose up to 3)

Select

Select the services your business specializes in from the list.

About Us

e.g., Welcome to OOO Dental Clinic.
We are a licensed dental clinic situated in California specializing mainly in aesthetic restorations and implantology.

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Notice
NFT Service Termination Notice
2023-04-19

To our users,

We want to thank all Medit Link users who have supported and used our NFT service.

Unfortunately, we must inform you that this service will be terminated following the upcoming program update.

Please see the details below for more information:

- **Feature in Question**
 - Share via NFT (in Medit Link Case Talk)
- **Effective Date**
 - Starting from the update in the first half of 2023 (scheduled for April 27)

※ The update schedule is subject to change.

We'll continuously work to provide better service.

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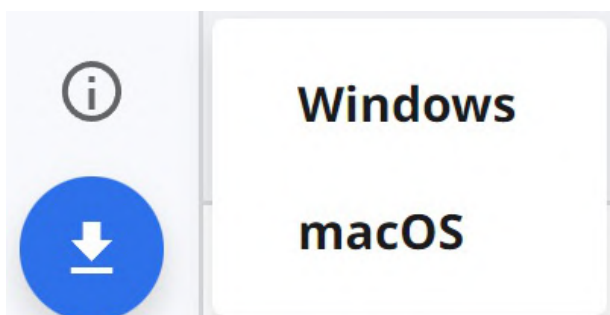
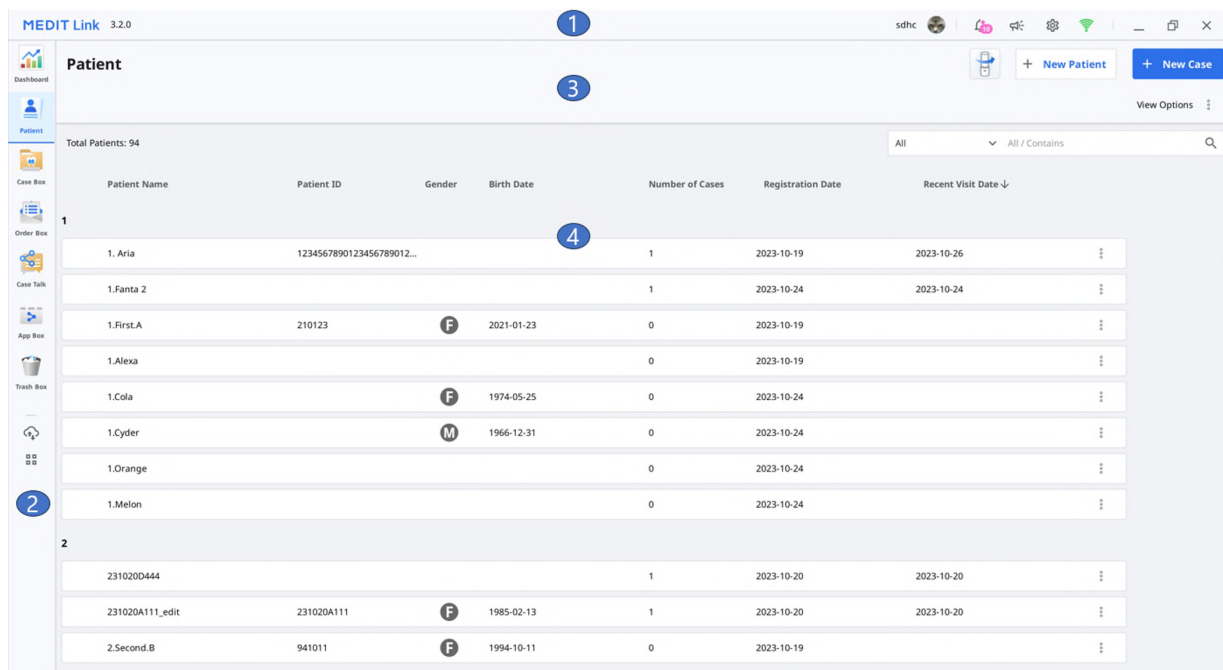
Mark All As Read

2

Delivery 2020-08-20 오후 5:20
Wonhoonchoi_Lab_Dev has dispatched the ordered package.

Order 2020-08-20 오후 5:18
Wonhoonchoi_Lab_Dev has accepted the order.

Order 2020-08-17 오후 4:05
Wonhoonchoi_Lab_Dev has accepted the order.

Order 2020-08-17 오후 4:05
Wonhoonchoi_Lab_Dev has accepted the order.

Order 2020-08-17 오후 4:05
Wonhoonchoi_Lab_Dev has accepted the order.

Order 2020-08-17 오후 4:05
Wonhoonchoi_Lab_Dev has accepted the order.

Work 2020-08-14 오후 6:33
Wonhoonchoi_Clinice_Dev_Member registered a new case test's Case.

Work 2020-08-14 오전 10:15
Wonhoonchoi_Clinice_Dev_Member registered a new case test's Case.

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Network speed is good.

Network speed is fast enough.

Network speed is poor. Can be switched to offline mode automatically.

Either network is not connected or Medit Link is in offline mode.

Synchronization is in progress. Some case lists may not match.

Synchronization failed. The case lists may not match.

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Merge all patient information to


Please select the patient you'd like to merge all data to.
After merging all data together, all information in other patient files will be deleted.

	Patient Name	Patient ID	Gender	Birth Date	Number of Cases	Date Registered	Recent Visit Date
☒	5-9_iScan_CAMOrd...				1	2020-09-18	2020-09-18
☐	test				2	2020-09-17	2020-09-17
☐	test				22	2019-07-26	2020-09-17

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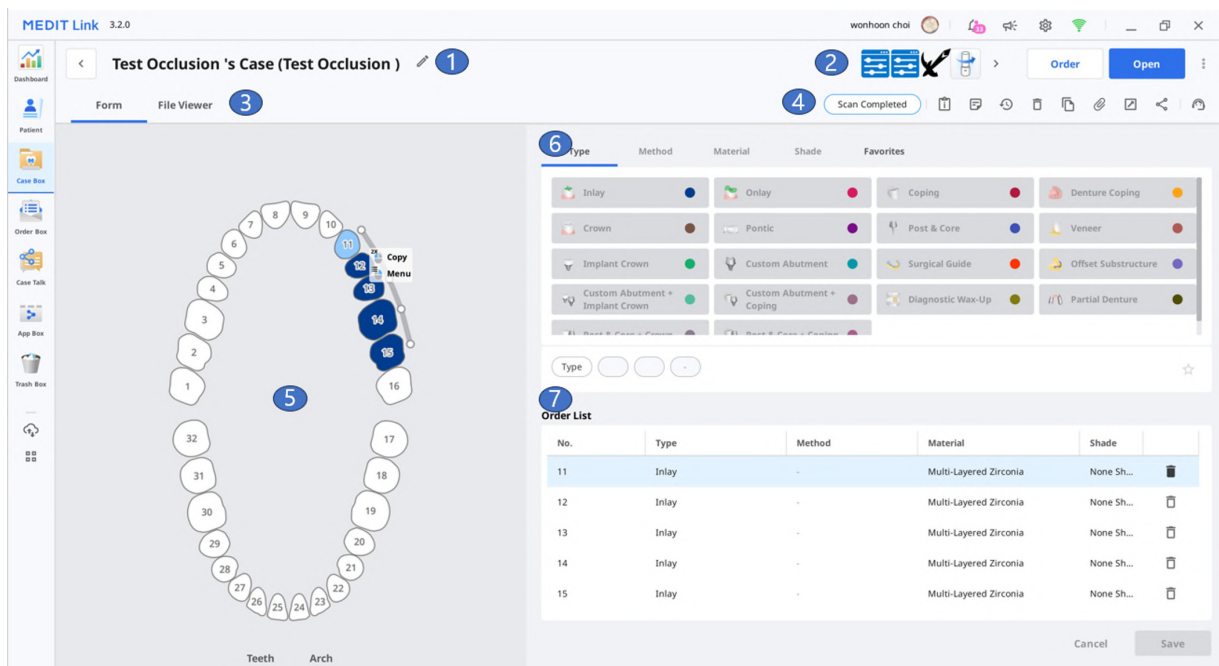Order Box/In Box

The screenshot shows the MEDIT Link 3.2.0 interface. The top header displays the case name 'Occlusion Test's Case (Occlusion Test)' and a 'Cancel Order' button. The main content area has two tabs: 'Order' and 'File Viewer'. The 'Order' tab is active, showing fields for Order ID, Patient Name, Partner Name, Scan Date, Order Date, Requested Delivery Date, and Expected Delivery Date. A 'Pending' status is shown. The 'File Viewer' tab is also visible, showing a table of order items.

Order List	Shipment		
Type	Method	Material	Shade
11 Inlay	-	Multi-Layered Zirconia	None Shade
12 Inlay	-	Multi-Layered Zirconia	None Shade
13 Inlay	-	Multi-Layered Zirconia	None Shade
14 Inlay	-	Multi-Layered Zirconia	None Shade

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Case Talk Name	Patient Name	Form Information	Scanned Date	Shared Date	Expiration Date
2022_04_06_MEDIT NFT	hk****	27 36	-	2022-04-06 13:40	unlimited
seolah.jeon	hk****	11, 12, 13, 21, 22, 23	-	2022-04-01 11:48	unlimited
2.6.0.38 TEST 2's Case - Clone - Clone_2022-03-31 18:46:02	2***** TE** 2	25, 26, 27, 33, 34, 35, 36	-	2022-03-31 18:46	unlimited

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Join the Medit Users Group on Facebook!
Connect with Experts, Learn Valuable Tips, and Elevate Your Skills
[Join Now](#)

Explore Medit Resources
News, tips, and expert insights on Digital Dentistry
[Visit Now](#)

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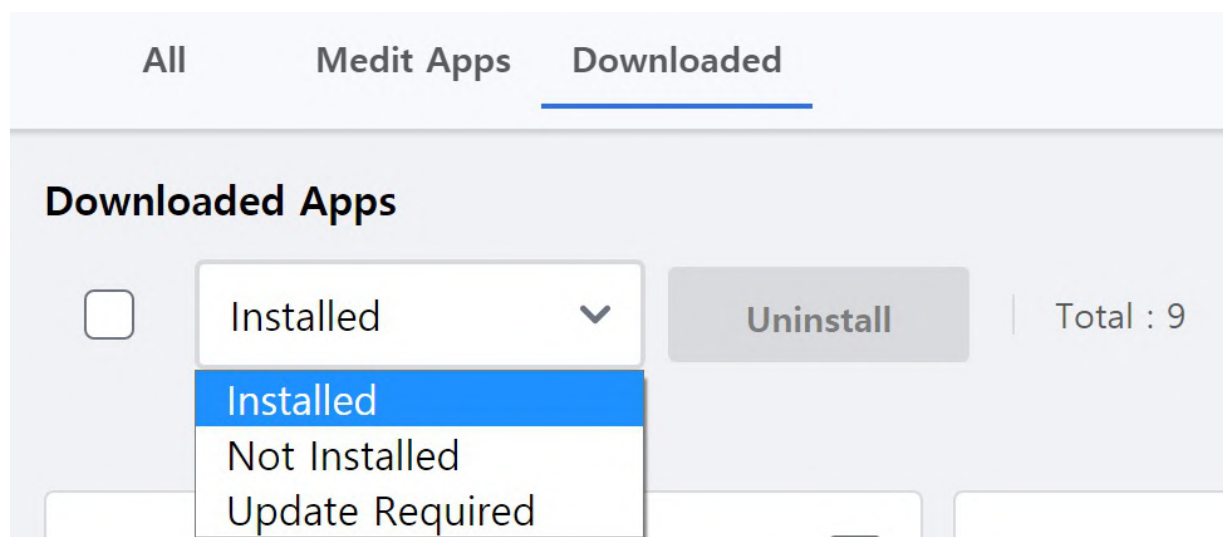
▼

All / Contains

Status	Case Name	Patient Name	Lab Name	Deletion Date ↓	Deleted By	Size
☐	Scan Completed	Jira_임플란트 46, 47's Case - Clone - Clone	임플란트 46, 47	2023-04-26 오전 9:12	wonhoon choi	66,654 KB
☐	Form Only	G11598492304180810429	test test	2023-04-18 오후 5:15	wonhoon choi	0 KB
☐	Form Only	G11598492304180806313	test test	2023-04-18 오후 5:15	wonhoon choi	0 KB
☐	Form Only	G11598492304180811349	tes test	2023-04-18 오후 5:15	wonhoon choi	0 KB
☐	Form Only	G11598492304180754083	Glidewell_fastdesign	2023-04-18 오후 5:00	wonhoon choi	0 KB
☐	Form Only	G11598492304180733332	Glidewell_fastdesign	2023-04-18 오후 5:00	wonhoon choi	0 KB
☐	Form Only	G11598492304180648146	test test	2023-04-18 오후 5:00	wonhoon choi	0 KB
☐	Case Completed	Multi Occlusion Normal Case's Case - Clone	Multi Occlusion Normal Case	2023-04-18 오전 10:50	wonhoon choi	23,652 KB
☐	CAD	Multi Occlusion Normal Case's Case - Clone - Clone	Multi Occlusion Normal Case	2023-04-18 오전 10:50	wonhoon choi	23,652 KB
☐	Form Only	Occlusion Test's Case	Occlusion Test	2023-04-17 오후 6:31	wonhoon choi	0 KB
☐	Form Only	Test Occlusion 's Case	Test Occlusion	2023-04-17 오후 6:31	wonhoon choi	0 KB
☐	Form Only	Occlusion Test's Case	Occlusion Test	2023-04-17 오후 6:31	wonhoon choi	0 KB
☐	Form Only	Occlusion Test's Case	Occlusion Test	2023-04-17 오후 6:26	wonhoon choi	0 KB

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Data Type

All

☐ Case Name	Patient Name	Scanned Date↓	Size	Status
☐ BLATRIX Charline's Case	BLATRIX Charline	2025-01-06 PM 6:28	729.30 MB	Upload Required
☐ BLATRIX Charline's Case	BLATRIX Charline	2025-01-06 PM 6:28	729.30 MB	Upload Required
☐ BLATRIX Charline's Case - 복제	BLATRIX Charline	2025-01-06 PM 6:28	733.35 MB	Upload Required
☐ BLATRIX Charline's Case - Clone	BLATRIX Charline	2025-01-06 PM 6:28	734.01 MB	Upload Required
☐ test의 케이스	Patient TEST	2023-04-26 PM 7:58	1.25 GB	Upload Required
☐ i700 HD's Case	i700 HD	2023-04-24 PM 5:23	911.24 MB	Upload Required
☐ Mandibular Movement	MEDIT	2023-04-24 PM 1:59	1.88 GB	Upload Required
☐ Mandibular Movement Case	MEDIT	2023-04-24 PM 1:59	1.75 GB	Upload Required
☐ Mandibular Movement Case - Clo...	MEDIT	2023-04-24 PM 1:59	1.75 GB	Upload Required
☐ 서라_Mandibular Movement_TEST	Mandibular Movement_export_test	2023-04-10 PM 3:39	34.73 MB	Upload Required
☐ 서라_Mandibular Movement_TEST...	Mandibular Movement_export_test	2023-04-10 PM 3:39	731.43 MB	Upload Required

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
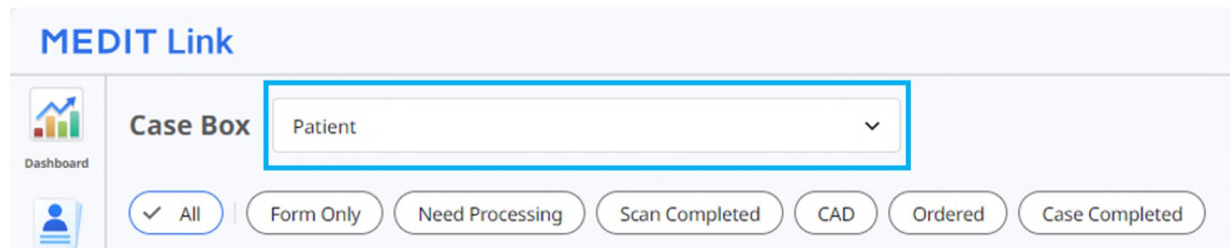
The screenshot shows the MEDIT Link interface. On the left is a sidebar with a 'Dashboard' icon and a user icon. The main area has a 'Case Box' dropdown menu currently set to 'Patient'. Below this are several filter buttons: 'All' (selected), 'Form Only', 'Need Processing', 'Scan Completed', 'CAD', 'Ordered', and 'Case Completed'.

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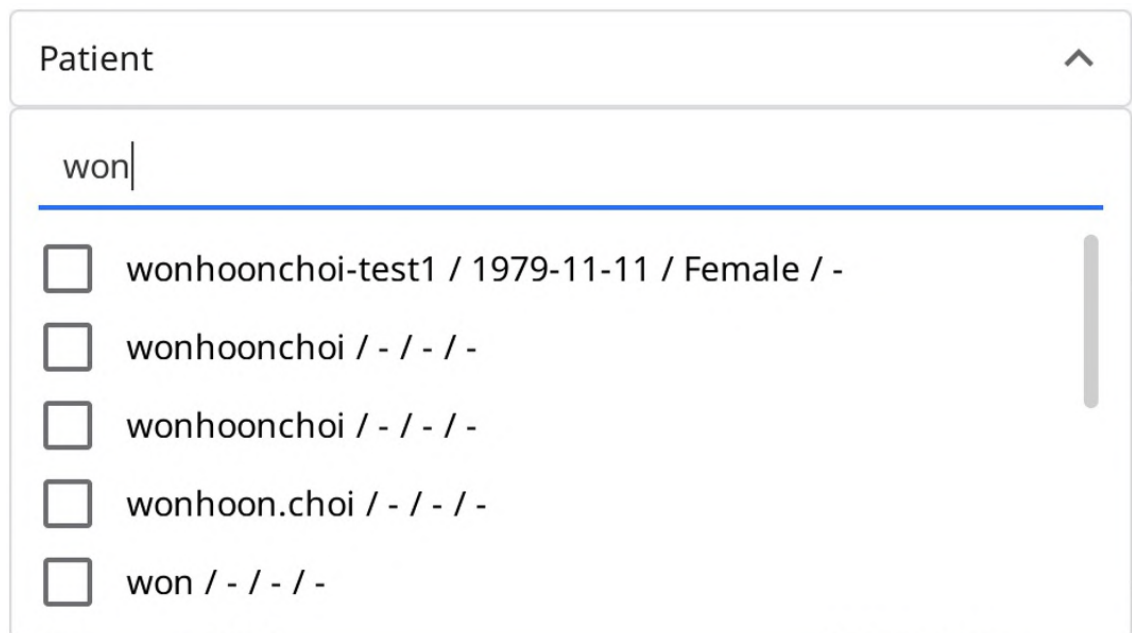This screenshot shows the dropdown menu for the 'Patient' filter. The search input field contains 'won'. Below the input, there is a list of search results, each preceded by a checkbox:

- ☐ wonhoonchoi-test1 / 1979-11-11 / Female / -
- ☐ wonhoonchoi / - / - / -
- ☐ wonhoonchoi / - / - / -
- ☐ wonhoon.choi / - / - / -
- ☐ won / - / - / -

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MEDIT Link 3.2.0 Wonhoon Chai Clinic

Case Box wonhoonchoi / - / - / - + 2

☒ All
 ☐ Form Only
 ☐ Need Processing
 ☐ Scan Completed
 ☐ CAD
 ☐ Ordered
 ☐ Case Completed

Total Cases: 20 2018-01-01-2023-10-24 All / Contains

Status	Patient Name	Case Name	Form Information	Last Modified Date ↓	Scan Date	Lab Name	Order Date
Form Only	wonhoonchoi	originalCase	-	2023-10-05 오전 8:10			
Form Only	wonhoonchoi	wonhoonchoi's Case	-	2023-10-05 오전 8:09			
Scan Completed	wonhoonchoi	FileViwerTest_1	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-06-15 오전 8:14	2023-02-09 오전 8:35		
Scan Completed	wonhoonchoi	FileViwerTest_1 - Clone	2-Inlay / 3-Inlay / 4-Inlay / 5-Inlay / 6-Inlay / 8...	2023-05-25 오후 2:05	2023-02-09 오전 8:35		
Form Only	wonhoonchoi	FileViwerTest_1 - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-03-15 오후 3:06			
Scan Completed	wonhoonchoi	FileViwerTest_1 - Clone - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-03-09 오후 4:41	2023-02-09 오전 8:35		
Need Processing	wonhoonchoi	FileViwerTest_1 - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-02-14 오전 8:13	2023-02-09 오전 8:35		
Scan Completed	wonhoonchoi	FileViwerTest_1 - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-02-10 오전 11:27	2023-02-09 오전 8:35		
Form Only	wonhoonchoi	testsamplecase	-	2023-02-10 오전 7:39			
Scan Completed	wonhoonchoi	Multi Occlusion	-	2023-02-09 오전 9:26	2023-02-09 오전 9:01		
Scan Completed	wonhoonchoi	FileViwerTest_1 - Clone - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-02-09 오전 8:50	2023-02-09 오전 8:35		
Case Completed	wonhoonchoi	wonhoonchoi's Case	12-Inlay / 13-Inlay / 14-Inlay	2022-10-13 오후 10:30	2022-10-11 오후 5:16		
Case Completed	wonhoonchoi	wonhoonchoi's Case	-	2022-10-13 오후 10:28			

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
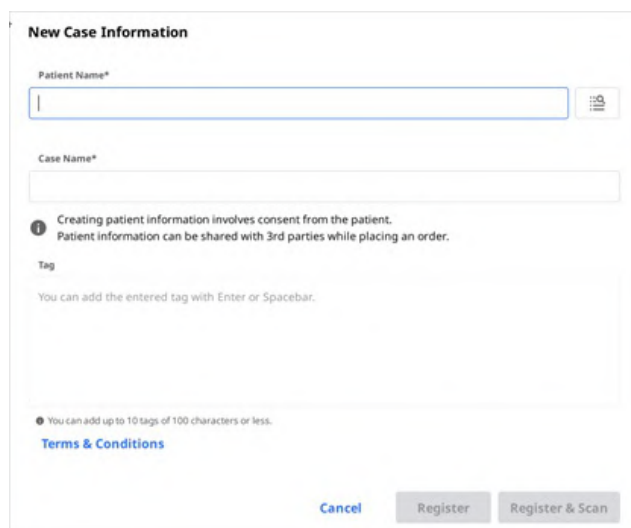
The form titled "New Case Information" contains the following fields and elements:

- Patient Name***: A text input field with a search icon on the right.
- Case Name***: A text input field.
- Information**: A note stating "Creating patient information involves consent from the patient. Patient information can be shared with 3rd parties while placing an order."
- Tag**: A text area with a hint "You can add the entered tag with Enter or Spacebar."
- Footer**: A note "You can add up to 10 tags of 100 characters or less." and a link "Terms & Conditions".
- Buttons**: "Cancel", "Register", and "Register & Scan".

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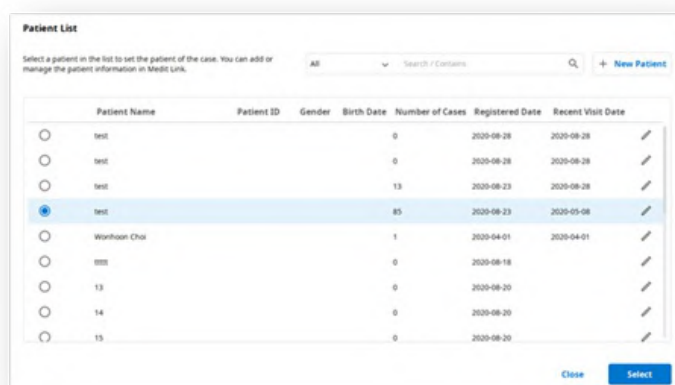
The "Patient List" dialog shows a table of existing patients. The table has columns: Patient Name, Patient ID, Gender, Birth Date, Number of Cases, Registered Date, and Recent Visit Date. A search bar and a "+ New Patient" button are at the top right.

	Patient Name	Patient ID	Gender	Birth Date	Number of Cases	Registered Date	Recent Visit Date	
☐	test				0	2020-06-28	2020-06-28	
☐	test				0	2020-06-28	2020-06-28	
☐	test				13	2020-06-23	2020-06-28	
☒	test				85	2020-06-23	2020-05-08	
☐	Wonhoon Choi				1	2020-04-01	2020-04-01	
☐	test				0	2020-06-18		
☐	13				0	2020-06-20		
☐	14				0	2020-06-20		
☐	15				0	2020-06-20		

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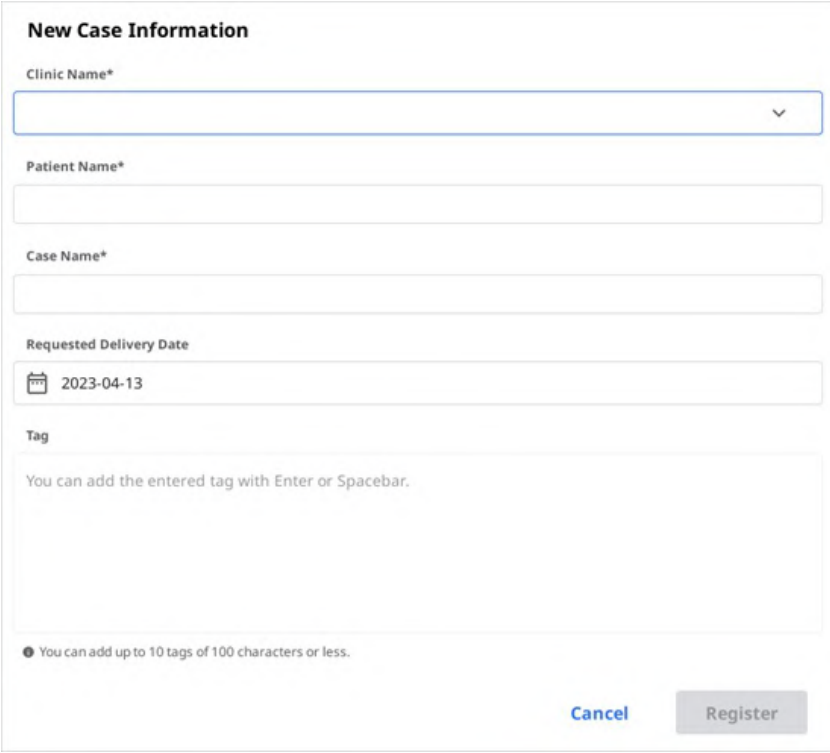The form is titled "New Case Information". It contains the following fields and elements:

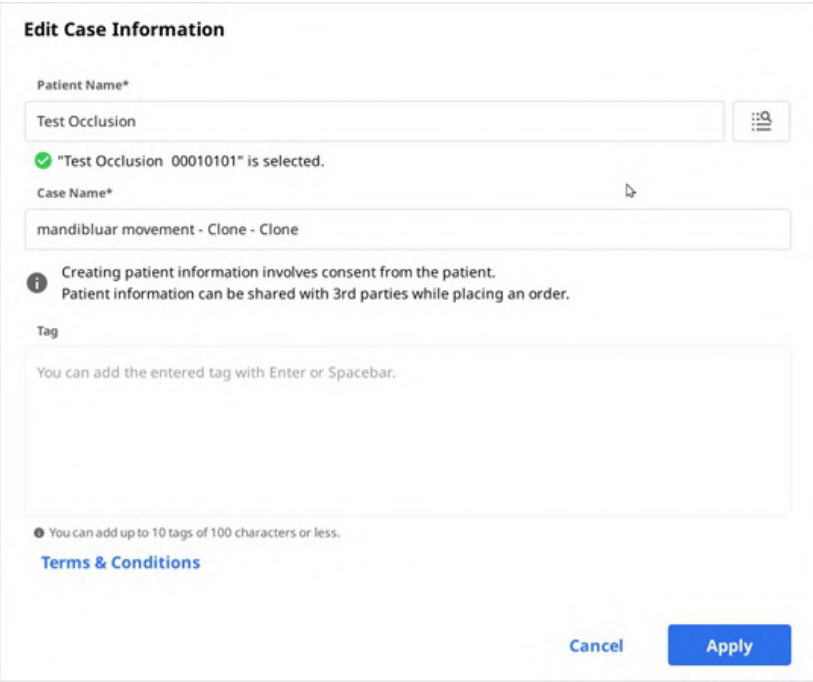
- Clinic Name***: A dropdown menu with a downward arrow on the right.
- Patient Name***: A text input field.
- Case Name***: A text input field.
- Requested Delivery Date**: A date picker showing "2023-04-13".
- Tag**: A text area with the instruction "You can add the entered tag with Enter or Spacebar." and a note at the bottom: "You can add up to 10 tags of 100 characters or less."
- At the bottom right, there are two buttons: "Cancel" (blue text) and "Register" (grey button).

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The form is titled "Edit Case Information". It contains the following fields and elements:

- Patient Name***: A text input field containing "Test Occlusion". To the right of the field is a menu icon (three horizontal lines).
- A green checkmark icon followed by the text: "Test Occlusion 00010101" is selected.
- Case Name***: A text input field containing "mandibluar movement - Clone - Clone".
- An information icon (i) followed by the text: "Creating patient information involves consent from the patient. Patient information can be shared with 3rd parties while placing an order."
- Tag**: A text area with the instruction "You can add the entered tag with Enter or Spacebar." and a note at the bottom: "You can add up to 10 tags of 100 characters or less."
- A link labeled "Terms & Conditions" in blue text.
- At the bottom right, there are two buttons: "Cancel" (blue text) and "Apply" (blue button).

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Patient List

Select a patient in the list to set the patient of the case. You can add or manage the patient information in Medit Link.

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2022-04-18

Tag

You can add the entered tag with Enter or Spacebar.

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MEDIT Link 3.2.0

Case Box wonhoonchoi-test1 / 1979-11-11 / Female / - + 2 ▼

☒ All |
 ☐ Form Only |
 ☐ Need Processing |
 ☐ Scan Completed |
 ☐ CAD |
 ☐ Ordered |
 ☐ Case Completed

☒ **Export** ▼ |
 Status ▼ |
 |
 Selected Cases: 4

	Status	Patient Name	Case Name	Form Information
☒	Scan Completed	wonhoonchoi-test1	wonhoonchoi-test1's Case	-
☒	Form Only	wonhoonchoi-test1	wonhoonchoi-test1's Case	-
☒	Form Only	wonhoonchoi	originalcase	-
☒	Scan Completed	wonhoonchoi-test1	wonhoonchoi-test1's Case	-

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☐ Convert images to the below format

☐ JPG
☐ JPEG
☒ PNG
☐ BMP

☐ Compress Files
☐ Combine Individual Mesh
☐ Export All Occlusal Relationship
☐ Save Case Information as PDF

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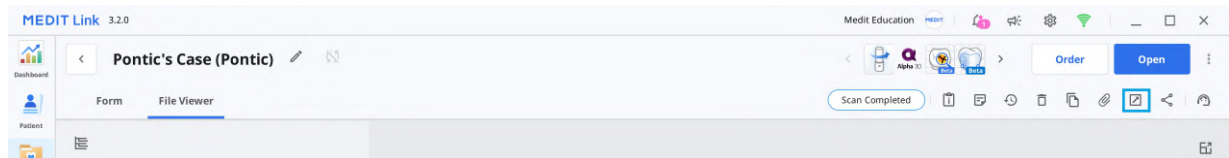
Options File List

Export Files To
C:/Users/Medit-Wonhoonchoi/Desktop/test/

Folder Name
Multi Occlusion - Scan - Before Scanning

File Name
Multi Occlusion - Scan - Before Scanning

CAD Type
Maestro

Mesh Format

Convert mesh files to the below formats

☐ meditMesh ☐ OBJ ☒ **PLY** ☐ STL

☒ Do not convert the attached files

Image Format

☐ Convert images to the below format

☐ JPG ☐ JPEG ☒ **PNG** ☐ BMP

☐ Compress Files

☐ Combine Individual Mesh

☐ Export All Occlusal Relationship

☐ Save Case Information as PDF [Preview](#)

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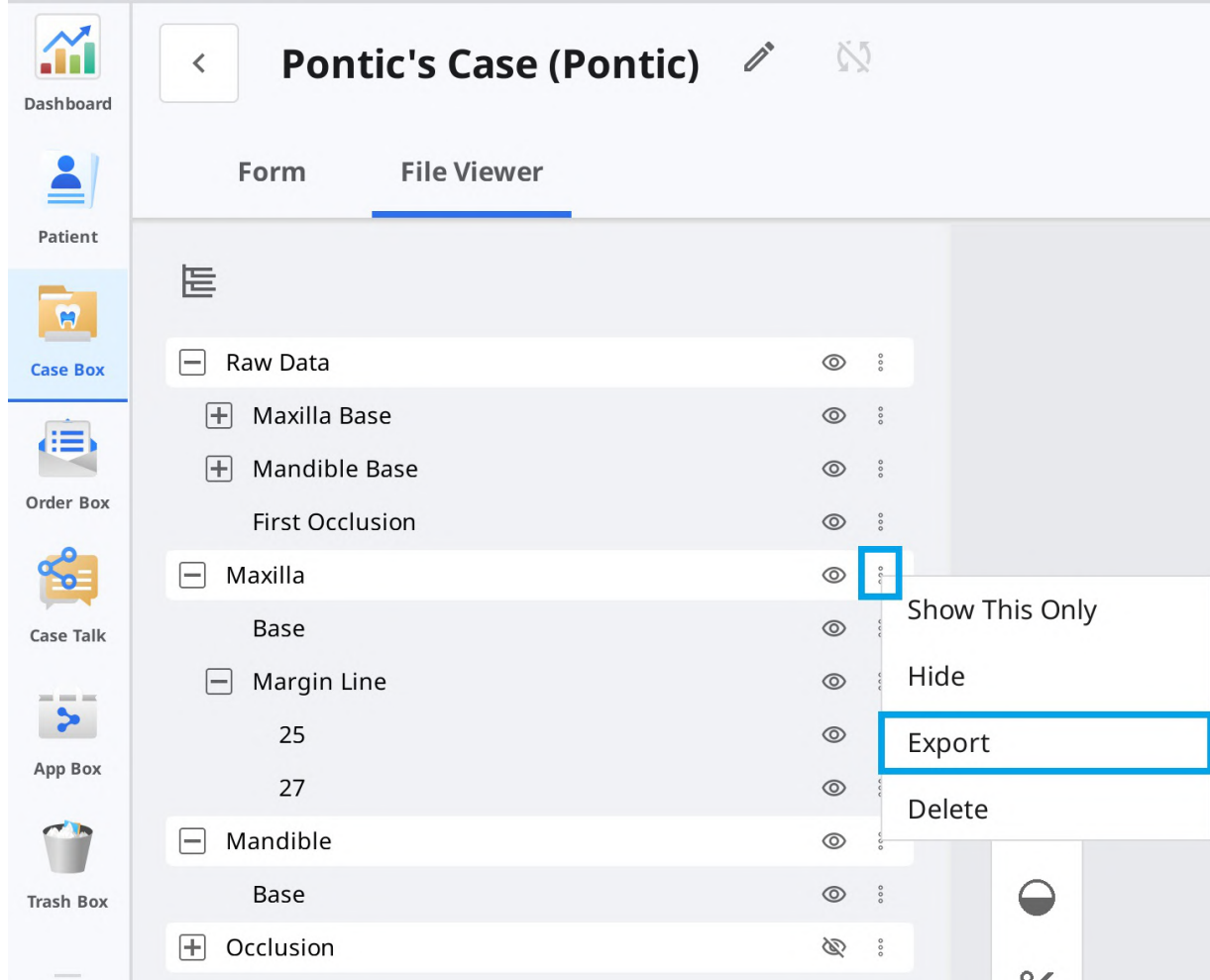
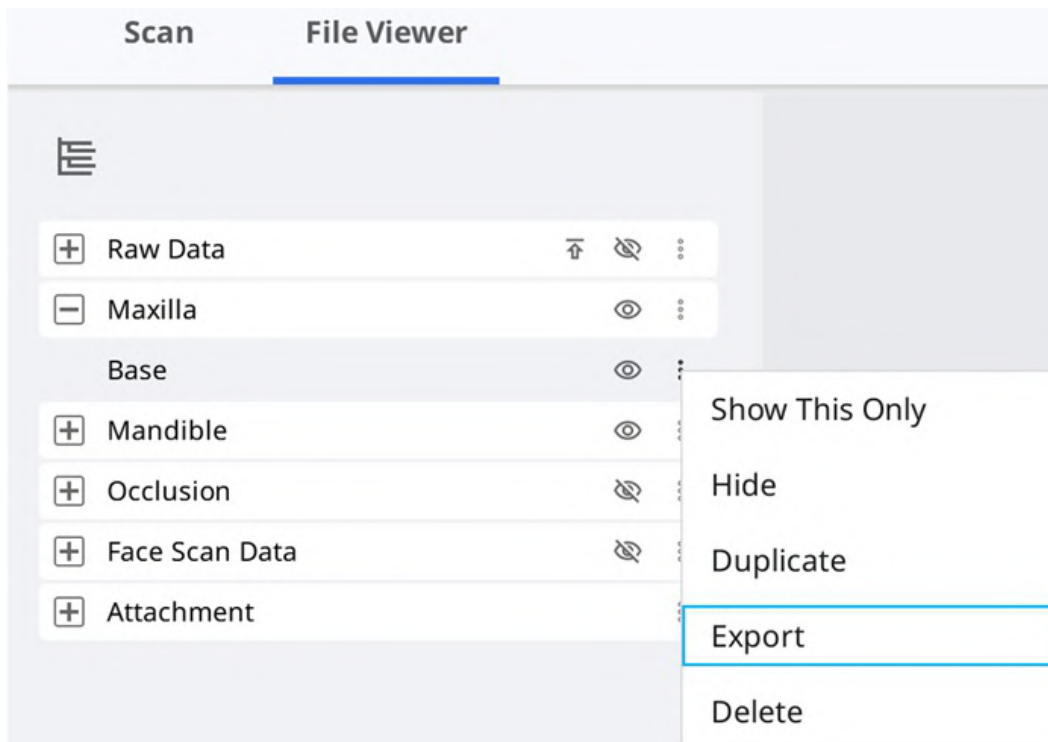
Question

? Exporting successfully completed.
Do you want to open the folder containing the exported file?

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MEDIT Link 3.2.0 Wonhoon Choi Clinic

Case Box wonhoonchoi-test1 / 1979-11-11 / Female / - +2

Export Status Selected Cases: 4 2018-01-01-2023-10-24 All / Contains

Status	Patient Name	Case Name	Form Information	Last Modified Date	Scan Date	Lab Name	Order Date
Scan Completed	wonhoonchoi-test1	wonhoonchoi-test1's Case	-	2023-10-19 오전 11:00			
Form Only	wonhoonchoi-test1	wonhoonchoi-test1's Case	-	2023-10-05 오전 8:15			
Form Only	wonhoonchoi	originalcase	-	2023-10-05 오전 8:10			
Scan Completed	wonhoonchoi-test1	wonhoonchoi-test1's Case	-	2023-10-05 오전 8:10	2023-08-18 오전 9:00		

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Share through Case Talk

Share with Anyone

Create a shared link and share it with anyone.

Expiration Date: 2022-04-27

PIN Code: ☐ PIN Code

You can extend the period up to 90 days from today.

Set the PIN code using six digits for privacy protection.

☒ Share the patient's name

☒ Grant Download Permission [Premium Feature](#)

If you check this option, all users who have access to the shared link can download the files.

The premium feature is provided to all users for a limited time. Feature availability is subject to change depending on the membership plan.

☒ Allow comments

If you check this option, you will be able to communicate through the shared link.

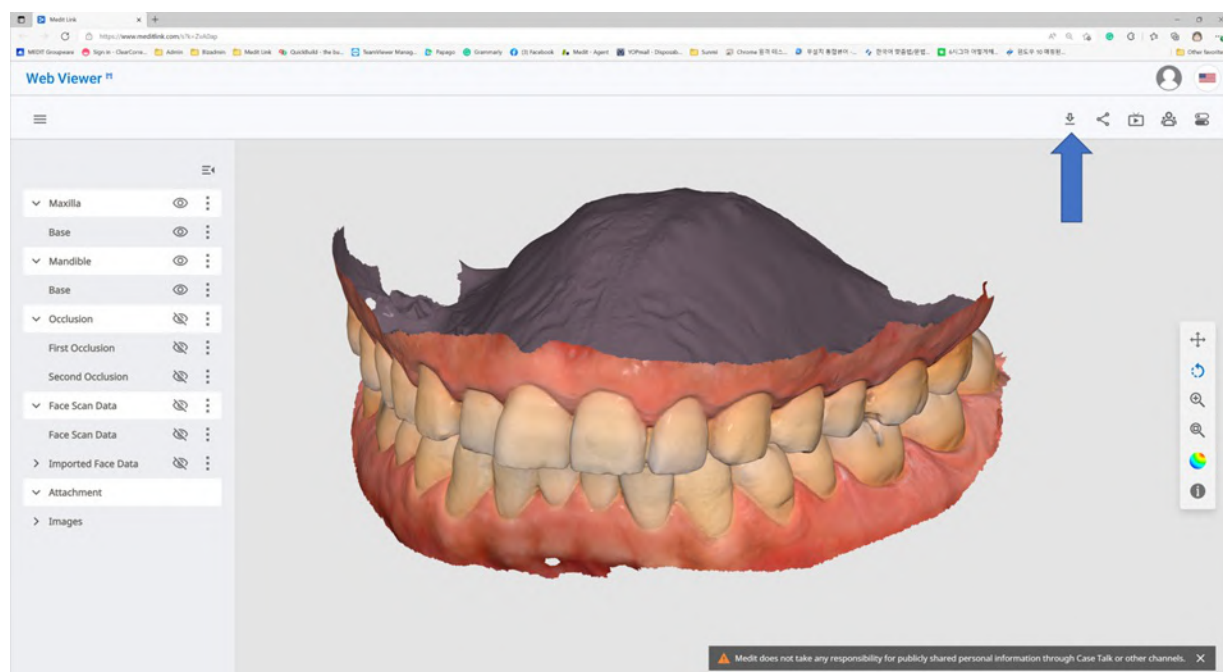
Comment: Please enter your comment.

0/1024

By pressing the "Get Link" button, you agree that you understand personal information can be collected and provided to a third party (or shared in any public domain) ONLY with the consent of the person whose images on which you have worked. You also agree the consent of that person was truthfully obtained, and Medit is not responsible for any problems which may arise there after.

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Download Premium Feature

The premium feature is provided to all users for a limited time.
Feature availability is subject to change depending on the membership plan.

Mesh Format

Convert mesh files to the below formats

☐ meditMesh ☐ OBJ ☒ PLY ☐ STL

i The selected format will be applied to the result files created in the Medit Link application.

i The attachment files will not be converted but will keep their original format.

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Case Name
mandibular movement - Clone - Clone

Patient Name
Test Occlusion

☒ Share patient information

Requested Delivery Date
2023-04-13

Delivery Time
오전 8:15 AM 8:00 PM 8:00

Memo

Tags
You can add the entered tag with Enter or Spacebar.

You can add up to 10 tags of 100 characters or less.

☒ Include all occlusion relationships in the order

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Selected Cases: 4

Case Completed

	Status		Case Name
☐	Ordered	test	test's Case
☒	Scan Completed	wonhoonchoi-test1	wonhoonchoi-test1's Case
☒	Scan Completed	test test(1234)	G11598492310180503424
☒	Scan Completed	test	test's Case
☒	Scan Completed	test	test's Case

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Patient Name*

Case Name*

Creating patient information involves consent from the patient.
Patient information can be shared with 3rd parties while placing an order.

Tag

Doctorname × Specialmaterial × SpecialComment ×

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Patient Name*

Case Name*

i Creating patient information involves consent from the patient.
Patient information can be shared with 3rd parties while placing an order.

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Memo

Tags

Doctorname × Specialmaterial × SpecialComment ×

i You can add up to 10 tags of 100 characters or less.

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