USER GUIDE

Medit Link (Revision 9) 2024.04

Table of Contents

Medit Link

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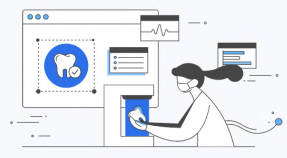
Clinic

Lab

Administrator Dentist, Manager Have the authority to create a new dental clinic account and have full access to Medit Link. ✓ Scan/Order/Manage orders ✓ Manage members Register	Administrator Laboratory Director, Manager Have the authority to create a new dental lab account and have full access to Medit Link. ✓ Scan/Manage ordered cases ✓ Manage members Register
Member Dental Hygienist, Dental Staff Belong to the dental clinic their admin created and have limited access to Medit Link after the admin's approval. ✓ Scan/Order/Manage orders Register	Member Dental Technician, Laboratory Staff Belong to the dental lab their admin created and have limited access to Medit Link after the admin's approval. ✓ Scan/Manage ordered cases Register

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Email Address

Check

ⓘ Please ensure your email address is active. Medit Link will send a verification email.
 ⓘ Registering with a disposable and/or fake e-mail address can make you vulnerable to security issues and make it difficult to receive information from Medit Link.

Password

ⓘ Password must be 8 to 16 characters long, including a combination of at least three letters, numbers, and special characters. (Passwords are accepted in English language only.)

Confirm Password

ⓘ Please type in the same password as above.

Select Country/Region

Select Country/Region

☐ I agree to all of the Terms and Conditions.
☐ Personal Information Collection Agreement (Mandatory)

View

☐ Terms & Conditions (Mandatory)

View

☐ Agree to the Marketing and Promotion Agreement (optional)

View

 ⓘ Check this option to receive marketing emails, which include product updates, software news, and promotional events.

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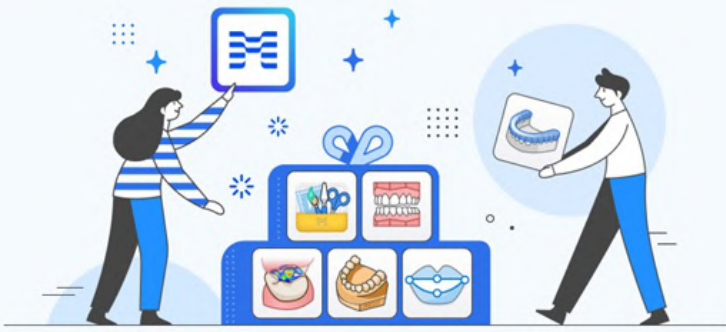
Welcome, stage-test-wh!

Utilize Various Apps

Download Medit Link to your desktop to utilize scanning programs and various apps.
All data generated is synced to the cloud and is also available on the web.

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The screenshot shows the MEDIT Link Case Box interface. The top navigation bar includes the MEDIT Link logo and a user profile for Wonhoon Choi. The main section is titled 'Case Box' and features a filter bar with buttons for 'All', 'Form Only', 'Need Processing', 'Scan Completed', 'CAD', 'Ordered', and 'Case Completed'. The 'Form Only' button is currently selected. Below the filter bar, a table displays a list of cases. The table has columns for Status, Case Name, Patient Name, Form Information, Last Modified Date, Scanned Date, Order Date, and Lab Name. The cases are filtered to show only those with the 'Form Only' status.

Status	Case Name	Patient Name	Form Information	Last Modified Date	Scanned Date	Order Date	Lab Name
Form Only	test's Case	test		2023-11-03 12:50	-	-	-
Scan Completed	wonhoonchoi-test1's Case - Clone - Clone	wonhoonchoi-test1	23, 24, 25, 26	2023-11-02 19:37	2023-11-02 09:01	-	-
Scan Completed	wonhoonchoi-test1's Case - Clone	wonhoonchoi-test1	23, 24, 25, 26	2023-11-02 19:21	2023-11-02 09:01	-	-
Ordered	wonhoonchoi-test1's Case	wonhoonchoi-test1	23, 24, 25, 26	2023-11-02 19:21	2023-11-02 09:01	2023-11-02 ...	Wonhoonchoi_lab2
Scan Completed	G11598492311021004395	test test (12345)	21	2023-11-02 19:09	-	-	-
Scan Completed	test's Case	test		2023-11-01 13:32	-	-	-
Scan Completed	G11598492311010319123	test test (test)	21	2023-11-01 12:20	-	-	-
Scan Completed	test's Case	test		2023-10-26 17:25	2023-10-26 17:24	-	-
Scan Completed	wonhoonchoi-test1's Case	wonhoonchoi-test1		2023-10-26 17:22	2023-10-26 17:21	-	-
Ordered	test's Case	test	14, 15, 16, 17	2023-10-24 15:45	-	2023-10-24 ...	Wonhoonchoi_lab2

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Case Box(診所)	All / Form Only / Need Processing / Scan Completed / CAD / Ordered / Case Completed
Work Box(技工所)	All / Form Only / Need Processing / Scan Completed / CAD / Done

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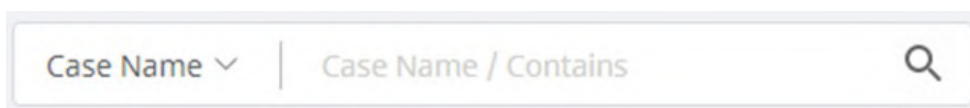
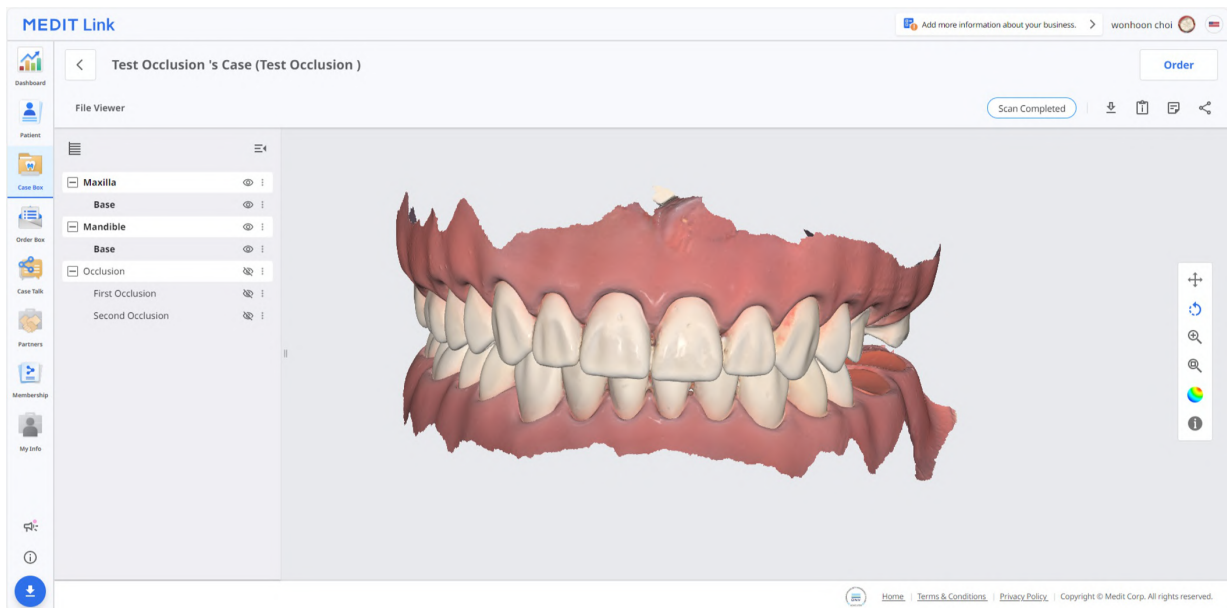
The screenshot displays the 'MEDIT Link' web application interface. The main header shows the title 'Occlusion Test's Case (Occlusion Test)' and a 'Cancel Order' button. The left sidebar contains navigation icons for Dashboard, Patient, Case Box, Order Box, Case Talk, Partners, Membership, and My Info. The main content area is divided into two sections: 'Order' and 'File Viewer'. The 'Order' section contains fields for Order ID (5996137), Patient Name (Occlusion Test), Partner Name (WonhoonChoi_Lab_Stage), Scanned Date (2023-04-27), Order Date (2023-04-20 PM 02:08), Requested Delivery Date (2023-04-27), and Expected Delivery Date (2023-04-27). The 'File Viewer' section displays a table of products with columns: No., Type, Method, Material, and Shade. The table lists four items (23, 24, 25, 26) all of Type 'Inlay', Method '-', Material 'Multi-Layered Zi...', and Shade 'None'. The status 'Pending' is shown at the top right of the main content area.

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A screenshot of the 'Create Annotation' dialog box. It has a title 'Create Annotation' and a prompt 'Please enter your annotation.' Below this is a text input field with placeholder text 'Please enter your message'. At the bottom are two buttons: 'Cancel' and 'OK'.

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A screenshot of the interface for replying to an annotation. At the top, it shows the user 'wonhoonchoi Beta wonhoonchoi' with delete and close icons. Below is the annotation title 'Annotation 1' and a timestamp '1 09-09 20:31'. At the bottom is a text input field with placeholder text 'Please enter your reply' and a 'Send' button.

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Invite Partner

Invite your desired partner to join Medit Link. When the lab accepts your invitation, a partnership is automatically formed.

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Membership

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The screenshot displays the Medit Link Membership page. The left sidebar contains navigation icons for Dashboard, Patient, Case Box, Order Box, Case Talk, Partners, Membership, and My Info. The main content area is titled 'Membership' and includes tabs for 'Membership', 'Billing Info', and 'Redeem'. The 'Plan' section shows the 'Premium' plan at \$0.99 per user per month, with benefits like a paid plan, 10 TB of cloud storage, and a one-month free trial. Below this, 'Premium Benefits' are listed, including 'Sample Coupon 1' (Medit Partner, expires 2023-02-25), 'Sample Coupon 2' (Medit Partner, expires 2023-02-25), '5% Off' (Dental Company, expires 2024-04-27), and 'Sample Boucher' (expires 2024-04-30). A 'Subscribe' button is present. The 'Cloud Storage' section shows a usage of 4.75 GB / 10 TB, with a 'Delete' button and a list of items to be removed: 'Raw Data' (30.51 GB) and 'Result Files & Attachments' (4.72 GB).

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This screenshot shows the 'Standard' plan details on the Medit Link Membership page. The 'Plan' section indicates a price of \$0.99 per user per month and lists benefits: a paid plan, 10 TB of cloud storage, efficiency for many cases, and a one-month free trial. The 'Premium Benefits' section includes 'Sample Coupon 1' (Medit Partner, expires 2023-02-25), 'Sample Coupon 2' (Medit Partner, expires 2023-02-25), '5% Off' (Dental Company, expires 2024-04-27), and 'Sample Boucher' (expires 2024-04-30). A 'Subscribe' button is visible. A note at the bottom states: 'The promotion has been applied. You can now use the plan until 2021-10-01. View More. If you have any difficulties with payment, select "Click" on the right. Click.'

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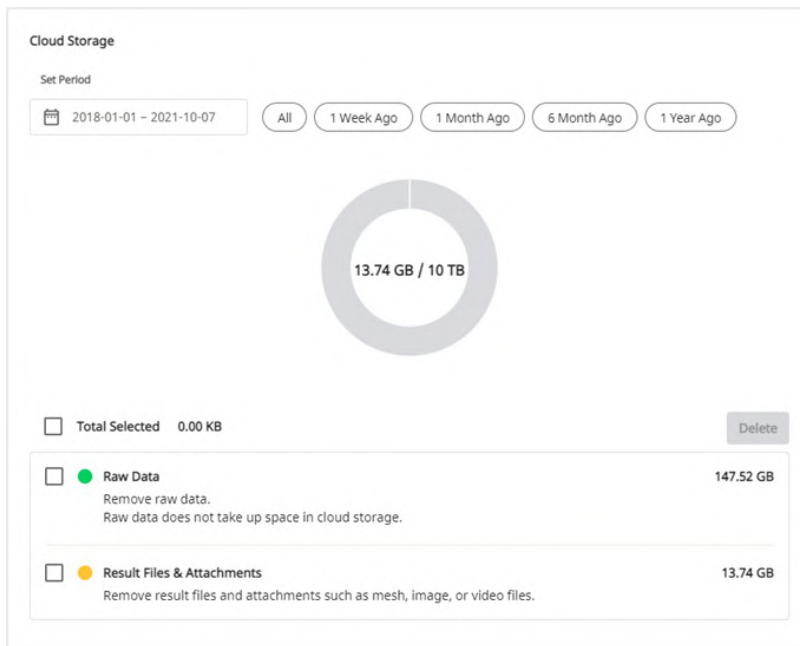
← MEDIT Link

TEST MODE

Subscribe to Premium membership for Clinic

\$0.00

Then \$0.99 per day after coupon expires

Premium membership for Clinic

Medit Link Membership

Billed daily

Subtotal

\$0.99

1 MONTH FREE – for first subscription

- \$0.99

100% off for a day

Total due today

\$0.00

Powered by stripe

Terms

Privacy

Pay with link ⇒

Or pay with card

Email

yhgam0829@gmail.com

Card information

1234 1234 1234 1234

VISA

MM / YY

CVC

Cardholder name

Full name on card

Country or region

South Korea

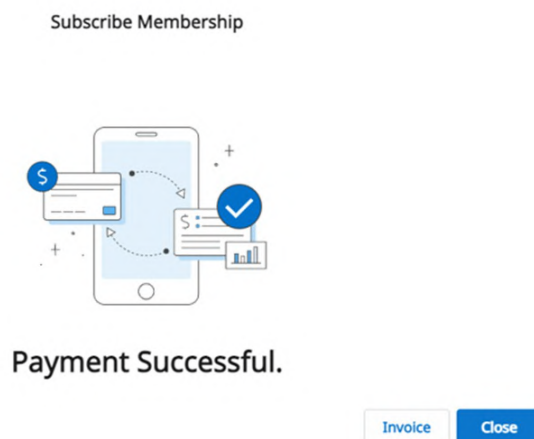
☐ Securely save my information for 1-click checkout

Pay faster on MEDIT SG PTE. LTD. and everywhere Link is accepted.

Subscribe

By confirming your subscription, you allow MEDIT SG PTE. LTD. to charge you for future payments in accordance with their terms. You can always cancel your subscription.

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
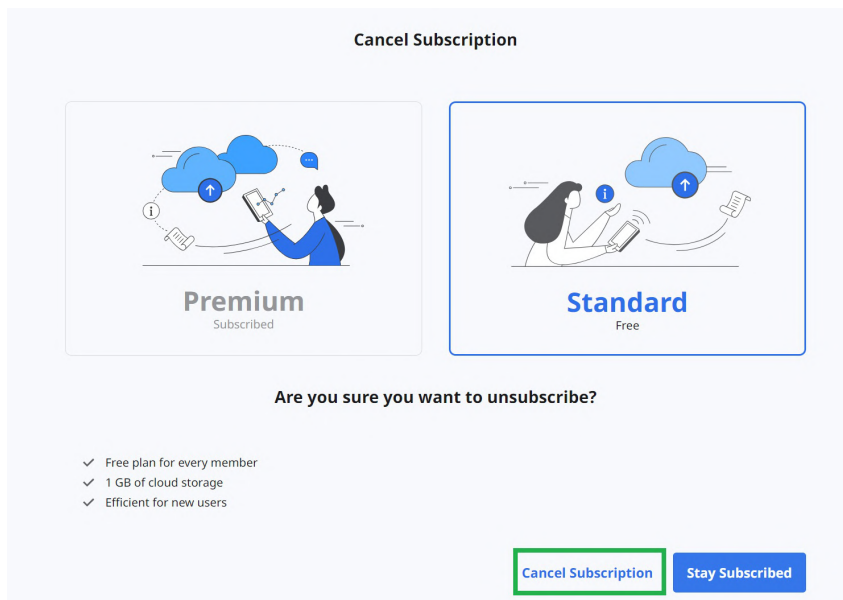
Confirm Unsubscribe

Your subscription will be canceled at the end of your billing period on 2021-09-10

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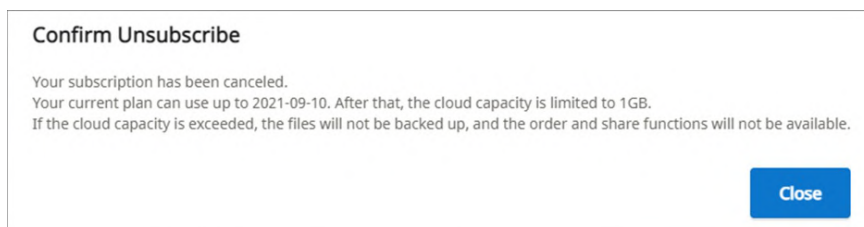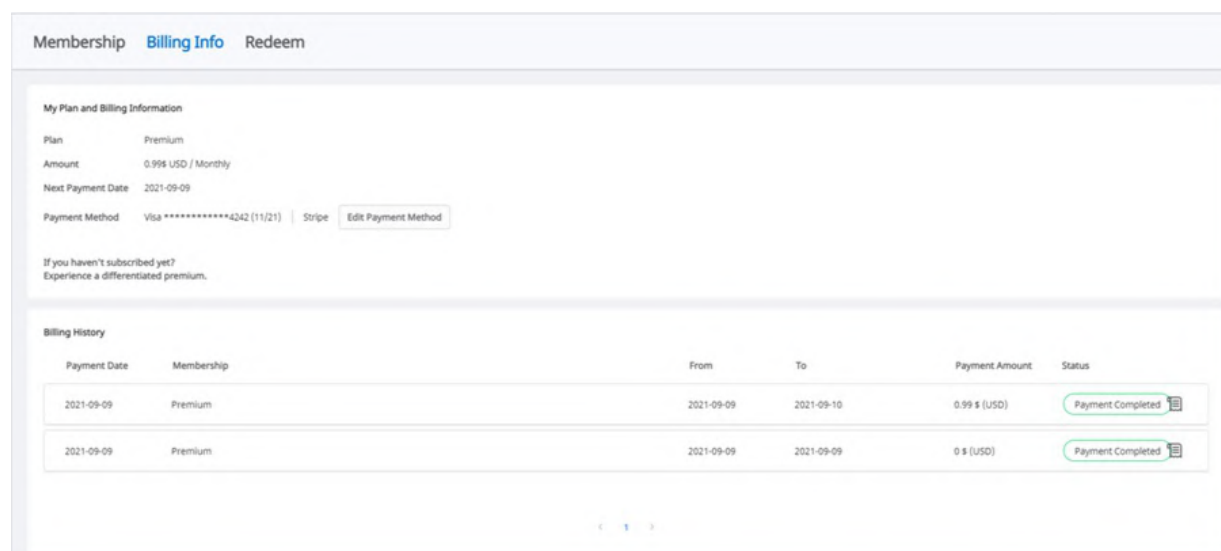
Amount 0.99 USD / Monthly

Next Payment Date 2021-09-09

Payment Method Visa **** *4342 (11/21) | Stripe **Edit Payment Method**

If you haven't subscribed yet?
Experience a differentiated premium.

Billing History

Payment Date	Membership	From	To	Payment Amount	Status
2021-09-09	Premium	2021-09-09	2021-09-10	0.99 \$ (USD)	Payment Completed
2021-09-09	Premium	2021-09-09	2021-09-09	0 \$ (USD)	Payment Completed

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Add

Go back

By adding a payment method, you agree to MEDIT SG PTE. LTD.'s
Terms of Service and Privacy Policy.

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Billing

CURRENT PLANS

Premium membership for Clinic
\$0.99 per day
Your plan renews on September 10, 2021.

Premium membership for Clinic
\$0.99 per day
Your plan renews on September 10, 2021.

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1

Home Terms & Conditions Privacy Policy Copyright © Medit Corp. All rights reserved.

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Register Redeem Code

You can register for the promotion or apply immediately.
Register allows you to apply and activate when you need it, If applied immediately, it will start after the expiration of the active promotion.
Periodic payments are stopped automatically.

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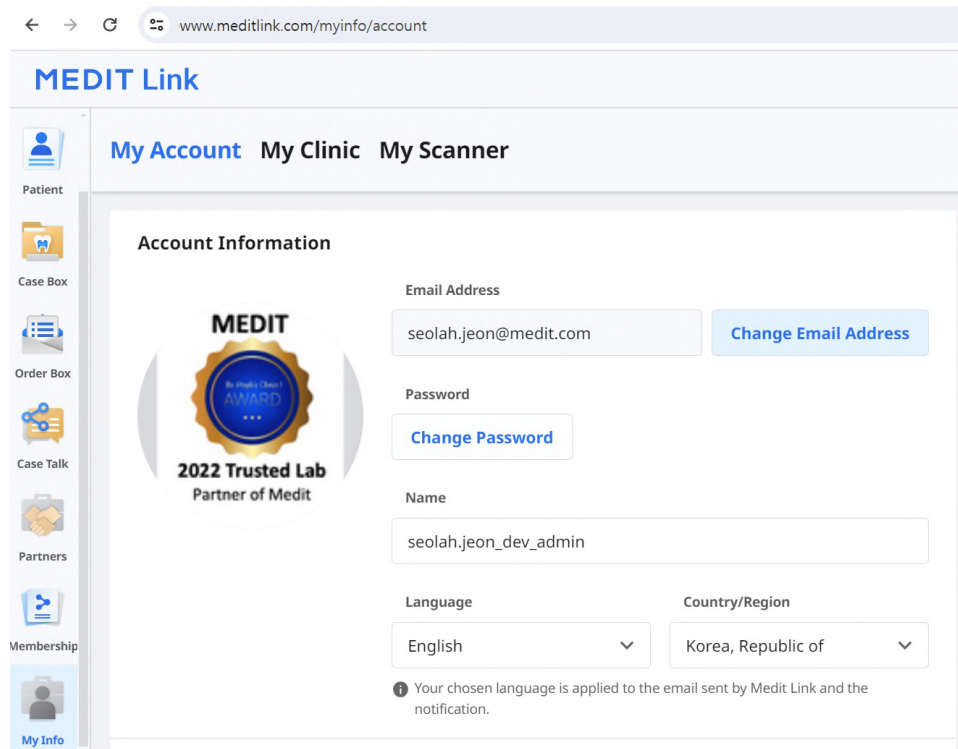
The screenshot shows a web browser at the URL www.meditlink.com/myinfo/account. The page is titled "MEDIT Link" and has a navigation bar with "My Account", "My Clinic", and "My Scanner". A left sidebar contains icons for Patient, Case Box, Order Box, Case Talk, Partners, Membership, and My Info. The main content area is titled "Account Information" and features a MEDIT logo with a "2022 Trusted Lab Partner of Medit" badge. The account details are as follows:

Field	Value	Action
Email Address	seolah.jeon@medit.com	Change Email Address
Password		Change Password
Name	seolah.jeon_dev_admin	
Language	English	
Country/Region	Korea, Republic of	

ⓘ Your chosen language is applied to the email sent by Medit Link and the notification.

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Edit

Cover and Logo

Looking for digitalized dental laboratories?

Find the most ordered Labs around your clinic with 'Featured Part

123

Services

Select

Specialties (choose up to 3)

Select

Select the services your business specializes in from the list.

About Us

e.g., Welcome to OOO Dental Clinic.
We are a licensed dental clinic situated in California specializing mainly in aesthetic restorations and implantology.

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MEDIT Link

My Account My Clinic My Scanner

Add more information about your business.

seolah.jeon

No.	Type	Serial Number	Usage Time	Last Connected	Last Calibrated	Warranty Start Date	Warranty Expiry Da...	Days Left	
1	i500	-	-	-	-	-	-	-	
2	i500	AA1911104453	0d 7h 18m	2023-04-14 12:48	2020-02-14 15:49	2019-12-02	2021-06-01	Expired	
3	i500	AA1911104480	0d 0h 49m	-	2019-11-26 16:33	2019-11-19	2021-05-18	Expired	
4	i500	AA1911104481	0d 5h 12m	-	2021-02-16 12:37	2019-11-21	2021-05-20	Expired	
5	i700	AZ2105102622	0d 8h 15m	2022-09-21 12:09	2022-09-15 11:25	2021-07-09	2022-07-08	Expired	Buy
6	i700w	BF2112100021	0d 7h 4m	2023-04-03 17:42	2023-03-07 17:29	-	-	-	Buy
7	i700w	BF2202100048	19152d 22h 15m	2023-04-03 18:30	2023-03-24 10:35	-	-	-	Buy
8	i600	BG2202100024	0d 5h 47m	2023-07-31 16:10	2023-06-26 13:43	2022-03-30	2023-03-29	Expired	Buy
9	i600	BG2202100026	0d 5h 40m	2022-09-05 16:34	2022-08-11 11:25	-	-	-	Buy
10	i600	BG2202100029	0d 3h 10m	2022-08-10 11:05	2022-06-16 11:58	-	-	-	Buy

The scanner information is automatically registered once your scanner is connected to a Medit scan program.

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2023-04-19

To our users,

We want to thank all Medit Link users who have supported and used our NFT service.

Unfortunately, we must inform you that this service will be terminated following the upcoming program update.

Please see the details below for more information:

- **Feature in Question**
 - Share via NFT (in Medit Link Case Talk)
- **Effective Date**
 - Starting from the update in the first half of 2023 (scheduled for April 27)

※ The update schedule is subject to change.

We'll continuously work to provide better service.

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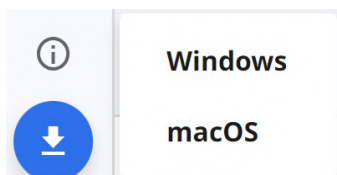
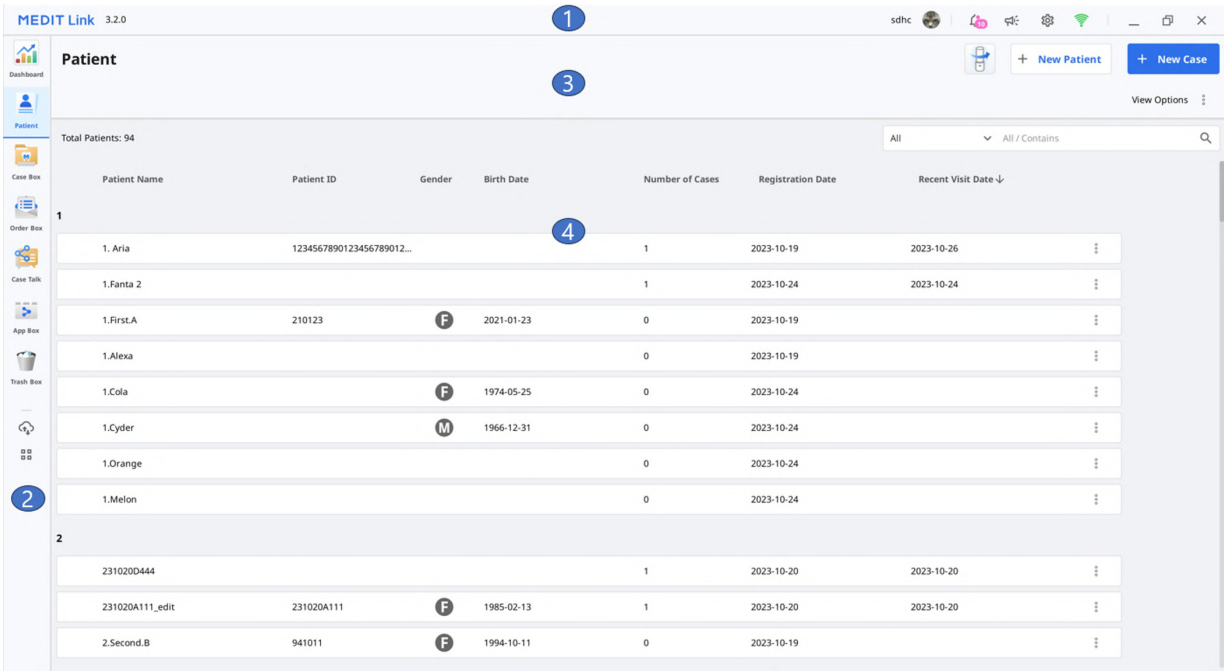
2

Delivery

Wonhoonchoi_Lab_Dev has dispatched the ordered package.

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Order

Wonhoonchoi_Lab_Dev has accepted the order.

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Order

Wonhoonchoi_Lab_Dev has accepted the order.

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Order

Wonhoonchoi_Lab_Dev has accepted the order.

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Order

Wonhoonchoi_Lab_Dev has accepted the order.

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Order

Wonhoonchoi_Lab_Dev has accepted the order.

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Work

Wonhoonchoi_Clinice_Dev_Member registered a new case test's Case.

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Work

Wonhoonchoi_Clinice_Dev_Member registered a new case test's Case.

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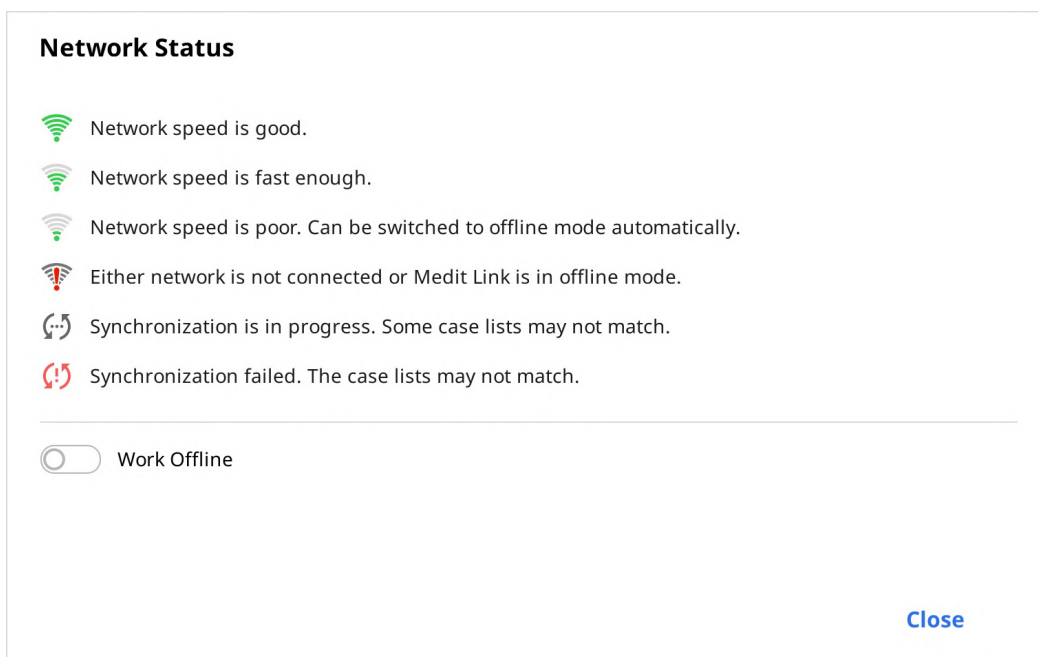
The screenshot shows the 'Settings' window with the 'General' tab selected. The left sidebar contains the following options: General, Storage Management, Set Connected Program, Export Options, Case Converting Tool, and Proxy Option. The main content area displays the following settings:

- Save Folder:** C:/ProgramData/Medit/Medit Link/Data/
- Apps Installation Folder:** C:/Program Files/Medit/
- Language Settings:** English
- Teeth Labeling:** FDI (selected), US (unselected)
- Notification:** Allow Notifications (checked)

An 'Apply' button is located at the bottom right of the settings area. A 'Close' button is located at the bottom right of the window frame.

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Compress & Combine

☒ Compress Files
☒ Combine Individual Mesh

Options when same folder or file already exists.

☒ Always ask when exporting
☐ Export a new file and overwrite the existing file. The file currently stored will be lost.
☐ Create a folder and export the files

Rule for Folder Name

☒ Case Creation Date
☐ Scanned Date
☐ Order Date

☒ Patient Name
☐ Case Name
☐ Partner Name

☐ Patient ID
☐ Order ID

Folder Name

Case Creation Date - Patient Name

Rule for File Name

☒ Case Creation Date
☐ Scanned Date
☐ Order Date

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Export

Import

A new case is created for every imported case. It contains case data, patient information, and scan file. However, it does not contain order information.

Import From

C:/ReleaseBuild/CaseBox_2023-7-5_171632_caries.meditex

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Export

Import

A new case is created for every imported case. It contains case data, patient information, and scan file. However, it does not contain order information.

Import From

C:\ReleaseBuild\CaseBox_2023-7-5_171632_series.meditex

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1/1/2018-11/13/2023 All / Contains

Files to Export

Case Name	Patient Name	Last Modified Date
☒ Medit Margin Line Sample (UG)	User Guide	11/13/2023 9:53 AM
☒ Daichi-Before's Case	Daichi-Before	11/10/2023 5:05 PM

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Question

Exporting successfully completed.
Do you want to open the folder containing the exported file?

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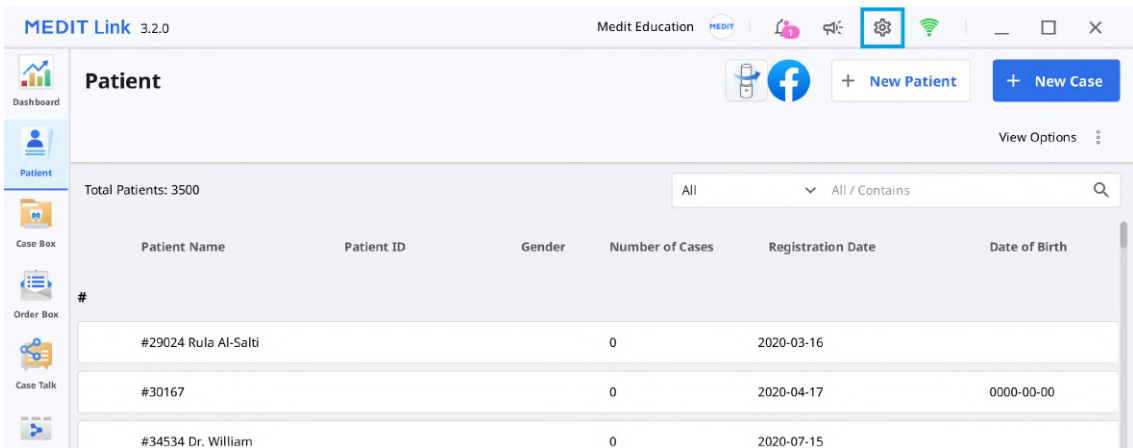
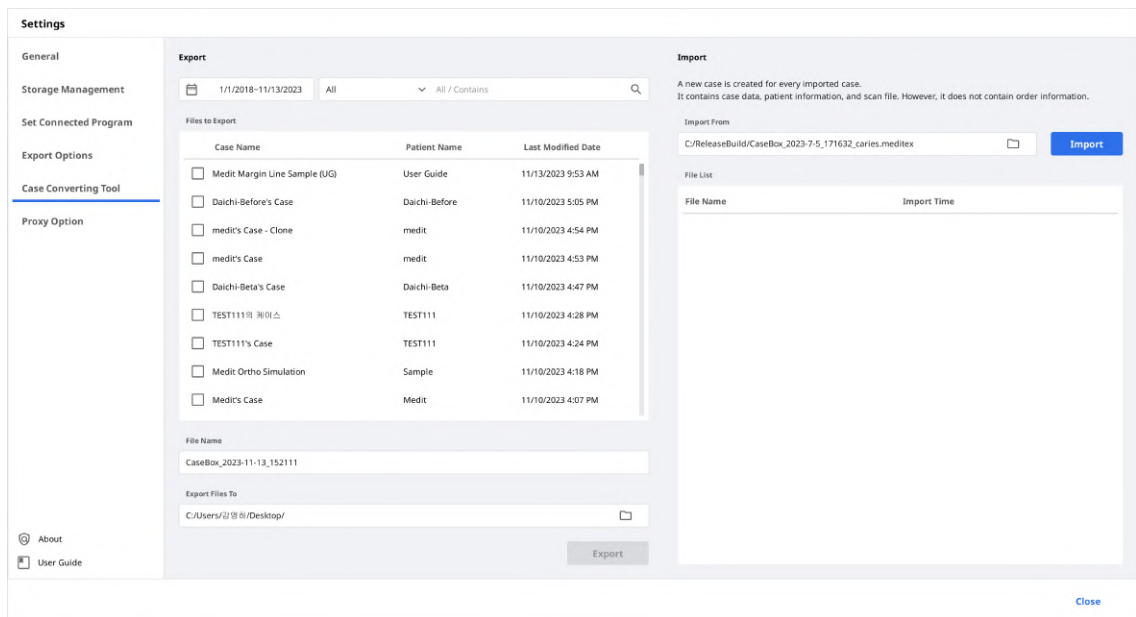
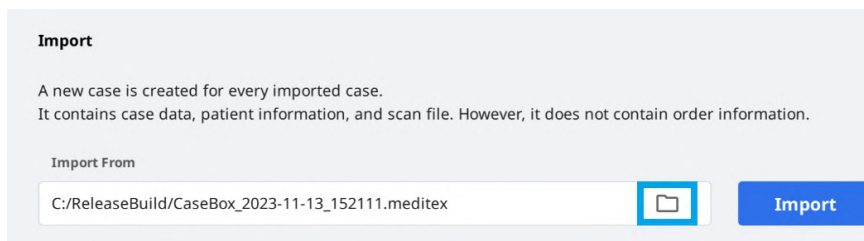
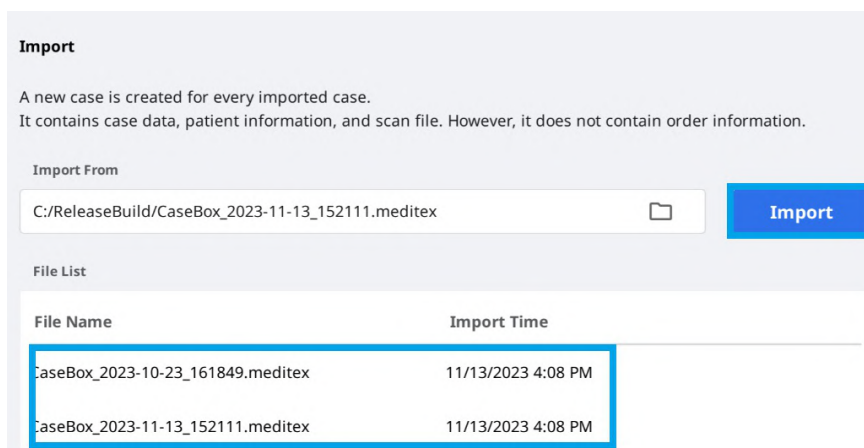
About

MEDIT Link

v 3.2.0 Build 178
Operation Korea
<https://www.medit.com/>
Copyright Medit Corp. All rights reserved.

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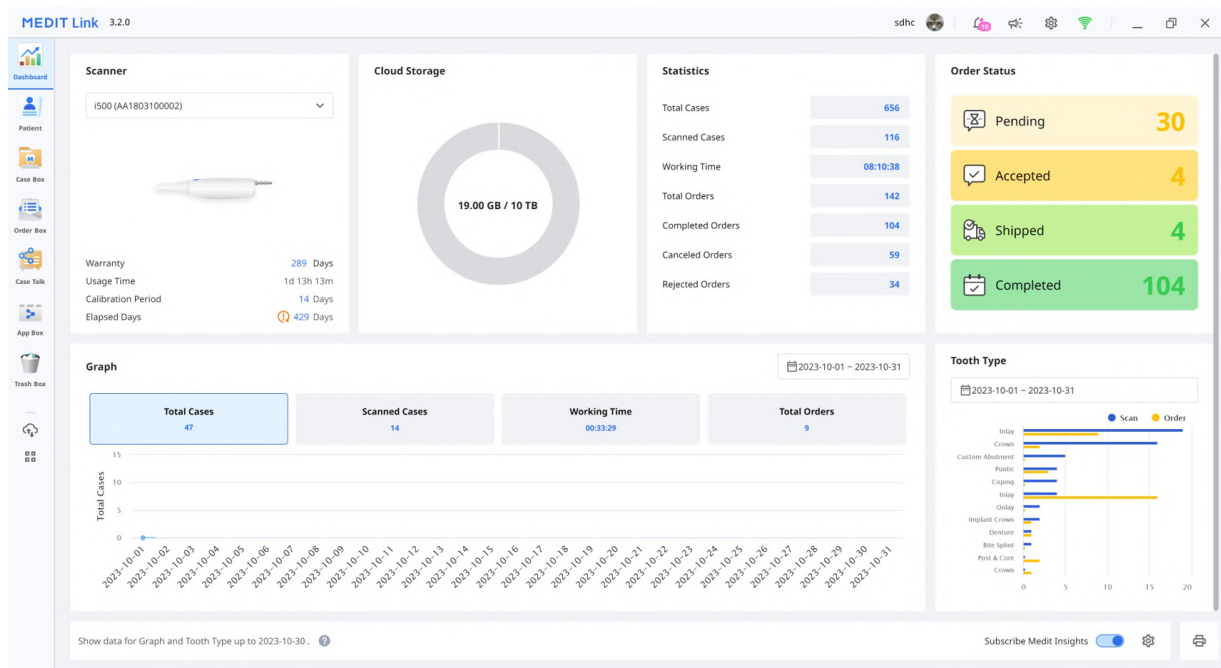
Total Patients: 106

All / Contains

Patient Name	Patient ID	Gender	Date of Birth	Number of Cases	Registration Date	Actions
KELLY				1	2021-07-02	Scan
Medit Sample				4	2023-11-05	Scan
MeditLink 3.0's				1	2023-01-17	Scan
Multi Occlusion Normal Case				1	2023-02-09	Scan
Michele Rinn				3	2021-04-26	Scan
newpatient				1	2022-10-13	Scan
ORG_Clinics[2.6.2] 03 Clinical				1	2023-06-14	Scan
Order Test - Change Name				2	2023-07-24	Scan

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Merge all patient information to

Please select the patient you'd like to merge all data to.
After merging all data together, all information in other patient files will be deleted.

	Patient Name	Patient ID	Gender	Birth Date	Number of Cases	Date Registered	Recent Visit Date
☒	5-9_jScan_CAMOrd...				1	2020-09-18	2020-09-18
☐	test				2	2020-09-17	2020-09-17
☐	test				22	2019-07-26	2020-09-17

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Status	Case Name	Patient Name	Lab Name	Deletion Date ↓	Deleted By	Size
Scan Completed	Jira_임플란트 46, 47's Case - Clone - Clone	임플란트 46, 47		2023-04-26 오전 9:12	wonhoon choi	66,654 KB
Form Only	G11598492304180810429	test test		2023-04-18 오후 5:15	wonhoon choi	0 KB
Form Only	G11598492304180806313	test test		2023-04-18 오후 5:15	wonhoon choi	0 KB
Form Only	G11598492304180811349	tes test		2023-04-18 오후 5:15	wonhoon choi	0 KB
Form Only	G11598492304180754083	Glidewell_fastdesign		2023-04-18 오후 5:00	wonhoon choi	0 KB
Form Only	G11598492304180733332	Glidewell_fastdesign		2023-04-18 오후 5:00	wonhoon choi	0 KB
Form Only	G11598492304180648146	test test		2023-04-18 오후 5:00	wonhoon choi	0 KB
Case Completed	Multi Occlusion Normal Case's Case - Clone	Multi Occlusion Normal Case		2023-04-18 오전 10:50	wonhoon choi	23,652 KB
CAD	Multi Occlusion Normal Case's Case - Clone - Clone	Multi Occlusion Normal Case		2023-04-18 오전 10:50	wonhoon choi	23,652 KB
Form Only	Occlusion Test's Case	Occlusion Test		2023-04-17 오후 6:31	wonhoon choi	0 KB
Form Only	Test Occlusion 's Case	Test Occlusion		2023-04-17 오후 6:31	wonhoon choi	0 KB
Form Only	Occlusion Test's Case	Occlusion Test		2023-04-17 오후 6:31	wonhoon choi	0 KB
Form Only	Occlusion Test's Case	Occlusion Test		2023-04-17 오후 6:26	wonhoon choi	0 KB

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
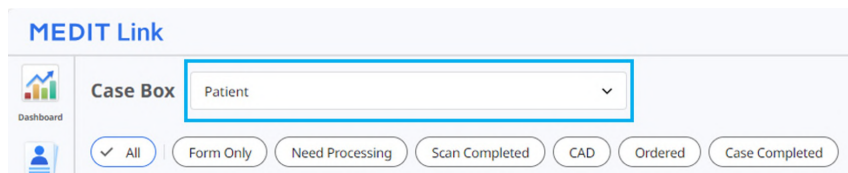
The screenshot shows the 'MEDIT Link' interface. On the left is a 'Dashboard' icon. The main area has a 'Case Box' dropdown menu currently set to 'Patient'. Below this are several filter buttons: 'All' (with a checkmark), 'Form Only', 'Need Processing', 'Scan Completed', 'CAD', 'Ordered', and 'Case Completed'.

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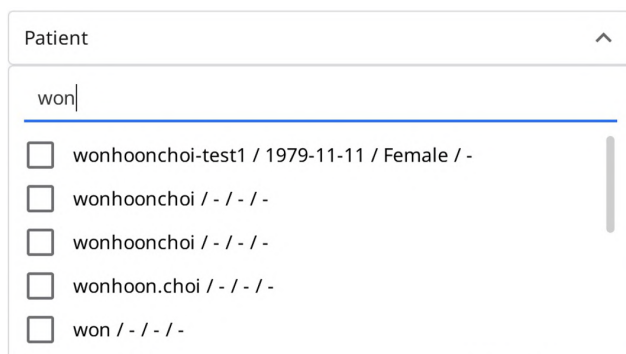
This screenshot shows the dropdown menu for the 'Patient' case box. The search input field contains 'won'. Below the input, there is a list of search results, each preceded by an unchecked checkbox:

- ☐ wonhoonchoi-test1 / 1979-11-11 / Female / -
- ☐ wonhoonchoi / - / - / -
- ☐ wonhoonchoi / - / - / -
- ☐ wonhoon.choi / - / - / -
- ☐ won / - / - / -

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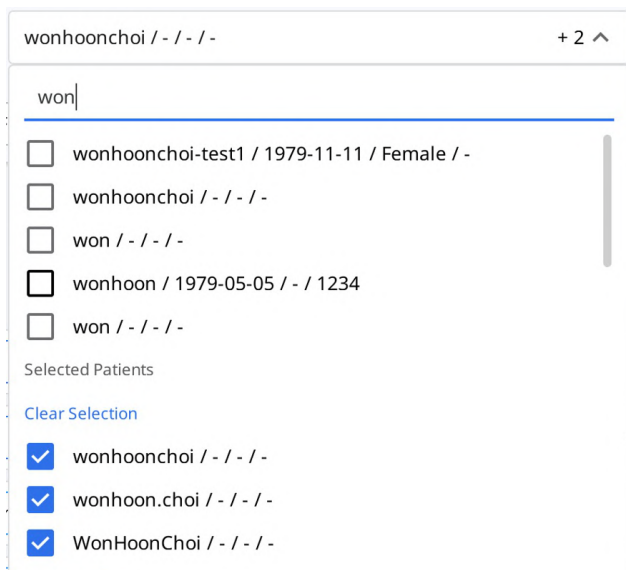This screenshot shows the dropdown menu with search results selected. The search input field still contains 'won'. The list of results is the same as in the previous step. Below the list, there is a section titled 'Selected Patients' with a 'Clear Selection' link. Three checkboxes are now checked:

- ☒ wonhoonchoi / - / - / -
- ☒ wonhoon.choi / - / - / -
- ☒ WonHoonChoi / - / - / -

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MEDIT Link 3.2.0 Wonhoon Choi_clinic

Case Box wonhoonchoi / - / - / - + 2

Dashboard Patient Case Box Order Box Case Task App Box Trash Box

✓ All Form Only Need Processing Scan Completed CAD Ordered Case Completed View Options

Total Cases: 20 2018-01-01-2023-10-24 All / Contains

Status	Patient Name	Case Name	Form Information	Last Modified Date	Scan Date	Lab Name	Order Date
Form Only	wonhoonchoi	originalCase	-	2023-10-05 오후 8:10			
Form Only	wonhoonchoi	wonhoonchoi's Case	-	2023-10-05 오후 8:09			
Scan Completed	wonhoonchoi	FileViewerTest_1	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-06-15 오전 8:14	2023-02-09 오전 8:35		
Scan Completed	wonhoonchoi	FileViewerTest_1 - Clone	2-Inlay / 3-Inlay / 4-Inlay / 5-Inlay / 6-Inlay / 8...	2023-05-25 오후 2:05	2023-02-09 오전 8:35		
Form Only	wonhoonchoi	FileViewerTest_1 - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-03-15 오후 3:06			
Scan Completed	wonhoonchoi	FileViewerTest_1 - Clone - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-03-09 오후 4:41	2023-02-09 오전 8:35		
Need Processing	wonhoonchoi	FileViewerTest_1 - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-02-14 오전 8:13	2023-02-09 오전 8:35		
Scan Completed	wonhoonchoi	FileViewerTest_1 - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-02-10 오전 11:27	2023-02-09 오전 8:35		
Form Only	wonhoonchoi	testsamplecase	-	2023-02-10 오전 7:39			
Scan Completed	wonhoonchoi	Multi Occlusion	-	2023-02-09 오전 9:26	2023-02-09 오전 9:01		
Scan Completed	wonhoonchoi	FileViewerTest_1 - Clone - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-02-09 오전 8:50	2023-02-09 오전 8:35		
Case Completed	wonhoonchoi	wonhoonchoi's Case	12-Inlay / 13-Inlay / 14-Inlay	2022-10-13 오후 10:30	2022-10-11 오후 5:16		
Case Completed	wonhoonchoi	wonhoonchoi's Case	-	2022-10-13 오후 10:28			

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
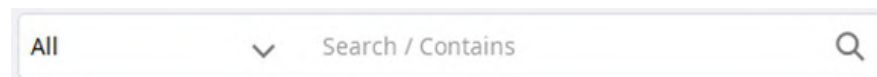
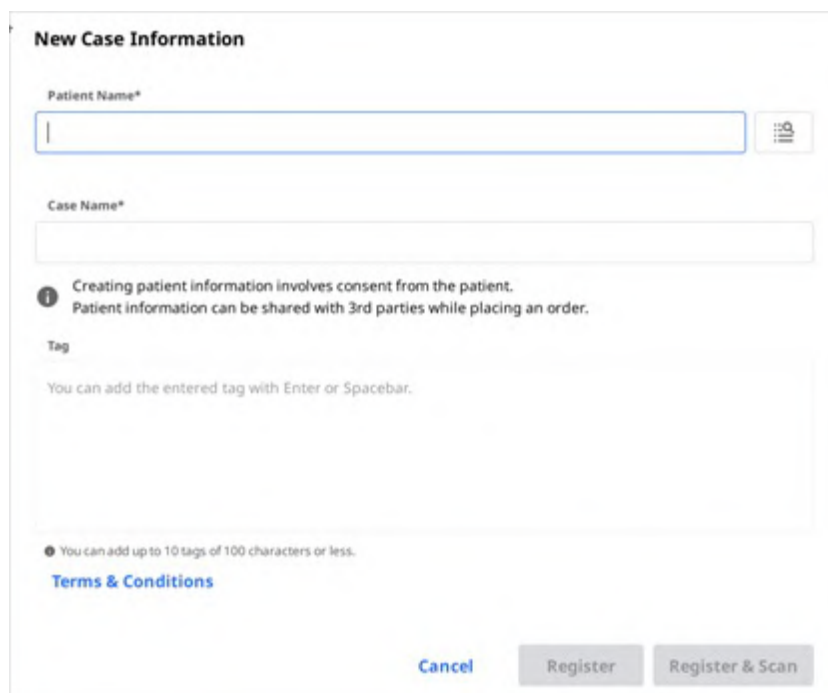
The form titled "New Case Information" contains the following fields and elements:

- Patient Name***: A text input field with a dropdown icon on the right.
- Case Name***: A text input field.
- Tag**: A text input field with a placeholder "You can add the entered tag with Enter or Spacebar."
- Terms & Conditions**: A link below the tag field.
- Buttons**: "Cancel", "Register", and "Register & Scan" at the bottom.
- Notes**: A note stating "Creating patient information involves consent from the patient. Patient information can be shared with 3rd parties while placing an order."
- Footer**: A note stating "You can add up to 10 tags of 100 characters or less."

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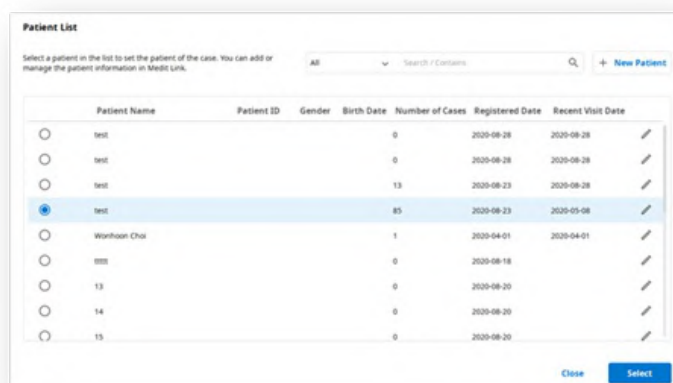The "Patient List" dialog box displays a table of existing patients. The table has the following columns: Patient Name, Patient ID, Gender, Birth Date, Number of Cases, Registered Date, and Recent Visit Date. The table contains 10 rows of data. The first four rows have "test" as the Patient Name. The fifth row has "Worhoon Choi". The last five rows have "13", "14", and "15" as Patient Names. The "Number of Cases" column shows values 0, 0, 13, 85, 1, 0, 0, 0, 0, and 0. The "Registered Date" column shows dates from 2020-08-28 to 2020-04-01. The "Recent Visit Date" column shows dates from 2020-08-28 to 2020-04-01. The fourth row is selected. There are radio buttons to the left of each row. At the bottom right, there are "Close" and "Select" buttons.

Patient Name	Patient ID	Gender	Birth Date	Number of Cases	Registered Date	Recent Visit Date
test				0	2020-08-28	2020-08-28
test				0	2020-08-28	2020-08-28
test				13	2020-08-23	2020-08-28
test				85	2020-08-23	2020-05-08
Worhoon Choi				1	2020-04-01	2020-04-01
13				0	2020-08-18	
13				0	2020-08-20	
14				0	2020-08-20	
15				0	2020-08-20	

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☐ Convert images to the below format

☐ JPG
 ☐ JPEG
 ☒ PNG
 ☐ BMP

☐ Compress Files
☐ Combine Individual Mesh
☐ Export All Occlusal Relationship
☐ Save Case Information as PDF

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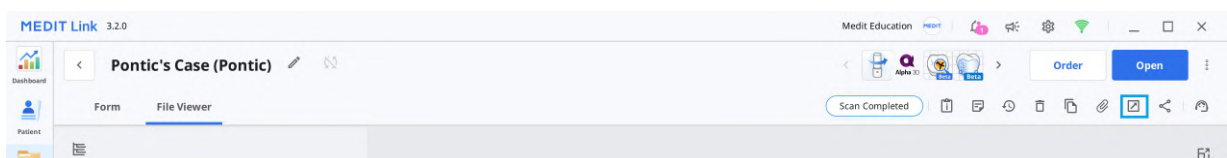
Options File List

Export Files To
C:/Users/Medit-Wonhoonchoi/Desktop/test/

Folder Name
Multi Occlusion - Scan - Before Scanning

File Name
Multi Occlusion - Scan - Before Scanning

CAD Type
Maestro

Mesh Format
Convert mesh files to the below formats
☐ meditMesh ☐ OBJ ☒ PLY ☐ STL
☒ Do not convert the attached files

Image Format
☐ Convert images to the below format
☐ JPG ☐ JPEG ☒ PNG ☐ BMP

☐ Compress Files
☐ Combine Individual Mesh
☐ Export All Occlusal Relationship
☐ Save Case Information as PDF [Preview](#)

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☒ [Icon] Maxilla
☒ Base

☒ [Icon] Mandible
☒ Base

☒ [Icon] Occlusion
☒ First Occlusion
☒ Second Occlusion

☒ [Icon] Dynamic Occlusion
☒ Free
☒ Maxilla
☒ Mandible

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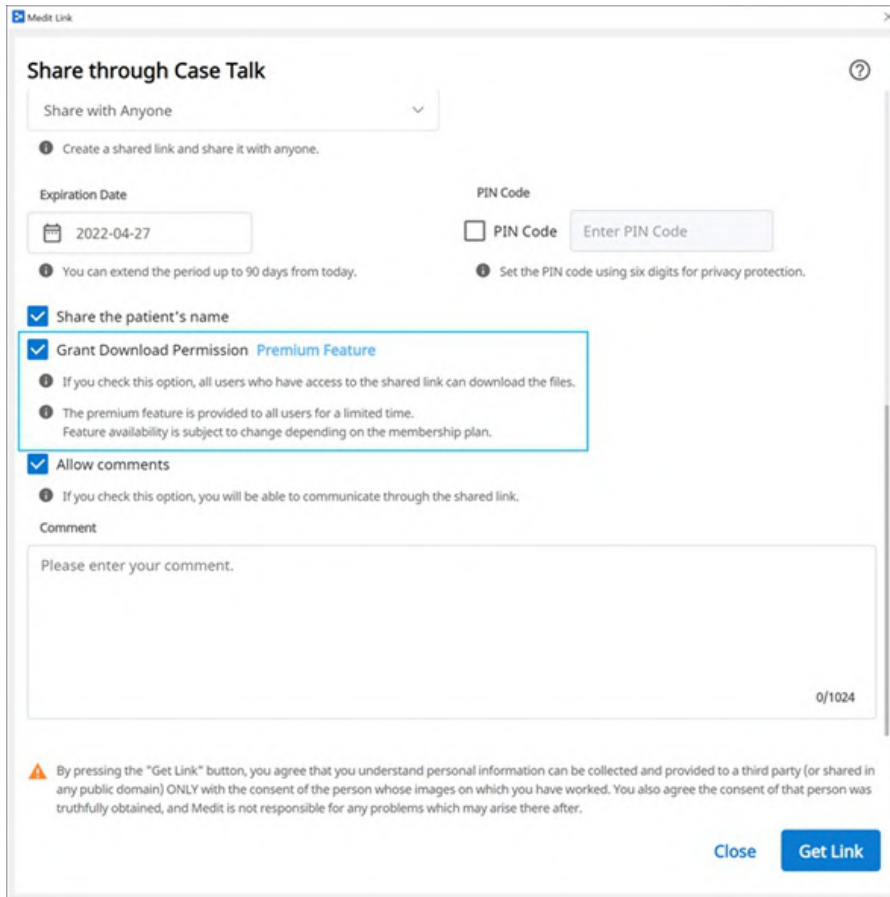
Share through Case Talk

Share with Anyone

Create a shared link and share it with anyone.

Expiration Date: 2022-04-27

PIN Code: ☐ PIN Code

You can extend the period up to 90 days from today.

Set the PIN code using six digits for privacy protection.

☒ Share the patient's name

☒ Grant Download Permission **Premium Feature**

If you check this option, all users who have access to the shared link can download the files.

The premium feature is provided to all users for a limited time. Feature availability is subject to change depending on the membership plan.

☒ Allow comments

If you check this option, you will be able to communicate through the shared link.

Comment

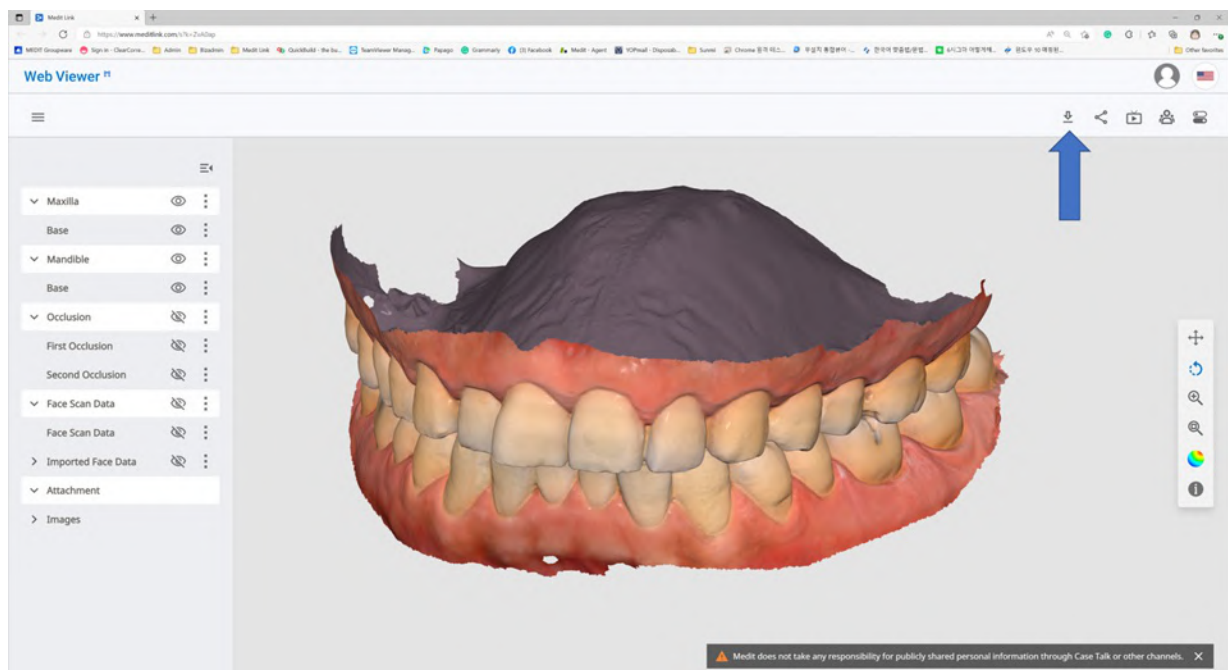
Please enter your comment.

0/1024

By pressing the "Get Link" button, you agree that you understand personal information can be collected and provided to a third party (or shared in any public domain) ONLY with the consent of the person whose images on which you have worked. You also agree the consent of that person was truthfully obtained, and Medit is not responsible for any problems which may arise there after.

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Download Premium Feature

The premium feature is provided to all users for a limited time.
Feature availability is subject to change depending on the membership plan.

Mesh Format

Convert mesh files to the below formats

☐ meditMesh ☐ OBJ ☒ PLY ☐ STL

i The selected format will be applied to the result files created in the Medit Link application.

i The attachment files will not be converted but will keep their original format.

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The screenshot shows the 'Order' form with the following details:

- Order Form Fields:**
 - Partner:** WonhoonChoi_Lab_Stage
 - Case Name:** mandibular movement - Clone - Clone
 - Patient Name:** Test Occlusion
 - Requested Delivery Date:** 2023-04-13
 - Delivery Time:** AM 8:00 to PM 8:00, currently set at 8:15.
 - Share patient information:** ☒
 - Tags:** A text area with a placeholder: "You can add the entered tag with Enter or Spacebar."
 - Include all occlusion relationships in the order:** ☒
 - Include Tags In The Order:** ☒
- Partner Information Card:**
 - WonhoonChoi_Lab_Stage**
 - Seoul, KR
 - [02835] 23 Incheon-ro 22-gil, Seongbuk-gu, Seoul, Korea
 - 01026088546

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