USER GUIDE

Medit Link (Revision 9) 2024.04

Table of Contents

Medit Link

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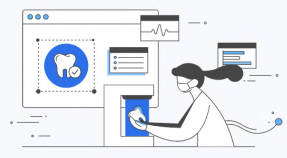
Administrator

Dentist, Manager

Have the authority to create a new dental clinic account and have full access to Medit Link.

- ✓ Scan/Order/Manage orders
- ✓ Manage members

Register

Administrator

Laboratory Director, Manager

Have the authority to create a new dental lab account and have full access to Medit Link.

- ✓ Scan/Manage ordered cases
- ✓ Manage members

Register

Member

Dental Hygienist, Dental Staff

Belong to the dental clinic their admin created and have limited access to Medit Link after the admin's approval.

- ✓ Scan/Order/Manage orders

Register

Member

Dental Technician, Laboratory Staff

Belong to the dental lab their admin created and have limited access to Medit Link after the admin's approval.

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Email Address

Check

ⓘ Please ensure your email address is active. Medit Link will send a verification email.
 ⓘ Registering with a disposable and/or fake e-mail address can make you vulnerable to security issues and make it difficult to receive information from Medit Link.

Password

ⓘ Password must be 8 to 16 characters long, including a combination of at least three letters, numbers, and special characters. (Passwords are accepted in English language only.)

Confirm Password

ⓘ Please type in the same password as above.

Select Country/Region

Select Country/Region

☐ I agree to all of the Terms and Conditions.
☐ Personal Information Collection Agreement (Mandatory)

View

☐ Terms & Conditions (Mandatory)

View

☐ Agree to the Marketing and Promotion Agreement (optional)

View

 ⓘ Check this option to receive marketing emails, which include product updates, software news, and promotional events.

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The screenshot shows the MEDIT Link Case Box interface. The 'Status' filter is set to 'Form Only'. The table displays the following cases:

Status	Case Name	Patient Name	Form Information	Last Modified Date	Scanned Date	Order Date	Lab Name	Share
Form Only	test's Case	test		2023-11-03 12:50	-	-	-	Share
Scan Completed	wonhoonchoi-test1's Case - Clone - Clone	wonhoonchoi-test1	23, 24, 25, 26	2023-11-02 19:37	2023-11-02 09:01	-	-	Share
Scan Completed	wonhoonchoi-test1's Case - Clone	wonhoonchoi-test1	23, 24, 25, 26	2023-11-02 19:21	2023-11-02 09:01	-	-	Share
Ordered	wonhoonchoi-test1's Case	wonhoonchoi-test1	23, 24, 25, 26	2023-11-02 19:21	2023-11-02 09:01	2023-11-02 ...	Wonhoonchoi_lab2	Share
Scan Completed	G11598492311021004395	test test (12345)	21	2023-11-02 19:09	-	-	-	Share
Scan Completed	test's Case	test		2023-11-01 13:32	-	-	-	Share
Scan Completed	G11598492311010319123	test test (test)	21	2023-11-01 12:20	-	-	-	Share
Scan Completed	test's Case	test		2023-10-26 17:25	2023-10-26 17:24	-	-	Share
Scan Completed	wonhoonchoi-test1's Case	wonhoonchoi-test1		2023-10-26 17:22	2023-10-26 17:21	-	-	Share
Ordered	test's Case	test	14, 15, 16, 17	2023-10-24 15:45	-	2023-10-24 ...	Wonhoonchoi_lab2	Share

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Case Box (치과 계정)	All / Form Only / Need Processing / Scan Completed / CAD / Ordered / Case Completed
Work Box (기공소 계정)	All / Form Only / Need Processing / Scan Completed / CAD / Done

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☰

1

Create a shared link and share it with specific people.

Expiration Date

📅

2022-06-28

1

You can extend the period up to 90 days from today.

PIN Code

☐

PIN Code

Enter PIN Code

1

Set the PIN code using six digits for privacy protection.

☒

Share the patient's name

☒

Grant Download Permission

Premium Feature

1

If you check this option, all users who have access to the shared link can download the files.

1

The premium feature is provided to all users for a limited time.
Feature availability is subject to change depending on the membership plan.

☒

Allow comments

1

If you check this option, you will be able to communicate through the shared link.

Comment

Please enter your comment.

0/1024

⚠

By pressing the "Get Link" button, you agree that you understand personal information can be collected and provided to a third party (or shared in any public domain) ONLY with the consent of the person whose images on which you have worked. You also agree the consent of that person was truthfully obtained, and Medit is not responsible for any problems which may arise there after.

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← **MEDIT Link** **TEST MODE**

Subscribe to Premium membership for Clinic

\$0.00

Then \$0.99 per day after coupon expires

Premium membership for Clinic **\$0.99**
Medit Link Membership
Billed daily

Subtotal **\$0.99**

1 MONTH FREE – for first subscription **-\$0.99**
100% off for a day

Total due today **\$0.00**

Powered by **stripe** | [Terms](#) | [Privacy](#)

Pay with **link** ⇒

Or pay with card

Email yhgam0829@gmail.com

Card information

1234 1234 1234 1234

MM / YY

CVC

Cardholder name

Full name on card

Country or region

South Korea

☐ **Securely save my information for 1-click checkout**
Pay faster on MEDIT SG PTE. LTD. and everywhere Link is accepted.

Subscribe

By confirming your subscription, you allow MEDIT SG PTE. LTD. to charge you for future payments in accordance with their terms. You can always cancel your subscription.

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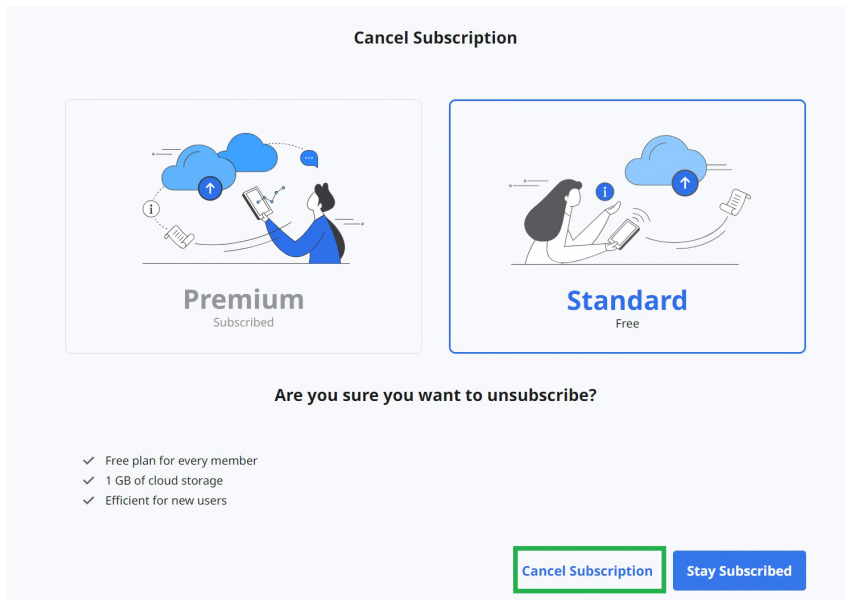
Confirm Unsubscribe

Your subscription will be canceled at the end of your billing period on 2021-09-10

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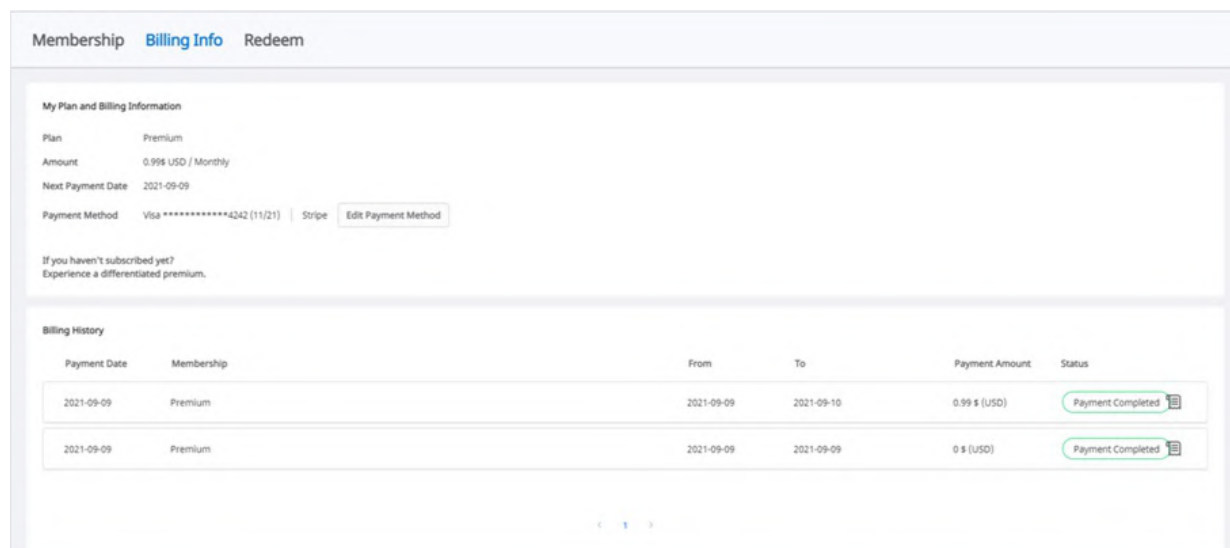
Membership Billing Info Redeem

My Plan and Billing Information

Plan: Premium
Amount: 0.99 USD / Monthly
Next Payment Date: 2021-09-09
Payment Method: Visa **** *4342 (11/21) | Stripe **Edit Payment Method**

If you haven't subscribed yet?
Experience a differentiated premium.

Billing History

Payment Date	Membership	From	To	Payment Amount	Status
2021-09-09	Premium	2021-09-09	2021-09-10	0.99 \$ (USD)	Payment Completed
2021-09-09	Premium	2021-09-09	2021-09-09	0 \$ (USD)	Payment Completed

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Add

Go back

By adding a payment method, you agree to MEDIT SG PTE. LTD.'s
Terms of Service and Privacy Policy.

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Billing

CURRENT PLANS

Premium membership for Clinic
\$0.99 per day
Your plan renews on September 10, 2021.

Premium membership for Clinic
\$0.99 per day
Your plan renews on September 10, 2021.

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Register Redeem Code

You can register for the promotion or apply immediately.
Register allows you to apply and activate when you need it, If applied immediately, it will start after the expiration of the active promotion.
Periodic payments are stopped automatically.

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The screenshot shows a web browser window with the URL www.meditlink.com/myinfo/account. The page is titled "MEDIT Link" and has a navigation bar with "My Account", "My Clinic", and "My Scanner". On the left is a sidebar with icons for Patient, Case Box, Order Box, Case Talk, Partners, Membership, and My Info (which is highlighted). The main content area is titled "Account Information" and features a MEDIT logo with a "2022 Trusted Lab Partner of MEDIT" badge. The form includes fields for Email Address (seolah.jeon@medit.com), Password (with a "Change Password" button), and Name (seolah.jeon_dev_admin). There are also dropdown menus for Language (English) and Country/Region (Korea, Republic of), each with a "Change" button. A note at the bottom states: "Your chosen language is applied to the email sent by MEDIT Link and the notification."

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Edit

Cover and Logo

Looking for digitalized dental laboratories?

Find the most ordered Labs around your clinic with 'Featured Part

1

2

3

Services

Select

Specialties (choose up to 3)

Select

Select the services your business specializes in from the list.

About Us

e.g., Welcome to OOO Dental Clinic.

We are a licensed dental clinic situated in California specializing mainly in aesthetic restorations and implantology.

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MEDIT Link

Add more information about your business. > seolah.jeon

Dashboard

Patient

Case Box

Order Box

Case Talk

Partners

Membership

My Info

My Account My Clinic My Scanner

No.	Type	Serial Number	Usage Time ⓘ	Last Connected	Last Calibrated	Warranty Start Date	Warranty Expiry Da...	Days Left ⓘ	
1	i500	-	-	-	-	-	-	-	
2	i500	AA1911104453	0d 7h 18m	2023-04-14 12:48	2020-02-14 15:49	2019-12-02	2021-06-01	Expired	
3	i500	AA1911104480	0d 0h 49m	-	2019-11-26 16:33	2019-11-19	2021-05-18	Expired	
4	i500	AA1911104481	0d 5h 12m	-	2021-02-16 12:37	2019-11-21	2021-05-20	Expired	
5	i700	AZ2105102622	0d 8h 15m	2022-09-21 12:09	2022-09-15 11:25	2021-07-09	2022-07-08	Expired	Buy
6	i700w	BF2112100021	0d 7h 4m	2023-04-03 17:42	2023-03-07 17:29	-	-	-	Buy
7	i700w	BF2202100048	19152d 22h 15m	2023-04-03 18:30	2023-03-24 10:35	-	-	-	Buy
8	i600	BG2202100024	0d 5h 47m	2023-07-31 16:10	2023-06-26 13:43	2022-03-30	2023-03-29	Expired	Buy
9	i600	BG2202100026	0d 5h 40m	2022-09-05 16:34	2022-08-11 11:25	-	-	-	Buy
10	i600	BG2202100029	0d 3h 10m	2022-08-10 11:05	2022-06-16 11:58	-	-	-	Buy

The scanner information is automatically registered once your scanner is connected to a Medit scan program.

Home

Terms & Conditions

Privacy Policy

Copyright © Medit Corp. All rights reserved.

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2023-04-19

To our users,

We want to thank all Medit Link users who have supported and used our NFT service.

Unfortunately, we must inform you that this service will be terminated following the upcoming program update.

Please see the details below for more information:

- **Feature in Question**
 - Share via NFT (in Medit Link Case Talk)
- **Effective Date**
 - Starting from the update in the first half of 2023 (scheduled for April 27)

※ The update schedule is subject to change.

We'll continuously work to provide better service.

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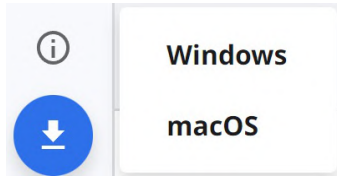
2

Delivery

Wonhoonchoi_Lab_Dev has dispatched the ordered package.

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Order

Wonhoonchoi_Lab_Dev has accepted the order.

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Order

Wonhoonchoi_Lab_Dev has accepted the order.

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Order

Wonhoonchoi_Lab_Dev has accepted the order.

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Order

Wonhoonchoi_Lab_Dev has accepted the order.

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Order

Wonhoonchoi_Lab_Dev has accepted the order.

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Work

Wonhoonchoi_Clinice_Dev_Member registered a new case test's Case.

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Work

Wonhoonchoi_Clinice_Dev_Member registered a new case test's Case.

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☒ Upload Automatically

Quick Scan

Choose if you would like to have the scan stages below automatically appear in Medit Scan for Clinics without having to fill in the form information.

☐ Pre-Op for Maxilla, Pre-Op for Mandible, Face Scan

Viewing Options

☒ Advanced Rendering

Adjust Color Texture

Medit User Experience Improvement Program

We value your feedback! Join the Medit User Experience Improvement Program and share your thoughts. The collected information will be used to improve the product and services. Medit Link does not collect any information that identifies individuals.

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Network speed is good.

Network speed is fast enough.

Network speed is poor. Can be switched to offline mode automatically.

Either network is not connected or Medit Link is in offline mode.

Synchronization is in progress. Some case lists may not match.

Synchronization failed. The case lists may not match.

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The screenshot shows the 'Settings' window with the 'General' tab selected. The left sidebar contains links for 'General', 'Storage Management', 'Set Connected Program', 'Export Options', 'Case Converting Tool', 'Proxy Option', 'About', and 'User Guide'. The main content area displays the following settings:

- Save Folder:** C:/ProgramData/Medit/Medit Link/Data/
- Apps Installation Folder:** C:/Program Files/Medit/
- Language Settings:** English
- Teeth Labeling:** FDI (selected) and US (unselected)
- Notification:** Allow Notifications (checked)

At the bottom right, there is an 'Apply' button. At the bottom right of the entire window, there is a 'Close' button.

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Compress & Combine

☒ Compress Files
☒ Combine Individual Mesh

Options when same folder or file already exists.

☒ Always ask when exporting
☐ Export a new file and overwrite the existing file. The file currently stored will be lost.
☐ Create a folder and export the files

Rule for Folder Name

☒ Case Creation Date
☐ Scanned Date
☐ Order Date

☒ Patient Name
☐ Case Name
☐ Partner Name

☐ Patient ID
☐ Order ID

Folder Name

Case Creation Date - Patient Name

Rule for File Name

☒ Case Creation Date
☐ Scanned Date
☐ Order Date

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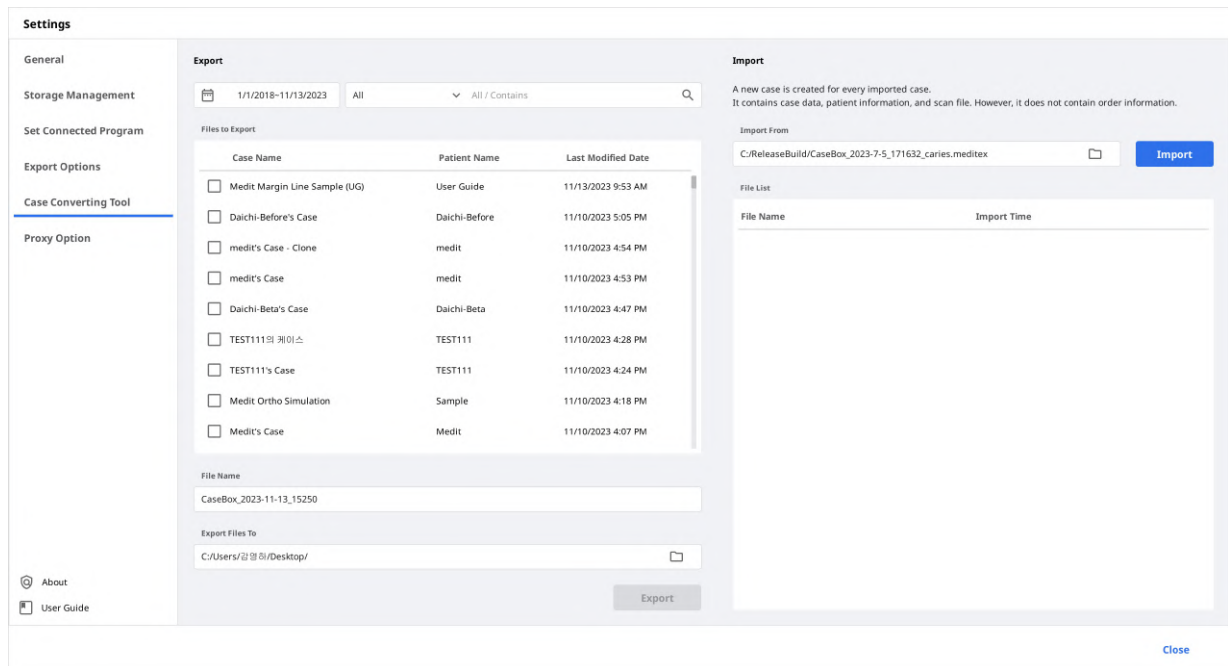
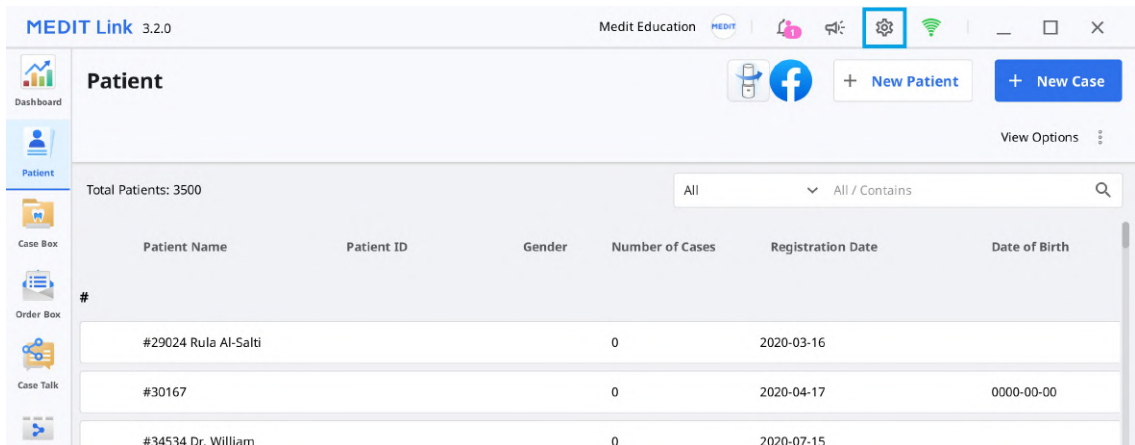
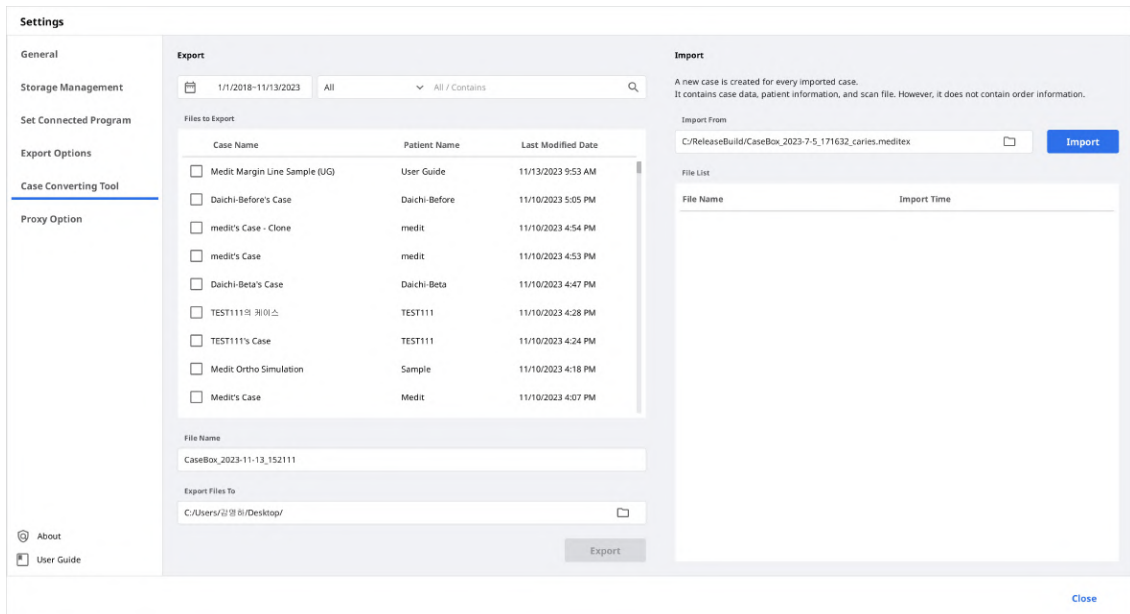
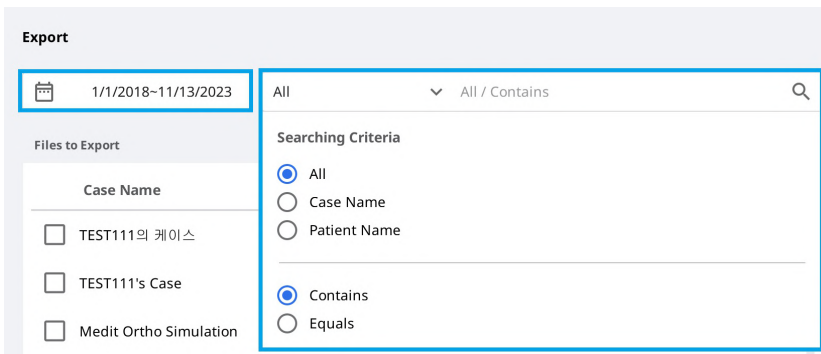
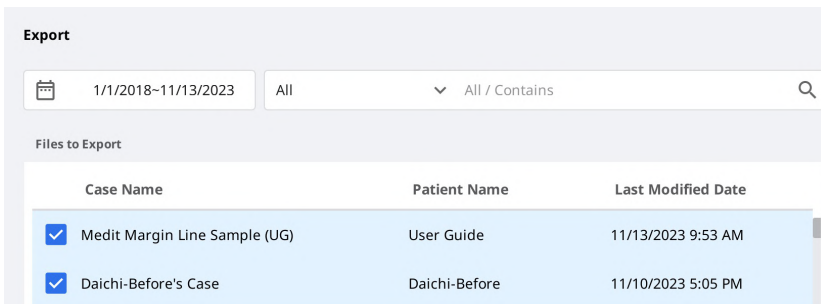
Question

Exporting successfully completed.
Do you want to open the folder containing the exported file?

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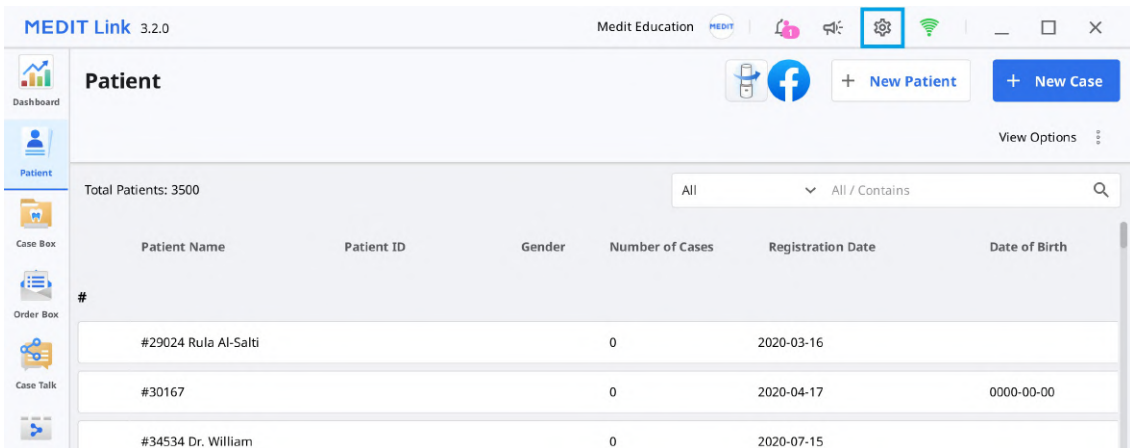
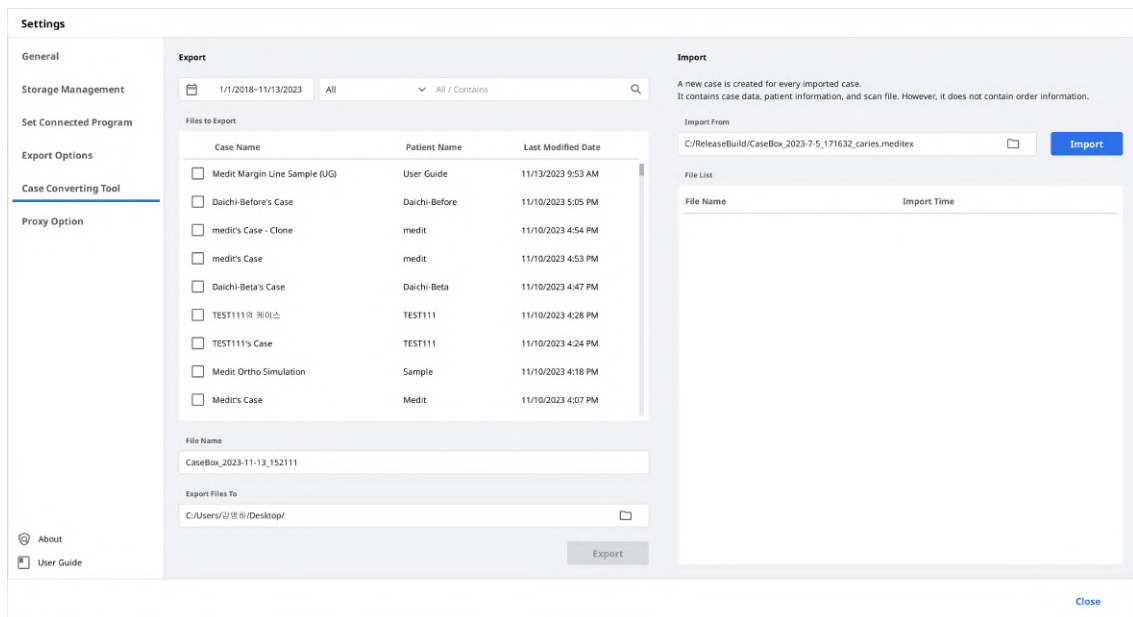
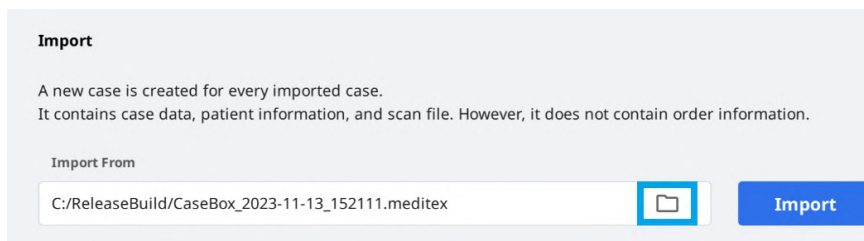
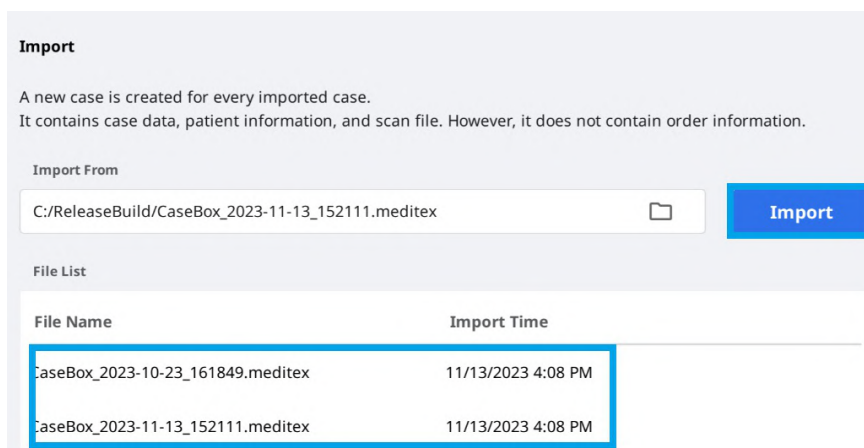
About

MEDIT Link

v 3.2.0 Build 178
Operation Korea
<https://www.medit.com/>
Copyright Medit Corp. All rights reserved.

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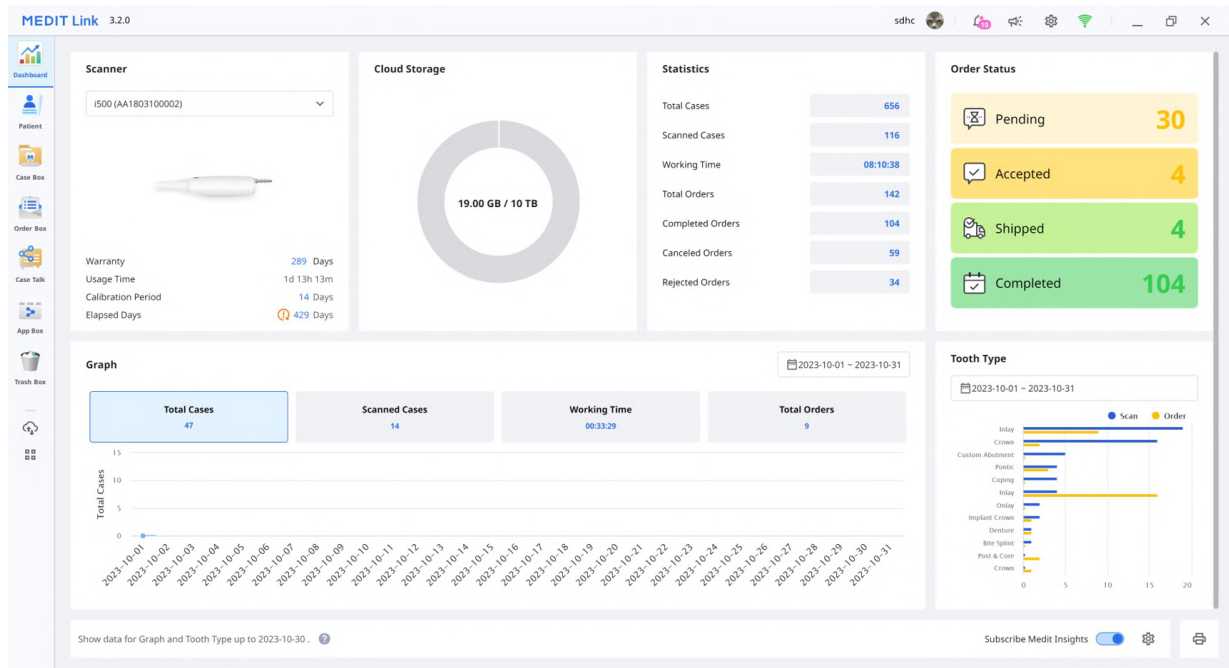
The screenshot shows the Patient management interface in Medit Link 3.2.0. It includes a sidebar with navigation icons and a main area with a 'Patient' header and a table of patients. The table has columns for Patient Name, Patient ID, Gender, Date of Birth, Number of Cases, and Registration Date. There are buttons for '+ New Patient' and '+ New Case' at the top right. The table lists several patients, including KELLY, Medit Sample, MeditLink 3.0's, Multi Occlusion Normal Case, Michele Rinn, newpatient, ORG_[Clinics][2.6.2] 03 Clinical, and Order Test - Change Name.

Patient Name	Patient ID	Gender	Date of Birth	Number of Cases	Registration Date
KELLY				1	2021-07-02
Medit Sample				4	2023-11-05
MeditLink 3.0's				1	2023-01-17
Multi Occlusion Normal Case				1	2023-02-09
Michele Rinn				3	2021-04-26
newpatient				1	2022-10-13
ORG_[Clinics][2.6.2] 03 Clinical				1	2023-06-14
Order Test - Change Name				2	2023-07-24

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Merge all patient information to

Please select the patient you'd like to merge all data to.
After merging all data together, all information in other patient files will be deleted.

	Patient Name	Patient ID	Gender	Birth Date	Number of Cases	Date Registered	Recent Visit Date
☒	5-9_jScan_CAMOrd...				1	2020-09-18	2020-09-18
☐	test				2	2020-09-17	2020-09-17
☐	test				22	2019-07-26	2020-09-17

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MEDIT Link 3.2.0

wonhoon choi

Test Occlusion's Case (Test Occlusion)

Form File Viewer

Scan Completed

6 Type Method Material Shade Favorites

Type	Method	Material	Shade	Favorites
Inlay	Onlay	Coping	Denture Coping	
Crown	Pontic	Post & Core	Veneer	
Implant Crown	Custom Abutment	Surgical Guide	Offset Substructure	
Custom Abutment + Implant Crown	Custom Abutment + Coping	Diagnostic Wax-Up	Partial Denture	

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Case Talk Name	Patient Name	Form Information	Scanned Date	Shared Date ↓	Expiration Date
2022_04_06_MEDIT NFT	hk****	27, 36	-	2022-04-06 13:40	unlimited
seolah.jeon	hk****	11, 12, 13, 21, 22, 23	-	2022-04-01 11:48	unlimited
2.6.0.38 TEST 2's Case - Clone - Clone_2022-03-31 18:46:02	2***** TE** 2	25, 26, 27, 33, 34, 35, 36	-	2022-03-31 18:46	unlimited

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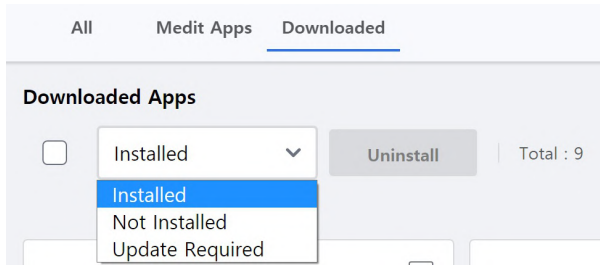
Status	Case Name	Patient Name	Lab Name	Deletion Date ↓	Deleted By	Size
Scan Completed	Jira_외출한트 46, 47's Case - Clone - Clone	외출한트 46, 47		2023-04-26 오전 9:12	wonhoon choi	66,654 KB
Form Only	G11598492304180810429	test test		2023-04-18 오후 5:15	wonhoon choi	0 KB
Form Only	G11598492304180806313	test test		2023-04-18 오후 5:15	wonhoon choi	0 KB
Form Only	G11598492304180811349	tes test		2023-04-18 오후 5:15	wonhoon choi	0 KB
Form Only	G11598492304180754083	Glidewell_fastdesign		2023-04-18 오후 5:00	wonhoon choi	0 KB
Form Only	G11598492304180733332	Glidewell_fastdesign		2023-04-18 오후 5:00	wonhoon choi	0 KB
Form Only	G11598492304180648146	test test		2023-04-18 오후 5:00	wonhoon choi	0 KB
Case Completed	Multi Occlusion Normal Case's Case - Clone	Multi Occlusion Normal Case		2023-04-18 오전 10:50	wonhoon choi	23,652 KB
CAD	Multi Occlusion Normal Case's Case - Clone - Clone	Multi Occlusion Normal Case		2023-04-18 오전 10:50	wonhoon choi	23,652 KB
Form Only	Occlusion Test's Case	Occlusion Test		2023-04-17 오후 6:31	wonhoon choi	0 KB
Form Only	Test Occlusion 's Case	Test Occlusion		2023-04-17 오후 6:31	wonhoon choi	0 KB
Form Only	Occlusion Test's Case	Occlusion Test		2023-04-17 오후 6:31	wonhoon choi	0 KB
Form Only	Occlusion Test's Case	Occlusion Test		2023-04-17 오후 6:26	wonhoon choi	0 KB

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
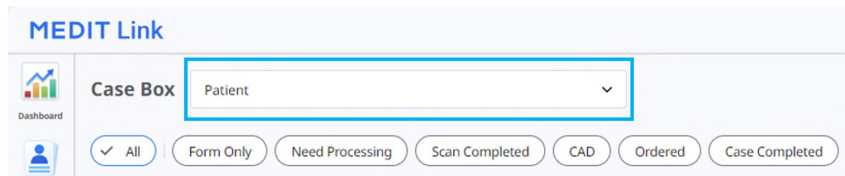
Case Box Patient

Dashboard

✓ All Form Only Need Processing Scan Completed CAD Ordered Case Completed

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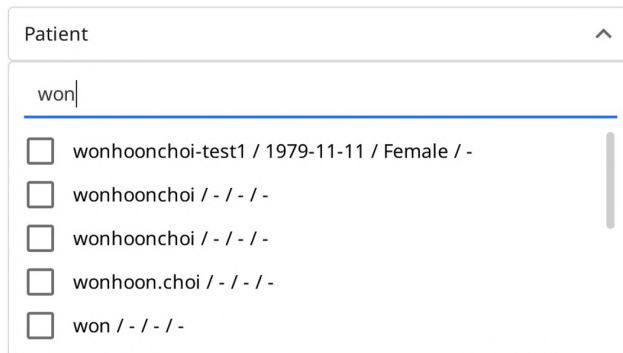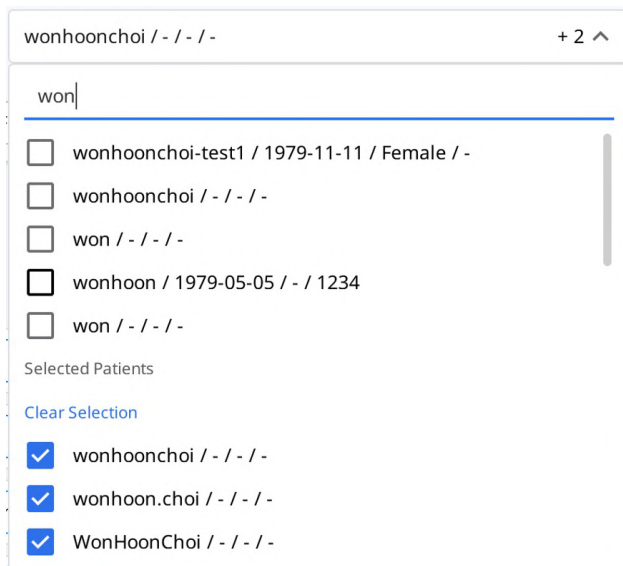
MEDIT Link 3.2.0 Wonhoon Choi_clinic

Case Box wonhoonchoi / - / - / - + 2

Filters: All (selected), Form Only, Need Processing, Scan Completed, CAD, Ordered, Case Completed

Total Cases: 20 | 2018-01-01-2023-10-24 | All / Contains

Status	Patient Name	Case Name	Form Information	Last Modified Date	Scan Date	Lab Name	Order Date
Form Only	wonhoonchoi	originalcase	-	2023-10-05 오전 8:10			
Form Only	wonhoonchoi	wonhoonchoi's Case	-	2023-10-05 오전 8:09			
Scan Completed	wonhoonchoi	FileViewerTest_1	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-06-15 오전 8:14	2023-02-09 오전 8:35		
Scan Completed	wonhoonchoi	FileViewerTest_1 - Clone	2-Inlay / 3-Inlay / 4-Inlay / 5-Inlay / 6-Inlay / 8...	2023-05-25 오후 2:05	2023-02-09 오전 8:35		
Form Only	wonhoonchoi	FileViewerTest_1 - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-03-15 오후 3:06			
Scan Completed	wonhoonchoi	FileViewerTest_1 - Clone - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-03-09 오후 4:41	2023-02-09 오전 8:35		
Need Processing	wonhoonchoi	FileViewerTest_1 - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-02-14 오전 8:13	2023-02-09 오전 8:35		
Scan Completed	wonhoonchoi	FileViewerTest_1 - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-02-10 오전 11:27	2023-02-09 오전 8:35		
Form Only	wonhoonchoi	testsamplecase	-	2023-02-10 오전 7:39			
Scan Completed	wonhoonchoi	Multi Occlusion	-	2023-02-09 오전 9:26	2023-02-09 오전 9:01		
Scan Completed	wonhoonchoi	FileViewerTest_1 - Clone - Clone	8-Inlay / 10-Inlay / 12-Inlay / 13-Inlay / 14-Inl...	2023-02-09 오전 8:50	2023-02-09 오전 8:35		
Case Completed	wonhoonchoi	wonhoonchoi's Case	12-Inlay / 13-Inlay / 14-Inlay	2022-10-13 오후 10:30	2022-10-11 오후 5:16		
Case Completed	wonhoonchoi	wonhoonchoi's Case	-	2022-10-13 오후 10:28			

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
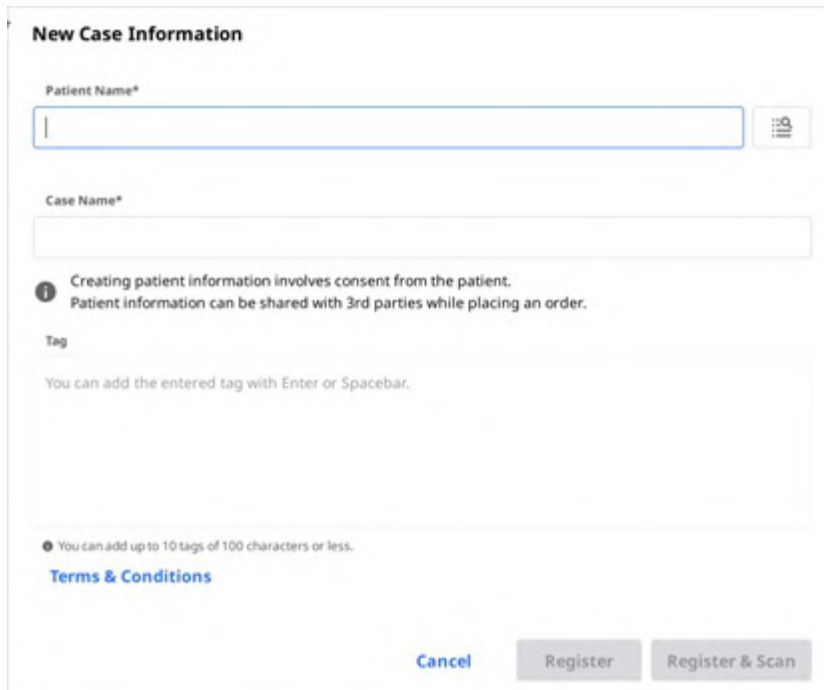
The 'New Case Information' form contains the following fields and elements:

- Patient Name***: A text input field with a dropdown icon on the right.
- Case Name***: A text input field.
- Information**: A note stating "Creating patient information involves consent from the patient. Patient information can be shared with 3rd parties while placing an order."
- Tag**: A text input field with a hint: "You can add the entered tag with Enter or Spacebar."
- Footer**: Includes a "Terms & Conditions" link, a "Cancel" button, and "Register" and "Register & Scan" buttons.

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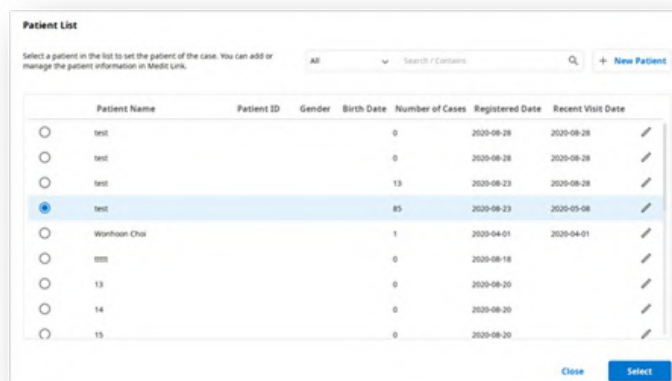The 'Patient List' dialog box displays a table of existing patients. The table has the following columns: Patient Name, Patient ID, Gender, Birth Date, Number of Cases, Registered Date, and Recent Visit Date. The fourth row is selected.

	Patient Name	Patient ID	Gender	Birth Date	Number of Cases	Registered Date	Recent Visit Date	
☐	test				0	2020-08-28	2020-08-28	
☐	test				0	2020-08-28	2020-08-28	
☐	test				13	2020-08-23	2020-08-28	
☒	test				85	2020-08-23	2020-05-08	
☐	Wonhoon Choi				1	2020-04-01	2020-04-01	
☐	test				0	2020-08-18		
☐	13				0	2020-08-20		
☐	14				0	2020-08-20		
☐	15				0	2020-08-20		

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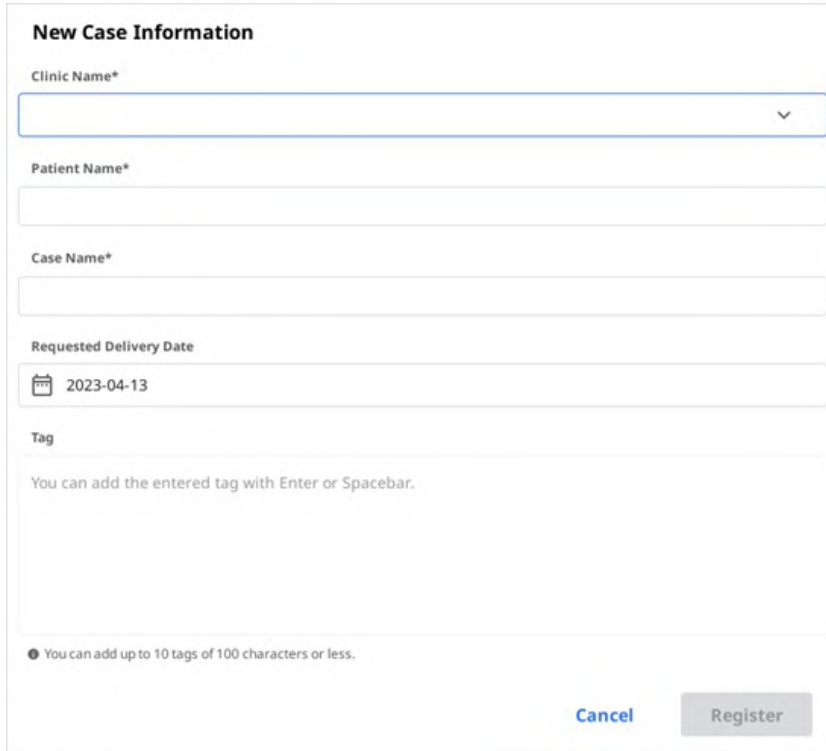
The form titled "New Case Information" contains the following fields and controls:

- Clinic Name***: A dropdown menu with a downward arrow icon.
- Patient Name***: A text input field.
- Case Name***: A text input field.
- Requested Delivery Date**: A date picker showing "2023-04-13".
- Tag**: A text area with the placeholder text "You can add the entered tag with Enter or Spacebar." and a note at the bottom: "You can add up to 10 tags of 100 characters or less."
- Buttons**: "Cancel" and "Register" buttons at the bottom right.

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✔ "Test Occlusion 00010101" is selected.

Case Name*

 Creating patient information involves consent from the patient.
 Patient information can be shared with 3rd parties while placing an order.

Tag

 You can add up to 10 tags of 100 characters or less.

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Patient List

Select a patient in the list to set the patient of the case. You can add or manage the patient information in Medit Link.

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Case Name*
test's Case

Requested Delivery Date
2022-04-18

Tag
You can add the entered tag with Enter or Spacebar.

You can add up to 10 tags of 100 characters or less.

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☐ Convert images to the below format

☐ JPG
 ☐ JPEG
 ☒ PNG
 ☐ BMP

☐ Compress Files
☐ Combine Individual Mesh
☐ Export All Occlusal Relationship
☐ Save Case Information as PDF

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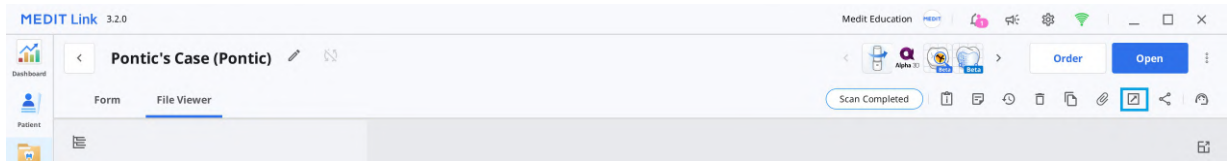
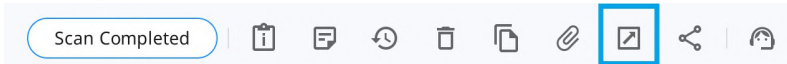
C:/Users/Medit-Wonhoonchoi/Desktop/test/

Folder Name

Multi Occlusion - Scan - Before Scanning

File Name

Multi Occlusion - Scan - Before Scanning

CAD Type

Maestro

Mesh Format

Convert mesh files to the below formats

☐ meditMesh
☐ OBJ
☒ **PLY**
☐ STL

☒ Do not convert the attached files

Image Format

☐ Convert images to the below format

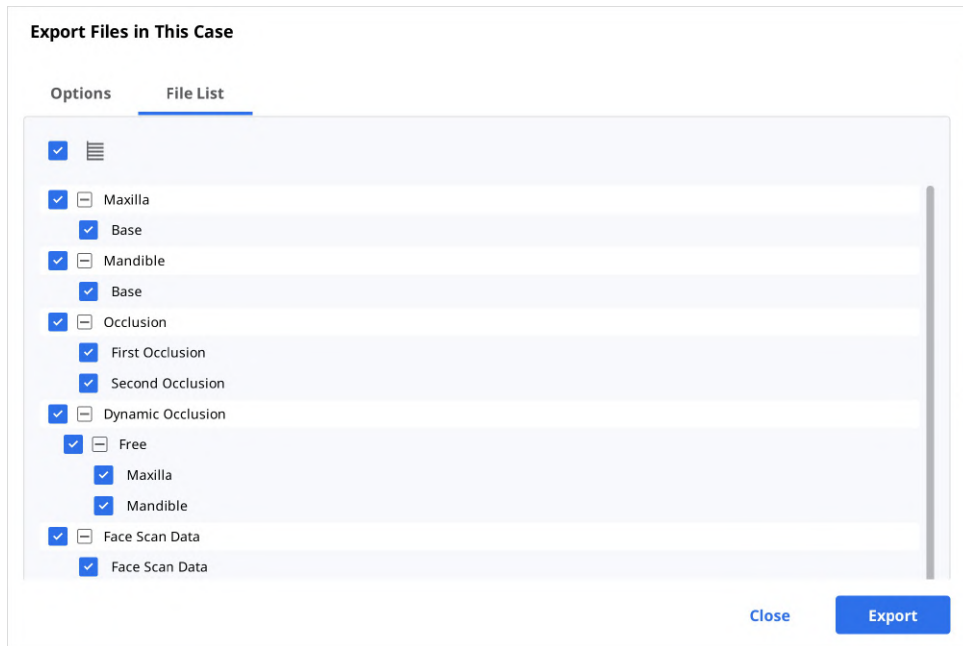
☐ JPG
☐ JPEG
☒ **PNG**
☐ BMP

☐ Compress Files
☐ Combine Individual Mesh
☐ Export All Occlusal Relationship
☐ Save Case Information as PDF [Preview](#)

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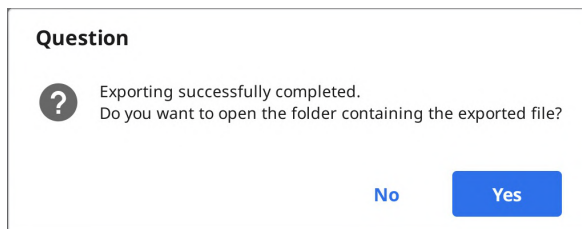
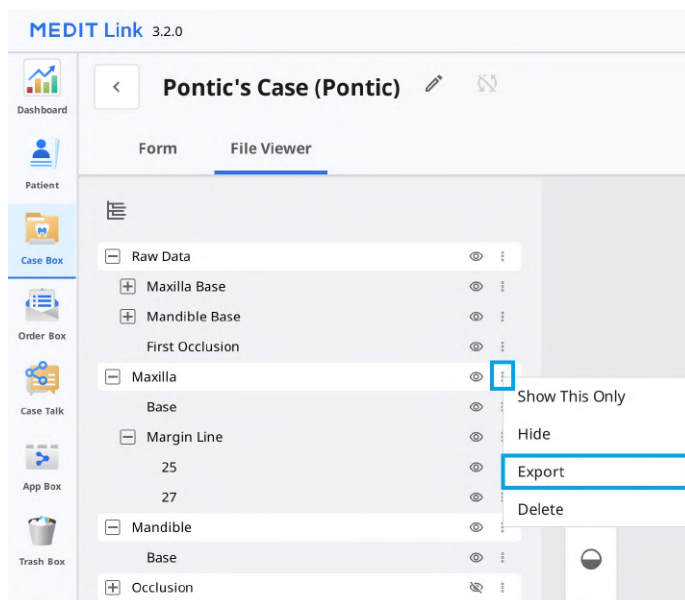
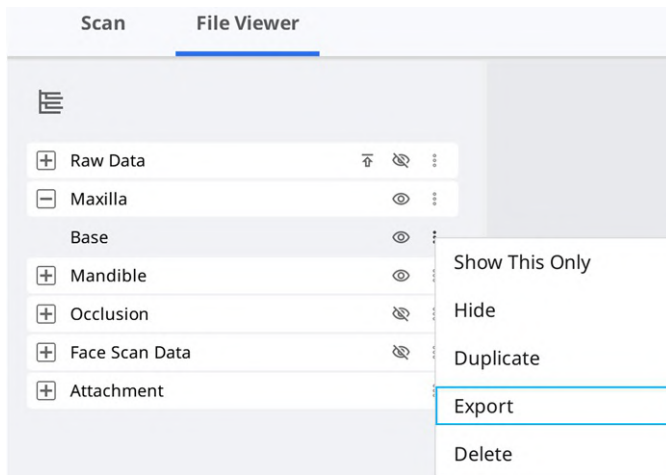
Options File List

Export Files To
C:/Users/Medit-Wonhoonchoi/Desktop/test/

Folder Name
Multi Occlusion - Scan - Before Scanning

File Name
Multi Occlusion - Scan - Before Scanning

CAD Type
Maestro

Mesh Format

Convert mesh files to the below formats

☐ meditMesh ☐ OBJ ☒ PLY ☐ STL

☒ Do not convert the attached files

Image Format

☐ Convert images to the below format

☐ JPG ☐ JPEG ☒ PNG ☐ BMP

☐ Compress Files

☐ Combine Individual Mesh

☐ Export All Occlusal Relationship

☐ Save Case Information as PDF [Preview](#)

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
The screenshot shows the 'Share through Case Talk' dialog box. It includes options for sharing with 'Anyone', setting an 'Expiration Date' (2022-04-27), and a 'PIN Code' section with a checkbox for 'PIN Code'. The 'Grant Download Permission' option is checked and highlighted with a blue box, with a note that it is a 'Premium Feature'. Below this, the 'Allow comments' option is also checked. At the bottom, there is a text area for a comment and a 'Get Link' button.

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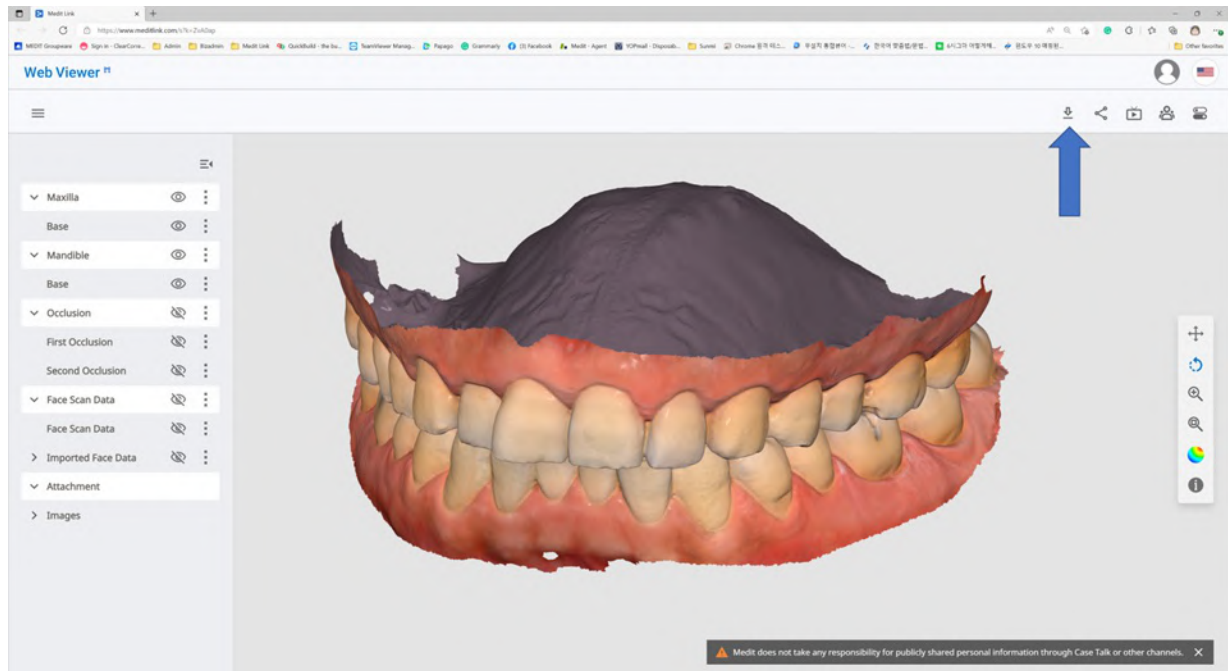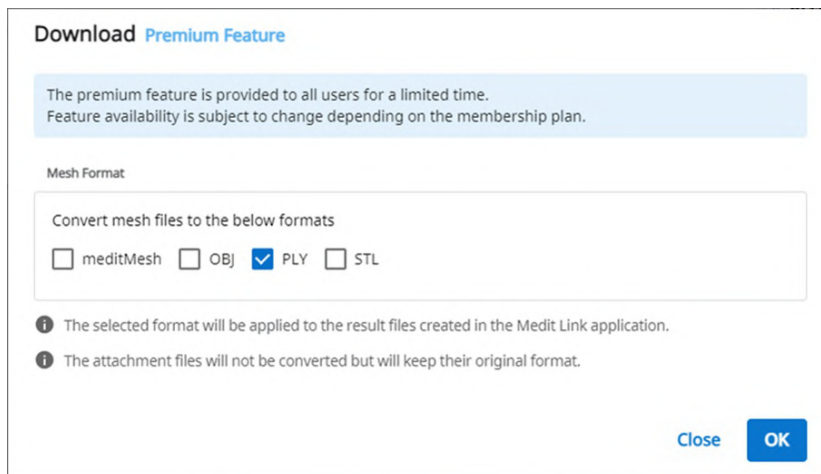
Case Name
mandibular movement - Clone - Clone

Patient Name
Test Occlusion

☒ Share patient information

Requested Delivery Date
2023-04-13

Delivery Time
AM 8:00 오전 8:15 PM 8:00

Memo

Tags
You can add the entered tag with Enter or Spacebar.

● You can add up to 10 tags of 100 characters or less.

☒ Include all occlusion relationships in the order

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