USER GUIDE

Medit Link

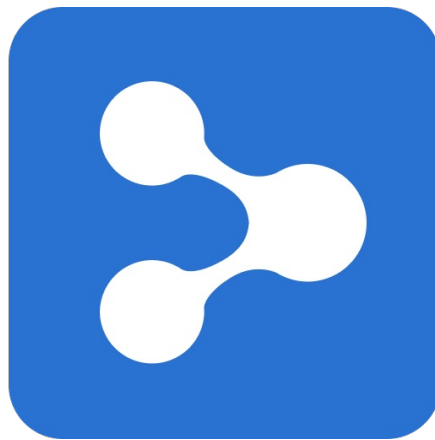
Table of Contents

Medit Link

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Administrator

Dentist (Manager)

All MEDIT Link functions are available.

- ✓ Order/Payment
- ✓ Member Management

Register

Administrator

Representative/Leader (Manager)

All MEDIT Link functions are available.

- ✓ Price Management for Dental Prostheses
- ✓ Order Management
- ✓ Member Management

Register

Member

Dental Hygienist and Dental Staff

In order to use MEDIT Link, join your group as a member. Case management and basic functions will be available after you join the group.

- ✓ Oral Scanning

Register

Member

Laboratory Employees

In order to use MEDIT Link, join your lab's group as a member. Case management and basic functions will be available after you join the group.

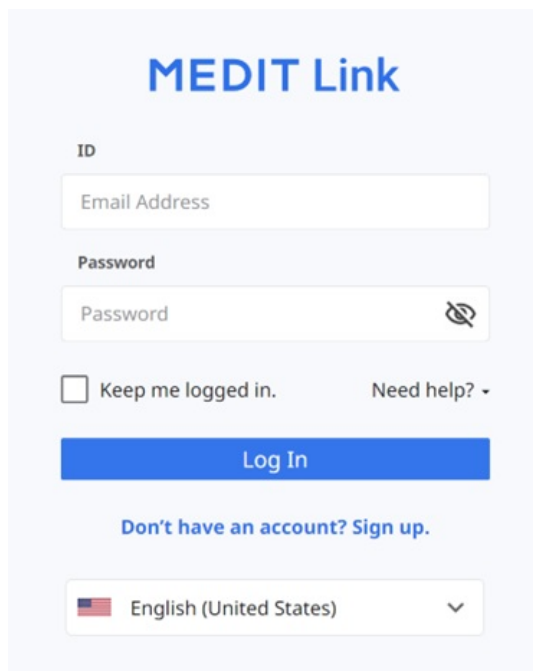
- ✓ Receiving Data for Making Dental Prostheses
- ✓ Sharing work files and production processes.

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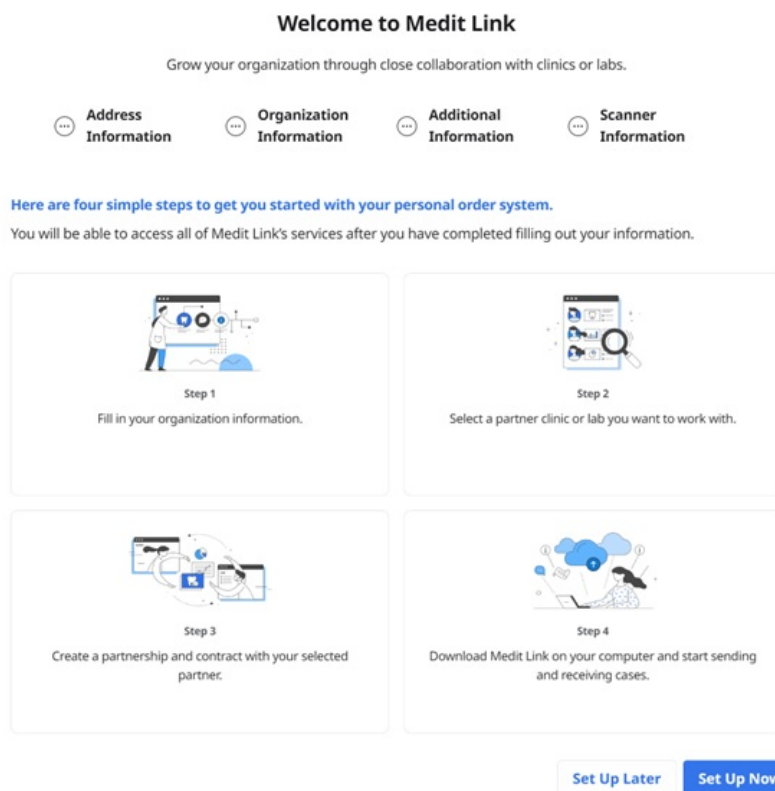
The login form for Medit Link features a light blue header with the logo. Below it, there are two input fields: 'ID' (labeled 'Email Address') and 'Password'. A 'Keep me logged in.' checkbox and a 'Need help?' link are positioned below the password field. A prominent blue 'Log In' button is centered below these elements. At the bottom, there is a link for 'Don't have an account? Sign up.' and a language selection dropdown menu currently set to 'English (United States)'.

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The 'Welcome to Medit Link' page provides a clear setup guide. It begins with the title and a subtitle: 'Grow your organization through close collaboration with clinics or labs.' Below this, four key areas of information are listed: Address, Organization, Additional, and Scanner. A four-step process is then outlined to get started with the personal order system. Each step includes an icon and a brief description: Step 1 (Fill in organization info), Step 2 (Select partner clinic/lab), Step 3 (Create partnership/contract), and Step 4 (Download app and start cases). The page concludes with two buttons: 'Set Up Later' and 'Set Up Now'.

Welcome to Medit Link

Grow your organization through close collaboration with clinics or labs.

Address Information Organization Information Additional Information Scanner Information

Here are four simple steps to get you started with your personal order system.

You will be able to access all of Medit Link's services after you have completed filling out your information.

Step 1
Fill in your organization information.

Step 2
Select a partner clinic or lab you want to work with.

Step 3
Create a partnership and contract with your selected partner.

Step 4
Download Medit Link on your computer and start sending and receiving cases.

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Medit Link

https://stage-www.meditlink.com/casebox?lang=en-US

MEDIT Groupware Sign in - ClearCorre... ChatGPT DeepL 번역: 세게... Papago Medit NAS Admin Medit Link Bloadmin Medit Link Enterprise QuickBuild - the bu... TeamViewer Manag... Grammarly (3) Facebook Medit - Agent > 다른 열거됨?

The more information you provide about your workplace, the higher we rank your reliability. wonhoon.choi

Case Box

Dashboard

Case Box

Order Box

Case Talk

Partners

Membership

Payment

My Info

View Options

Total Cases : 18

2023-01-06 ~ 2023-04-06

Case Name Case Name / Contains

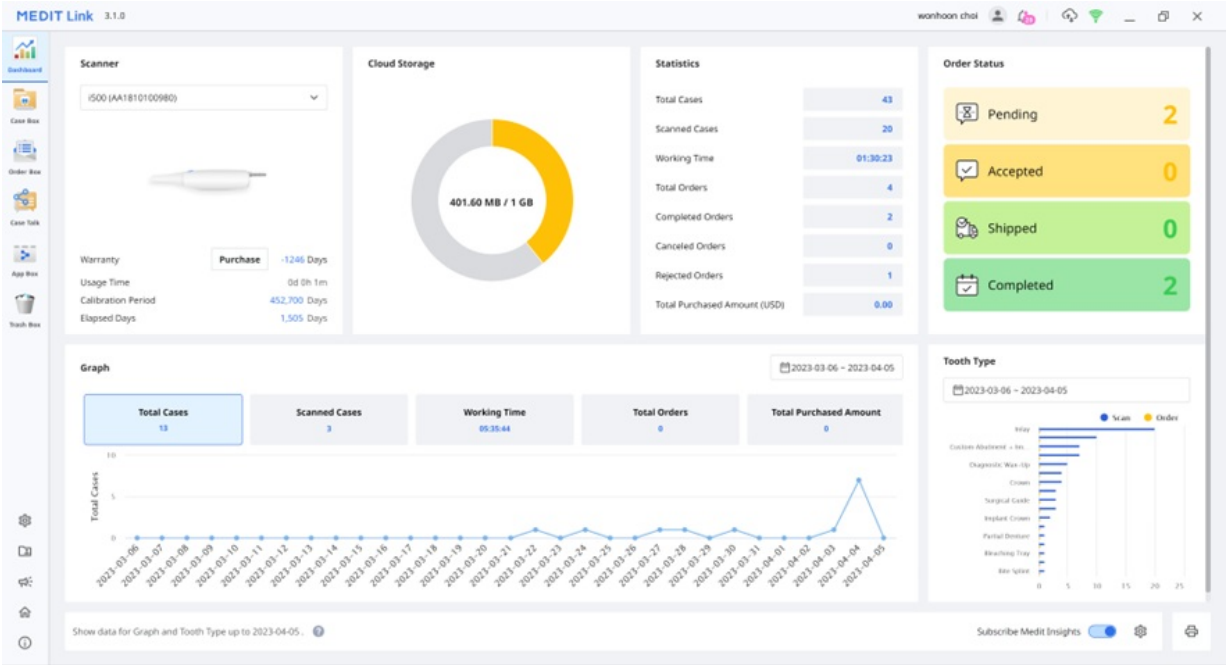
Status	Case Name	Patient Name	Form Information	Last Modified Date	Scanning Date	Order Date	Lab Name	
Scan Completed	mandibular movement - Clone -...	Test Occlusion	13, 14, 15, 16	2023-04-06 16:48	2023-04-03 10:11	-	-	Share
Form Only	Occlusion Test's Case	Occlusion Test	14, 15, 16, 17	2023-04-06 14:40	-	-	-	Share
Ordered	Occlusion Test's Case	Occlusion Test	21, 22, 23, 24, 25, 26	2023-04-06 14:13	2023-04-04 14:17	2023-04-06 14:13	WonhoonChoi_Lab...	Share
Case Completed	mandibular movement - Clone	Test Occlusion	13, 14, 15, 16	2023-04-06 11:55	2023-04-03 10:11	-	-	Share
Case Completed	test_initial's Case	test_initial		2023-04-06 11:55	-	-	-	Share
Case Completed	mandibular movement	Test Occlusion	13, 14, 15, 16	2023-04-06 11:55	2023-04-03 10:11	-	-	Share
Case Completed	COL-20532 test	test	12, 13, 14, 15, 16	2023-04-06 11:53	-	-	-	Share
Case Completed	Order-test	test	23, 24, 25, 26, 27	2023-04-05 18:36	-	2023-03-28 16:46	WonhoonChoi_Lab...	Share
Form Only	test_initial's Case	test_initial		2023-04-04 13:38	-	-	-	Share
Form Only	test_initial's Case	test_initial		2023-03-31 08:18	-	-	-	Share

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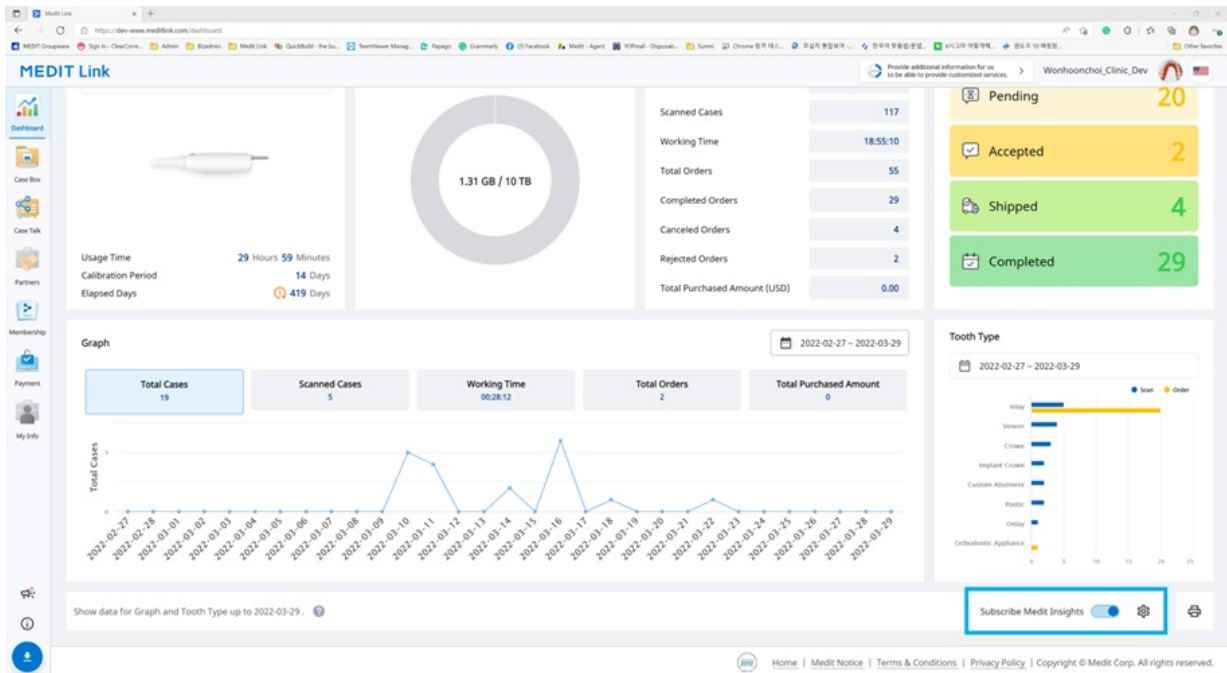
Settings

Receive the report via

☒ Medit Link account
 ☐ Another email address

Enter another email address to receive the report email.

The report will be sent on the 2nd day of every month.

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The screenshot shows the Medit Link application interface. The top bar displays the user name 'wonhoon choi'. The sidebar on the left contains navigation icons. The main content area is divided into two sections: 'Order' (left) and 'Order List' (right). The 'Order' section contains fields for Order ID, Patient Name, Partner Name, Scanning Date, Order Date, Requested Delivery Date, and Expected Delivery Date. The 'Order List' section contains a table with columns: No., Type, Method, Material, Shade, and Price. The table lists 8 items, all with a price of 0.00. The total price is USD 0.00.

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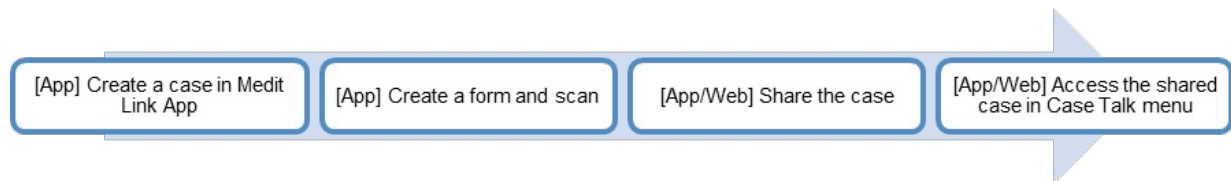
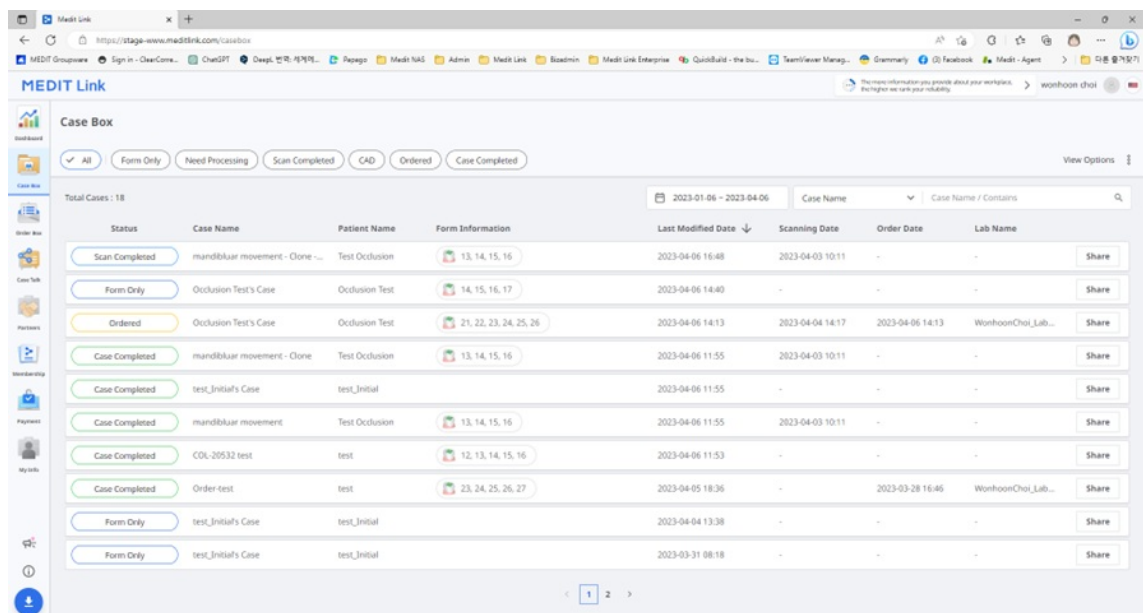
Share With

Share with specific people

Create a shared link and share it with specific people.

Expiration Date

2022-06-28

You can extend the period up to 90 days from today.

Share the patient's name

Grant Download Permission Premium Feature

If you check this option, all users who have access to the shared link can download the files.

The premium feature is provided to all users for a limited time. Feature availability is subject to change depending on the membership plan.

Allow comments

If you check this option, you will be able to communicate through the shared link.

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PIN Code

☐ PIN Code
Enter PIN Code

Set the PIN code using six digits for privacy protection.

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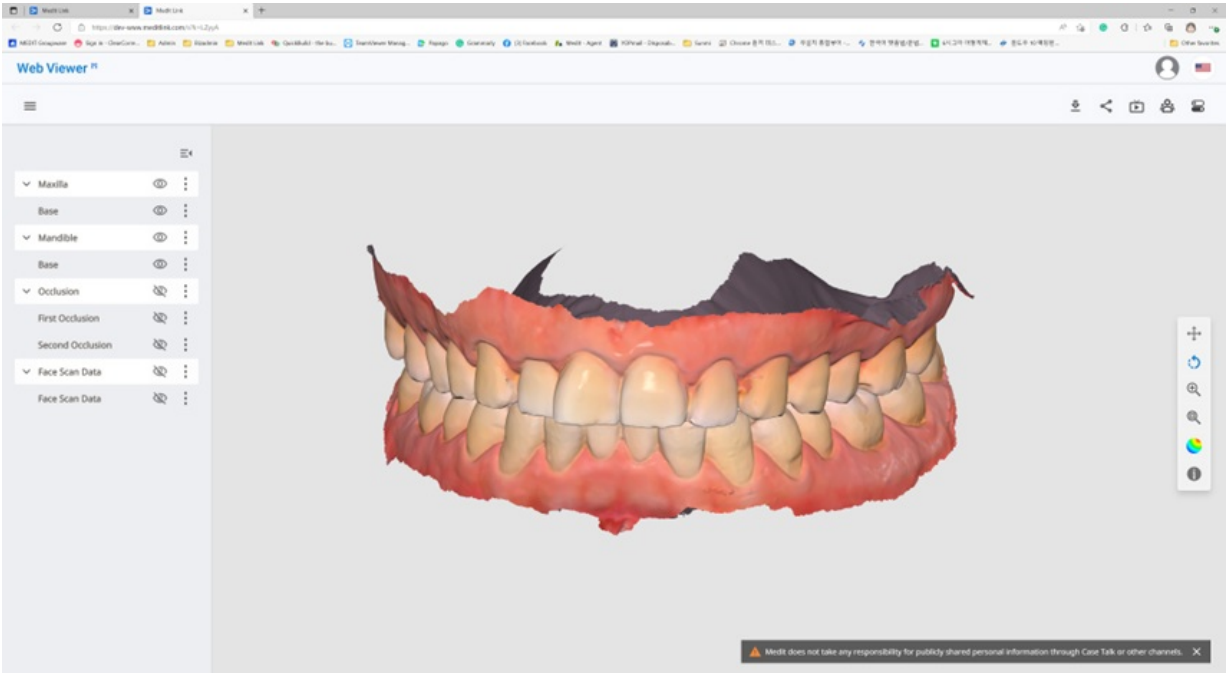
This is a close-up of the 'Create Annotation' dialog box. It features the title 'Create Annotation', the instruction 'Please enter your annotation.', a text input field with the placeholder 'Please enter your message', and two buttons at the bottom: 'Cancel' and 'OK'.

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This screenshot shows a chat-style interface for an annotation. At the top, it says 'wonhoonchoi Beta wonhoonchoi' with trash and close icons. Below is 'Annotation 1' with a timestamp '09-09 20:31'. At the bottom, there is a text input field with the placeholder 'Please enter your reply' and a 'Send' button.

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Invite Partner

Invite your desired partner to join Medit Link. When the lab accepts your invitation, a partnership is automatically formed.

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The screenshot displays the Medit Link user interface for the Membership section. On the left, a sidebar lists navigation options like 'Dashboard', 'Case Note', 'Order Note', 'Case Talk', 'Resources', 'Membership', 'Payment', and 'My Info'. The main content area is titled 'My Plan' and features an illustration of a person using a smartphone. Below the illustration, it states 'Standard Free' and lists benefits: 'Free plan for every member', '1 GB of cloud storage', and 'Efficient for new users'. A 'Subscribe' button is visible. To the right, the 'Cloud Storage' section shows a 'Set Period' from 2018-01-01 to 2023-04-06. A donut chart indicates that 353.21 MB out of 1 GB is used. Below the chart, there are checkboxes for 'Total Selected' (0.00 KB), 'Raw Data' (11.97 GB), and 'Result Files & Attachments' (353.21 MB). At the bottom, there is a note about payment difficulties and a 'Click' link.

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MEDIT Link

TEST MODE

Subscribe to Premium membership for Clinic

\$0.00

Then \$0.99 per day

Medit Link Membership

Premium membership for Clinic

\$0.99

Billed daily

Subtotal

\$0.99

👉 1 MONTH FREE – for first subscription

- \$0.99

100% off for a day

Total due today

\$0.00

Pay with card

Email

Card information

1234 1234 1234 1234

VISA

MM / YY

CVC

Name on card

Country or region

South Korea

☐ Save information to pay faster next time

Subscribe

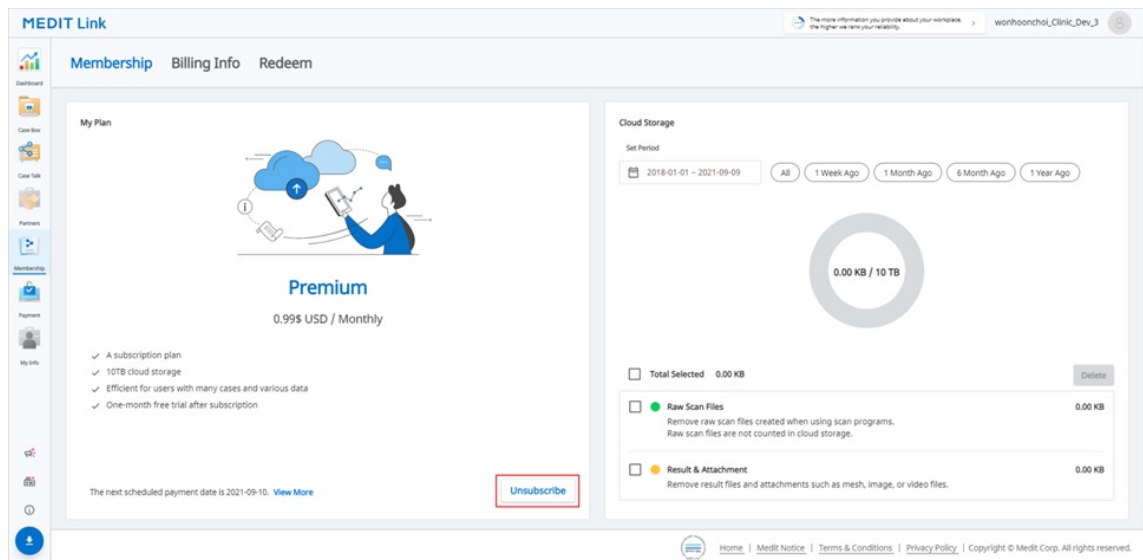
By confirming your subscription, you allow MEDIT SG PTE. LTD. to charge your card for this payment and future payments in accordance with their terms.

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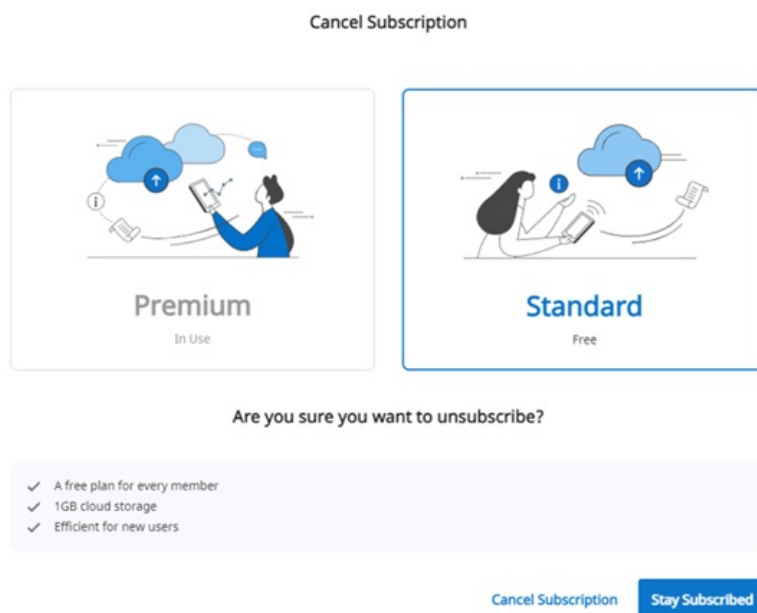
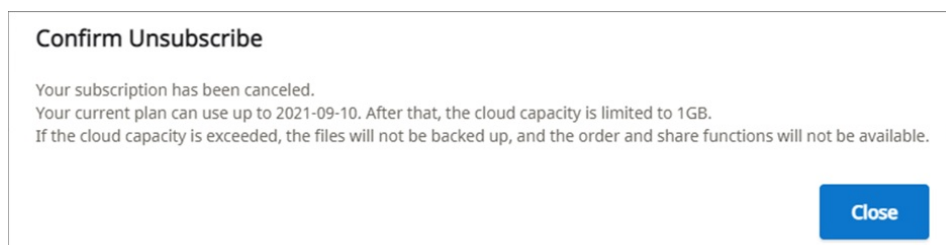
Confirm Unsubscribe

Your subscription will be canceled at the end of your billing period on 2021-09-10

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The screenshot shows the 'Billing Info' tab in the MEDIT Link interface. The 'My Plan and Billing Information' section displays the following details:

- Plan: Premium
- Amount: 0.99\$ USD / Monthly
- Next Payment Date: 2021-09-09
- Payment Method: Visa *****4242 (11/21) | Stripe | [Edit Payment Method](#)

Below this, a message states: 'If you haven't subscribed yet? Experience a differentiated premium.'

The 'Billing History' section contains a table with the following data:

Payment Date	Membership	From	To	Payment Amount	Status
2021-09-09	Premium	2021-09-09	2021-09-10	0.99 \$ (USD)	Payment Completed
2021-09-09	Premium	2021-09-09	2021-09-09	0 \$ (USD)	Payment Completed

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This screenshot is identical to the previous one, but the 'Edit Payment Method' button in the 'Payment Method' section is highlighted with a red rectangular box to indicate the next step in the process.

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Add

Go back

By adding a payment method, you agree to MEDIT SG PTE. LTD.'s Terms of Service and Privacy Policy.

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Billing

CURRENT PLANS

Premium membership for Clinic
\$0.99 per day
Your plan renews on September 10, 2021.

Premium membership for Clinic
\$0.99 per day
Your plan renews on September 10, 2021.

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Register

My Redeem Code

Promotion	Code Number	Dealer Name	Registration Date	From	To	Status
No search results were found.						

Home | Medit Notice | Terms & Conditions | Privacy Policy | Copyright © Medit Corp. All rights reserved.

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Register Redeem Code

You can register for the promotion or apply immediately.
Register allows you to apply and activate when you need it, If applied immediately, it will start after the expiration of the active promotion.
Periodic payments are stopped automatically.

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The screenshot shows the 'Medit Link' web application interface. The top navigation bar includes 'Membership', 'Billing Info', and 'Redeem'. The 'Redeem' section is active, displaying a 'Register Redeem Code' form with a 'Redeem Code' input field and a 'Register' button. Below this, the 'My Redeem Code' section contains a table with the following data:

Promotion	Code Number	Dealer Name	Registration Date	From	To	Status
Test Promotion - 1 month	[Redacted]	Medit Link	2021-09-09	-	-	Registered Activate
Test Promotion - 1 month	[Redacted]	Yoshida	2021-09-09	-	-	Registered Activate
Test Promotion - 1 month	[Redacted]	Medit Link	2021-09-09	-	-	Registered Activate
Test Promotion - 1 month	[Redacted]	Medit Link	2021-09-09	-	-	Registered Activate

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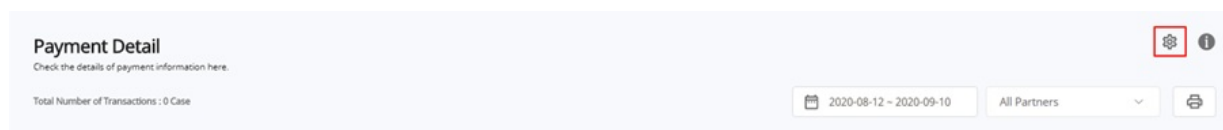
Account Information

① JPG, GIF, or PNG only. Max size of 10 MB.

Email Address
coldgreentea@naver.com [Change Email Address](#)

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Country
Korea, Republic of ▼

① Your chosen language is applied to the email sent by Medit Link and the notification.

Subscription Information

[Subscribe](#)

Standard

Organization Information

[Edit Information >](#)

Organization Name
wonhoonchoi_Lab3

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[My Account](#) [My Organization](#) [My Scanner](#) [My Price List](#)

[✓ Address Information](#) [✓ Organization Information](#) [⊖ Additional Information](#)

Address Information (Mandatory) *

Address 1 *
23 Incheon-ro 22-gil, Seongbuk-gu, Seoul, Korea [View Location on Map](#)

① Street Address, P.O. Box, Company Name, C/O
① Please enter the correct address so labs can check your company's location on the map.

Address 2
Apartment, Suite, Unit, Building, Floor, etc.

① Apartment, Suite, Unit, Building, Floor, etc.

Zip/Postal Code *
02835

City *
Seoul

Country *
Korea, Republic of ▼

State/Province *
Seoul

① If you have a contract history, you cannot change the country information. Please contact Medit Link's operation team.

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My Account My Organization My Scanner My Price List ⓘ

Teeth Arch AED (₩) ▼

✓ All (99/557) Inlay (8/12) Onlay (8/12) Coping (10/17) Denture Coping (5/5) Crown (15/26) Pontic (23/36) Post & Core (3/4) Veneer (8/11) Implant Crown (12/53) Custom Abutment (2/3) Surgical Guide (1/1) Offset Substructure (4/5) Partial Denture (0/7) Diagnostic Wax-Up (0/1) Custom Abutment + Implant Crown (0/159) Custom Abutment + Coping (0/45) Post & Core + Crown (0/100) Post & Core + Coping (0/60)

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Notice
NFT Service Termination Notice
2023-04-19

To our users,

We want to thank all Medit Link users who have supported and used our NFT service.

Unfortunately, we must inform you that this service will be terminated following the upcoming program update.

Please see the details below for more information:

- **Feature in Question**
 - Share via NFT (in Medit Link Case Talk)
- **Effective Date**
 - Starting from the update in the first half of 2023 (scheduled for April 27)

※ The update schedule is subject to change.

We'll continuously work to provide better service.

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Mark All As Read

2

Delivery
Wonhoonchoi_Lab_Dev has dispatched the ordered package.

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Order
Wonhoonchoi_Lab_Dev has accepted the order.

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Order
Wonhoonchoi_Lab_Dev has accepted the order.

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Order
Wonhoonchoi_Lab_Dev has accepted the order.

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Order
Wonhoonchoi_Lab_Dev has accepted the order.

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Order
Wonhoonchoi_Lab_Dev has accepted the order.

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Work
Wonhoonchoi_Clinice_Dev_Member registered a new case test's Case.

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Work
Wonhoonchoi_Clinice_Dev_Member registered a new case test's Case.

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Patient Management

Allows to select patients to delete or merge their information from different cases.
Allows to merge together information of a single patient spread across multiple cases. After merging, all cases of the patient are combined together under the patient name you selected.
Press 'Delete' to remove patients from the system.

AB Search / Contains + New Patient

Delete Merge Patients Patients with No Cases

	Patient Name	Patient ID	Gender	Birth Date	Number of Cases	Date Registered	Recent Visit Date	
☒	5-9_Scan_CAMOrder_Teeth2				3	2020-09-18	2020-09-18	
☒	5-9_Scan_CAMOrder_Teeth2				1	2020-09-18	2020-09-18	
☒	test				2	2020-09-17	2020-09-17	
☒	test				22	2019-07-26	2020-09-17	
☒	test				1	2020-09-17	2020-09-17	
☒	ASD Demo				1	2020-09-16	2020-09-16	
☒	ASD Demo				1	2020-09-16	2020-09-16	
☐	gfm				1	2020-09-15	2020-09-15	
☐	Medit Link 2.3.0			2020-9-24	2	2020-09-14	2020-09-14	
☐	test				1	2020-09-13	2020-09-13	
☐	株式会社				2	2020-09-13	2020-09-13	
☐	test3				1	2020-09-13	2020-09-13	
☐	test1				1	2020-09-13	2020-09-13	
☐	testq1231				26	2019-07-25	2020-09-11	

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Email Address

Memo

Creating patient information involves consent from the patient.
Patient information can be shared with 3-parties on order.
[Terms & Conditions](#)

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Patient Management

Allows to select patients to delete or merge their information from different cases.
 Allows to merge together information of a single patient spread across multiple cases. After merging, all cases of the patient are combined together under the patient name you selected.
 Press 'Delete' to remove patients from the system.

All Search / Contains + New Patient

Delete Merge Patients Patients with No Cases

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Merge all patient information to

Please select the patient you'd like to merge all data to.
After merging all data together, all information in other patient files will be deleted.

	Patient Name	Patient ID	Gender	Birth Date	Number of Cases	Date Registered	Recent Visit Date
☒	5-9_jScan_CAMOrd...				1	2020-09-18	2020-09-18
☐	test				2	2020-09-17	2020-09-17
☐	test				22	2019-07-26	2020-09-17

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Shown

Hide All

- :: Status
- :: Case Name
- :: Patient Name
- :: Form Information
- :: Last Modified Date
- :: Scanning Date

Hidden

Show All

- :: Birth Date
- :: Order Date
- :: Lab Name

Cancel

Apply

MEDIT Link 3.0.0 Wonhoon Choi_clinic

Case Box

Total Count: 514 2018-01-01-2022-09-29 All / Contains

Status	Case Name	Patient Name	Form Information	Last Modified Date	Scanning Date
Form Only	test's Case	test	-	2022-09-29 오전 8:45	
Ordered	test's Case	test	14-Implant Crown / 15-Implant Crown / 16-Implant Crown	2022-09-29 오전 8:13	2022-09-29 오전 7:56
Form Only	test's Case	test	-	2022-09-28 오후 5:56	
Form Only	test's Case	test	-	2022-09-28 오전 10:41	
Scan Completed	test's Case	test	24-Inlay / 25-Inlay / 26-Inlay	2022-09-28 오전 9:27	
Scan Completed	Multi Occlusion - Clone - Clone - Clone ...	test	24-Inlay / 25-Inlay / 26-Inlay	2022-09-27 오후 7:03	2021-05-14 오전 9:27
Scan Completed	Preop 사용된 케이스 - Clone	straumann	15-Inlay / 26-Pontic	2022-09-27 오후 5:09	2021-11-18 오후 6:53

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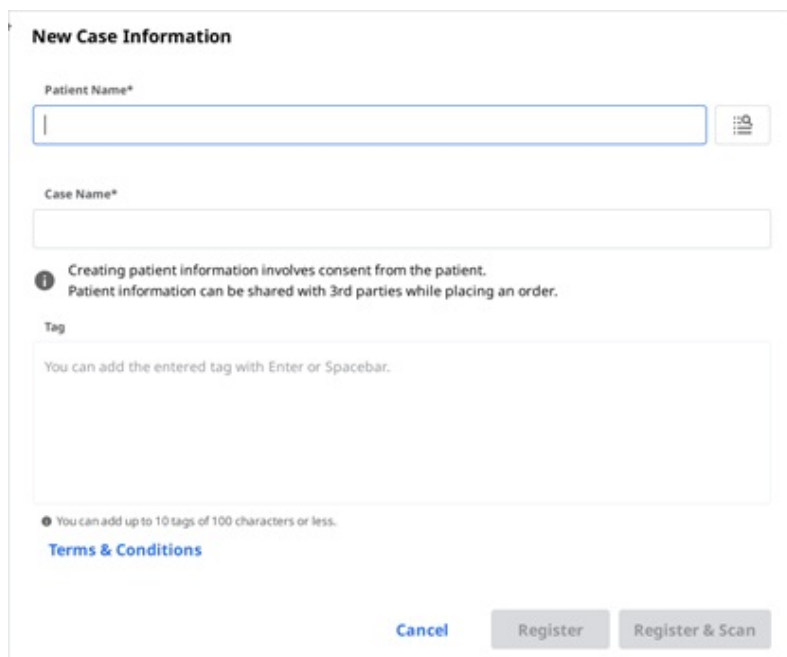
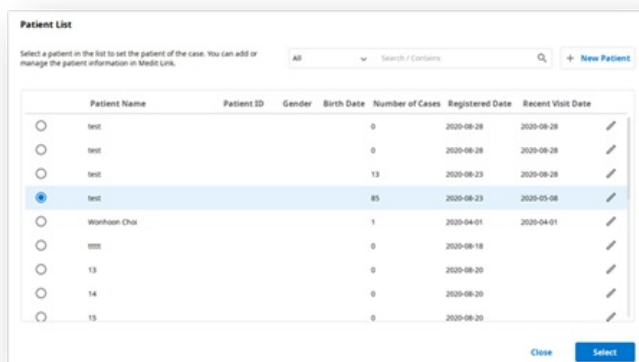
The form titled "New Case Information" contains the following fields and elements:

- Patient Name***: A text input field with a search icon button to its right.
- Case Name***: A text input field.
- Information**: A note stating "Creating patient information involves consent from the patient. Patient information can be shared with 3rd parties while placing an order."
- Tag**: A text area with the instruction "You can add the entered tag with Enter or Spacebar."
- Footer**: Includes a note "You can add up to 10 tags of 100 characters or less.", a link for "Terms & Conditions", and three buttons: "Cancel", "Register", and "Register & Scan".

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The "Patient List" dialog shows a table of existing patients. The first four rows have a radio button for selection. The fifth row is highlighted with a blue background.

	Patient Name	Patient ID	Gender	Birth Date	Number of Cases	Registered Date	Recent Visit Date	
☐	test				0	2020-08-28	2020-08-28	
☐	test				0	2020-08-28	2020-08-28	
☐	test				13	2020-08-23	2020-08-28	
☒	test				85	2020-08-23	2020-05-08	
☐	Worham Choi				1	2020-04-01	2020-04-01	
☐	test				0	2020-08-18		
☐	13				0	2020-08-20		
☐	14				0	2020-08-20		
☐	15				0	2020-08-20		

Buttons at the bottom: "Close" and "Select".

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The screenshot shows a form titled "New Case Information". It contains the following fields and elements:

- Clinic Name***: A dropdown menu with a downward arrow.
- Patient Name***: A text input field.
- Case Name***: A text input field.
- Requested Delivery Date**: A date picker showing "2023-04-13".
- Tag**: A text area with the instruction "You can add the entered tag with Enter or Spacebar." and a note "You can add up to 10 tags of 100 characters or less."
- At the bottom right, there are two buttons: "Cancel" and "Register".

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The screenshot shows a form titled "Edit Case Information". It contains the following fields and elements:

- Patient Name***: A text input field containing "Test Occlusion". To its right is a search icon.
- A green checkmark message: "Test Occlusion 00010101" is selected.
- Case Name***: A text input field containing "mandibular movement - Clone - Clone".
- An information icon with a note: "Creating patient information involves consent from the patient. Patient information can be shared with 3rd parties while placing an order."
- Tag**: A text area with the instruction "You can add the entered tag with Enter or Spacebar." and a note "You can add up to 10 tags of 100 characters or less."
- A link: "Terms & Conditions".
- At the bottom right, there are two buttons: "Cancel" and "Apply".

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Patient List

Select a patient in the list to set the patient of the case. You can add or manage the patient information in Medit Link.

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2022-04-18

Tag

You can add the entered tag with Enter or Spacebar.

You can add up to 10 tags of 100 characters or less.

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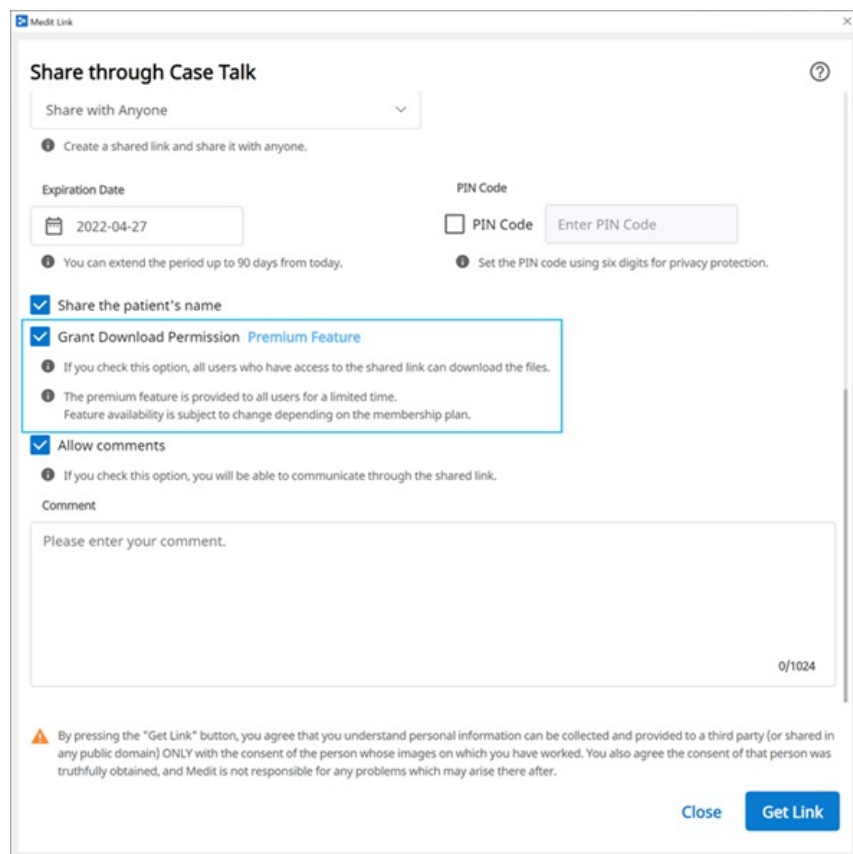
Share through Case Talk

Share with Anyone

1 Create a shared link and share it with anyone.

Expiration Date: 2022-04-27

PIN Code: ☐ PIN Code

1 You can extend the period up to 90 days from today. 1 Set the PIN code using six digits for privacy protection.

☒ Share the patient's name

☒ Grant Download Permission [Premium Feature](#)

1 If you check this option, all users who have access to the shared link can download the files.

1 The premium feature is provided to all users for a limited time. Feature availability is subject to change depending on the membership plan.

☒ Allow comments

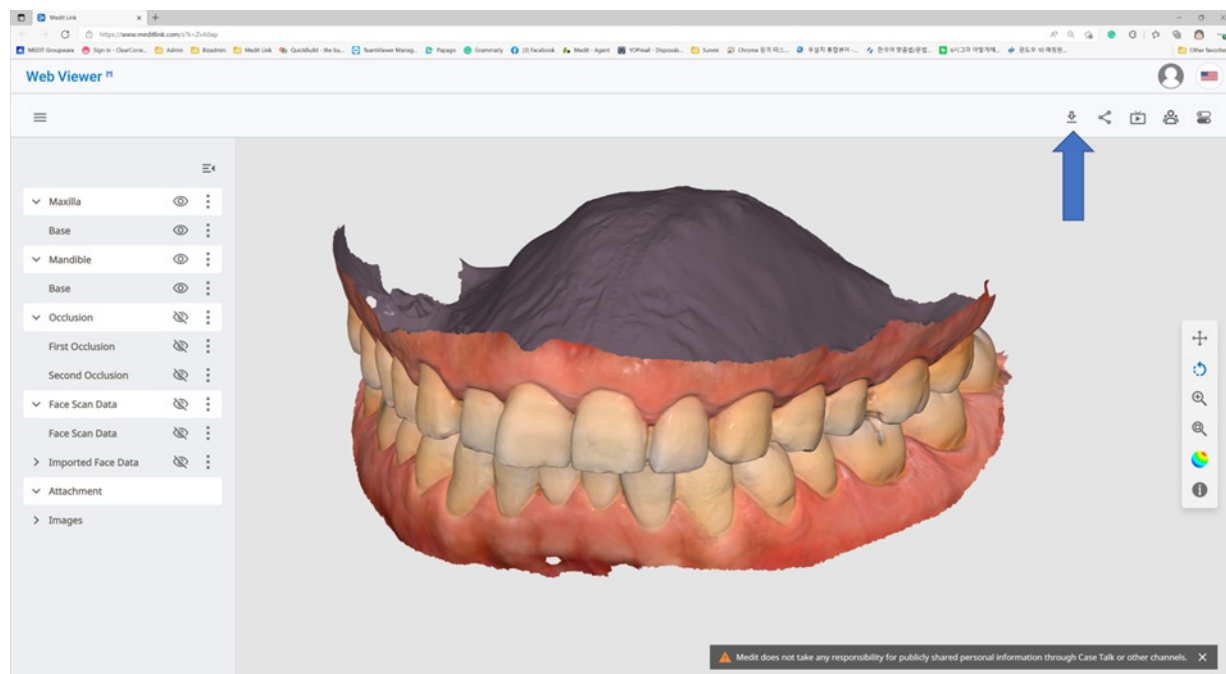
1 If you check this option, you will be able to communicate through the shared link.

Comment: Please enter your comment. 0/1024

By pressing the "Get Link" button, you agree that you understand personal information can be collected and provided to a third party (or shared in any public domain) ONLY with the consent of the person whose images on which you have worked. You also agree the consent of that person was truthfully obtained, and Medit is not responsible for any problems which may arise there after.

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Download **Premium Feature**

The premium feature is provided to all users for a limited time.
Feature availability is subject to change depending on the membership plan.

Mesh Format

Convert mesh files to the below formats

☐ meditMesh ☐ OBJ ☒ PLY ☐ STL

i The selected format will be applied to the result files created in the Medit Link application.

i The attachment files will not be converted but will keep their original format.

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Case Name
mandibular movement - Clone - Clone

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☒ Include all occlusion relationships in the order
☒ Include Tags In The Order

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Patient Name*

Case Name*

i Creating patient information involves consent from the patient.
Patient information can be shared with 3rd parties while placing an order.

Tag

Doctorname × Specialmaterial × SpecialComment ×

i You can add up to 10 tags of 100 characters or less.

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MEDIT Link 3.1.0

mandibluar movement - Clone - Clone (Test Occlusion)

Form File Viewer

Scan Completed

Occlusal Relationship 1

Memo

Tags
Doctorname Specialmaterial SpecialComment

The image shows a screenshot of the MEDIT Link 3.1.0 software interface. The main window displays the 'mandibluar movement - Clone - Clone (Test Occlusion)' case. A 'File Viewer' tab is active, showing a 3D model of the occlusion. A 'Memo' dialog box is open, displaying the 'Tags' section with three tags: 'Doctorname', 'Specialmaterial', and 'SpecialComment'. A blue box highlights the 'Tags' section in the dialog box. A blue arrow points from the 'Tags' section in the dialog box to the 'Tags' section in the main window's 'File Viewer' tab.

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Copyright (c) Medit Corp. All rights reserved.